



PRESSALIT®

Product Item number	Diaper basket, 15.8" x 8.4" x 6.9" RT767
Product description	VVS:615801767 NRF:7501034 EAN:5708590171074 HMI: 124570 NBS:N13:449 Basket for diapers, A:15.8" x B:8.4" x C: 6.9". Basket for diapers, A:15.8" x B:8.4" x C: 6.9".
Materials	Core material: Steel Surface material: Rilsan
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT®

Product	Diaper basket, 15.8" x 8.4" x 6.9"
Item number	RT767
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: - Clinimax Difficil-S - Erisan OXY+ - Sprit (husholdningsprit) - Suma Bac D10 - Virkon - Rodalon Desinfektionsspray - Lysoformin Spezial - CaviCide - Sufanios Premium - Tvättssprit 70% - WipeClean UniversalCleaning - Chlorine Disinfection Wipes - Klorilli (Kiilto Clean) NON-DISINFECTANTS: - Suma Star free D1 - Taski 100 - Taski 200 - Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.

pressalit.com

PRESSALIT®