


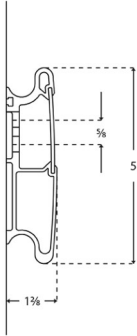

<p>Product Item number</p>	<p>PLUS wall track, 23.6" R9806112</p>
<p>Color Product description</p>	<p>VVS:615841106 NRF:7501013 EAN:5708590305738 HMI: 71739</p> <p>Anthracite gray</p>  <p>PLUS wall tracks are used for mounting of support arms, shower seats and sink brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track.</p> <p>PLUS wall track, 23.6", for horizontal mounting. The cover plate hides bolts, wires and simplifies cleaning. Materials: Aluminium and polystyrene. Excl. mounting kit.</p>
<p>Adjustability Product capacity</p>	<p>Fixed mounting</p> <p>Maximum load: Up to 2200 lbs. depending on the wall type.</p>
<p>Materials Ergonomy Features</p>	<p>The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.</p> <p>The track distributes the weight of the individual bathroom product, for example a sink bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions.</p> <p>The horizontal track makes it possible to shift the product along the wall to generate extra space as needed. The front cover plate conceals mounting bolts and any cables.</p>
<p>Warranty</p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



Product	PLUS wall track, 23.6"
Item number	R9806112
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>



Product	PLUS wall track, 23.6"
Item number	R9806112

Mounting in different wall types

	<p>Concrete >B25 V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p>Solid Brick >= Mz12 V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p>Solid Sand Lime Brick >= Hlz12 V8620 - Mounting kit, 3 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p>Aerated Concrete P4 V8670 - Mounting kit, 3 bolts M8 x 35, plugs FTP M 8. To be ordered separately.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8620 - Mounting kit, 6 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.</p>	



PRESSALIT®

Product	PLUS wall track, 23.6"
Item number	R9806112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®