R2120 - R2121
R2140 - R2141
R2142 - R2143

R4820, R482019
R4825, R482519
R4826, R4840
R484019, R4842
R484219, R4845
R484519, R4847
R484719

MATRIX

Wash basin lifter – Operation and maintenance manual	en
Vaskeophæng – Brugs- og vedligeholdelsesvejledning	dk
Waschtisch-Lifter – Gebrauchs- und Pflegeanleitung	de
Support de lavabo – Manuel d'utilisation et d'entretien	fr
Muurframe voor wastafel – Gebruiks- en onderhoudshandleiding	nl
Upphängning till tvättställ – Bruks- och underhållsanvisning	sv
Oppheng til servant – Bruks- og vedlikeholdsveiledning	no
Soporte de lavabo - Instrucciones de uso y mantenimiento	es
Supporto per lavabo - Manuale d'uso e manutenzione	it
Wspornik umywalki – Instrukcja obsługi i konserwacji	pl
洗面盆支架 - 操作和维护手册	中文

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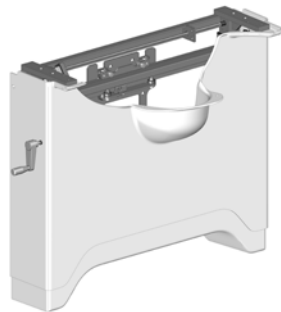
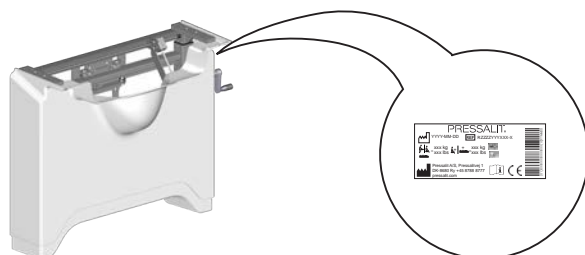
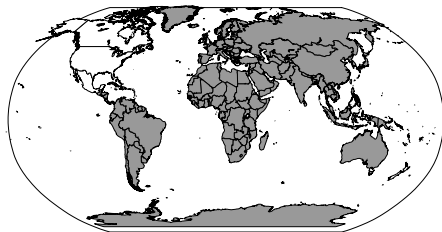
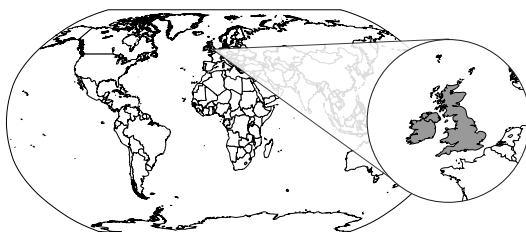
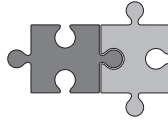
REF
R482019, R484019, R484219

REF
R482519, R484519, R484719

		
R4820, R4825, R4826	19 kg / 42 lbs	150 kg / 330 lbs
R4840, R4842, R4845, R4847	15 kg / 33 lbs	
R482019, R482519, 484019, R484519, 484219, R484719	20,5 kg / 45 lbs	

R2140 > R4840, R4845,
R484019, R484519

R2141 > R4840, R4845,
R484019, R484519

R2142 > R4842, R4847,
R484219, R484719

R2143 > R4842, R4847,
R484219, R484719

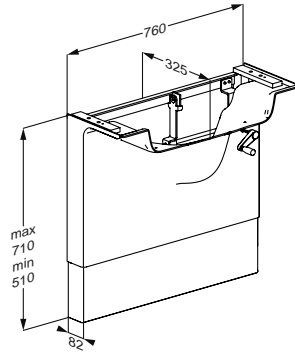
R2120 > R4820, R4825, R4826,
R482019, R482519


R2121 > R4820, R4825, R4826,
R482019, R482519


mm


R4820, R4825, R4826

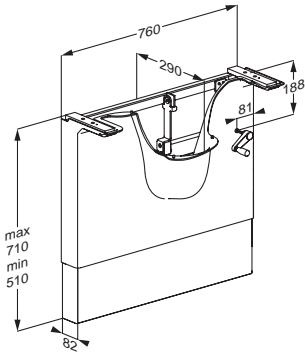
R4840, R4845

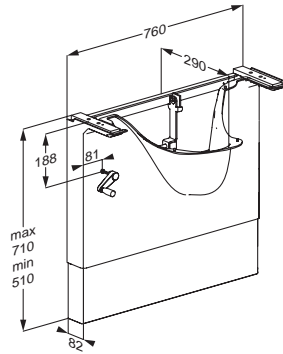
R4842, R4847

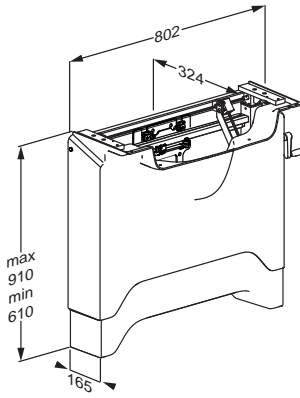
R482019, R482519

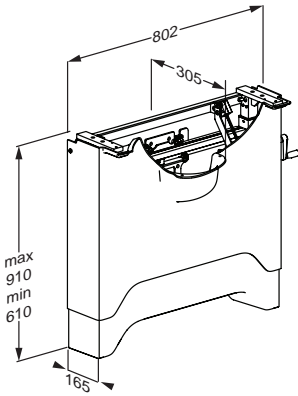
R484019, R484519

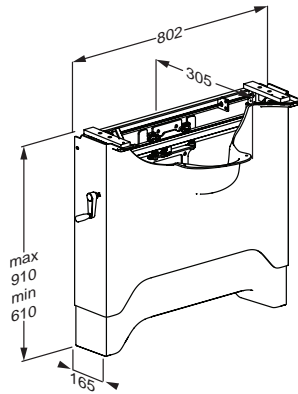
R484219, R484719

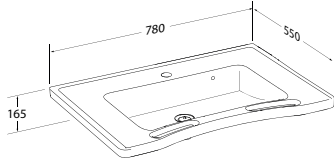
R2120

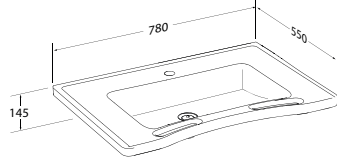
R2121

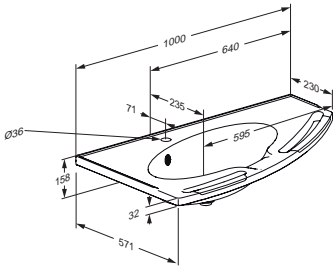
R2140

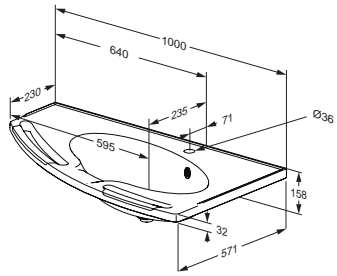R2142


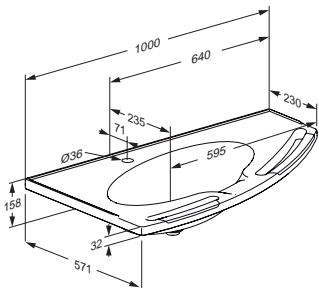
R2141

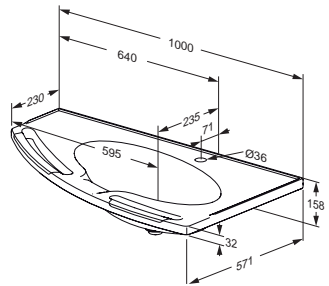
R2143

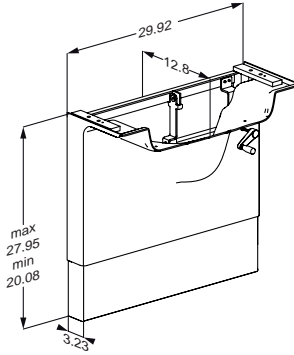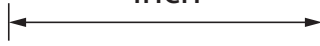
inch


R4820, R4825, R4826

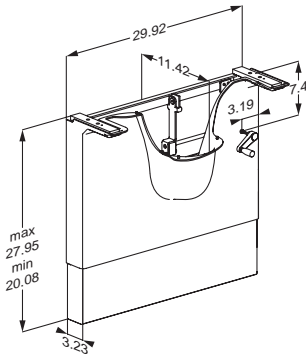
R4840, R4845

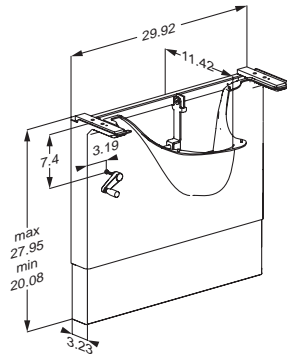
R4842, R4847

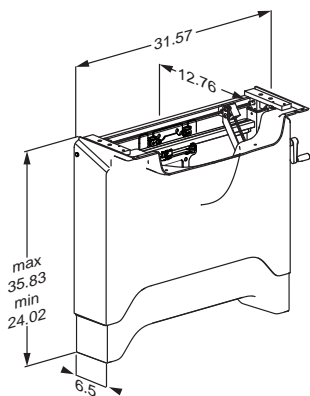
R482019, R482519

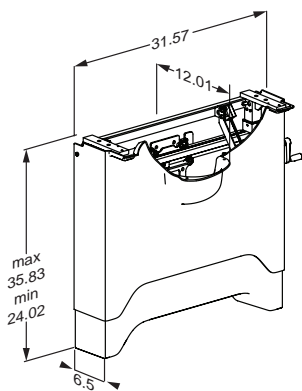
R484019, R484519

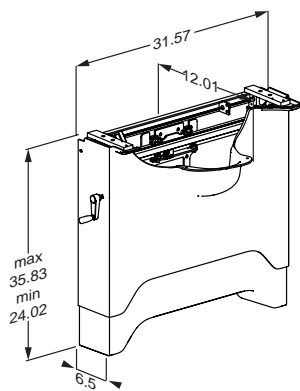
R484219, R484719

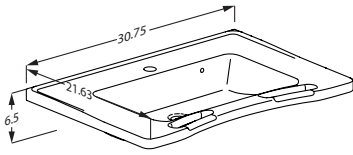
R2120

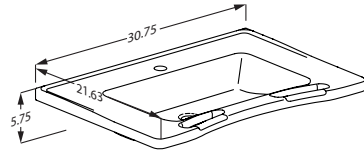
R2121

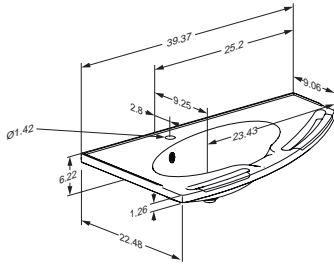
R2140

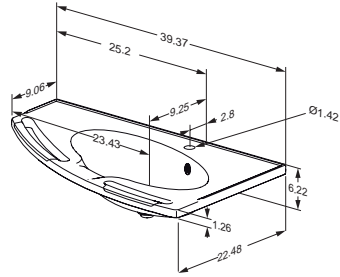R2142

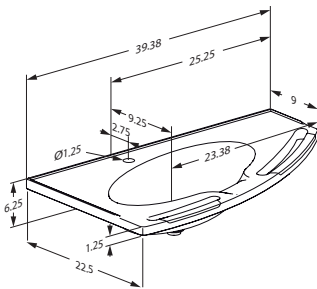R2141

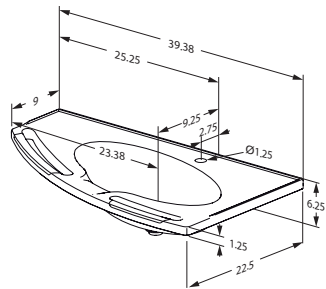R2143

Content

Mounting instruction	13
Symbols used in this manual	14
General safety	15
Safety distances	16
Intended use	18
Intended operator profile	18
Intended user profile	18
Operating environment	18
Contraindications for use	18
Operating instructions	19
Sideways adjustment	20
Cleaning	21
Maintenance	22
Maintenance inspection	22
Product label	25
Trouble-shooting	26
Disposal and recycling	26

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

 WARNING				<p>WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>
 CAUTION				<p>CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>
IMPORTANT				<p>IMPORTANT indicates important information about handling and use of the product.</p>
NOTICE				<p>NOTICE is used to address practices not related to personal safety.</p>
 <p>Head injury warning</p>	 <p>Hands injury warning</p>	 <p>Genitalia injury warning</p>	 <p>Feet injury warning</p>	
 <p>Compatible products</p>	 <p>Article number/ versions</p>	 <p>Product weight</p>	 <p>Max. user weight</p>	

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

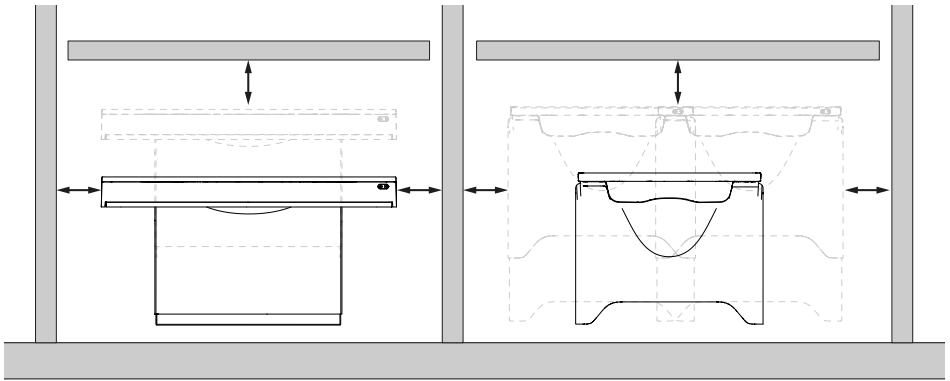
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.





Safety distances

Make sure that the wash basin lifter is at safe distance from surrounding objects. Check the distances before any use.

en

ISO 17966:2016		
	Max 120 mm (4.72") Min 300 mm (11.81")	Max 60 mm (2.36") Min 300 mm (11.81")
	Max 8 mm (0.31") Min 25 mm (0.98")	Max 4 mm (0.16") Min 25 mm (0.98")
	Max 35 mm (1.38") Min 120 mm (4.72")	Max 25 mm (0.98") Min 120 mm (4.72")
	Max 8 mm (0.31") Min 75 mm (2.95")	Max 8 mm (0.31") Min 75 mm (2.95")

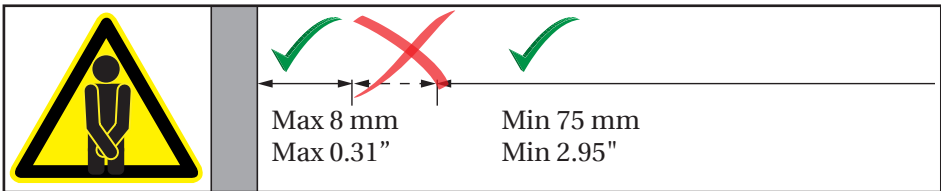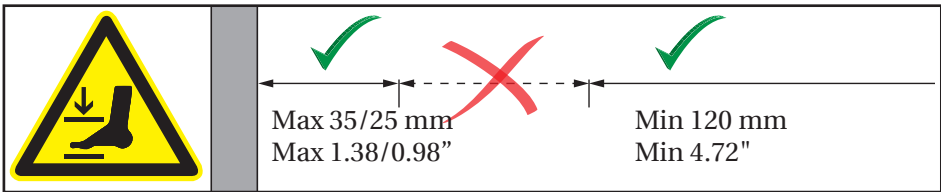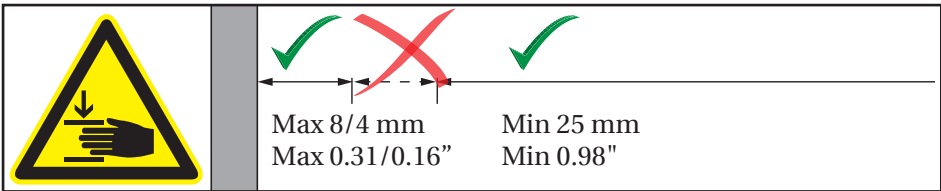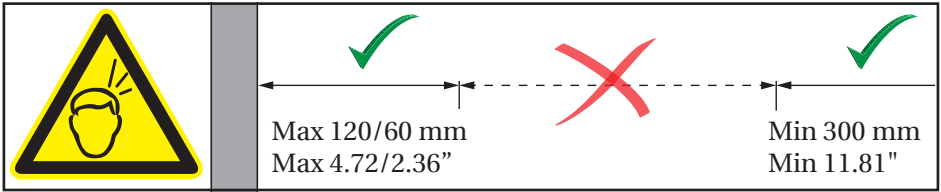 **WARNING**

If the unit moves within the safety distances, you must keep an especially close eye on surrounding people and objects to avoid crushing hazards.

en

Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Intended user profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions.

Contraindications for use

People who are not capable of keeping their balance in an upright position are not recommended to use this wash basin lifter.

en

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Operating instructions

The wash basin lifter models R4820, R4825, R4826, R4840, R4845, R4842 and R4847 can be adjusted 200 mm (7.87") up and down.

Max load is 150 kg/330 lbs.

Raise

Lower

Turn the crank handle clockwise to raise the wash basin, and counterclockwise to lower it.

CAUTION

Never place objects underneath the wash basin.

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

en

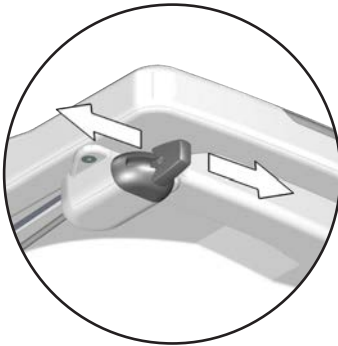
Sideways adjustment

The wash basin lifter models R482019, R482519, R484019, R484519, R484219 and R484719 can be adjusted 300 mm (11.81") up and down, and also 500 mm in total (19.69") to either the left or right.

Max load is 150 kg/330 lbs.

Unlock the sliding mechanism by pressing and holding the side adjustment button to either the left or right.

en

Grasp the handles of the sink and push the wash basin lifter to either the left or right. The lifter slides quite easily, so you don't have to use much effort.

When the wash basin lifter is in the desired position, release the side adjustment button and the lifter will lock in place with a click.

CAUTION

Make sure the locking mechanism is properly engaged before any use or height adjustment. This is ensured by pushing the wash basin lifter a little to the right or left until a click is heard.

Be careful not to injure your feet when moving the wash basin lifter sideways

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

en

Maintenance

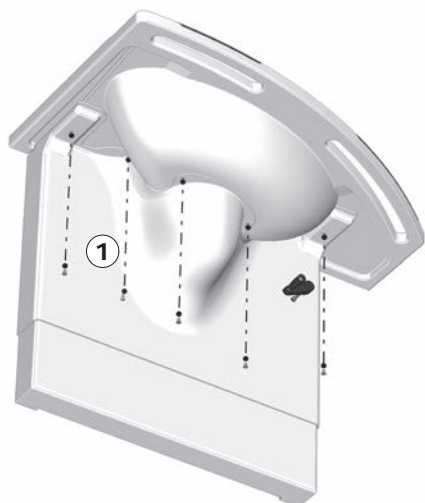
Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a monthly basis to adjust the product to its maximum positions up and down to lubricate the internal parts.

It is recommended that maintenance inspection is done once a year.


Maintenance inspection

For R4820, R4825, R4826, R4840, R4845, R4842 and R4847

Take the wash basin to its highest position.

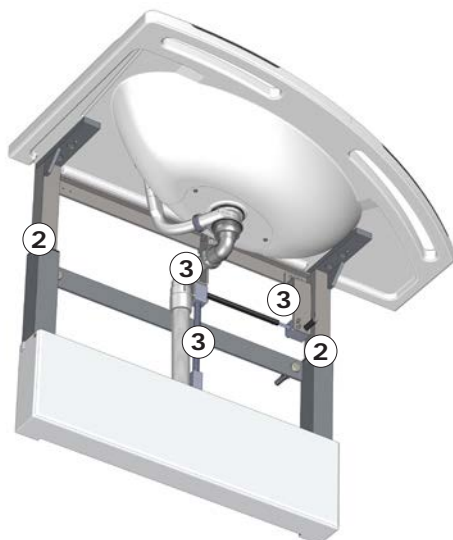Remove the crank handle, the screws and the upper front cover ①.


Lubricate the inner square tubes ② and the hoisting mechanism ③.

Take the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

Reattach the front covers.


For R482019, R482519, R484019, R484519, R484219 and R484719

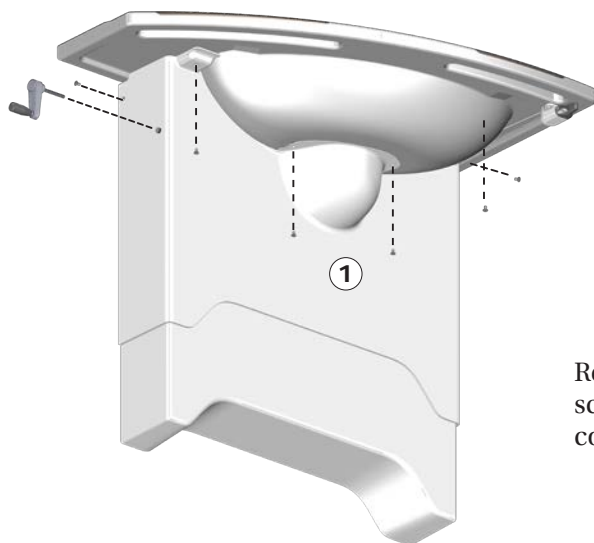Take the wash basin to its highest position.


Remove the crank handle, the screws and the upper front cover ①.

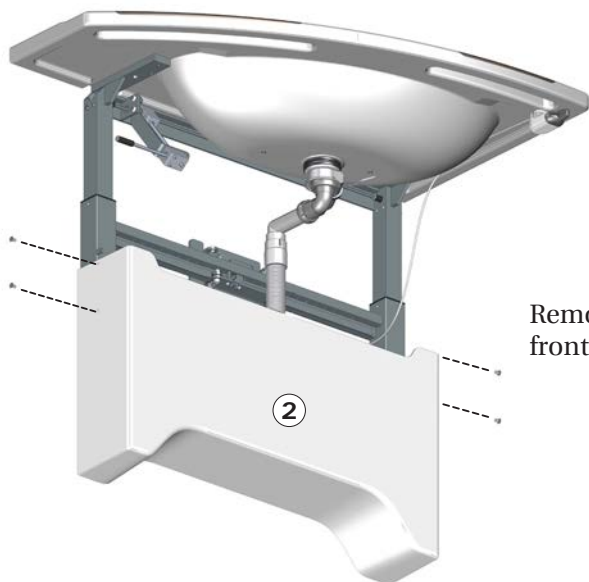en


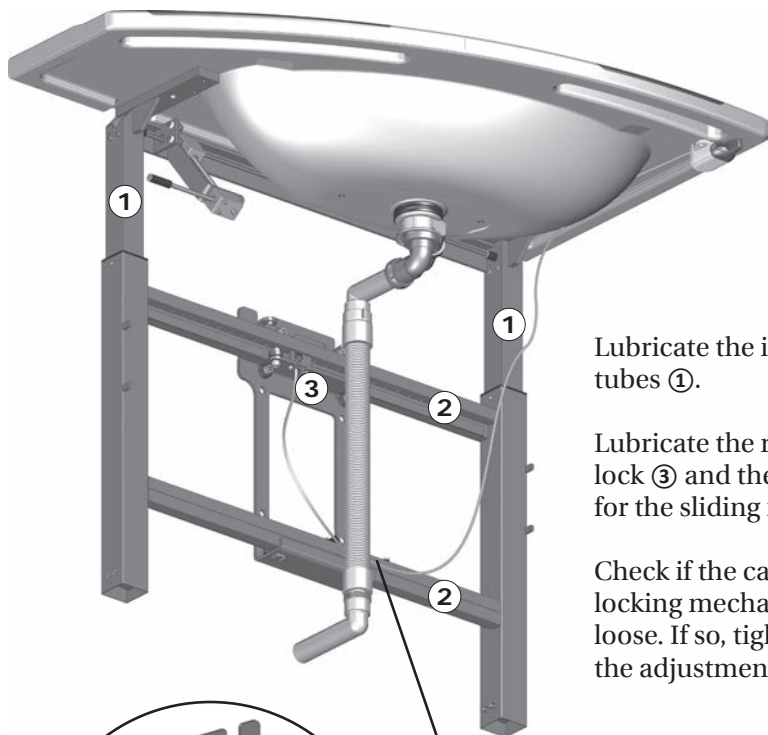
Remove the screws and the lower front cover ②.

Lubricate the inner square tubes ①.

Lubricate the rails ②, the lock ③ and the bearings ④ for the sliding mechanism.

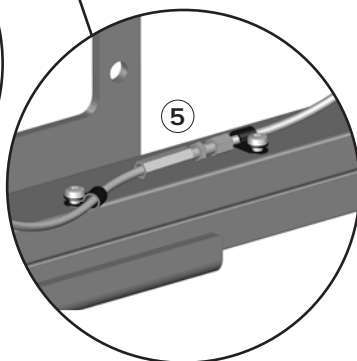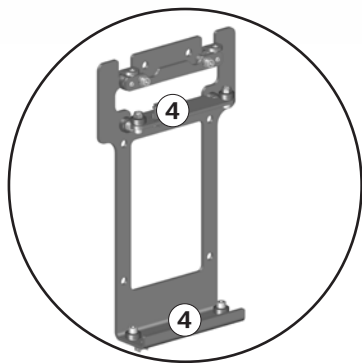Check if the cable for the locking mechanism is too loose. If so, tighten it using the adjustment screw ⑤.


Reattach the front and the top cover.

Lower the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

Also slide the wash basin lifter to its extreme left and right positions.

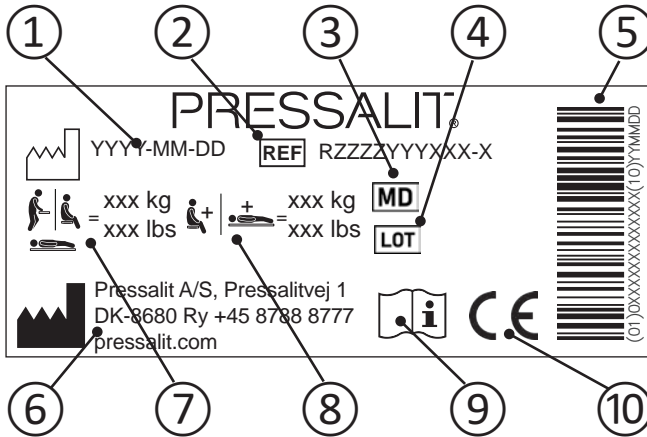Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

⑨ Consult manual for use

These products are CE-marked in accordance with relevant requirements in:

- ⑩
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
- A declaration of conformity can be found at pressalit.com

Trouble-shooting


Symptom	Cause	Action
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".
The product cannot be moved sideways.	Hoses or cables may be stuck.	Remove the cover(s) and check if anything is jammed.
	The locking mechanism may be stuck.	Check the lock and the cable as described under "Maintenance".

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Technical data

Materials	Powder laquered stainless steel Plastics (PVC, Polystyrene, ABS)
-----------	---

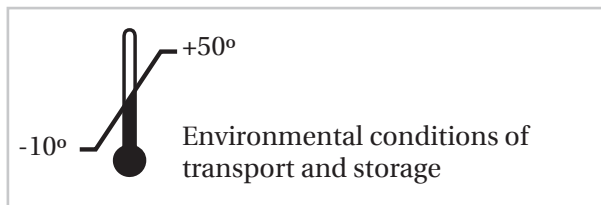Storage and transportation

- The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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