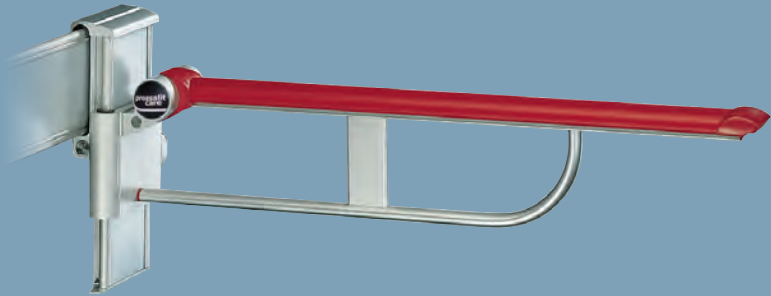
keep living

- UK:** Support arm
Operating and maintenance manual
- DK:** Toiletstøtte
Brugs- og vedligeholdelsesvejledning
- NO:** Toalettstøtte
Bruks- og vedlikeholdsveiledning
- SE:** Toalettstöd
Bruks- och underhållsanvisning
- DE:** Stützklappgriff
Gebrauchs- und Pflegeanleitung
- FR:** Barre de relèvement
Manuel d'utilisation et d'entretien
- ES:** Apoyabrazos
Instrucciones de uso y mantenimiento
- NL:** Toiletsteun
Gebruiks- en onderhoudshandleiding

R3016 – RF016
R3018 – RF018
R3146 – RF146
R3148 – RF148
R3516
R3518
R3546 – RF546
R3548 – RF548

English.....	3
Dansk	7
Norsk.....	11
Svenska	15
Deutsch	19
Français	23
Español.....	27
Nederlands	31

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Thank you for choosing this Pressalit Care support arm.¹

The arm provides extra reassurance and comfort when using the toilet, when sitting or getting up and for sideways movement.

We hope that helpers and users will benefit from the multiple functions and advantages of the product:

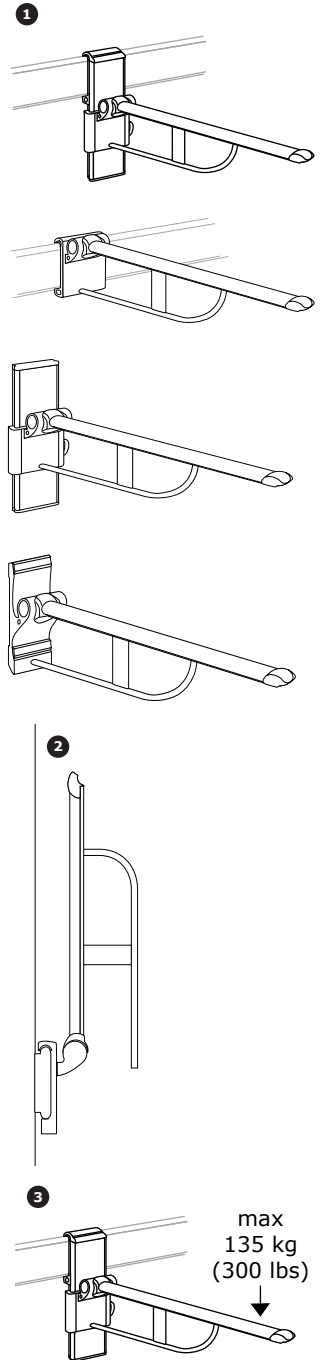
- The end of the arm is designed to provide secure grip.
- The end is comfortably rounded and a good support when the user stands or sits. It also provides a clear indication of where the arm ends.
- The materials are aluminium and nylon.
- Depending on model, the arm can be supplemented with accessories such as toilet paper holder, shower curtain or flush buttons. Accessories can be ordered from Pressalit Care.
- The arm is available in a special version with lock button.

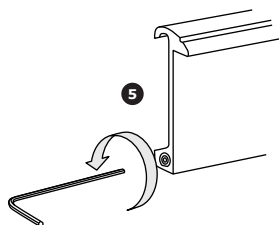
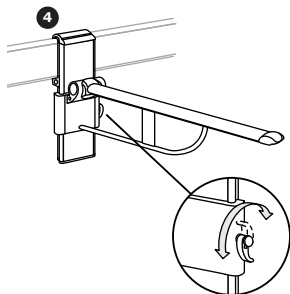
Important safety information

- Please read the user and maintenance guide thoroughly before using the arm. Pressalit Care cannot be held liable if the product is used in any way which differs from that described in the guide.
- The information in the guide is based on correct fitting of the support arm in accordance with our installation guide.
- If the product is out of order or damaged, it must not be used until it is repaired.
- When moving a support arm without any cushion spring from vertical to horizontal, hold the arm all the way down until in the horizontal position.
- When moving the arm from horizontal to vertical, do not release the arm until it is locked in the vertical position.² The arm can easily be pulled down to horizontal from this position again.
- When the arm is in the horizontal position, avoid lifting it up.
- If the arm is fitted with a lock button, it can be locked in either horizontal or vertical position. The horizontal lock ensures that the arm cannot be lifted up, e.g. when a user gets up. The vertical lock means a standing user can hold on - e.g. for men when using the toilet.
- The product is not intended for play, and we therefore recommend that it is not operated by children without adult supervision.
- We reserve the right to amend this guide without prior notice.

Maximum load

135 kg ³

Before using the product for the first time

- Clean the support arm after fitting.
- Check that the arm moves smoothly if fitted on a horizontal and/or vertical track.

Operation

- Lower the arm from its vertical to horizontal position.

Adjustment

- Adjusting the height of the vertical track: Press lock handle inwards until it engages. Turn the handle downwards. Adjust support arm and turn lock handle upwards to lock in position. ④
- Adjusting the height of the horizontal track: Use an Allen key to slacken the lock. ⑤ Adjust the arm and retighten the lock if necessary.

Cleaning

Daily:

- Use ordinary soapy water. Avoid use of abrasive, corrosive or chlorine-based cleaning products. Rinse off with clean water and dry with a soft cloth.
- This product cannot be cleaned in an autoclave.

Occasional:

- Removal of lime deposits on aluminium parts: Use 3 parts 30% household vinegar to 7 parts water. Wipe off with a cloth dipped in clean water and wrung out.

Maintenance

- If a product mounted on a horizontal track does not move smoothly, it may be necessary to clean and lubricate the slide. Use a mild detergent, wipe off with a soft cloth and allow to dry. Apply a lubricant in the form of silicon spray, and wipe off with a dry cloth.

Service inspection

We recommend that a service inspection is performed once annually.

- 1: Check all screws and tighten if necessary.
- 2: Check surfaces for damage, and clean the product in accordance with cleaning guide.
- 3: Check that the arm locks into position when raised.
- 4: Check all moving parts, lubricate if necessary with silicon spray.

Faultfinding

Fault	Cause	Remedy
Arm seems unstable when in horizontal position.	The horizontal slide is not tightened.	Tighten the slide using an Allen key.
Arm seems unstable when in vertical position.	The vertical slide is not tightened.	Tighten the vertical slide fully. Use the lock handle.
The cushion spring in the arm does not cushion properly.	The spring may be broken.	Contact Pressalit Care or your dealer.
Support arm on a horizontally-adjustable product cannot be moved on the wall track.	Dirt on the wall track.	Clean using a mild detergent and dry with a soft cloth. Apply a lubricant in the form of silicon spray, and wipe off with a soft cloth.
Support arm on a vertically-adjustable product cannot be moved on the track.	Dirt on the track.	Clean using a mild detergent and dry with a soft cloth. Apply a lubricant in the form of silicon spray, and wipe off with a soft cloth.
The arm on a vertically-adjustable product cannot be tightened.	Lock handle stuck.	Free handle by turning clockwise. Retighten the adjustable product.

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

CE

The CE mark indicates that this product has been manufactured according to the Machine Directive 2006/42/EC of 17 May 2006.

Information on disposal and recycling

Electrical and electronic equipment (EEE) contains materials, components and substances which can be hazardous to human health and the environment when waste from electric and electronic equipment (WEEE) is not disposed of correctly. Such equipment must not be disposed of with unsorted domestic refuse, but must be collected separately. Waste must be processed according to local authority instructions.

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Pressalit Care A/S
a division of Pressalit Group A/S