R7034, R7304
R7334, R7364

R7114, R7115, R7124
R7125, R7134, R7135
R7404, R7414, R7415
R7424, R7425, R7434
R7435, R7454, R7455
R7464, R7465

PLUS

Shower seat 310/450 – Operation and maintenance manual	en
Brusesæde 310/450 – Brugs- og vedligeholdelsesvejledning	dk
Duschsitz 310/450 – Gebrauchs- und Pflegeanleitung	de
Siège de douche 310/450 – Manuel d'utilisation et d'entretien	fr
Douchezitting 310/450 – Gebruiks- en onderhoudshandleiding	nl
Duschsits 310/450 – Bruks- och underhållsanvisning	sv
Dusjsete 310/450 – Bruks- og vedlikeholdsveiledning	no
Asiento de ducha 310/450 - Instrucciones de uso y mantenimiento	es
Sedile per doccia 310/450 - Manuale d'uso e manutenzione	it
Siedzisko prysznicowe 310/ 450 – Instrukcja obsługi i konserwacji	pl
Sedátko do sprchy 310/450 – Návod k obsluze a údržbě	cs

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English	14
Dansk	26
Deutsch	38
Français	50
Nederlands	62
Svenska	74
Norsk	86
Español	98
Italiano	110
Język polski	122
Čeština	134
中文	146

Pressalit's Environmental and Quality Management Systems are certified by LRQA

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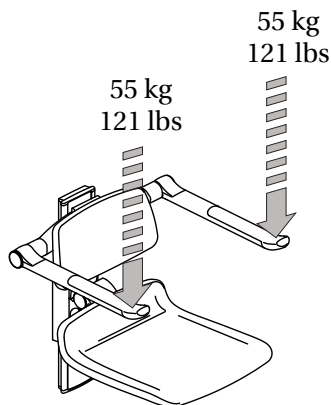
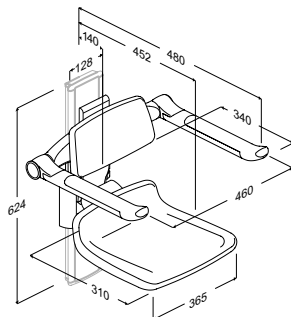
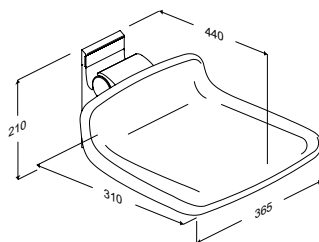
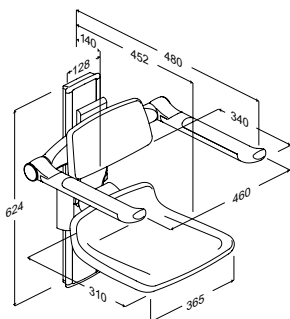
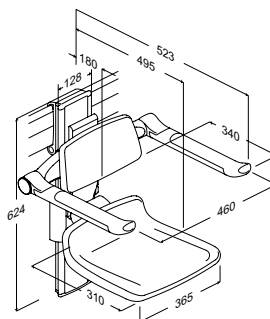
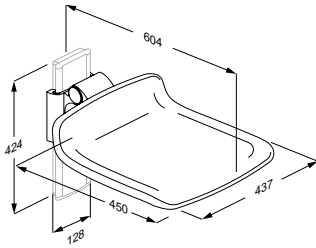
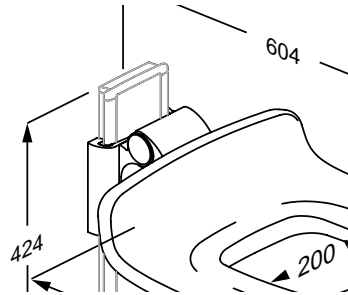
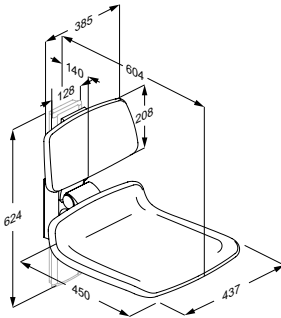
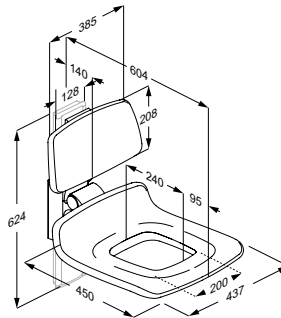
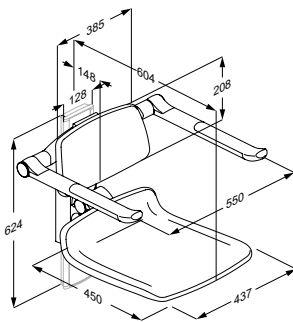
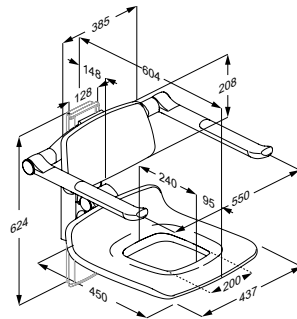
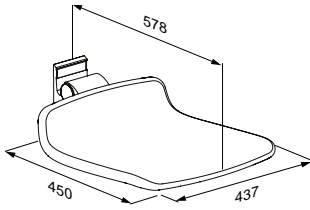
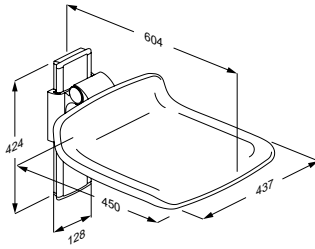
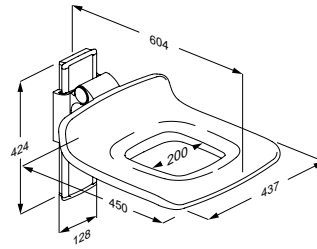
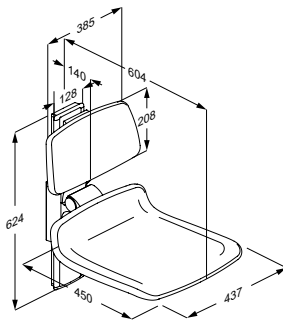
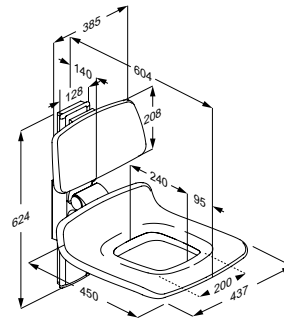
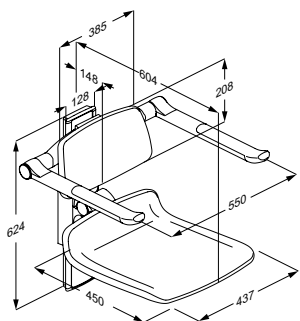
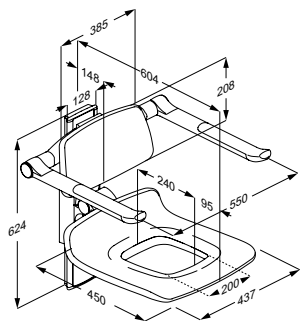
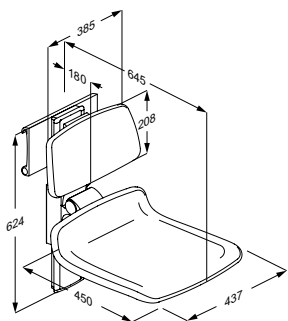
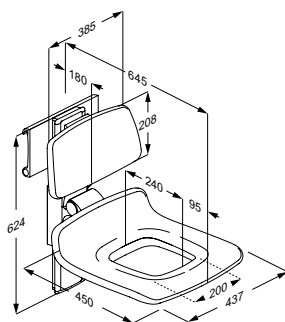
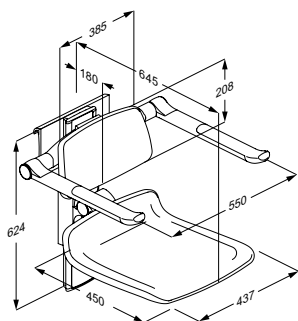
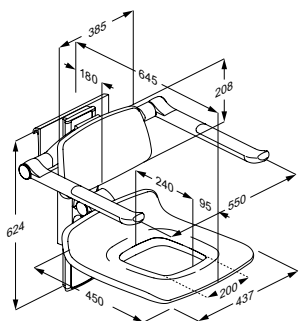
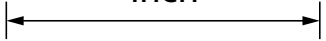
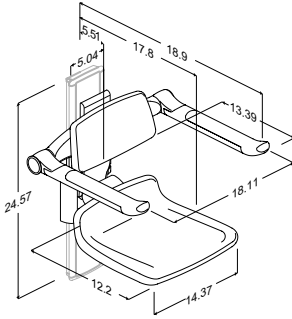
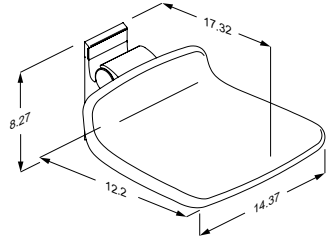
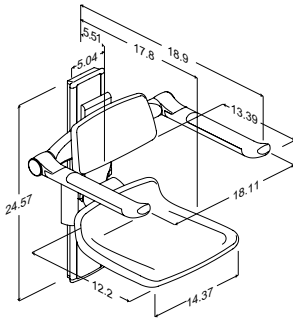
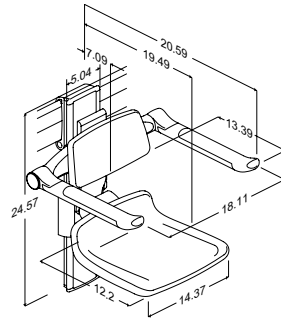
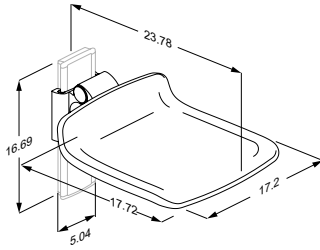
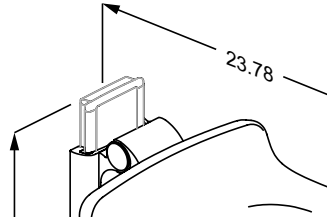
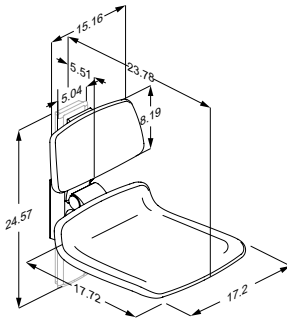
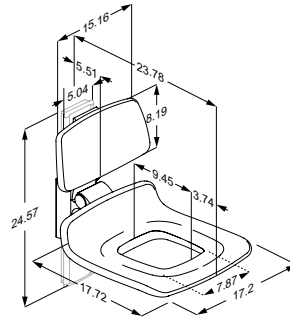
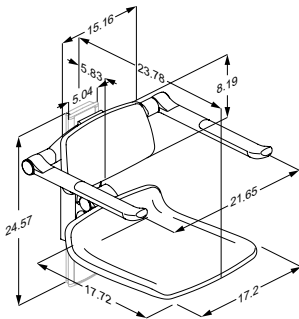
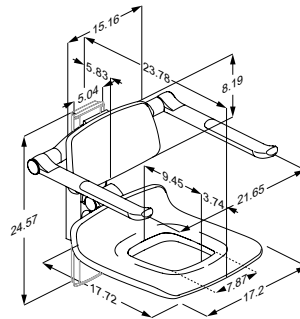
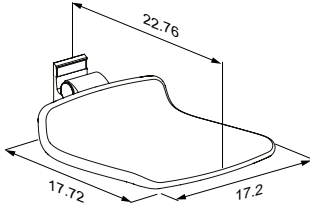
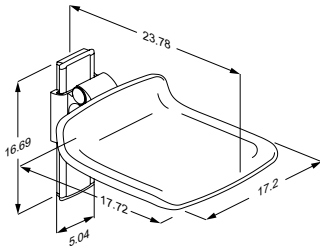
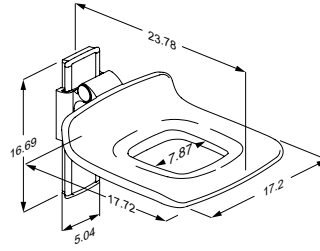
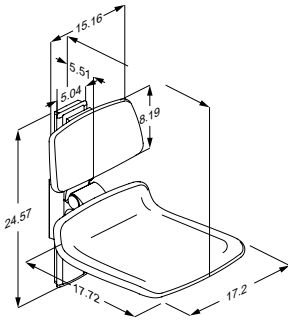
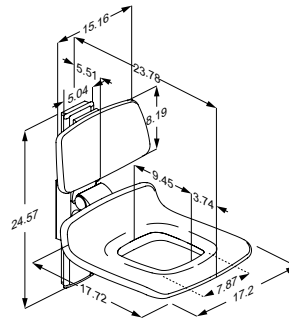
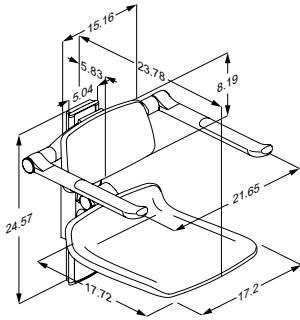
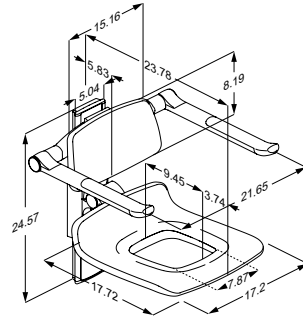
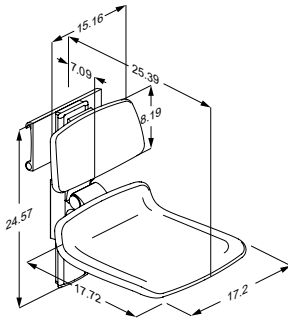
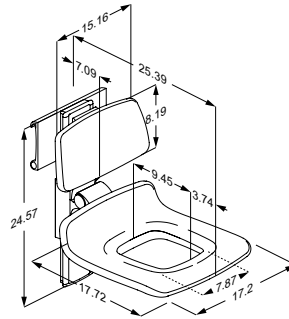
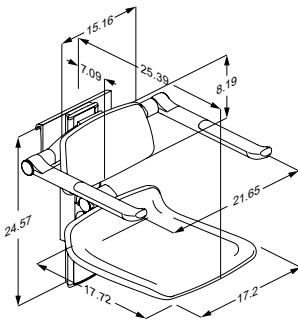
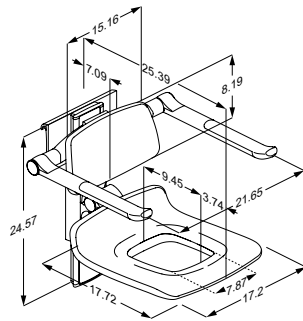
R7465

Content

Mounting instruction	14
Operational signage	14
Symbols used in this manual	15
General safety	16
Intended use	17
Intended user profile	17
Operating environment	17
Operating instructions	18
Folding the seats and armrests	18
Adjusting the seat and the backrest	19
Adjusting the armrests	20
Adjusting sideways	20
Cleaning	21
To remove limescale	21
Maintenance	22
Lubrication	22
Product label	23
Service inspection	24
Materials	24
Trouble-shooting	24
Disposal and recycling	25

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage can be downloaded from pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Article number/ versions

Product weight

Max. user weight

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

The shower seat is used to sit on during showering. The armrests are intended for support when sitting and standing up from the shower seat.

Intended user profile

The 310 seat is suitable for children and small adults, while the 450 seat can carry large people. Both shower seats can be used by people whose weight does not exceed **200 kg / 440 lbs.**

The maximum vertical load for the armrests is 55 kg / 120 lbs.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

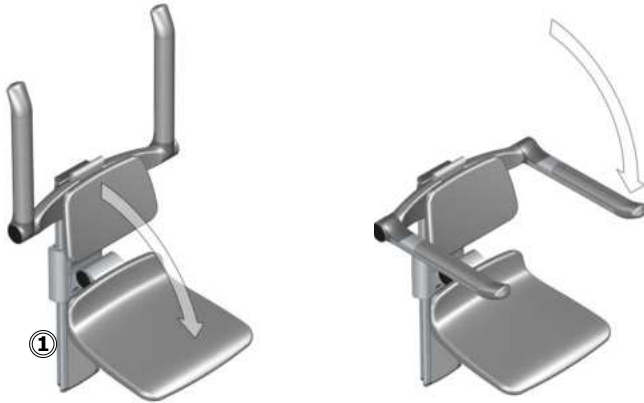
NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year.

Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

en

Operating instructions

Folding the seats and armrests

Fold the shower seat and the armrests (if fitted) fully down.

Fold it back up when not in use. You can hear a click when the shower seat and the armrests are correctly folded up.

Take care to avoid injury to hands when in the vicinity of closing moveable parts ①.

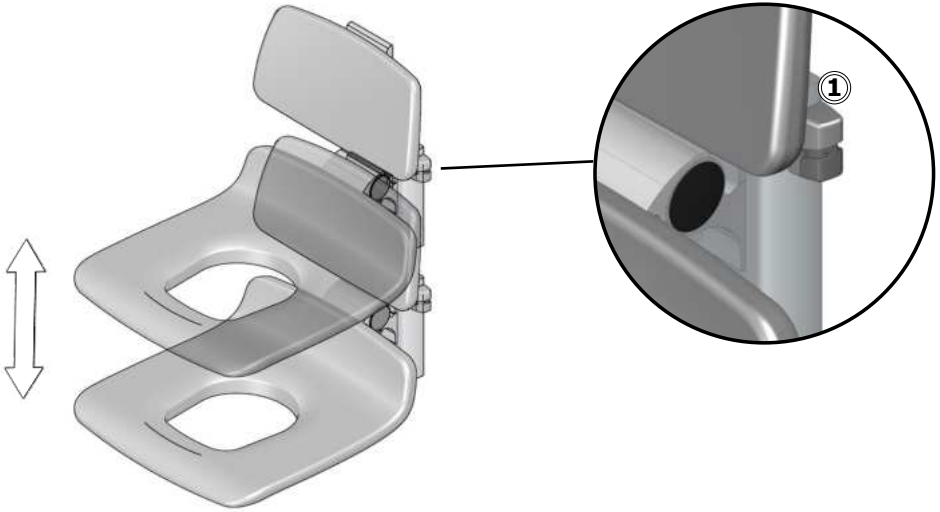
The armrests are not intended for horizontal load, e.g. as an aid for lateral transfer from a wheelchair to a shower seat.

Make sure that seat is dried off whenever it has been exposed to water.



Adjusting the seat and the backrest

To adjust upwards, pull the shower seat upwards.

To adjust downwards, pinch the locking button ① and hold it closed while moving shower seat down. Release the locking button to secure the shower seat in place.

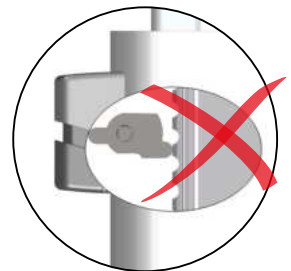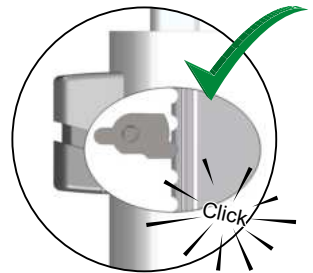Make sure that the locking button pin is engaged in the notch.


en

Adjusting the armrests


Armrests can also be adjusted independently from seat and backrest.

To adjust the armrests upwards, pull upwards at the shoulders ① until desired height is reached and a click is heard.

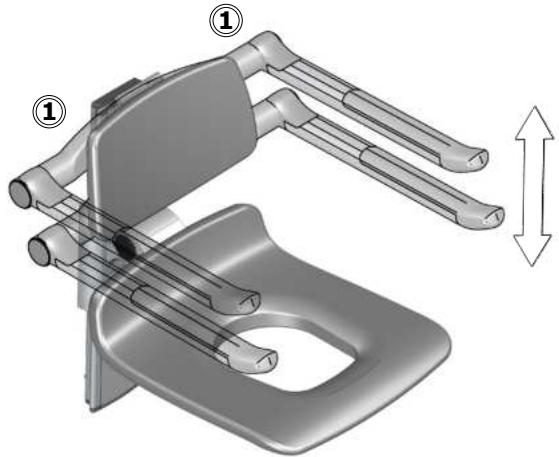To adjust the armrests downwards, pull the armrests all the way to the top, then support them all the way to the lowest position, and then pull upwards again to the desired position until a click is heard.

Adjusting sideways

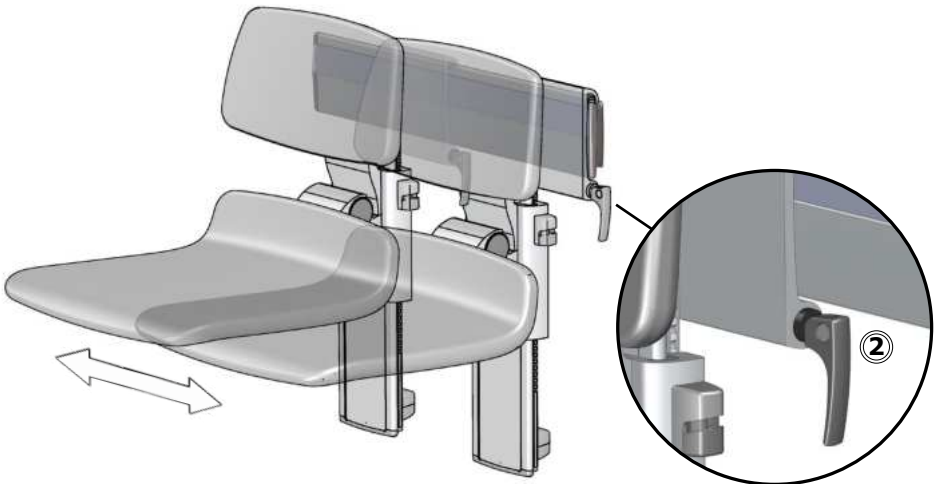Release the locking handle ② and move the shower seat sideways. Tighten the locking handle again to secure the seat in place.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

Maintenance

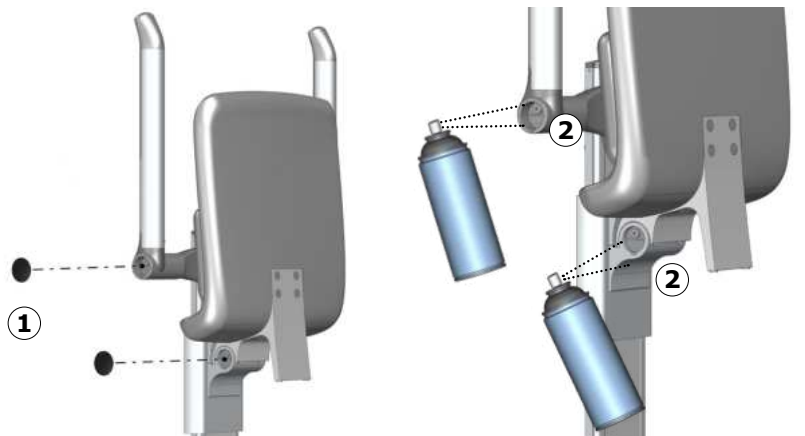
Expected service life of the shower seat is 10 years, provided that maintenance is carried out as described below:

Remove dust from the wall tracks ① on a regular basis. Wipe the wall tracks with silicone.

It is recommended on a weekly basis to adjust the product to its maximum positions and to fold the seat and armrests (if fitted) up and down.


Lubrication

It is recommended to lubricate the joints of the seat and armrests once a year.


Use a screwdriver to remove the cover ① for the joints.

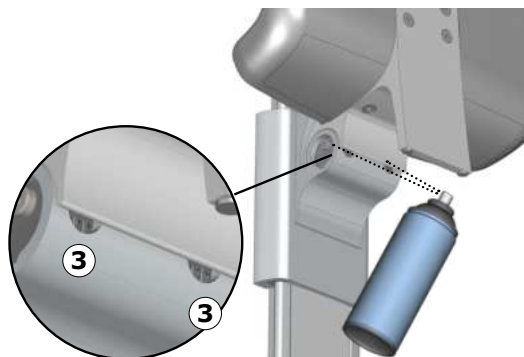Lubricate the joint ② and the two joint holes ③.

Product label

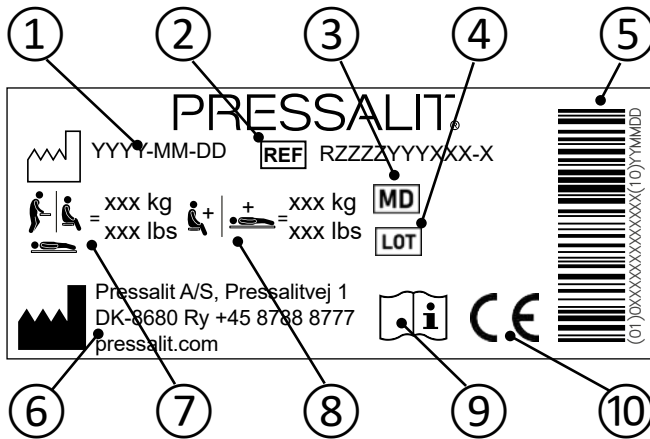① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

⑨ Consult manual for use

These products are CE-marked in accordance with relevant requirements in:

- ⑩
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
- A declaration of conformity can be found at pressalit.com

Service inspection

Ensure that all screws and bolts are properly tightened, and check the functionality of the product.

Materials

Materials	Anodized aluminium Stainless steel Polyurethane foam ABS SEBS
-----------	---

Trouble-shooting

Symptom	Cause	Action
The product can't be moved sideways or up and down.	There might be dirt or dust on the walltracks.	Clean the walltracks and grease with silicone.
The locking handle for horizontal adjustment doesn't lock.	The lock is not properly adjusted.	See mounting instruction.
The armrests and/or the seat is difficult to fold up and down.	The joints for the seat and the armrests needs lubrication.	Lubricate the joints for the seat and the armrests as described in Lubrication .

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
SEBS	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.
ABS	As virgin ABS is somewhat expensive, recycling ABS is economically very attractive. Recycled ABS can be blended with virgin material to produce products with lower cost while preserving the high quality.
Polyurethane	Like other plastics, polyurethane can be recycled in two primary ways: mechanical recycling, in which the material is reused in its polymer form, and chemical recycling that takes the material back to its various chemical constituents.

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