

PRESSALIT.

Product	PLUS wall track, 400 mm, left-grooved, for vertical mounting		
Item number	R9873112		
	VVS:615843134		
	NRF:7501072		
	EAN:5708590369280		
	HMI: 124111		
Colour	Anthracite grey		
Product description	Below a selection of products for fitting to the wall track.		
	PLUS wall track, 400 mm, left-grooved, for vertical mounting. For height adjustable, left-hand operated support arms. The cover plate hides bolts and wires and facilitates cleaning. Materials: Aluminium and polystyrene. Excl. mounting kit.		
Adjustability	Fixed mounting		
Materials	The wall track is primarily made of 10-micrometre anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer		
Ergonomy	The vertical track enables height adjustment of the different products. If combined with the horizontal track, the available space in the bathroom can be utilized optimally. In environments with multiple users, the vertical track makes it possible to adjust the different products to individual user heights.		
Features	The front cover plate conceals mounting bolts and any cables.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		



PRESSALIT.

Product Item number	PLUS wall track, 400 mm, left-grooved, for vertical mounting R9873112
Cleaning	PLUS wall tracks The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.



PRESSALIT.

Product	
Item numb	er

PLUS wall track, 400 mm, left-grooved, for vertical mounting

R9873112


Installation in different wall types


D. ∇. D. D. D. C.	C	α. V	
		D : C	

Concrete >B25

V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.

To be ordered separately.

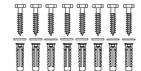


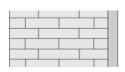


Solid Brick >= Mz12

V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.

To be ordered separately.

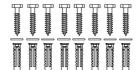


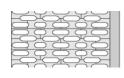

Solid Sand Lime Brick >= HIz12

V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10.

To be ordered separately.

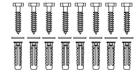





Aerated Concrete P4

V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8.

To be ordered separately.

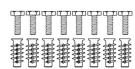


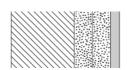

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs

NB! Plugs are not needed. To be ordered separately.

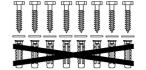



PRESSALIT®

Product Item number	PLUS wall track, 400 mm, left-grooved, for vertical m	nounting
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	!

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com