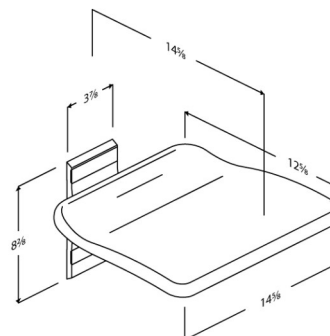




# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS folding seat, fixed height R5510000
<b>Color</b>  <b>Product description</b>	VVS:777531000 NRF:7501001 RSK:7973939 EAN:5708590306339 HMI: 130159 NBS:N13:448 FDA:INN/D135865 White  <p>The seat is suitable for situations where additional seating is required and space is limited, - e.g. in kitchens, lifts and hallways.          The folding seat is not suitable for use in the shower area.</p> <p>PLUS folding seat. For fixed mounting.          Materials: Aluminium, thermoset plastic, plastic.          Excl. mounting kit.</p>
<b>Adjustability</b> <b>Product capacity</b>	Fixed mounting  Maximum user mass: 960 lbs. When the seat is loaded with 960 lbs, each wall fastener is loaded with 436 lbs.  <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
<b>Materials</b>  <b>Ergonomy</b> <b>Features</b>	The folding seat is made out of thermoset plastic, and the mounting and suspension bracket are made from 3/8" anodised aluminium. The end caps are made from ABS plastic.  Ergonomically shaped seat, which offers good and comfortable support.  The seat can be folded away when not in use, - or to free up space. Stays up by itself when folded away.
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.





# PRESSALIT®

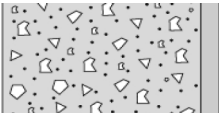
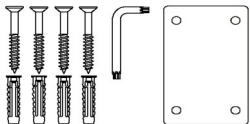

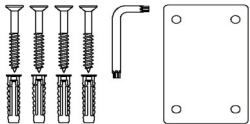
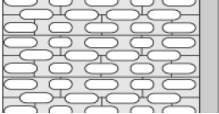
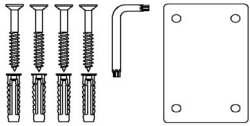
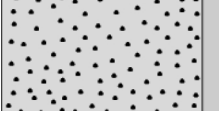
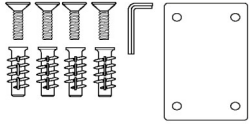
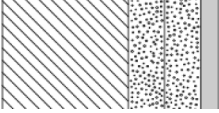
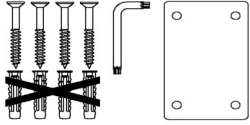
<b>Product</b>	PLUS folding seat, fixed height
<b>Item number</b>	R5510000
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>



PRESSALIT®

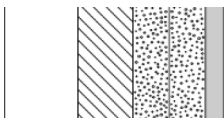
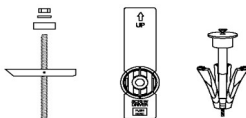
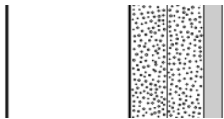
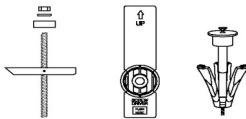
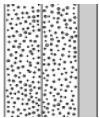

<b>Product</b>	PLUS folding seat, fixed height
<b>Item number</b>	R5510000

## Mounting in different wall types

	<p><b>Concrete &gt;B25</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. To be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b></p> <p>V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8604 - Mounting kit, 4 countersunk bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.</p>	



# PRESSALIT®

<b>Product</b> <b>Item number</b>	PLUS folding seat, fixed height R5510000
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></p> <p>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p> 
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b></p> <p>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p> 
	<p><b>Drywall (2 x 12,5 mm) water-resistant</b></p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p> 

pressalit.com

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®