R9504717
R9504721

MSCT 4

Mobile shower change trolley – Operation and maintenance manual	en
Mobilt bruse-/skifteleje – Brugs- og vedligeholdelsesvejledning	dk
Duschwagen – Gebrauchs- und Pflegeanleitung	de
Chariot de douche et de change – Manuel d'utilisation et d'entretien	fr
Mobiël douche-/aankleedbed – Gebruiks- en onderhoudshandleiding	nl
Mobil dusch/skøtbord – Bruks- och underhållsanvisning	sv
Mobilt dusjställebord – Bruks- og vedlikeholdsanvisning	no
Camilla móvil para ducha/cambiador – Instrucciones de uso y mantenimiento	es
Doccia/fasciatoio mobile - Manuale d'uso e manutenzione	it
Suihku- ja vaatteidenvaihtovaunu – käyttö- ja huolto-opas	fi
Wózek toaletowy – Instrukcja obsługi i konserwacji	pl

English	8
Dansk	27
Deutsch	46
Français	65
Nederlands	84
Svenska	103
Norsk	122
Español	141
Italiano	160
Suomi	179
Język polski	198

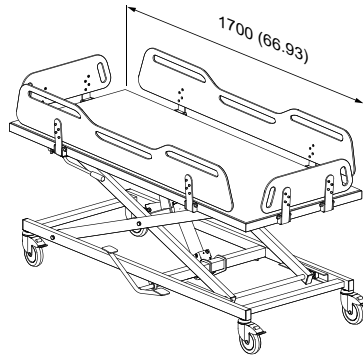
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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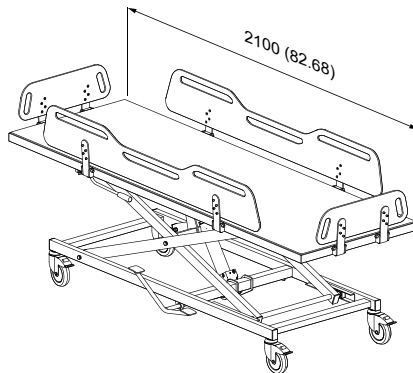
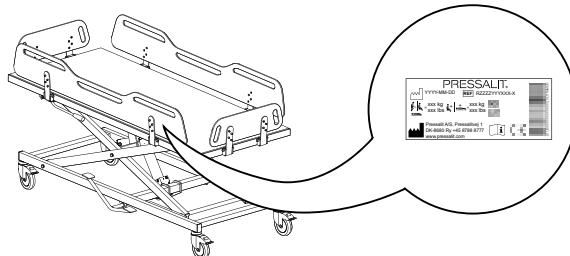
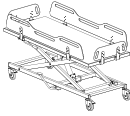
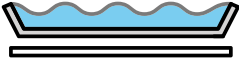
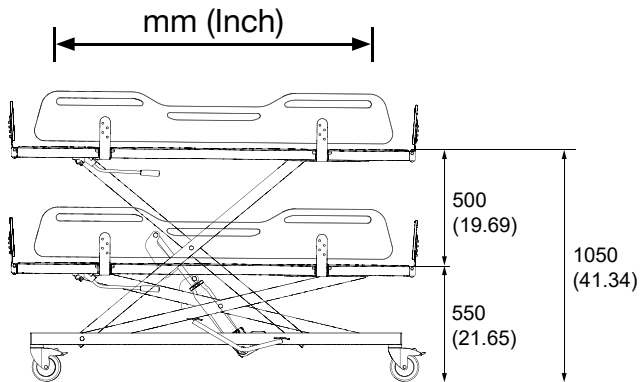
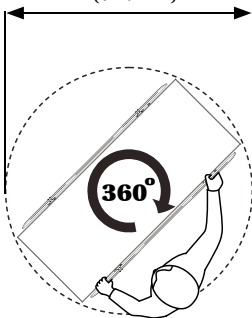
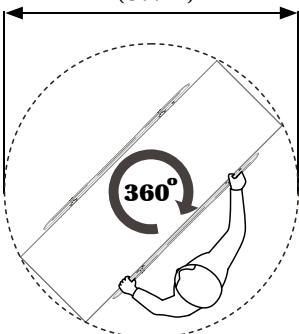
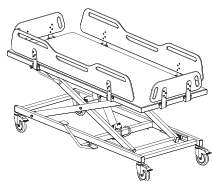
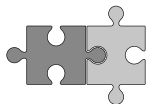
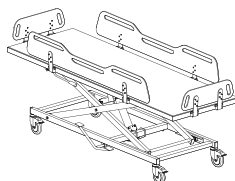
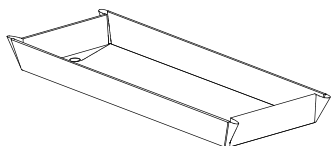
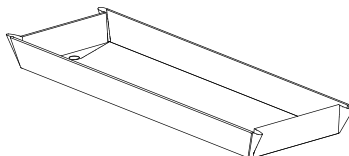
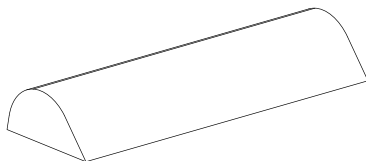
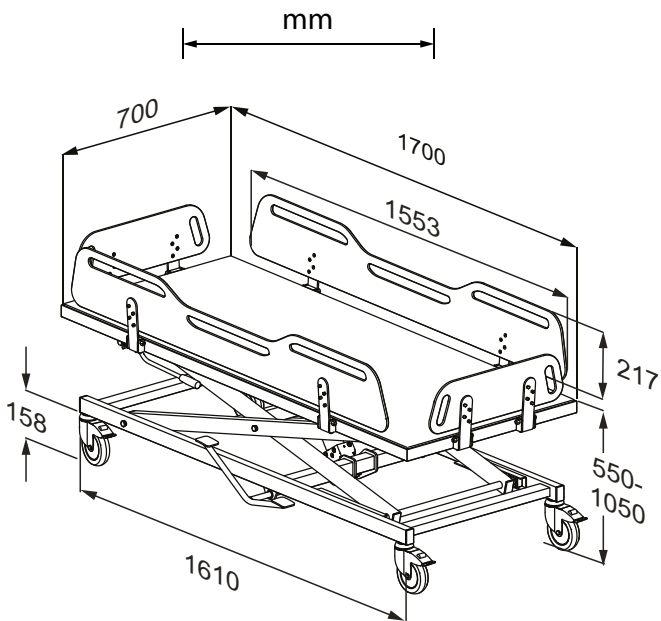
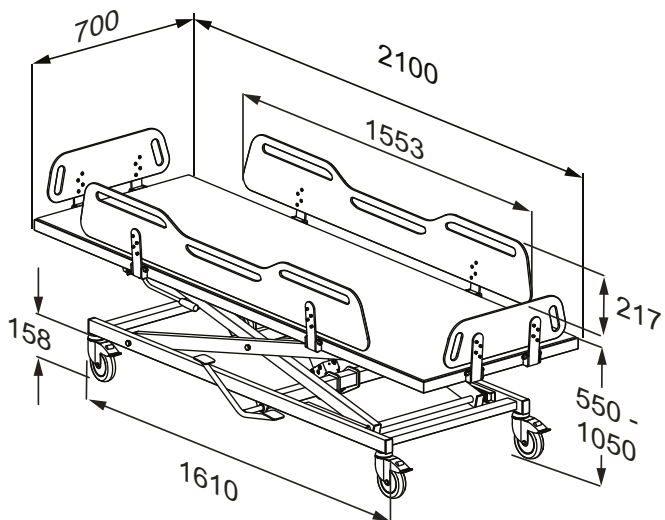
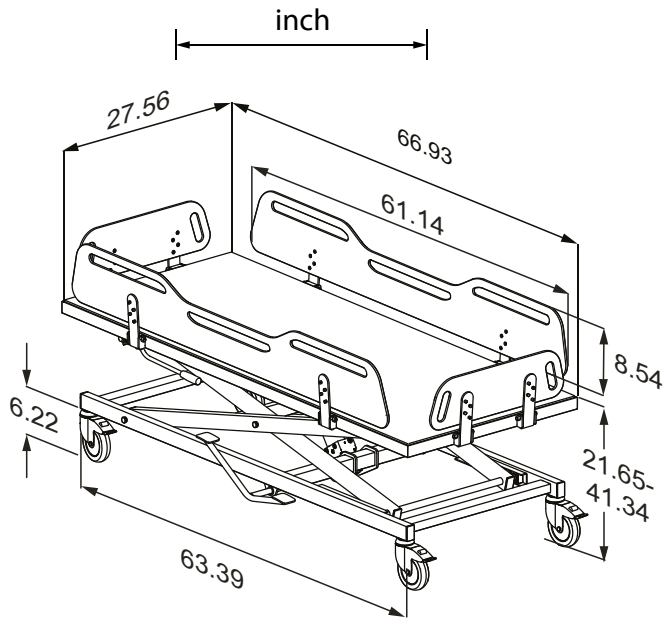
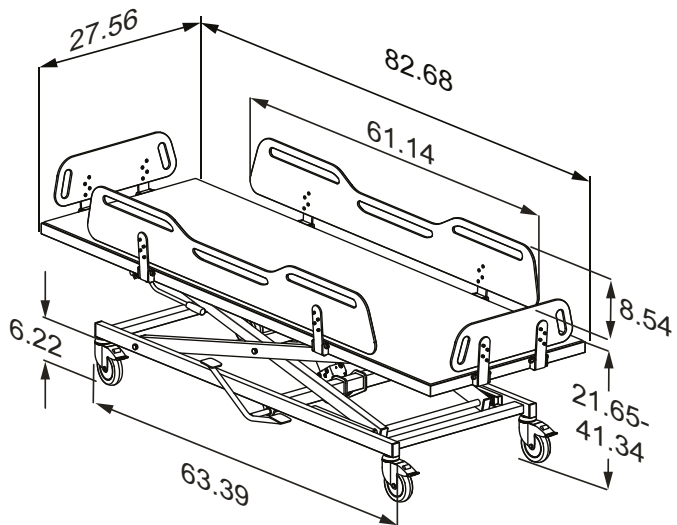
R9504721

Content

Symbols used in this manual	9
General safety	10
Safety distances	11
Intended use	12
Intended operator profile	12
Intended user profile	12
Operating environment	12
Contraindications for use	13
Biocompatibility	13
Flammability	13
Operating instructions	14
Safety brakes	14
Safety rails	15
Height adjustment	16
Tilt function	17
Moving	18
Accessories	19
Shower mattress	19
Drain plug	20
Hose hanger	20
Head rest	21
Cleaning	22
Disinfection	22
Maintenance	23
Maintenance inspection	23
Product label	24
Storage and transportation	25
Troubleshooting	25
Technical data	26
Disposal and recycling	26

en

Operational signage

A Quick-guide is included with the product. It can also be downloaded from pressalit.com.


Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

	<p>WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>
	<p>CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>
	<p>IMPORTANT indicates important information about handling and use of the product.</p>
	<p>NOTICE is used to address practices not related to personal safety.</p>

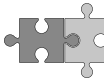en

 <p>Head injury warning</p>	 <p>Hand injury warning</p>	 <p>Genitalia injury warning</p>	 <p>Feet injury warning</p>
 <p>Spare parts</p>	 <p>Do not leave patient unattended</p>	 <p>Compatible products</p>	 <p>Product weight</p>
 <p>Max. user weight</p>	 <p>Article number / versions</p>		

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.
- If you suspect that any individual parts may be damaged, contact your Pressalit dealer or service provider. Do not remove, attempt to repair or install parts yourself, unless directed to do so by Pressalit or its agents.

CAUTION

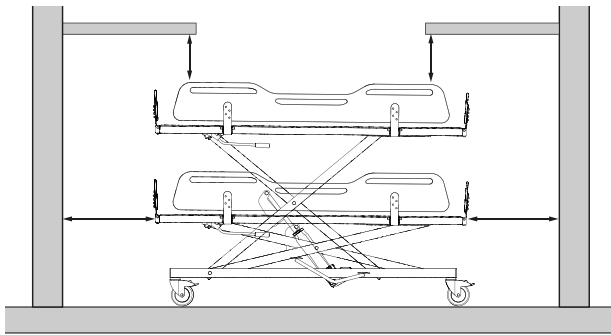
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

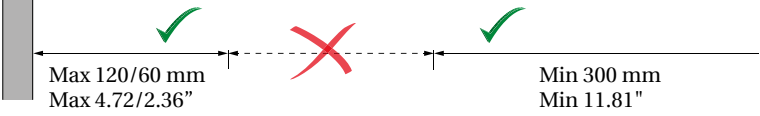
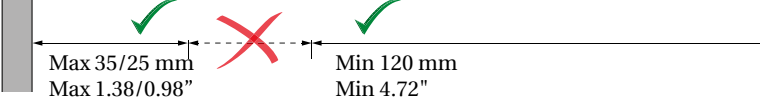
NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Safety distances

Make sure that the unit is at safe distance from surrounding objects. Check the distances before any use.

ISO 17966:2016		 / 	
		Max 120/60 mm Max 4.72/2.36"	Min 300 mm Min 11.81"
		Max 8/4 mm Max 0.31/0.16"	Min 25 mm Min 0.98"
		Max 8 mm Max 0.31"	Min 75 mm Min 2.95"
		Max 35/25 mm Max 1.38/0.98"	Min 120 mm Min 4.72"

WARNING

If the unit moves within the safety distances, you must keep a close eye on especially surrounding people and objects to avoid crushing hazards.

en

Intended use

This product is intended for bathing or changing a person who is unable to stand or sit. The person is lying horizontally on the shower trolley.

Intended operator profile

Only personnel who are trained in healthcare and/or in bathing or changing someone under their care should use this product.

Intended user profile

The product can be used by all – adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

Operating environment

For indoor use in a wet environment, such as a bathroom or washroom.

The product is intended to be used at room temperature in public changing places, schools, private homes, nursing homes and hospitals under normal working conditions.

The product surface may become hot or cold if exposed to external sources of heat or cold, such as sunlight or frost.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Contraindications for use

It is not recommended that pregnant women are bathed in the shower change trolley.

It is not recommended that patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions are bathed in the shower change trolley.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may be bathed in the shower change trolley only after consulting with the attending physician.

Do not use if the patient's weight exceeds 200 kg / 440 lbs.

Biocompatibility

The biocompatibility of the materials that come in contact with the user's skin during the intended use has been evaluated according to the procedures outlined in DS/EN ISO 10993-1:2020, which involves assessing the materials for harmful substances. Based on this evaluation, the materials are considered suitable for their intended use. en

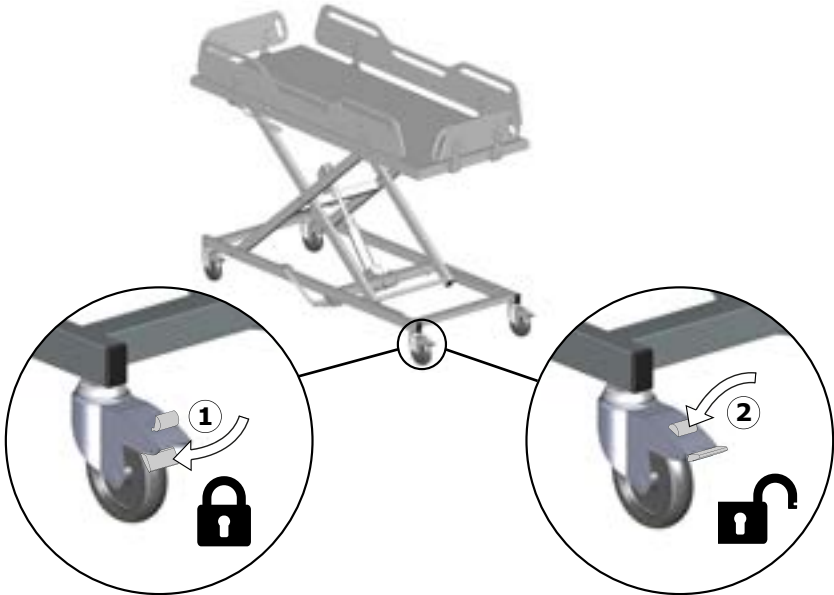
Flammability

The product materials have undergone flammability testing in accordance with ISO 8191-1:1987 and ISO 8191-2:1988.

Operating instructions

Safety brakes

The change trolley is equipped with brakes on all four wheels to prevent accidental movement.

Lock the brakes①: Step on the lock pedal on all four wheels.

Unlock the brakes②: Step on the release button on all four wheels.

Always use your foot to lock/unlock the wheels. Wear closed-toe shoes and avoid using your hands. Be mindful of the brakes to avoid tripping or bumping into them.

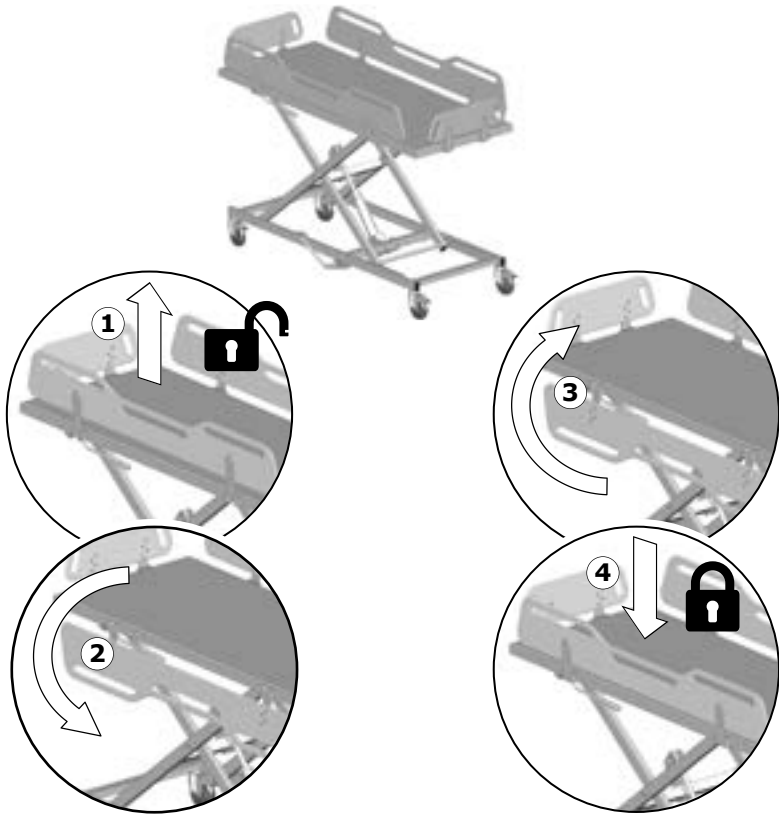
Ensure all four wheels are locked before transferring a patient to/from the mobile change trolley.

Never leave a patient unattended on the trolley.

Safety rails

The safety rails can be folded up and down individually.

Fold down: Lift the safety rail upwards to unlock it①, and swing it down②.

Fold up: Swing the safety rail up③ and push it down to lock it in place④.

CAUTION

To protect the patient, it is recommended that the side rails are upright and locked when using the mobile shower change trolley.

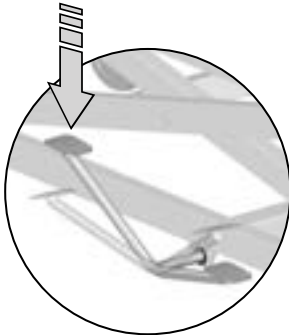
Be careful when adjusting the side rails to avoid pinching hands, feet, or other body parts.

Height adjustment


The mobile shower change trolley can be adjusted 500 mm / 19.69" up and down. The maximum adjustable height is 1050 mm / 41.34". The shower change trolley features a double foot pedal for adjusting its height.

en

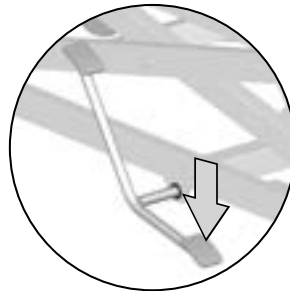Step repeatedly on the long pedal to raise the unit.

Depress the short pedal to lower the unit.

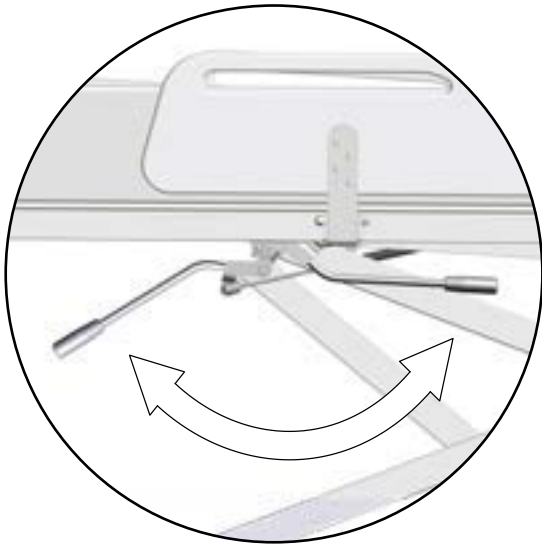
Do not place your foot on the bearing frame and be especially cautious of the risk of pinching hands, feet, or objects when adjusting the height of the unit.

! WARNING

Never let children or pets be near the unit during height adjustment.

Tilt function

The shower change trolley can tilt approximately 3 degrees to facilitate water drainage when using the optional shower mattress.

To use the tilt function, turn the tilting handle clockwise as far as it will go.

NOTICE

When a person is lying on the shower change table during tilting, turn the tilting handle with controlled, gentle movements to ensure a comfortable experience for the person. Ensure that all side supports are folded up while tilting.

en

Moving

To move the trolley, unlock all four wheels. Grab the lying surface itself, not the safety rails, and proceed with caution while moving the unit.

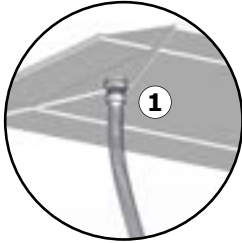
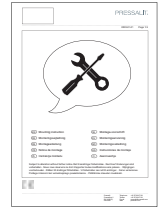
To prevent the trolley from tipping over, avoid using it on floors with lowered drains, holes, or significant slopes.

Accessories

Shower mattress

The shower change trolley can be used as a bathtub with the optional shower mattress (R844721 and R844717). An instruction for attaching the drain hose is included with the shower mattress and can also be downloaded from pressalit.com.

Attach the drain hose to the mattress drain①.

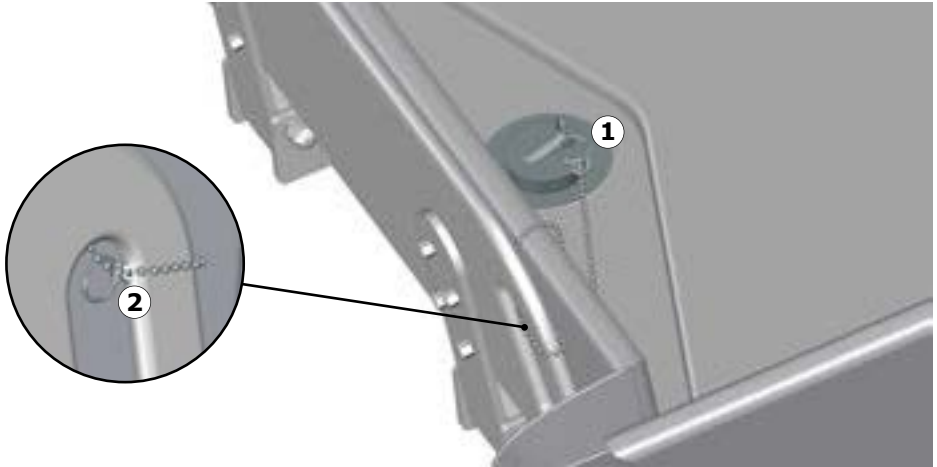
Pull the drain hose through the shower trolley's drain hole②, fold the mattress and place it on the trolley. Make sure that all four side supports are folded up and locked.

IMPORTANT

- To maintain hygiene, it is recommended to check for any visible damages or signs of wear and tear on the mattress regularly. If any damages are found, the mattress should be replaced immediately to prevent any risk of contamination or infection.
- Use gloves while handling the waste hose for the mattress. It may have been in contact with human waste.

en


Drain plug

Attach one key ring of the chain to the plug①. Make a loop around one of the footboard handles with the other ring②.

en

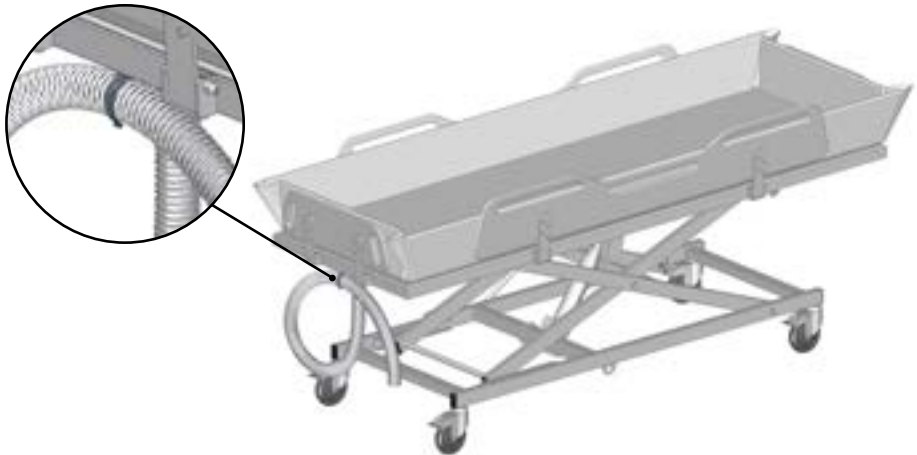Hose hanger


Coil up the hose and attach it to the hose hanger.

Head rest

The shower mattress can be used with the optional headrest for optimal comfort (R844521).

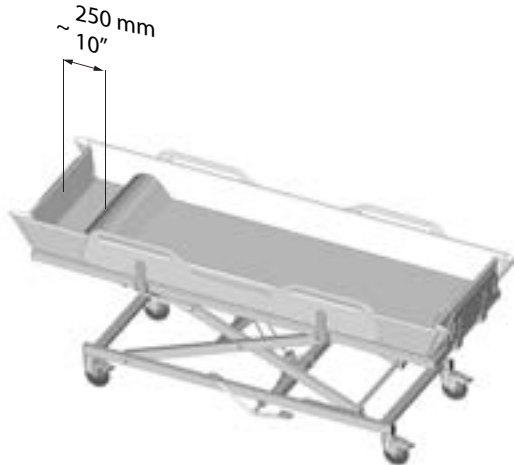Place the headrest at the head end of the shower trolley, approx. 250 mm (10") from the headboard.

The mattress and the head rest are to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents/disinfectants tested by Pressalit can be found on pressalit.com

IMPORTANT

Regularly inspect the headrest for any visible damage or signs of wear to maintain hygiene. If any issues are found, replace these items immediately to minimize the risk of contamination or infection.

Cleaning

To clean the product, use a damp cloth that has been wrung out in a mild soap mixture, such as universal cleaning solution. Rinse with water where necessary.

After cleaning, it is essential to wipe the product with a dry, soft cloth. This step is crucial to prevent the materials from discoloration. Additionally, it helps prevent any potential degradation of the surface paint.

A list of suitable cleaning agents/disinfectants tested by Pressalit can be found in the technical datasheet or on pressalit.com under the specific product.

CAUTION

- Always read the supplied instructions of the cleaning agent and comply with these instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.
- Do not perform any type of cleaning or disinfection of the change trolley while the user is lying on it.

IMPORTANT

- Never spray the product directly with a high pressure cleaner.
- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acid or base.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

Disinfection

When using the mobile shower change trolley for multiple patients, it is necessary to both clean and disinfect it between uses on different patients to prevent the risk of cross infection.

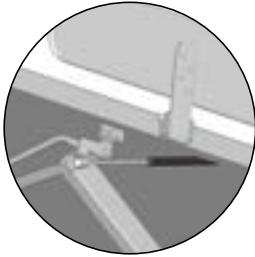
NOTICE

If the product is used in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.


Maintenance

Maintenance inspection should be carried out once a year. The inspection should be recorded in a logbook describing what has been checked and replaced. Spare parts should always be obtained from your supplier or Pressalit in order to ensure a continuously safe product and the guarantee. Contact your supplier or Pressalit for purchase of spare parts and drawings. Perform a visual inspection of steel parts for penetrating rust.

Maintenance inspection


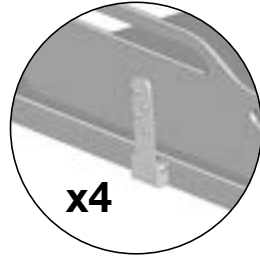
Check whether the gas spring can still tilt the trolley.

x4

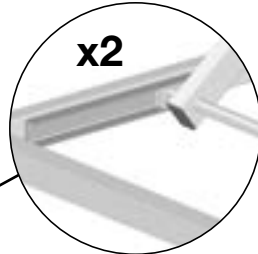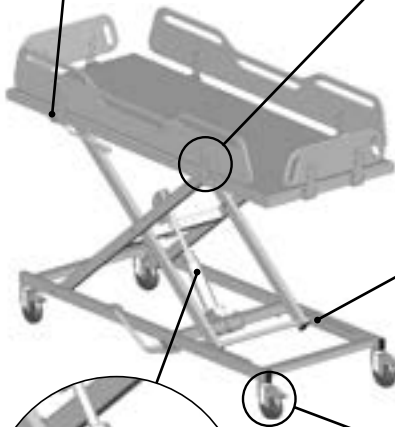Check whether all nuts, screws and hinges have become loose. Tighten if necessary.

x2

Lubricate the sliding rails at least every six months. Use a food grade grease.

Inspect the hydraulic pump for any signs of oil leaks. If minor leaks are detected, they can be cleaned with a dry cloth.

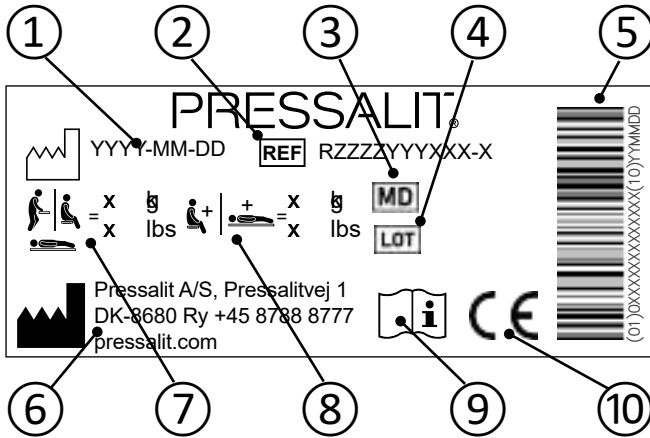
x4

Check the trolley and wheels for corrosion. Replace the wheels if necessary.

en

Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

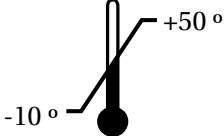
⑧ Max load

⑨ Consult manual for use

⑩ These products are CE-marked in accordance with relevant requirements in regulation (EU) 2017/745 (5 April 2017) on medical devices.

A declaration of conformity can be found at pressalit.com.

Storage and transportation

	<p>The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.</p>
-----------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------

IMPORTANT

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Troubleshooting

Symptom	Cause	Action
The trolley has difficulty moving up and/or down.	The lifting unit lacks lubrication.	Lubricate the sliding rails. See Maintenance .
The trolley will not tilt.	The gas spring is defective.	Replace the gas spring. See Maintenance .

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the warranty. Expected service life of the shower change trolley is 10 years.

en

Technical data

Max. user weight	200 kg / 440 lbs	
Materials	Powder-lacquered steel Plastics CDF	
Weight	R9504717	84 kg / 185 lbs
	R9504721	88 kg / 194 lbs

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected. When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion. The following table provides information on the recycling and handling of the product.

en

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available. If such systems are not available, incineration of plastic waste can be an alternative, as it can generate energy that can be used for power production.

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