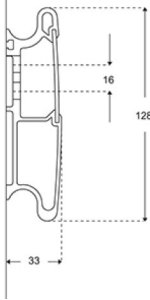




Product Item number	PLUS wall track, 1800 mm R9818112	
Colour	VVS:615841118 NRF:7501017 EAN:5708590305813 HMI: 71743 Anthracite grey	
Product description	PLUS wall tracks are used for mounting of support arms, shower seats and wash basin brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track. PLUS wall track, 1800 mm, for horizontal mounting. The cover plate hides bolts and wires and facilitates cleaning. Materials: Aluminium and polystyrene. See below a selection of products for fitting to the wall track. Excl. mounting kit.	
Adjustability Product capacity	Fixed mounting Maximum load: Up to 1000 kg depending on the wall type.	
Materials Ergonomy Features	The wall track is primarily made of 10-micrometre anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer. The track distributes the weight of the individual bathroom product, for example a wash basin bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions. The horizontal track makes it possible to shift the product along the wall to constantly optimize the available space in the bathroom. The front cover plate conceals mounting bolts and any cables.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



PRESSALIT®

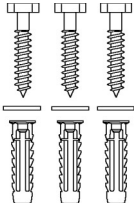
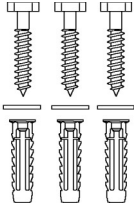
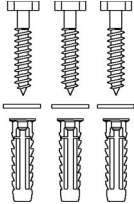
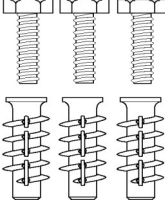
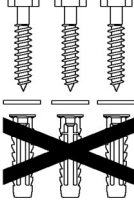
Product	PLUS wall track, 1800 mm
Item number	R9818112
Cleaning	The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.



PRESSALIT®

Product	PLUS wall track, 1800 mm
Item number	R9818112

Installation in different wall types

	Concrete >B25 V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Brick >= Mz12 V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Aerated Concrete P4 V8670 - Mounting kit, 3 bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



PRESSALIT®

Product	PLUS wall track, 1800 mm
Item number	R9818112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®

9. september 2026