




PRESSALIT®

Product Item number	PLUS backrest, fixed height R6404112
Colour	VVS:777530212 NRF:6317034 EAN:5708590365398 HMI: 124215 FDA: INN/D135865 Anthracite grey
Product description	 PLUS backrests are offered in a number of colours and versions, they are available as both fixed height and adjustable. They are suitable for wall-mounting behind a toilet without cistern or for mounting on one of Pressalit's toilet lifters. PLUS backrest, 385 x 209 mm, soft feel. Fixed height, for mounting directly on the wall behind a wall-mounted elongated toilet without cistern. Materials: Aluminium and polyurethane foam with a plywood core. Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.
Adjustability Product capacity	Fixed mounting Maximum load: 50 kg. Maximum user weight: 100 kg. When the backrest is loaded with 50 kg, each wall fastener will be carrying 118 kg. <i>Backrests need to be able to support the weight of the person who is leaning against it.</i> <i>The backrest is not intended to carry the full body weight of a person, and the test methods applicable to backrests reflect this: ISO 17966 prescribes a static test at 50% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 50 kg, it is – according to ISO 17966 – the equivalent of a user weighing 100 kg leaning on the backrest.</i>
Materials Ergonomy Features	The backrest is made of polyurethane (PU) foam which is covered with an in-mould coated surface that is protected using a clear PU lacquer. Soft and slip-resistant material that has a warm feel. For toilet without cistern
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

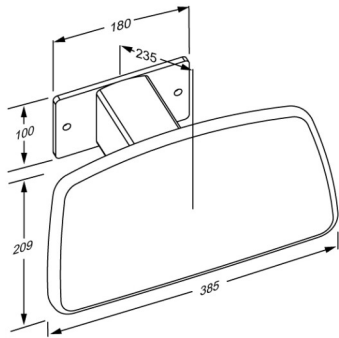

Product	PLUS backrest, fixed height
Item number	R6404112
Cleaning	PLUS backrest The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Erisan OXY+ Tvättspit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH) NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa) Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water. The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water. Expected service life: 10 years. Contraindications for use: It is not recommended that people who need to lie down while being changed use the backrest for support.



PRESSALIT®

Product	PLUS backrest, fixed height
Item number	R6404112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®