




PRESSALIT®

Product Item number	VALUE fold down grab bar, 33.5" R1081000
Color	VVS:615892102 NRF:6191054 RSK:7828176 EAN:5708590346847 HMI: 138190 FDA: ILS/D135866 White
Product description	 VALUE and VALUE II fold down grab bars with the ALBA wing mounting profile are specifically designed for use with SELECT TL3 and TL4 toilet lifters. The height adjustment of the toilet lifter is done only with the control panel on the fold down grab bar. VALUE fold down grab bar, 33.5", foldable, height adjustable 5.9" without the use of tools (right hand operated). Provides firm support when sitting, standing, or transferring to and from a toilet. Materials: Rilsan-coated steel, diecast aluminum and end plug in nylon (PA). For fitting to TL3/TL4/TL5 electrical/manual toilet lifter, mounting profile included. For mounting on user's left side.
Adjustability Adjustment height Product capacity	Height adjustable 5,91 inch Maximum load: 265 lbs
Materials	The fold down grab bar is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA). The mounting profile is made from diecast aluminum.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

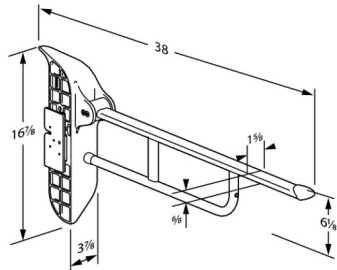

Product	VALUE fold down grab bar, 33.5"
Item number	R1081000
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean) NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.