Toilet lifter – Operation and maintenance manual	en
Toiletløfter – Brugs- og vedligeholdelsesvejledning	dk
Toalettupphäng – Bruks- och underhållsanvisning	sv
Oppheng til toalett – Bruks- og vedlikeholdsveiledning	no

English	3
Dansk	21
Svenska	38
Norsk	55

en	The guarantee covers faults or defects in material or manufacture within a period of 5 years. Products are subject to minor technical modifications and design deviations. E. & O.E.
dk	Garantien dækker i 5 år fra fakturadato for materiale- og fabriktionsfejl. Der tages forbehold for trykfejl, tekniske ændringer og modelafvigelser.
sv	Garantin gäller i 5 år från fakturadatum för material- och fabriktionsfel. Med reservation för tryckfel, tekniska ändringar och modellavvikelser.
no	Garantien gjelder i 5 år fra fakturadato for material- og fabriktionsfeil. Med forbehold for trykkfeil, tekniske endringer og modellforskjeller.

Content

Symbols used in this manual	4
General safety	5
Product label	6
Intended use	8
Intended operator profile	8
Operating environment	8
Assembly and installation	9
Versions	10
Compatible support arms	10
Operating instructions	11
Support arms	12
Cleaning	13
Maintenance	14
Maintenance inspection	14
Dimensions - mm	17
Support arms	17
Dimensions - inch	18
Support arms	18
Technical data	19
Trouble-shooting	20
Disposal and recycling	21

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressalitcare.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Feet injury warning

Head injury warning

Electricity or potentially dangerous voltage.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.


CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

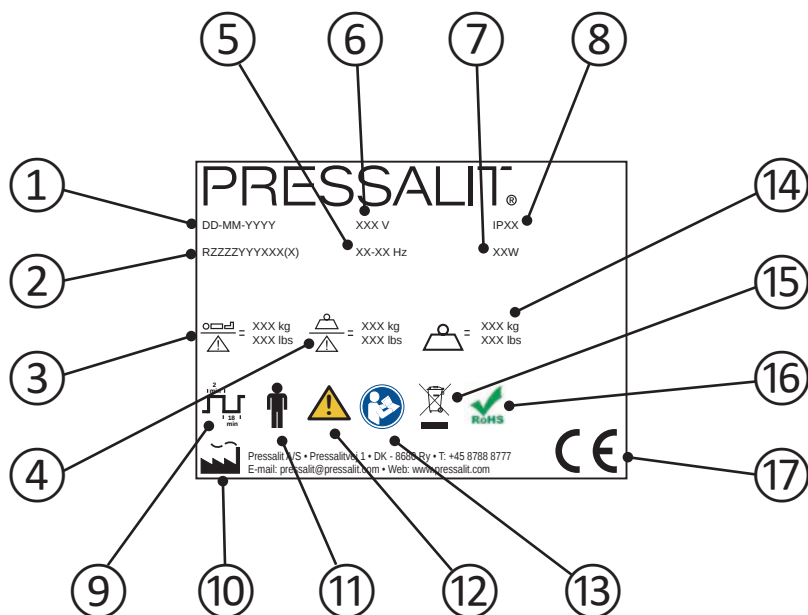- ① Production date
- ② Article number
- ③ Maximum user weight
- ④ Safe working load
- ⑤ Frequency
- ⑥ Mains voltage

-
- ⑦ Maximum power consumption
-
- ⑧ IP class
-
- ⑨ Duty cycle The motor on height-adjustable products can be used for a maximum of 2 minutes of continuous operation, followed by 18 minutes pause.
-
- ⑩ Name and address of manufacturer
-
- ⑪ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock.
-
- ⑫ Consult the manual for use for important cautionary information such as warnings and precautions.
-
- ⑬ Consult manual for use
-
- ⑭ Product weight
-
- ⑮ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
-
- ⑯ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
-
- ⑰ This product is CE-marked in accordance with the European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and with Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at www.pressalitcare.com.
-

Product label placement ①.

Intended use

This product is intended for optimum ergonomics at a toilet, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

Never place objects underneath the toilet.

Versions

R8273, R8283


Compatible support arms

R3886, R3686

R3871, R3671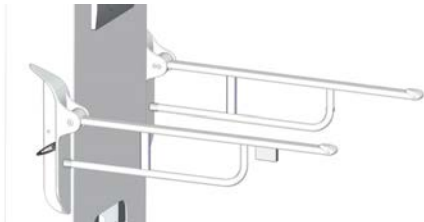

R1081, R1083

en

Operating instructions

The toilet lifter can be adjusted 200 mm up and down. Max load toilet lifter: 200 kg / 440 lbs. Max load support arms: 150 kg / 330 lbs.

The height adjustment is controlled by operating buttons attached to the right support arm.

Press and hold the Up button to raise the toilet, and press and hold the Down button to lower it. When the button is released, the motion stops.

Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

CAUTION

Never place objects underneath the toilet.

Support arms

The support arms can be folded up and down. Fold them up when not in use.

For support arms R3686, R3786, R3671 and R3771: You can hear a click when the support arm is correctly folded up.

Take care to avoid injury to hands when in the vicinity of closing moveable parts.

The support arms can be adjusted 150 mm in height.

Fold up the arm.

Press and turn the height adjustment locking handle ①.

Move the wing ② with the support arm up or down.

Tighten the locking handle again. It will slide back into place by itself.

en

Cleaning

Clean the product after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth.

For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids or chloride.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of aggressive, abrasive, corrosive or chlorine-based cleaning or disinfecting products.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

Maintenance

Expected service life of the toilet lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that maintenance inspection is done once a year.

Maintenance inspection

Take the toilet to its highest position.


**Disconnect mains
before removing
the covers.**

Remove the control panel ①.
Remove the toilet ②.

en


Unscrew the locking handles for the support arms ③.

Unscrew the screws for the support arms ④ and remove both the support arms.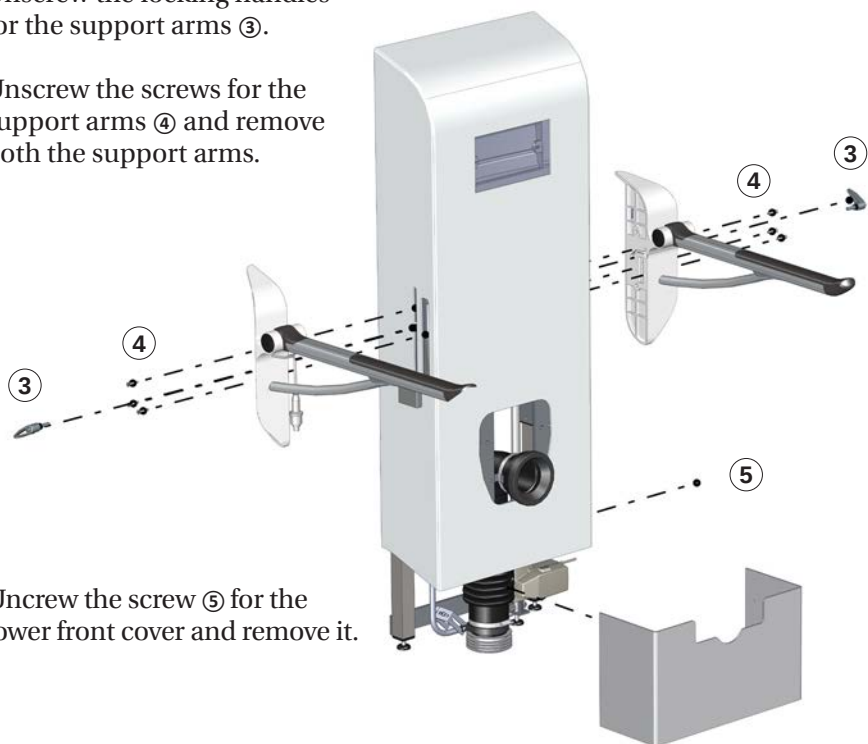


Unscrew the screw ⑤ for the lower front cover and remove it.


en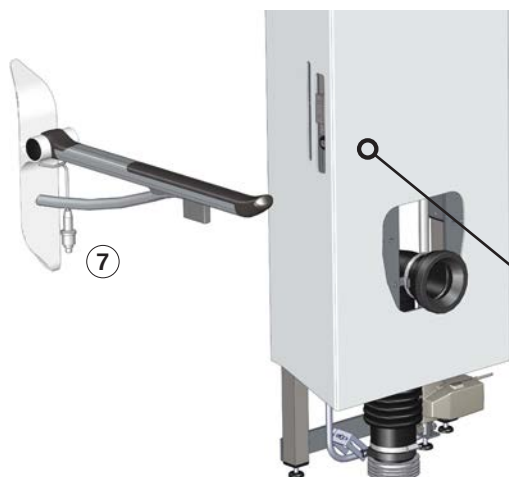

Reach up under the front cover to unplug the wire ⑥ from the control buttons to the control box inside the cabinet. The plug ⑦ can pass through the hole in the cabinet side.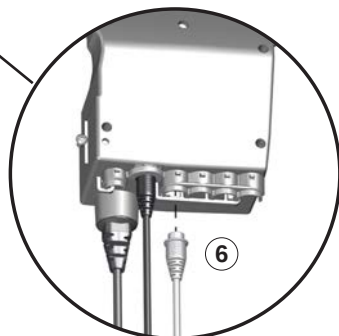

Remove the front cover.

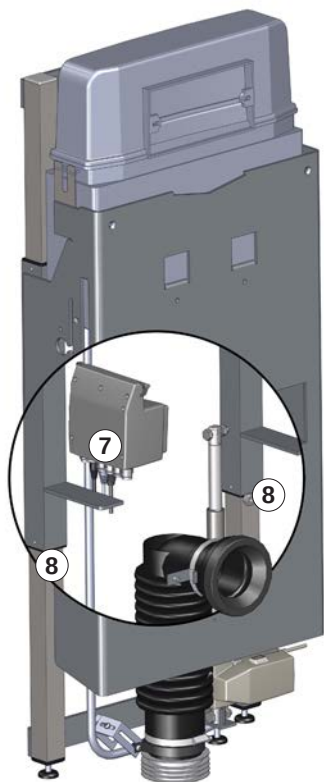Check all electrical connections ⑦.


Lubricate the inner square tubes ⑧ with WD-40 or equivalent.

Reattach the front and lower front cover. en
Mount the support arms and the toilet.

Reconnect mains.


Take the toilet to its lowest position, and then run it to the top position again to distribute the lubricant.

Dimensions - mm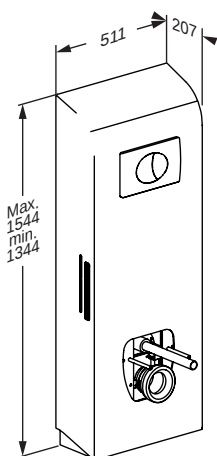


R8273, R8283


Support arms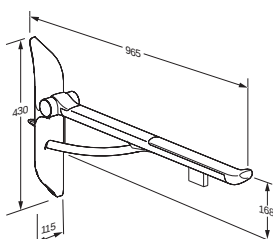

R3886, R3686

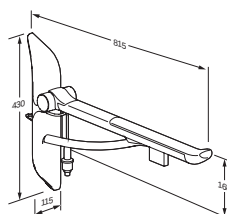
R3871, R3671

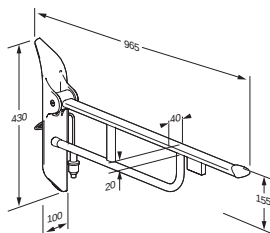
R1083, R1081

Dimensions - inch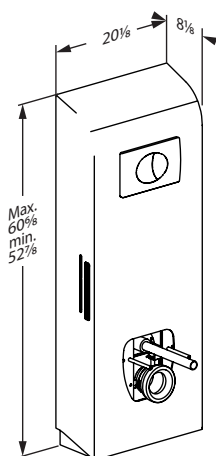


R8273, R8283


Support arms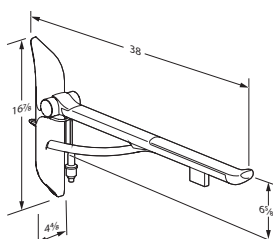

R3886, R3686

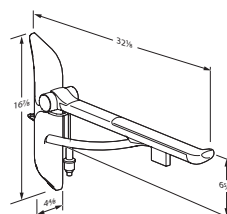R3871, R3671

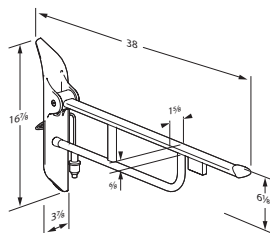
R1083, R1081

Technical data

Max load	Toilet lifter: 200 kg / 440 lbs
	Support arms: 150 kg / 330 lbs
Materials	Stainless steel Plastics
Weight	
R8273, R8283	55,9 kg / 123.2 lbs
R3886	5,34 kg / 11.7 lbs
R3686	5,24 kg / 11.5 lbs
R3871	5 kg / 11 lbs
R3671	4,9 kg / 10.8 lbs
R1083	4,57 kg / 10 lbs
R1081	3,15 kg / 6.9 lbs

Trouble-shooting

Symptom	Cause	Action
Operating buttons does not work.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".
Height adjustment of support arms is not possible.	The screws ① holding the support arm in place are tightened too hard.	Loosen the screws until height adjustment is possible. Check mounting instruction.

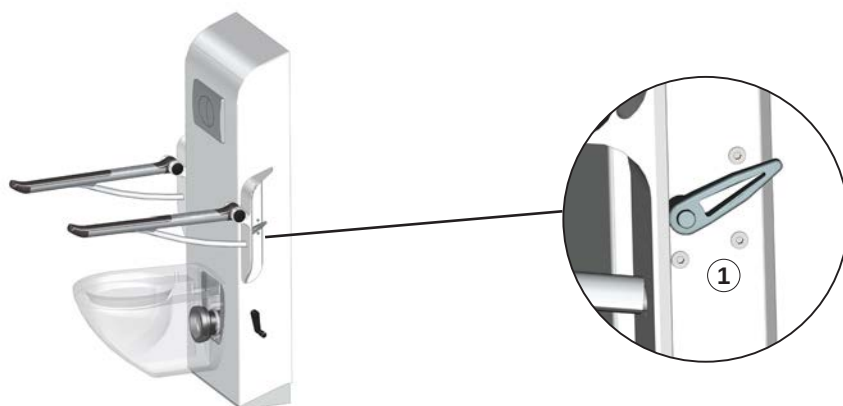Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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