

## PRESSALIT.

| PLUS 90° angle grab bar 15.8" x 18.7"   |
|---|
| RT115108  |
| VVS:777567108   |
| NRF:6317175   |
| EAN:5708590154015   |
| HMI: 25432  |
| NBS:N13:436   |
| Dark blue (RAL 5011)  |
|   |
|   |
| PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom A reliable solution with many different options, which can easily lay the grounds for increased independence. |
| PLUS grab bar, angle 90° - left, diamter 1.3", A:15.8" x B:18.7".<br>Material: Powder-coated aluminium and plastic.<br>Excl. mounting kit.  |
| Maximum load: 440 lbs.  When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs.  Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.  |
| Core material: Aluminium Surface material: Polyester lacquer Cover cap: Cycoloy plastic Strap: Nylon (PA)   |
| The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.  |
| Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.   |
|   |



# PRESSALIT.®

| Product                   | PLUS 90° angle grab bar 15.8" x 18.7"  |
|---------------------------|--|
| Item number               | RT115108   |
| Cleaning                  | This product should be cleaned with hot water or a mild cleaning solution.  Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.   |
|                           | The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.  DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%  |
|                           | NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro  The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.  |
| Declaration of conformity | DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water. This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices. |
|                           | 1  |



## PRESSALIT.

| Product    |   |  |  |
|------------|---|--|--|
| Item numbe | r |  |  |

PLUS 90° angle grab bar 15.8" x 18.7"

RT115108

### Mounting in different wall types

#### Concrete >B25

V8617 - Mounting kit, 12 bolts  $\emptyset$ 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.






#### Solid Brick >= Mz12

V8617 - Mounting kit, 12 bolts  $\emptyset$ 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.



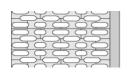

### Solid Sand Lime Brick >= Hlz12

V8617 - Mounting kit, 12 bolts  $\emptyset$ 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.






#### **Aerated Concrete P4**

V8667 - Mounting kit, 12 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.



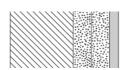

### Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8617 - Mounting kit, 12 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed.

To be ordered separately.





## PRESSALIT®

| Product<br>Item number | PLUS 90° angle grab bar 15.8" x 18.7"<br>RT115108  |           |
|------------------------|--|-----------|
|                        | Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood  Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.           |           |
|                        | Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)  Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. |           |
|                        | Drywall (2 x 12,5 mm) water-resistant  Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.  | <u>^!</u> |

pressalit.com