




PRESSALIT®

Product Item number	SCT 2100 shower change table, coated canvas, electrically height adjustable R8581418000
Colour	VVS: 777543740 EAN: 5708590376110 HMI: 140138 White
Product description	 <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>SCT 2100 shower change table, powered, height adjustable 700 mm; from 300 mm - 1000 mm, with wired hand control Foldable. Incl. safety rail and water collection tray with integrated outlet. Outer measurements 910 mm x 1800 mm. Excl. mounting kit.</p>
Adjustability Adjustment height Product capacity Speed upwards Noise level	Height adjustable 700 mm Maximum user weight (MUM): 200 kg. Safe working load (SWL): 200 kg. When the table is loaded with 200 kg, each mounting bolt supports 26 kg <i>This product has been tested statically with a load of 500 kg as the IEC 60601-1 standard requires testing with safety factor 2.5.</i> Speed up mm: 19 mm/s Speed down mm: 21 mm/s 50 dB
Materials	Construction in powder-laquered stainless steel, canvas cover in PVC-coated polyester (for extra durability), water collection tray in PVC-U and carriage housing in polystyrene.

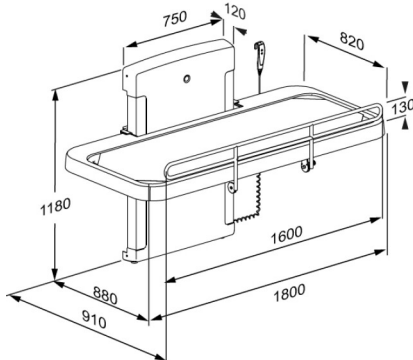

Product	SCT 2100 shower change table, coated canvas, electrically height adjustable
Item number	R8581418000
Ergonomy	The 2000 series shower and changing tables has a low start position, to make transfers to the bench easier, and with washable coated canvas for easy cleaning. The flexibility of the shower and changing table provides for optimal working conditions for the carer as well as a high level of comfort and safety for the user.
Features	<p>A water collection tray with integrated water outlet prevents water from splashing on carers or the floor. The flexible hose attached to the centre mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. The integrated gas cylinders enable the table to be folded out of the way when not in use to maximise floor space in front of the unit.</p> <p>Electric with wired hand control – the red buttons clearly indicate “Up” and “Down”. The wired hand control comes with a holder for fitting to the wall.</p> <p>The stretcher of the table is very easy to clean, and the stretcher cover can easily be removed for washing or easy replacement.</p>
Mains connection (V)	230
Actuator (motor)	LA40 Careline (Linak), 4000 N, IPX6
Control box	CA63 Careline, (Linak), IPX6
Wired remote control	HB30 (Linak), IP66
Power consumption (maximum load)	<p>Power consumption by maximum load, watt: 420 W</p> <p>Power consumption by standby: 0,5 W</p>
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.




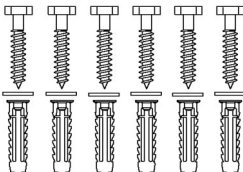

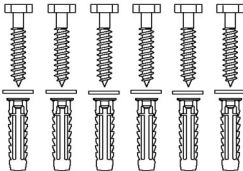
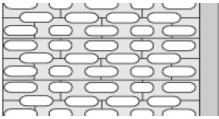
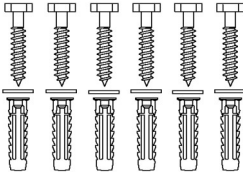
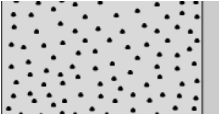
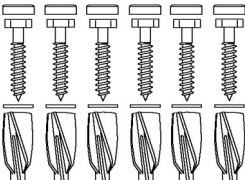
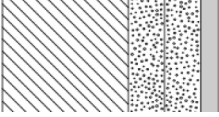
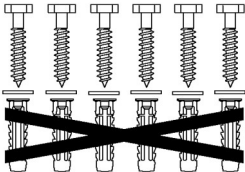
PRESSALIT®

Product	SCT 2100 shower change table, coated canvas, electrically height adjustable
Item number	R8581418000
Cleaning	<p>SCT 1000 & 2000</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS:</p> <p>Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes OxiGenesis from ENVIROX</p> <p>NON-DISINFECTANTS:</p> <p>Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>



Product	SCT 2100 shower change table, coated canvas, electrically height adjustable
Item number	R8581418000

Installation in different wall types

	<p>Concrete >B25</p> <p>V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p>Solid Brick >= Mz12</p> <p>V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p>Solid Sand Lime Brick >= Hlz12</p> <p>V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. To be ordered separately.</p>	
	<p>Aerated Concrete P4</p> <p>V8658 - Mounting kit, 6 coach screws Ø10 x 70 - plugs GB14. To be ordered separately.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</p> <p>V8608 - Mounting kit, 6 coach screws Ø10 x 70 - plugs SX12. NB! Plugs are not needed. To be ordered separately.</p>	



PRESSALIT®

Product Item number	SCT 2100 shower change table, coated canvas, electrically height adjustable R8581418000
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</p> <p>Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
 Pressalitvej 1
 DK-8680 Ry
 Tel.: +45 8788 8788
 Fax: +45 8788 8789
 E-mail: pressalit@pressalit.com

PRESSALIT®