




PRESSALIT®

Product	VALUE IV back rest, fixed height
Item number	R1661000112
Colour	VVS: 777525090 NRF: 6311837 EAN: 5708590355115 HMI: 124224 DB: 2084079 FDA: INN/D135865 Anthracite grey 
Product description	VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering. VALUE IV back rest. Fixed height. Materials: Diecast aluminium and vacuum-moulded polyethylene (PE). Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.
Adjustability	Fixed mounting
Materials	The wall profile is made of diecast aluminium, and the backrest is made of vacuum-moulded polyethylene (PE)
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

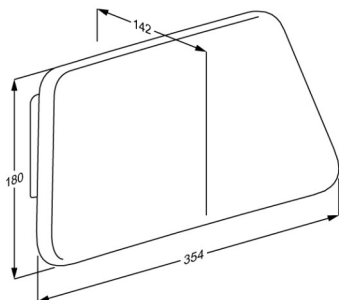

Product	VALUE IV back rest, fixed height
Item number	R1661000112
Cleaning	VALUE backrest The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean) NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.



PRESSALIT®

Product	VALUE IV back rest, fixed height
Item number	R1661000112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®