R8455512

## CT 5000

Change table – Operation and maintenance manual	en
Skifteleje – Brugs- og vedligeholdelsesvejledning	dk
Pflegeliege – Gebrauchs- und Pflegeanleitung	de
Table de change – Manuel d'utilisation et d'entretien	fr
Aankleedtafels – Gebruiks- en onderhoudshandleiding	nl
Skötbord – Bruks- och underhållsanvisning	sv
Stellebord – Bruks- og vedlikeholdsveiledning	no
Cambiator - Instrucciones de uso y mantenimiento	es
Stół do pielęgnacji – instrukcja użytkowania i konserwacji	pl
换洗台- 操作和维护手册	中文

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Pressalit's Environmental and Quality Management Systems are certified by LRQA

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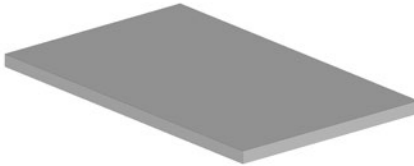
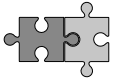
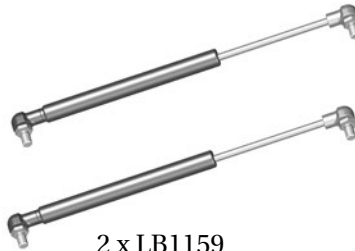
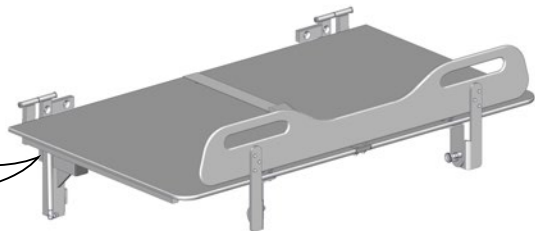
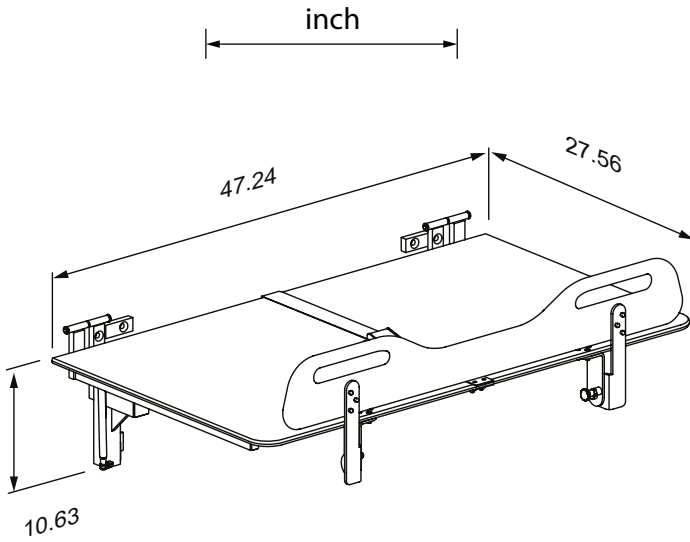
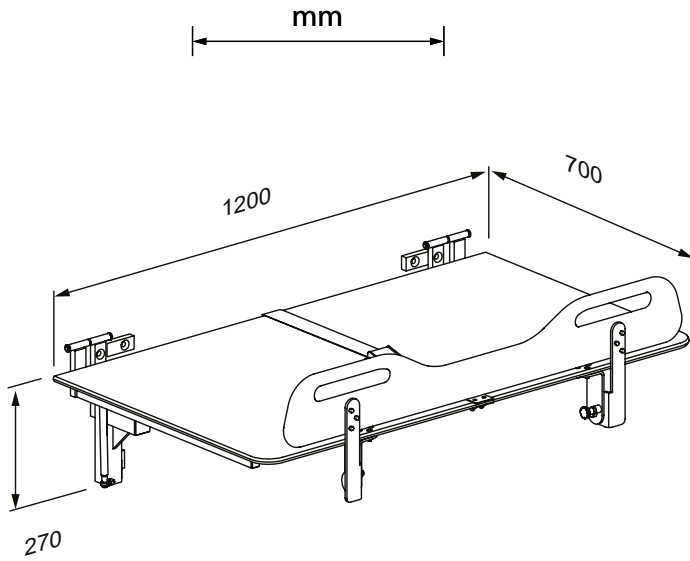
## Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from [pressalit.com](http://pressalit.com).

Original instructions

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

**WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

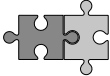
**IMPORTANT** indicates important information about handling and use of the product.

**NOTICE** is used to address practices not related to personal safety.

en

 <p>Hand injury warning</p>	 <p>Do not leave patient unattended</p>	 <p>Compatible products</p>
 <p>Article number/ versions</p>	 <p>Product weight</p>	 <p>Max. user weight</p>
 <p>Spare parts</p>		

# General safety

## **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

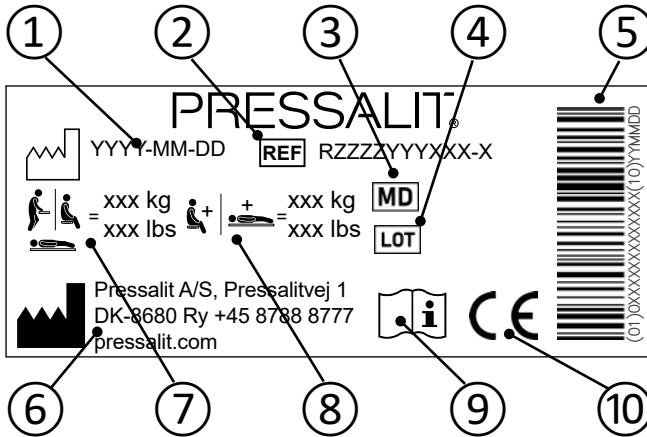
## **CAUTION**

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

## **NOTICE**

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

# Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

⑨ Consult manual for use

⑩ These products are CE-marked in accordance with relevant requirements in regulation (EU) 2017/745 (5 April 2017) on medical devices.

A declaration of conformity can be found at [pressalit.com](http://pressalit.com).

## Intended use

This product is designed for users who can partially assist themselves while getting dressed.

## Intended user profile

The product is suitable for all users—adults, children, the elderly—whose weight does not exceed the **maximum load of 150 kg / 331 lbs.**

## Operating environment

The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

## Contraindications for use

The use of the change table is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions. Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the change table only after consulting with the attending physician. Do not use if the patient's weight exceeds 150 kg / 331 lbs.

## Biocompatibility

The biocompatibility of the materials that come into contact with the user's skin during the intended use has been evaluated according to the procedures outlined in DS/EN ISO 10993-1:2020, which involve assessing the materials for harmful substances. Based on this evaluation, the materials are considered suitable for their intended use.

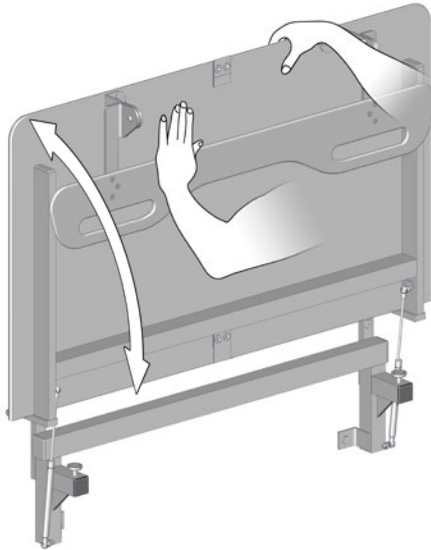
The product's surface may become hot and/or cold when exposed to external sources of heat or cold, such as sunlight or frost.

## NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities. A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

# Operating instructions

en

Support the bench using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the table up against the wall after use. Be sure to fold the safety rail away first.


**Take care to avoid injury to hands and head when in the vicinity of moving mechanical parts.**

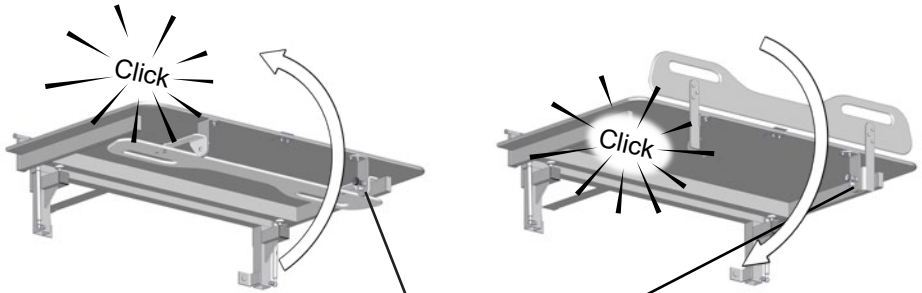**Do not leave the patient unattended.**

## The safety rail


**To pull the safety rail up:** Release the lock ① and pull the safety rail from the underside of the table, and swing it upwards. The safety rail will automatically lock into position.


**To fold the safety rail away:** Release the lock ① and swing the safety rail down to the underside of the table. The safety rail will automatically lock into position.

en

## The body belt

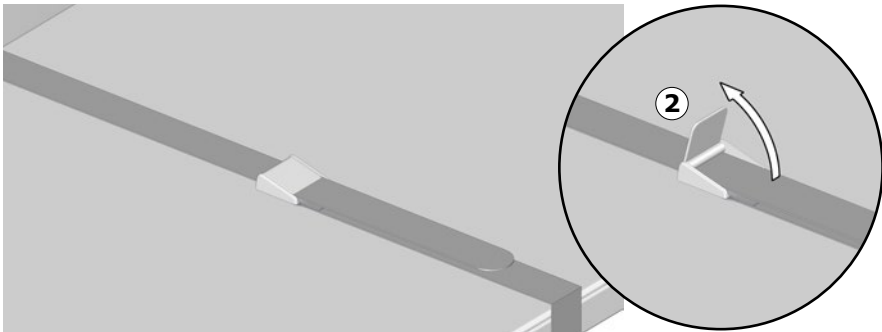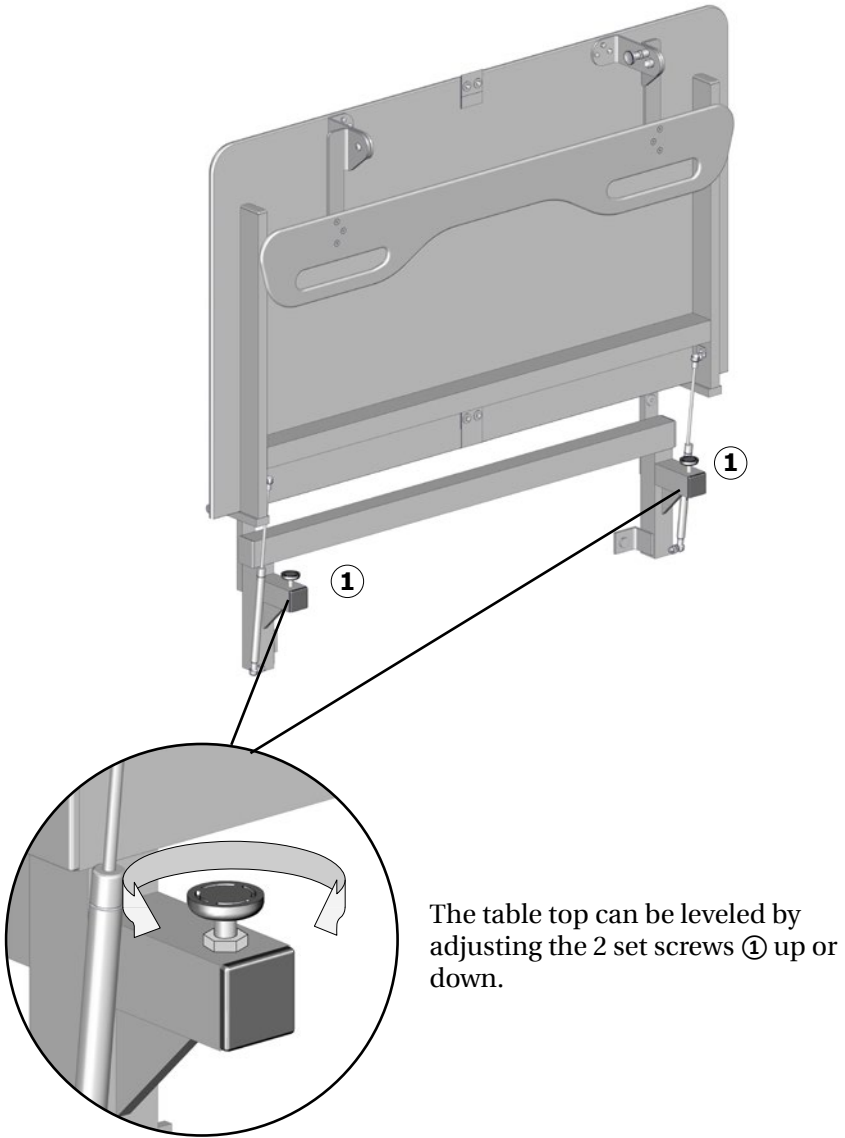The product is equipped with a body belt, that can be used to secure the user. Pull the belt lock lever ② up to release the belt lock. Press the lever down again to lock the body belt.

# Cleaning

Clean the change table after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth. Wipe off any water on the bench surface.

## **CAUTION**

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

## **IMPORTANT**

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids or chloride.
- Never spray the change table directly with a high pressure cleaner.
- Avoid use of aggressive, abrasive, corrosive or chloride-based cleaning or disinfecting products.

# Disinfection

When using the change table for multiple patients, it is necessary to both clean and disinfect it between uses to prevent the risk of cross-infection. A list of suitable disinfectants can be found at [pressalit.com](http://pressalit.com).

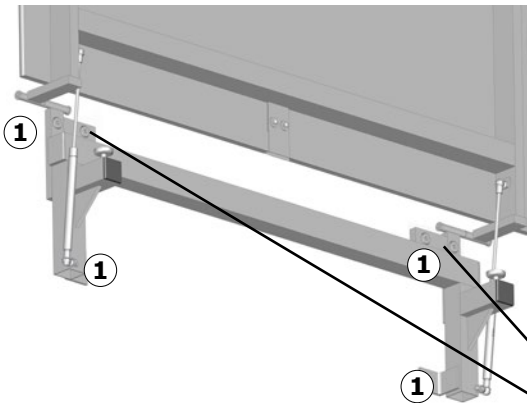
en

# Maintenance

Expected service life of the change table is 10 years, provided that maintenance is carried out as described below:



It is recommended that service inspection is done once a year.

## Maintenance inspection

Check the wall mounting screws ① and re-tighten if necessary.

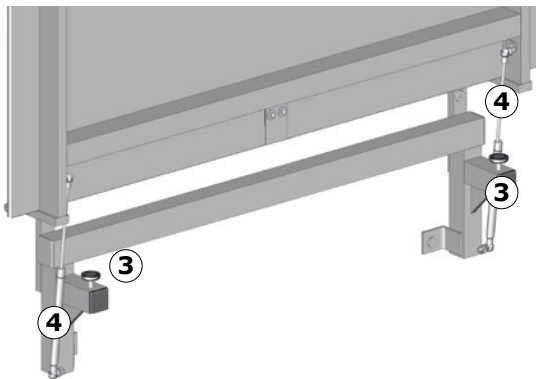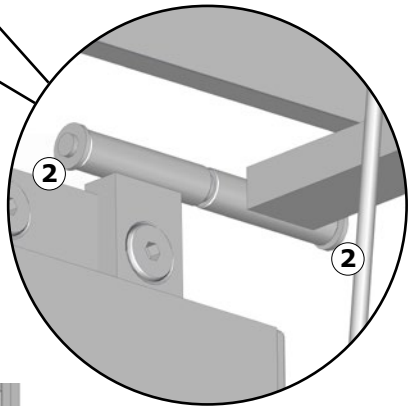Make sure that the star lock washers ② are firmly attached to the hinge axles.

Check the adjustment of the 2 set screws ③.

Check the gas cylinders ④ for wear.

Perform a visual inspection of steel parts for penetrating rust.

## NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty of the shower table will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.

## Trouble-shooting

Symptom	Cause	Action
The change table does not stay in upright position.	Gas cylinder is defective.	Contact Pressalit A/S or your local dealer.
The change table top is not level.	The set screws are too high/low.	Adjust the set screws.
	The set screws are defective.	Contact Pressalit A/S or your local dealer.
The star lock washers are loose or fallen out.	The axle can push them out over time	Use a hammer to put the lock washers back in place. Replacement star lock washers can be ordered, article no. LB3502

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

## Storage and transportation

- The temperature for transportation and storage of the unit must be between 5 and 40 degrees Celsius / 41 and 104 degrees Fahrenheit.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

## Technical data

Max load	150 kg / 331 lbs
Materials	Powder-lacquered stainless steel, Melamine Compact laminate, Polyamid (nylon)
Weight	27 kg / 60 lbs

en

# Disposal and recycling

The unit contains reusable materials, and there are no known hazards or risks associated with the disposal of the product. All components can be discarded after being cleaned and disinfected. When disposing of the product, we recommend disassembling it and breaking down its components into different waste groups for recycling or combustion.

<b>Recyclable components</b>	<b>Material</b>
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Utilize plastic sorting and recycling systems where locally available. Alternatively, incinerating plastic has the added benefit of generating energy, which can be used, for example, in combined heat and power production.

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