

PRESSALIT.

Product	INDIVO lift for worktop 2401-3000 mm		
Item number	RK1254202		
	EAN:5708590378602	No.	
		Min 600 Max 70	
		2401 - 3000	
Colour	Grey (RAL 9006)		
	,		
		•	
Product description		be easily integrated into the private home, ensuring	
		ne full potential and use the height adjustable o help provide a flexible kitchen solution for e.g. care	
		ool science labs, wherever there is a need for	
		·	
	INDIVO lift for worktop, with three Incl. cross brace and five bracket		
		with switch or wired remote control. m. Please indicate length and depth of worktop wher	
	ordering.	The rease majorite length and depth of worklop when	
	Excl. mounting kit.		
Adjustability	Height adjustable		
Adjustment height	300 mm		
Product capacity	Maximum load: 150 kg, this inclu	des the weight of the workton	
	The load must be evenly distribut		
Speed upwards	Speed up mm: 20 mm/s Speed down mm: 20 mm/s		
Noise level	45 dB		
Materials	Lifting columns: Powder-coated s	steel	
	Cross brace: Aluminium Brackets: Powder-coated steel		
Ergonomy		hen worktop is ideal in all situations where various	
-	users - both seated and standing	- use the same kitchen.	
	facilities such as group homes ar	nes as well as larger centres with common kitchen	



PRESSALIT.

Product Item number	INDIVO lift for worktop 2401-3000 mm RK1254202	
Features	Lifts are height adjusted either via switches in the front edge or with a wired remote control.	
Actuator (motor)	DL17 Deskline (Linak) 70N, IP20	
Control box	CBD6S (Linak) IP20	
Wired remote control	HB53 (Linak), IPX1	
Power consumption (maximum load)	Power consumption by maximum load, watt: 300 W Power consumption by standby: 2 W	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date.	



PRESSALIT®

Product	INDIVO lift for worktop 2401-3000 mm
Item number	RK1254202
Cleaning	INDIVO lift for kitchen worktop The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS:
	Suma Star free D1 Taski 100 Taski 200 Milizid Citro
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.
WEEE	Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.



PRESSALIT®

Product	INDIVO lift for worktop 2401-3000 mm
Item number	RK1254202
Declaration of conformity	CE UK RoHS_LVD_EMC This product is in compliance with relevant requirements in: - the RoHS Directive 2011/65/EU - the EMC Directive 2014/30/EU - the Low Voltage Directive 2014/35/EU. Furthermore, for the UK, in compliance with relevant requirements in: - The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012 - The Electromagnetic Compatibility Regulations 2016 - The Electrical Equipment (Safety) Regulations 2016



PRESSALIT.

Product

INDIVO lift for worktop 2401-3000 mm

Item number RK1254202

Installation in different wall types


a .	V. 0:1	۲	
Σ.	, v . a	Ø.	20.
ļ	22	: ₫, ▽	. ₽.
	· D · B	<u>``.c</u>	

Concrete >B25

V8607 - Mounting kit, 9 coach screws Ø10 x 70 - plugs SX12.

To be ordered separately.




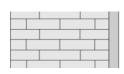


Solid Brick >= Mz12

V8607 - Mounting kit, 9 coach screws Ø10 x 70 - plugs SX12.

To be ordered separately.

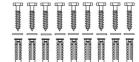


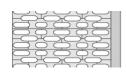

Solid Sand Lime Brick >= HIz12

V8607 - Mounting kit, 9 coach screws Ø10 x 70 - plugs

To be ordered separately.

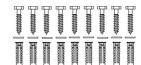





Aerated Concrete P4

V8657 - Mounting kit, 9 coach screws Ø10 x 70 - plugs GB14.

To be ordered separately.

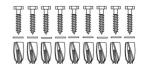


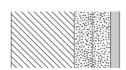

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8607 - Mounting kit, 9 coach screws \emptyset 10 x 70 - plugs SX12. NB! Plugs are not needed.

To be ordered separately.

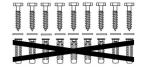



PRESSALIT®

Product Item number	INDIVO lift for worktop 2401-3000 mm RK1254202	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (9 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (9 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant Wall fasteners (9 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or	
	drywall it is possible to use e.g. fischer KD8 toggles or similar.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT.