




<p><b>Product</b> <b>Item number</b></p>	<p>SCT 2000 shower change table, coated mesh cover, fixed height R8425218000</p>
<p><b>Color</b>  <b>Product description</b></p>	<p>VVS:777543020 EAN:5708590372167 HMI: 130360 FDA:INT/D124320</p> <p>White</p>  <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>SCT 2000 shower change table, fixed height, foldable. Incl. safety rail and water collection tray with integrated outlet. Outer measurements 33.9" x 70.9". Excl. mounting kit.</p>
<p><b>Adjustability</b> <b>Product capacity</b></p>	<p>Fixed mounting</p> <p>Maximum user weight (MUM): 386 lbs. Maximum load (SWL): 386 lbs. When the table is loaded with 386 lbs, each mounting bolt supports 115 lbs. <i>This product has been tested statically with a load of 579 lbs as the ISO 17966 standard requires testing with a safety factor 1.5.</i></p>
<p><b>Materials</b> <b>Ergonomy</b></p>	<p>Construction in powder-laquered stainless steel, mesh cover in PVC-coated polyester (for extra durability) and water collection tray in PVC-U.</p> <p>The 2000 series changing table has a low start position to make transfers to the table easier. The washable, perforated mesh cover lets the water pass through to the water collection tray underneath.</p> <p>The flexibility of the changing table provides for optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p>

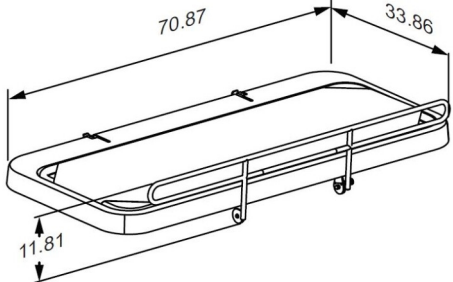

<b>Product</b> <b>Item number</b>	SCT 2000 shower change table, coated mesh cover, fixed height R8425218000
<b>Features</b>	<p>A water collection tray with integrated water outlet prevents water from splashing on caregiver or the floor. The flexible hose attached to the center mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space in front of the unit.</p> <p>The stretcher of the table is very easy to clean, and the stretcher cover can be unzipped for washing or easy replacement.</p>
<b>Warranty</b>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.</p>
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes OxiGenesis from ENVIROX</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>









<b>Product</b>	SCT 2000 shower change table, coated mesh cover, fixed height
<b>Item number</b>	R8425218000

## Mounting in different wall types

	<p><b>Concrete &gt;B25</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  To be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  To be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  To be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b>  V8658 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs GB14.  To be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  NB! Plugs are not needed.  To be ordered separately.</p>	

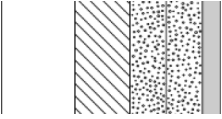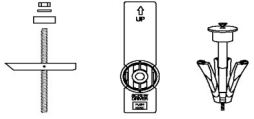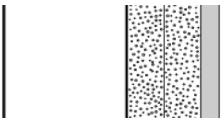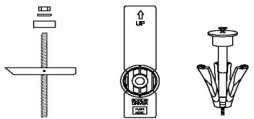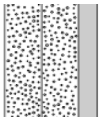

<b>Product</b> <b>Item number</b>	SCT 2000 shower change table, coated mesh cover, fixed height R8425218000
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar. 
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar. 
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

It is the responsibility of the installer to ensure that suitable wall fasteners are used.