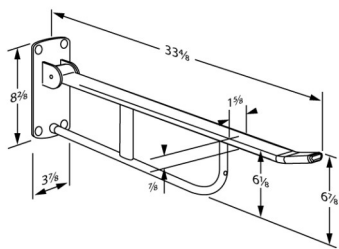




PRESSALIT®

Product Item number	VALUE II fold down grab bar, fixed height R1109000
Color	VVS:615891100 NRF:6191057 RSK:8487434 EAN:5708590367576 HMI: 123560 FDA: ILS/D135866 White
Product description	<div>  </div> <p>VALUE II fold down grab bar, fixed, foldable, length 33.5". The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p> <p>VALUE II fold down grab bar, fixed, foldable, length 33.5". The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Materials: Rilsan-coated steel, pressure-cast aluminium, plastic. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p>
Adjustability Product capacity	Fixed mounting Maximum load: 265 lbs. When the fold down grab bar is loaded with 265 lbs, each mounting bolt is loaded with 534 lbs.
Materials Ergonomy Features	Basic material: Steel Surface material: Rilsan. Provides firm support when you are sitting, standing, or transferring to and from a toilet. Good, broad support surface for the hand and arm. The ergonomic form gives a firm and comfortable grip. The grab bar can fold up when not in use or to make more space.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT®

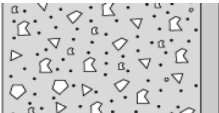
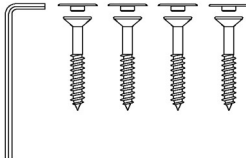

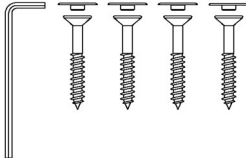
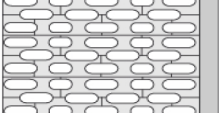
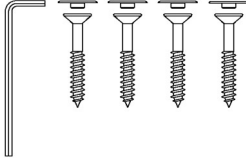
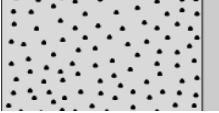
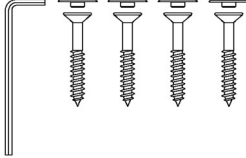
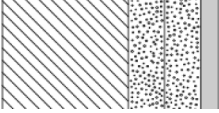
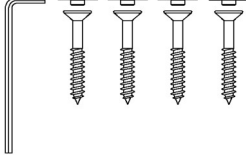
Product	VALUE II fold down grab bar, fixed height
Item number	R1109000
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättssprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean)</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
Declaration of conformity	<p>This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices.</p>



PRESSALIT®

Product	VALUE II fold down grab bar, fixed height
Item number	R1109000

Mounting in different wall types

	<p>Concrete >B25</p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Solid Brick >= Mz12</p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Solid Sand Lime Brick >= Hlz12</p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Aerated Concrete P4</p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</p> <p>V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.</p>	



PRESSALIT®

Product Item number	VALUE II fold down grab bar, fixed height R1109000
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</p> <p>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.

pressalit.com

Pressalit A/S
 Pressalitvej 1
 DK-8680 Ry
 Tel.: +45 8788 8788
 Fax: +45 8788 8789
 E-mail: pressalit@pressalit.com

PRESSALIT®