


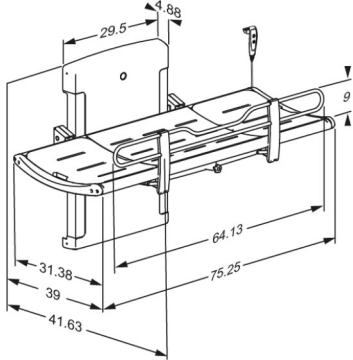

<p><b>Product</b> <b>Item number</b></p>	<p>SCT 3000 shower change table, electrically height adjustable R8592318301</p>
<p><b>Color</b>  <b>Product description</b></p>	<p>EAN:5708590374161 FDA:INQ/D124319</p> <p>Graphite gray (RAL 7024)</p>  <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>The SCT 3000 series has a stylish and award-winning design focused on comfort and ease of use.</p> <p>IAPMO-certified. SCT 3000 shower change table, powered, height adjustable 27.6"; from 11.8" - 39.4", with wired hand control Foldable. Incl. safety rail, flexible hose (Ø1.26") and water collection tray with integrated outlet. Surface length 70.9", outer measurements 41.6 x 75.2". Excl. mounting kit.</p>
<p><b>Adjustability</b> <b>Adjustment height</b>  <b>Product capacity</b>  <b>Speed up</b></p>	<p>Height adjustable</p> <p>27,58 inch</p> <p>Maximum user weight (MUM): 500 lbs. Maximum load (SWL): 522 lbs. (User weight + weight of water) When the table is loaded with 500 lbs, each mounting bolt supports 65 lbs. <i>This product has been tested statically with a load of 1306 lbs, as the IEC 60601-1 standard requires testing with a safety factor 2.5.</i></p> <p>Speed up: 0,59 inch/s Speed down: 0,79 inch/s</p>



<b>Product</b>	SCT 3000 shower change table, electrically height adjustable
<b>Item number</b>	R8592318301
<b>Noise level</b>	48 dB
<b>Materials</b>	Powder-lacquered stainless steel, aluminium and plastics (PVC, Polystyrene). Pads in polyurethane foam, are tested according to the DIN EN 1021-1 standard (flame resistance).
<b>Ergonomy</b>	<ul style="list-style-type: none"><li>• The ergonomically designed safety rails have a practical lower central section, which provides better access and work environment for the carer while transferring, showering, drying, changing and dressing the user.</li><li>• One hand operated safety rails allows the carer to maintain eye contact as well as physical contact with the user during the whole process.</li><li>• Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.</li></ul>
<b>Features</b>	<ul style="list-style-type: none"><li>• A water collection tray with integrated water outlet prevents water from splashing on caregiver or the floor. The flexible hose (Ø1.26") attached to the center mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall.</li><li>• The slightly curved shape makes the table comfortable and safe to lie on.</li><li>• Adjustable neck support which can be moved to either end of the 51.2" changing tables.</li><li>• The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space.</li><li>• The powered shower change table is supplied with a wired hand control. The red buttons clearly indicate "up" and "down".</li><li>• The wired hand control comes with a practical holder for placement on the wall.</li></ul>
<b>Mains connection (V)</b>	120
<b>Actuator (motor)</b>	LA40 Careline (Linak) 4000 N, IPX6
<b>Control box</b>	CA63 Careline (Linak), IPX6
<b>Wired remote control</b>	HB30 (Linak), IP66
<b>Power consumption (maximum load)</b>	Power consumption by maximum load: 420 W Power consumption by standby: 2 W



<b>Product</b>	SCT 3000 shower change table, electrically height adjustable
<b>Item number</b>	R8592318301
<b>Warranty</b>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.</p>
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes OxiGenesis from ENVIROX</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
<b>WEEE</b>	<p>Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.</p>









<b>Product</b>	SCT 3000 shower change table, electrically height adjustable
<b>Item number</b>	R8592318301

## Mounting in different wall types

	<p><b>Concrete &gt;B25</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  To be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  To be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  To be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b>  V8658 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs GB14.  To be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b>  V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12.  NB! Plugs are not needed.  To be ordered separately.</p>	

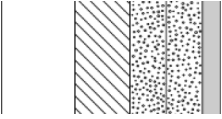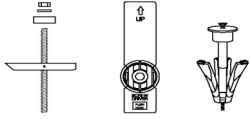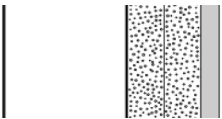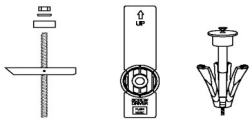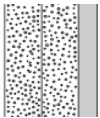

<b>Product</b> <b>Item number</b>	SCT 3000 shower change table, electrically height adjustable R8592318301	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.