R1196
R1197
R1198
R1199

TMA 4

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Pressalit's Environmental and Quality Management Systems are certified by LRQA

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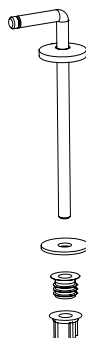
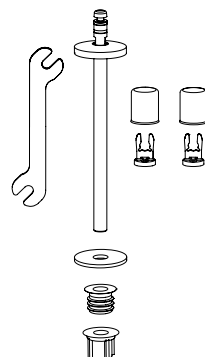
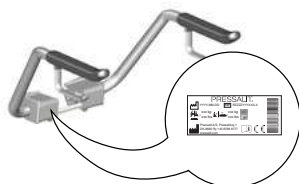
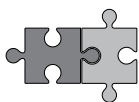
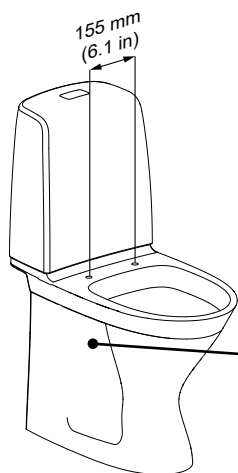
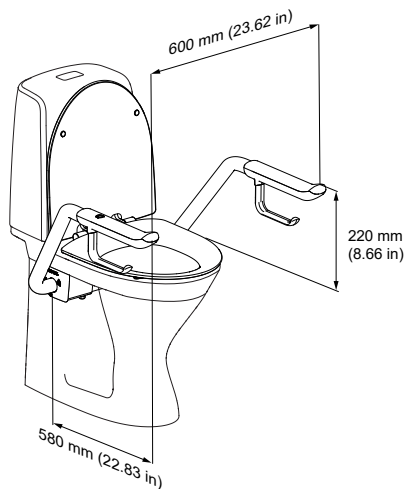
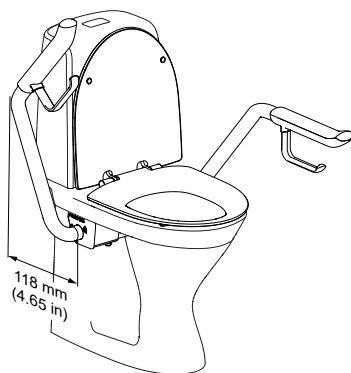
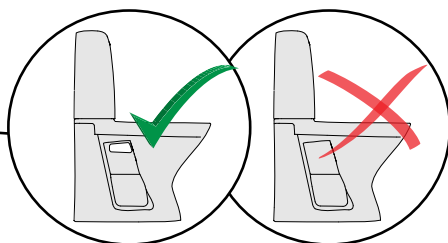
Content

Mounting instruction	5
Symbols used in this manual	6
General safety	7
Intended use	8
Intended user profile	8
Operating environment	8
Operating instructions	9
Nurse call button	9
Adjustment	9
Maintenance	10
Maintenance inspection	10
Cleaning	11
Trouble-shooting	11
Product label	12
Technical data	14
Disposal and recycling	14

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Test the load-bearing capacity of the floor before installation. Floor-standing WC ceramic appliances must be securely fixed to the floor with screws.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Product weight

Max. user weight

Article number/
versions

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

The arm support is used as support for a person when he or she sits down or gets up from a toilet.

Intended user profile

The product can be used by everyone - adults, children, teenagers, and the elderly who need support when getting up or sitting down on the toilet.

The arm supports have been tested according to ISO/FDIS 17966:2025 with a safety factor of 1.5, corresponding to an allowable maximum load of 65 kg/143.3 lbs.

Operating environment

For indoor use in a wet environment, such as a bathroom or washroom, in private homes, nursing homes and hospitals. The expected lifetime is 10 years. *Life cycle test performed according to ISO/FDIS 17966:2025.*

CAUTION

The surface of the product may become hot and/or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

CAUTION

Fire hazard. Keep open flame away from the grey handles.

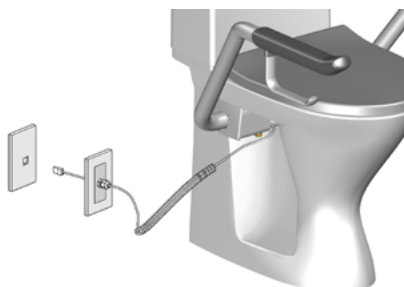
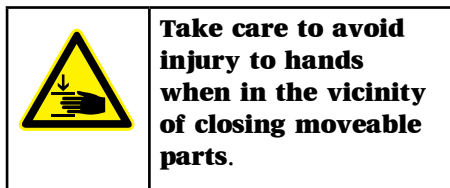
NOTICE

Any serious incident that has occurred in relation to the product should be reported to Pressalit and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Operating instructions

The two arm supports can be folded up and down individually. Fold them both up when not in use.


en

Nurse call button

The support arm models R1197 and R1198 are equipped with a nurse call button on either left or right arm. Press the button to call for help. The button must be connected to a call system.


Adjustment

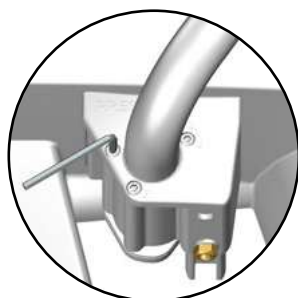The movement of the support arms can be adjusted using the three screws on each side. It is recommended to tighten the screws so that the arms can only be lowered when a certain amount of force is applied.

Maintenance

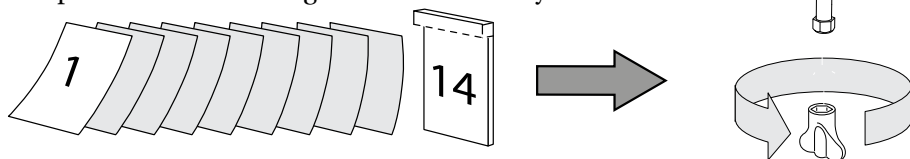
The product must be retightened after 14 days of use.

It is recommended to carry out a maintenance inspection once a year.

Maintenance inspection

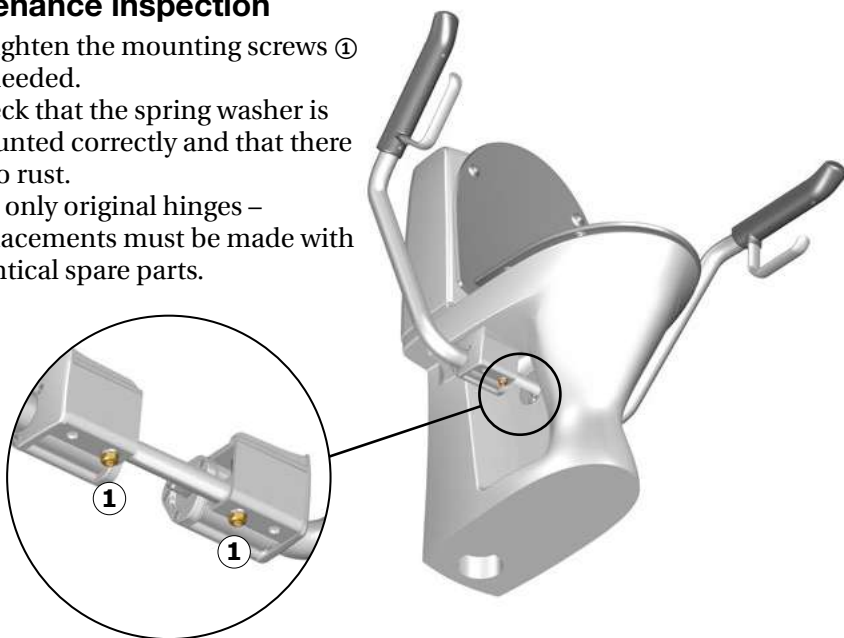- Retighten the mounting screws ① as needed.
- Check that the spring washer is mounted correctly and that there is no rust.
- Use only original hinges – replacements must be made with identical spare parts.

CAUTION

- Do not use the product if it is defective or damaged.
- Repairs may only be performed on the components mentioned in this section.

Contact Pressalit A/S or your local dealer if the problem cannot be resolved. Do not attempt to repair the product yourself, as this will void the warranty.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT


- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

en

Trouble-shooting

Symptom	Cause	Action
The call button does not work.	The wire is damaged.	Contact Pressalit.
The support arms drop down too fast.	The adjustment screws are too loose.	Tighten the three adjustment screws. See "Adjustment".
The support arms are loose/unstable.	The mounting screws have become loose.	Tighten the mounting screws. See "Maintenance".

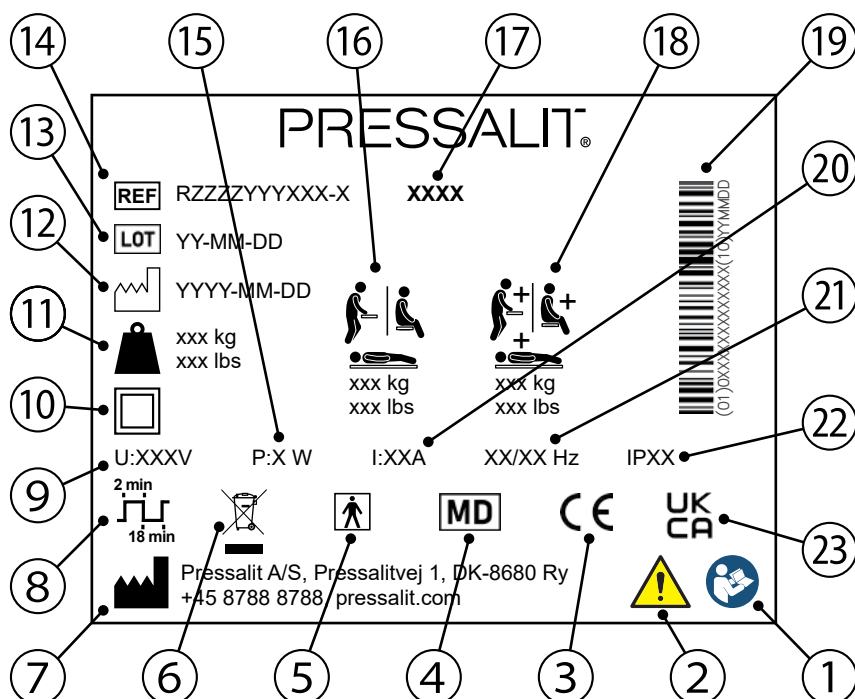Product label

- 1 Read the manual before use
- 2 Consult the manual for important cautionary information such as warnings and precautions
- 3 These products are CE-marked in accordance with relevant requirements in applicable legislation.
A declaration of conformity can be found at pressalit.com
- 4 Medical device
- 5 Type BF applied part. Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock
- 6 Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑦	Name and address of manufacturer
⑧	Duty cycle
⑨	Mains voltage
⑩	Class II Product, double insulated
⑪	Product weight
⑫	Production date
⑬	Batch code
⑭	Article number
⑮	Maximum power consumption
⑯	Maximum user weight
⑰	Model name
⑱	Max load
⑲	GS1-128 barcode
⑳	Electric Current
㉑	Frequency
㉒	IP class
㉓	UK Conformity Assessed. UK responsible representative: Pressalit Ltd., 100, Longwater Avenue, Green Park, Reading, Berkshire, RG2 6GP, United Kingdom.

Technical data

Materials	Stainless steel Aluminium Polyurethane	
Operating temperature	-30°C – +80°C (-22°F - 176°F)	
Weight	R1196 R1199	4,37 kg /9.63 lbs
	R1197 R1198	4,47 kg /9.85 lbs

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Polyurethane	Like other plastics, polyurethane can be recycled in two primary ways: mechanical recycling, in which the material is reused in its polymer form, and chemical recycling that takes the material back to its various chemical constituents.

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