




PRESSALIT®

Product	VALUE IV back rest, fixed height
Item number	R1661000112
Color	VVS: 777525090 NRF: 6311837 RSK: 8565071 EAN: 5708590355115 HMI: 124224 DB: 2084079 FDA: INN/D135865 Anthracite gray
Product description	VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering. VALUE IV back rest. Fixed height. Materials: Diecast aluminium and vacuum-moulded polyethylene (PE). Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.
Adjustability	Fixed mounting
Materials	The wall profile is made of diecast aluminium, and the backrest is made of vacuum-moulded polyethylene (PE)
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

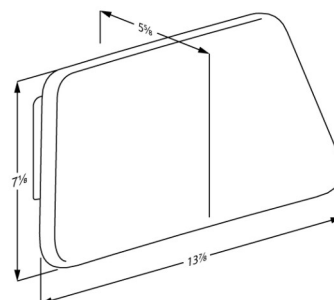



Product	VALUE IV back rest, fixed height
Item number	R1661000112
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean) NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.