R9414717 - R9414721  
R9434717 - R9434721  
R9454717 - R9454721  
R9464717 - R9464721

## MSCT 1

<b>Shower change trolley – Operation and maintenance manual</b>	<b>en</b>
<b>Mobilt bruse-/skifteleje – Brugs- og vedligeholdelsesvejledning</b>	<b>dk</b>
<b>Duschwagen – Gebrauchs- und Pflegeanleitung</b>	<b>de</b>
<b>Chariot de douche et de change – Manuel d'utilisation et d'entretien</b>	<b>fr</b>
<b>Mobiel douchebrancard – Gebruiks- en onderhoudshandleiding</b>	<b>nl</b>
<b>Mobil duschbrits – Bruks- och underhållsanvisning</b>	<b>sv</b>
<b>Mobilt dusjstøllebord – Bruks- og vedlikeholdsanvisning</b>	<b>no</b>
<b>Camilla para ducha/cambio – Instrucciones de uso y mantenimiento</b>	<b>es</b>
<b>Barella per cambio di indumenti e doccia - Manuale d'uso e manutenzione</b>	<b>it</b>
<b>Suihku- ja vaatteiden vaihtovaunu – käyttö- ja huolto-opas</b>	<b>fi</b>
<b>Wózek toaletowy – Instrukcja obsługi i konserwacji</b>	<b>pl</b>

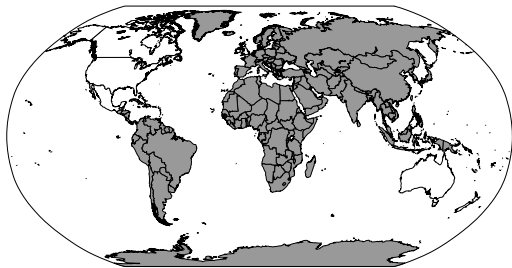
English . . . . .	11
Dansk . . . . .	38
Deutsch . . . . .	63
Français . . . . .	88
Nederlands . . . . .	113
Svenska . . . . .	138
Norsk . . . . .	163
Español . . . . .	188
Jзык polski . . . . .	213

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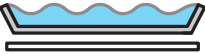
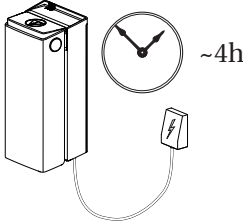
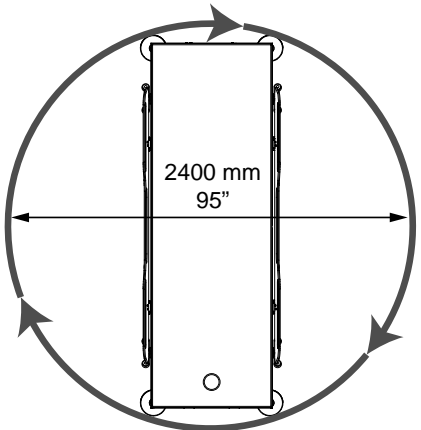
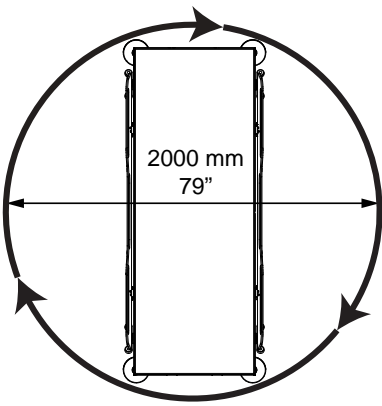
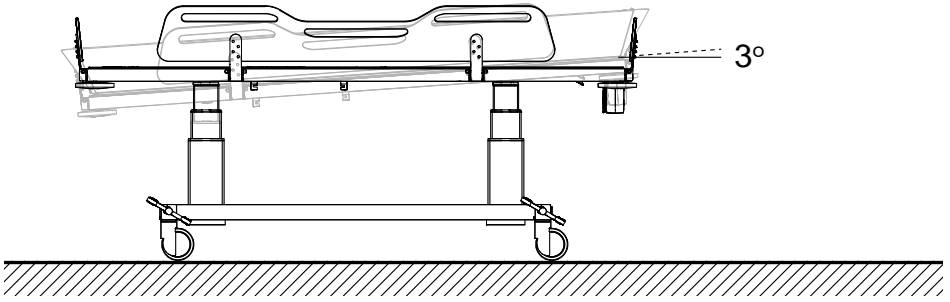
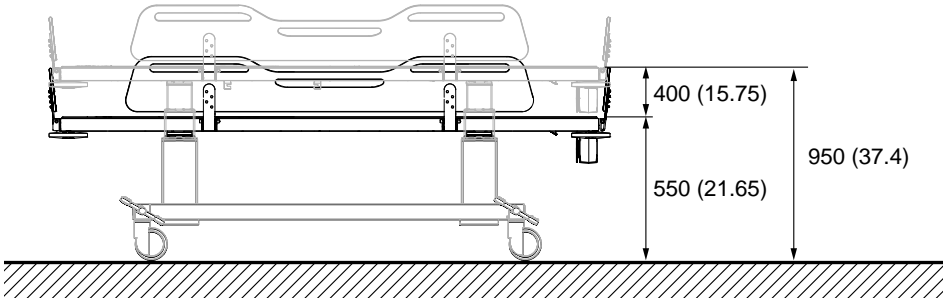
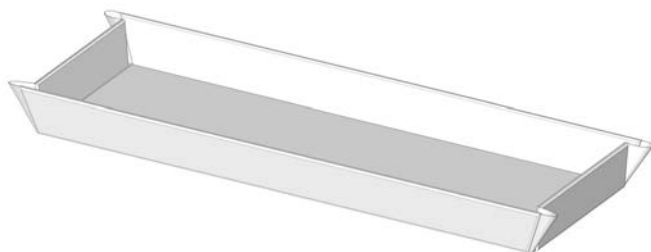
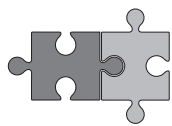
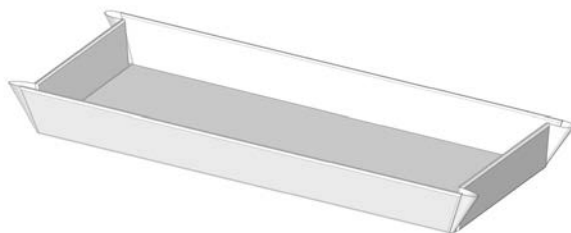
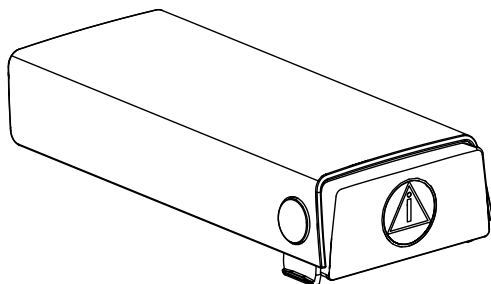
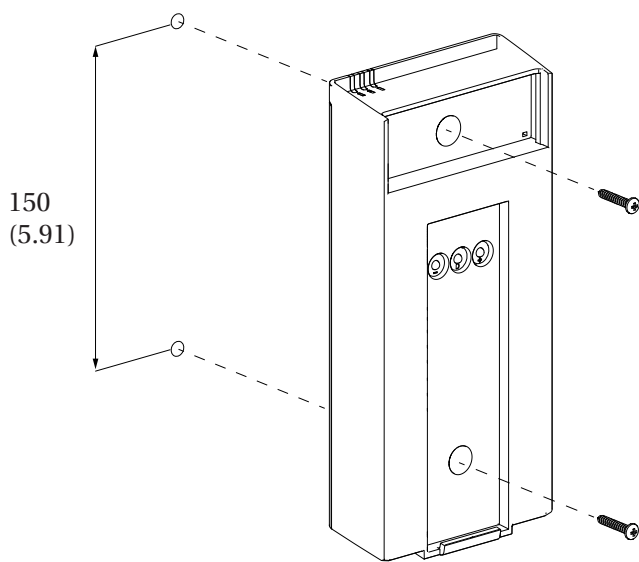
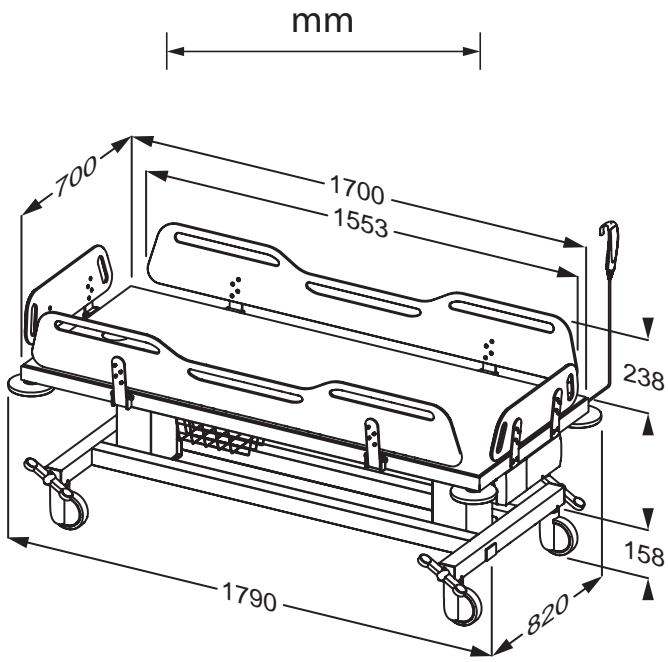
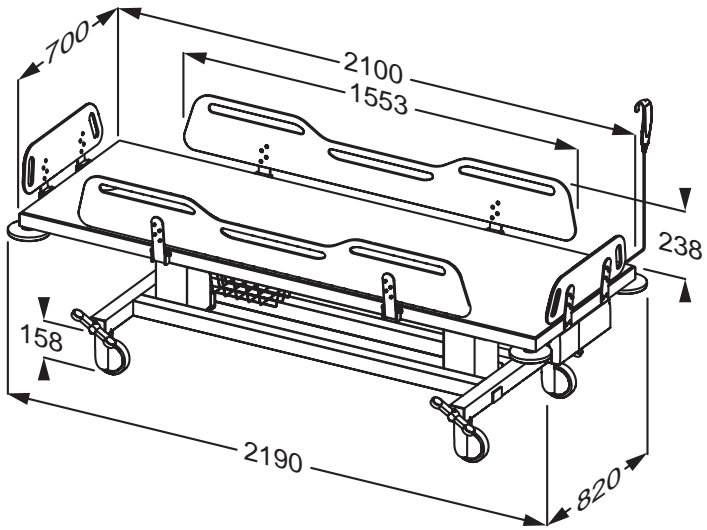
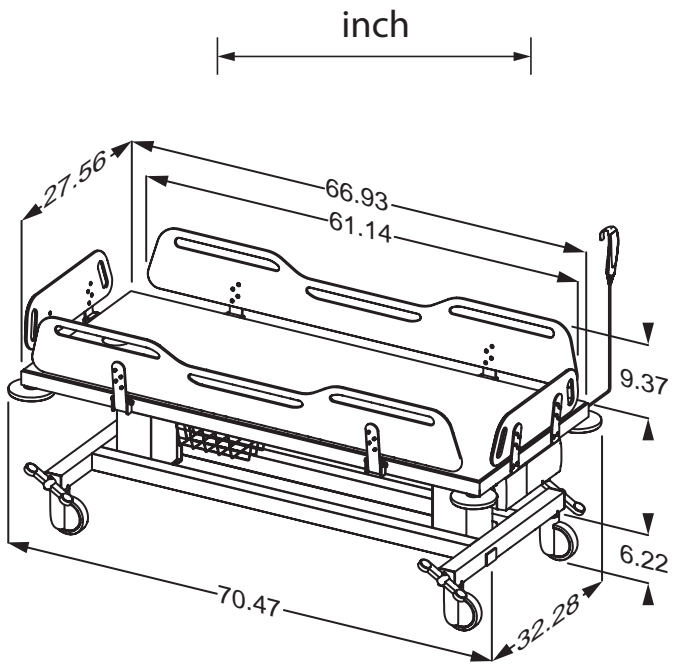
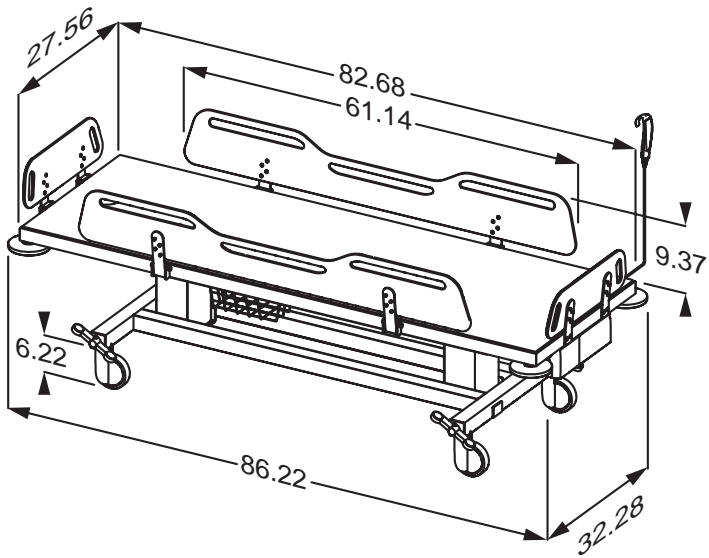
# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

**WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

**IMPORTANT** indicates important information about handling and use of the product.

**NOTICE** is used to address practices not related to personal safety.

Head injury warning

Hand injury warning

Genitalia injury warning

Feet injury warning

Do not leave the patient unattended.

Compatible products

Spare parts

Max. user weight

Article number/ versions

Product weight

Electrical data

en

# General safety

## **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

## **CAUTION**

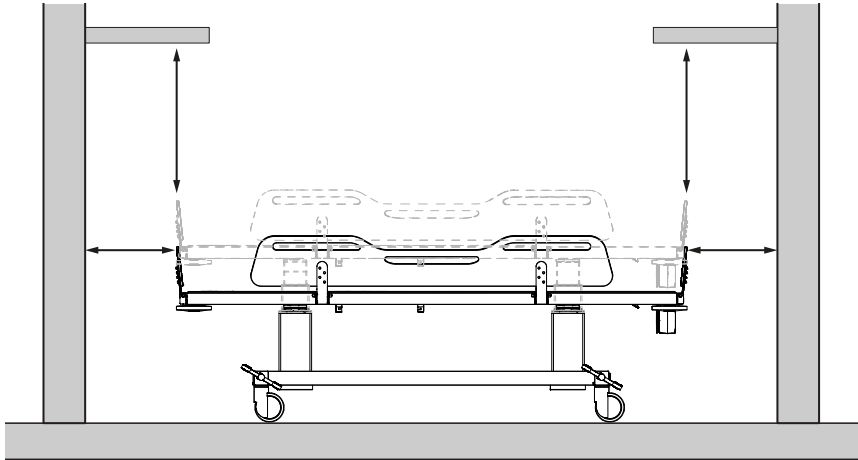
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

## **NOTICE**


- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

# Safety distances



Make sure that the unit is at safe distance from surrounding objects. Check the distances before any use.

ISO 17966:2016		
	Max 120 mm (4.72") Min 300 mm (11.81")	Max 60 mm (2.36") Min 300 mm (11.81")
	Max 8 mm (0.31") Min 25 mm (0.98")	Max 4 mm (0.16") Min 25 mm (0.98")
	Max 8 mm (0.31") Min 75 mm (2.95")	Max 8 mm (0.31") Min 75 mm (2.95")
	Max 35 mm (1.38") Min 120 mm (4.72")	Max 25 mm (0.98") Min 120 mm (4.72")

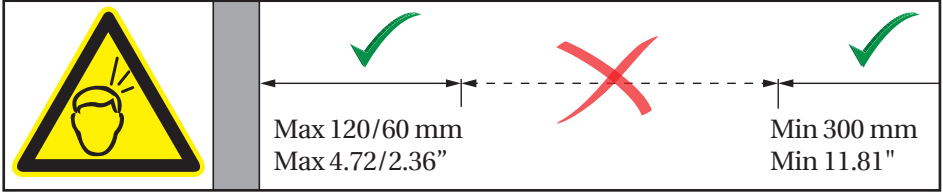en

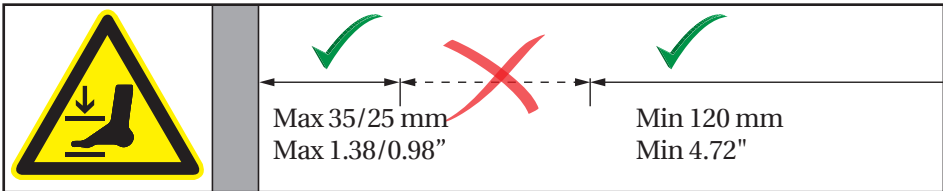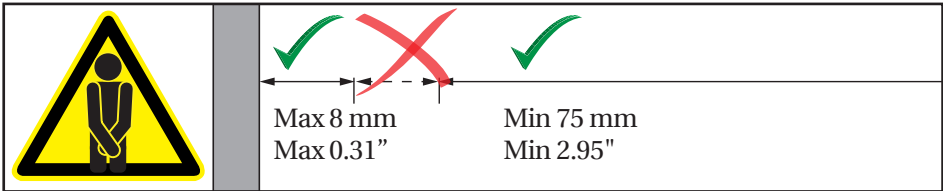 **WARNING**

If the unit moves within the safety distances, you must keep an especially close eye on surrounding people and objects to avoid crushing hazards.

## Intended use

This product is intended for changing a person who is unable to stand or sit. The person is lying horizontally on the change trolley.

## Intended operator profile

Only personnel who are trained in healthcare and/or in changing someone under their care should use this product.

## Intended user profile

The product can be used by all – adults, children, young, old, whose weight does not exceed the **maximum load of 200 kg / 440 lbs.**

## Operating environment

The change trolley is intended to be used at room temperature in public changing places, schools, private homes, day/residential/nursing facilities and hospitals under normal working conditions.

The change trolley may be used dry rooms and not as a shower table in showers. If the relative humidity is above 70%, the expected life of the unit will be significantly reduced. The change trolley must not be exposed to chlorine-containing fumes.

en

The product surface may become hot or cold if exposed to external sources of heat or cold, such as sunlight or frost.

## NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

## **Contraindications for use**

It is not recommended that pregnant women are bathed in the shower change trolley.

It is not recommended that patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions are bathed in the shower change trolley.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerabral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may be bathed in the shower change trolley only after consulting with the attending physician.

Do not use if the patient's weight exceeds 200 kg / 440 lbs.

## **Biocompatibility**

The materials in contact with the skin of the patient or the user during the changing procedure, have been tested for their content of harmful substances according to ISO 10993 Biocompatibility, and are therefore considered suitable for this purpose.

en

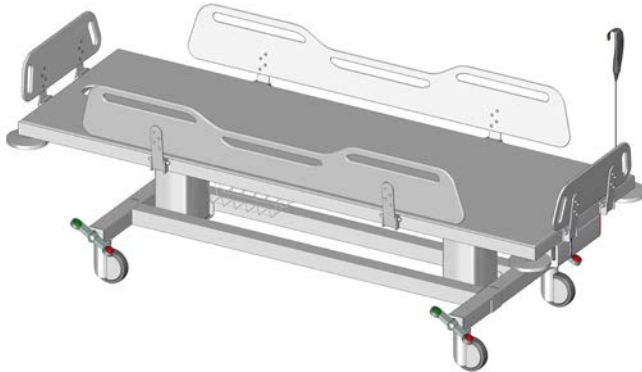
## **Flammability**

The product materials have undergone flammability testing in accordance with ISO 8191-1:1987 and ISO 8191-2:1988.


# Operating instructions

## Safety brakes

The shower change trolley is equipped with safety brakes on all four wheels to ensure that it cannot move unintentionally.

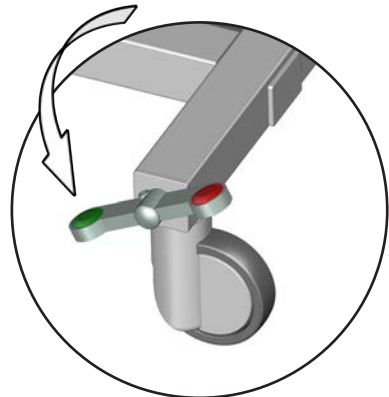**To lock the brakes:** Step on the red button on one of the four rocker pedals to lock the brakes on all four wheels.



**To unlock the brakes:** Step on the green button on one of the four rocker pedals to unlock the brakes on all four wheels.

Always ensure that the brakes are locked when transferring a patient to and from the shower change trolley.

# Moving

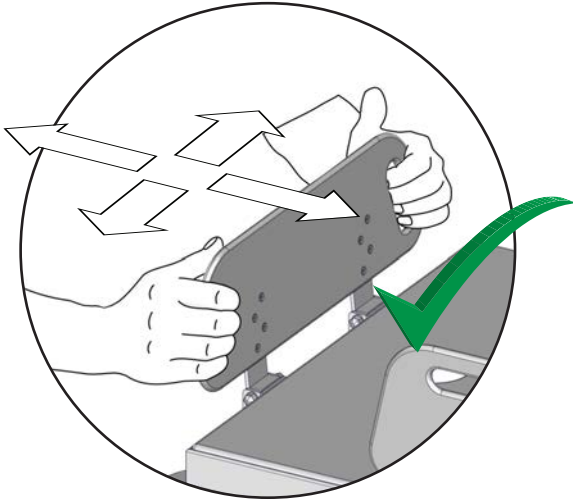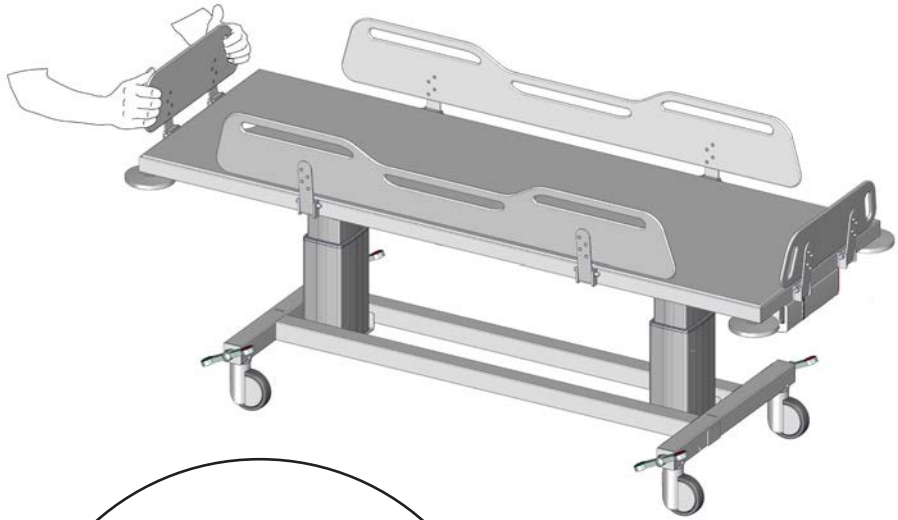en


Use the cut-out grips on the side supports when moving the shower change trolley. Push straight ahead in the desired direction. Do not twist or turn the side support.

**Do not leave the patient unattended.**
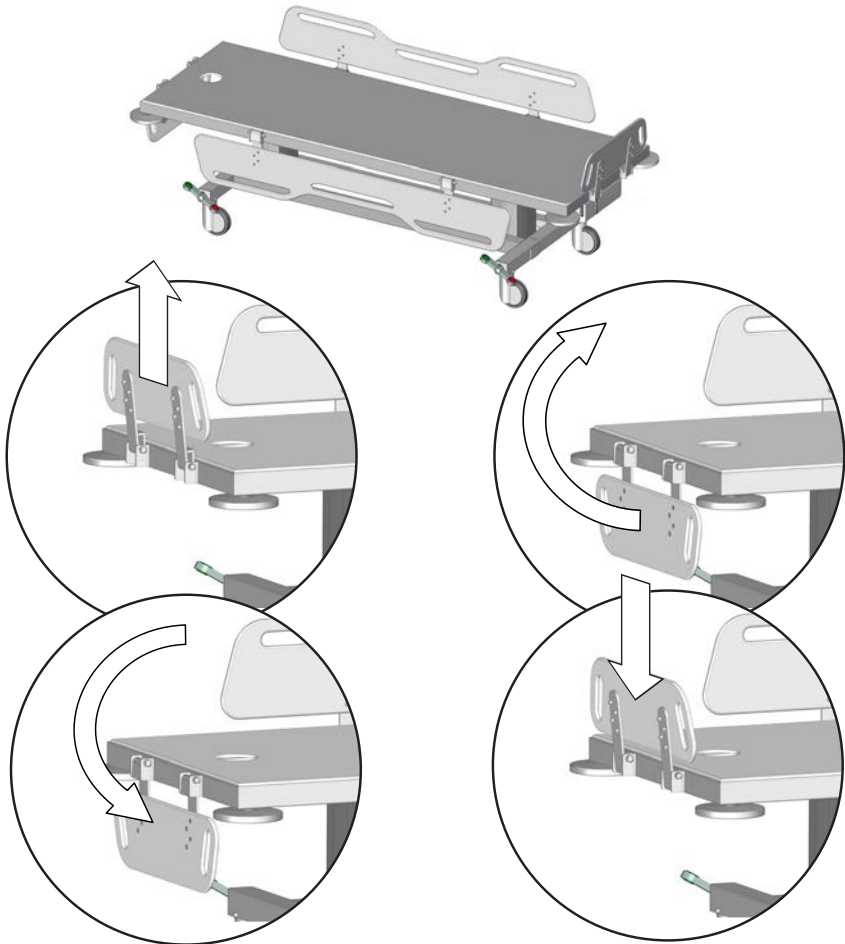
## Side supports

The side supports can be folded up and down individually for easy transfer of the patient. The end supports can be folded up and down individually for easy access to the patient's feet/legs (putting on stockings, etc).


**To fold down:** Lift the side support upwards to unlock it, and swing it down.

**To fold up:** Swing the side support up and push it down to lock it in place.

When the trolley is in use, make sure that all side supports are erect and in locked position to protect the patient from falling out of the unit.

## Height adjustment

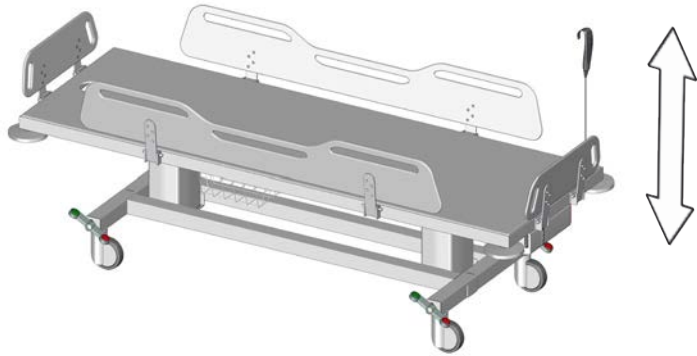The shower change trolley can be adjusted 400 mm / 15.75" up and down.

**Max user weight: 200 kg / 440 lbs**

**Max load: 250 kg / 550 lbs**

The height adjustment is controlled by the wired remote control unit.

en

Press and hold the Up button to raise the unit. When the button is released, the motion stops.

Press and hold the Down button to lower the unit. When the button is released, the motion stops.

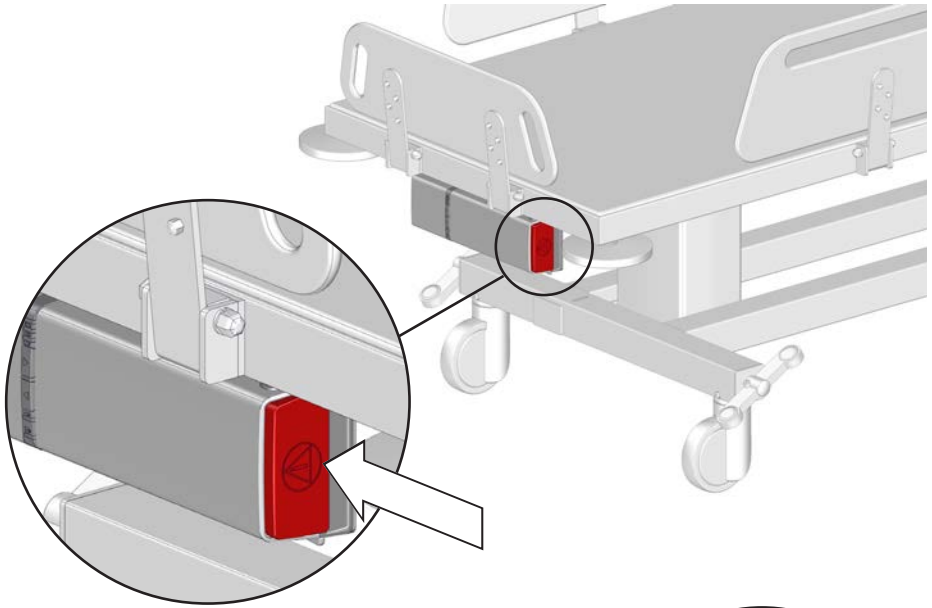
**To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.**

**Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.**

## Emergency stop

In case of an emergency, push the emergency stop on the battery. The upward or downward movement will stop immediately.


en

## Releasing the emergency stop

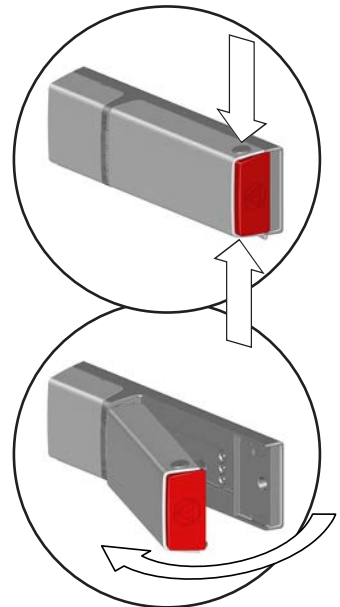
Push the two release buttons to remove the battery.

Reattach the battery again. Make sure the battery lock clicks into place.

The emergency stop is now released.

## Tilt function

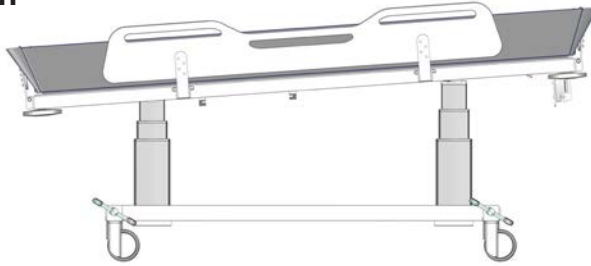The shower change trolley can be tilted approximately 3 degrees to allow the water to drain when using the optional shower mattress.

The tilt function is controlled by the wired remote control unit.

Press and hold the Tilt up button to tilt the unit. The motion stops when the unit is tilted approx. 3 degrees, or when the tilt up button is released.

Press and hold the Tilt down button to level the unit. The motion stops when the unit is level, or when the tilt down button is released.

en

It is still possible to raise and lower the unit when it is in tilted position.

## Resetting

If the lifting function does not work properly or if the two lifting columns are out of sync, try resetting the lifting system.

Press and hold the up button and the down button simultaneously.

The columns will move down after a few seconds while a beep tone is heard.

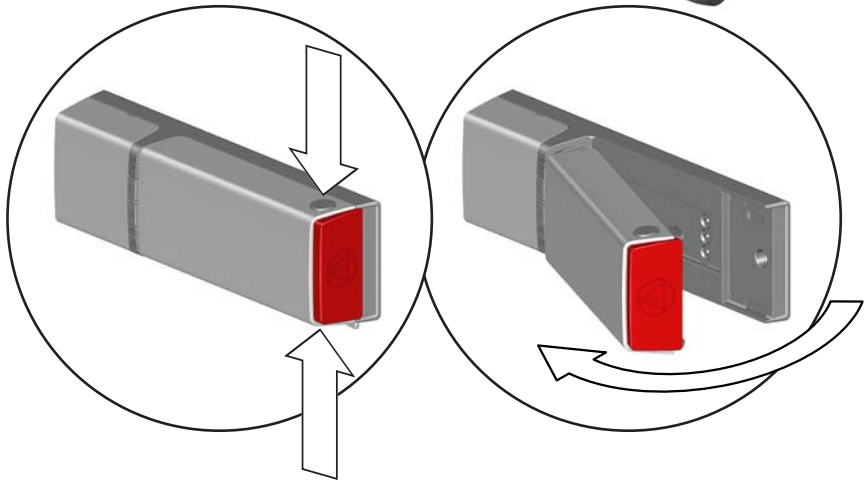
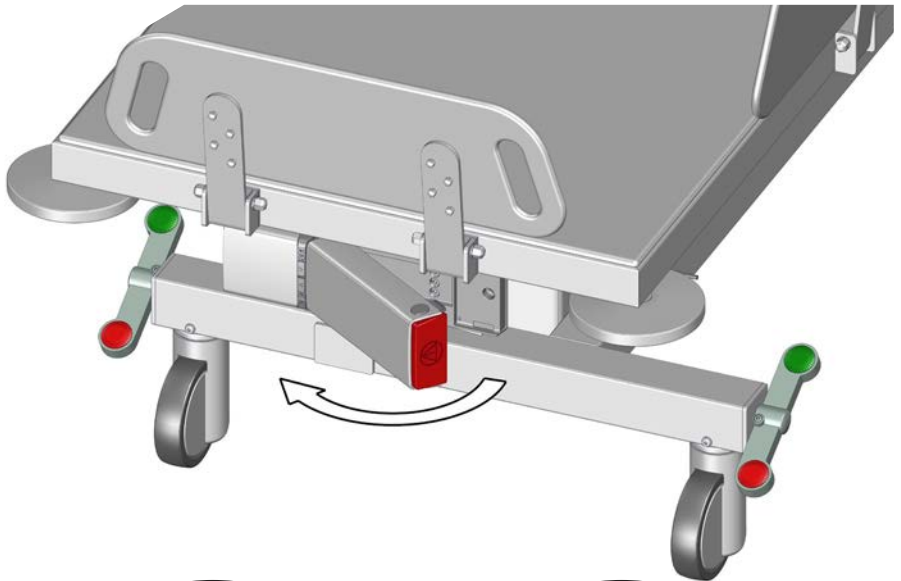
Keep holding both buttons until the beep tone disappears.

The lifting columns are now reset.


# Charging the battery

The battery can height adjust the shower change trolley up to 30 times when fully charged.


Push the two release buttons to remove the battery for charging in the external charger.

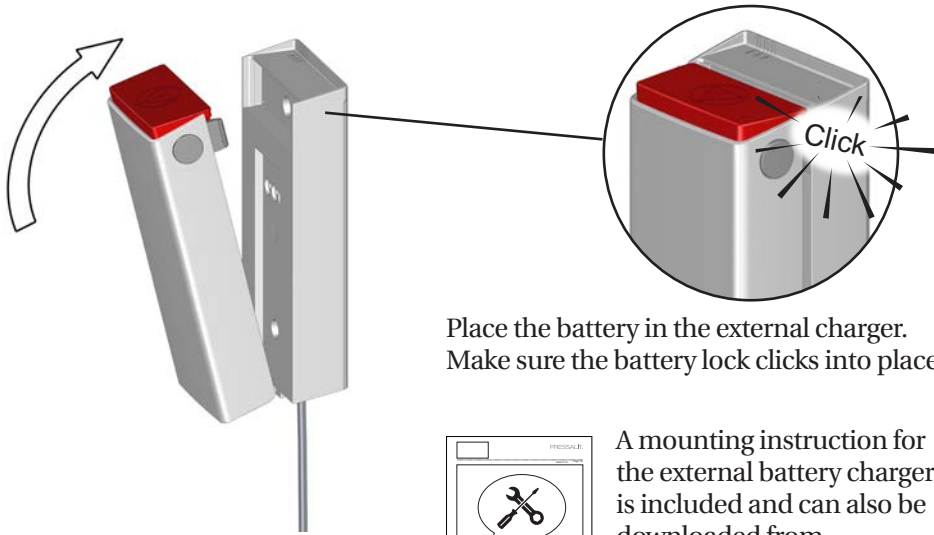en

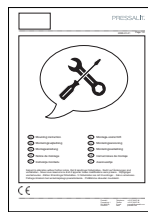
Place the battery in the external charger. Make sure the battery lock clicks into place.

A mounting instruction for the external battery charger is included and can also be downloaded from [pressalit.com](http://pressalit.com).

## Charging LED indicators

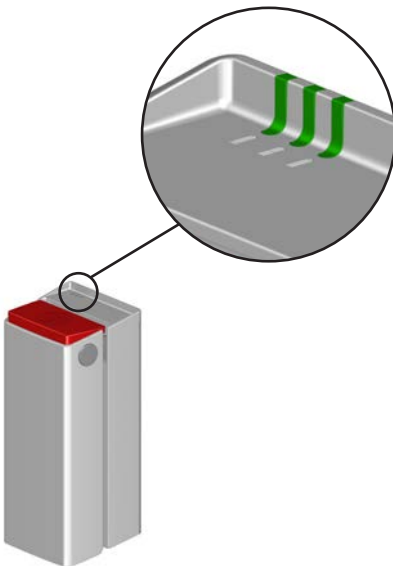The three LEDs on the external charger show the battery charging status.


	Charging status
	90-100%
	65-90%
	40-65%
	0-40%
	Charging stopped due to low battery temperature, high battery temperature or other error conditions
	Charging stopped due to lost communication with battery

# Accessories


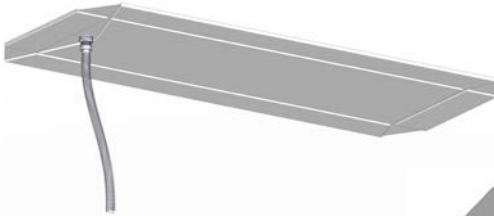
## Shower mattress

The shower change trolley can be used as a bathtub with the optional shower mattress (R844721 and R844717).

Attach the drain hose to the mattress drain, as per the included mounting instruction.

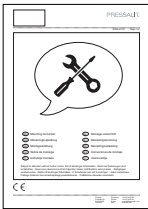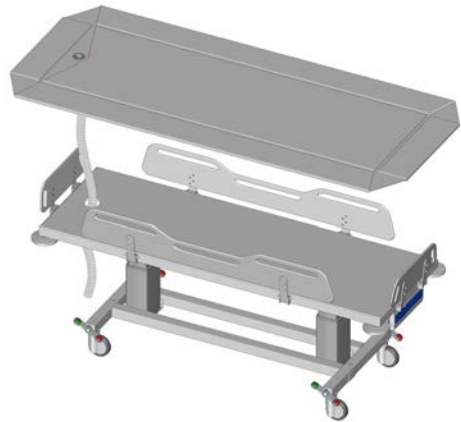Pull the drain hose through the shower trolley's drain hole, fold the mattress and place it on the trolley. Make sure that all four side supports on the shower trolley are folded up and locked.


An instruction for attaching the drain hose is included with the shower mattress and can also be downloaded from [pressalit.com](http://pressalit.com).


## IMPORTANT

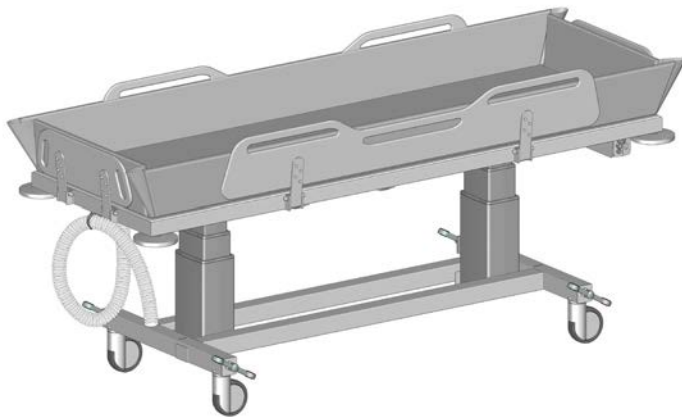- To maintain hygiene, it is recommended to check for any visible damages or signs of wear and tear on the mattress regularly. If any damages are found, the mattress should be replaced immediately to prevent any risk of contamination or infection.
- Use gloves while handling the waste hose for the mattress. It may have been in contact with human waste.

en

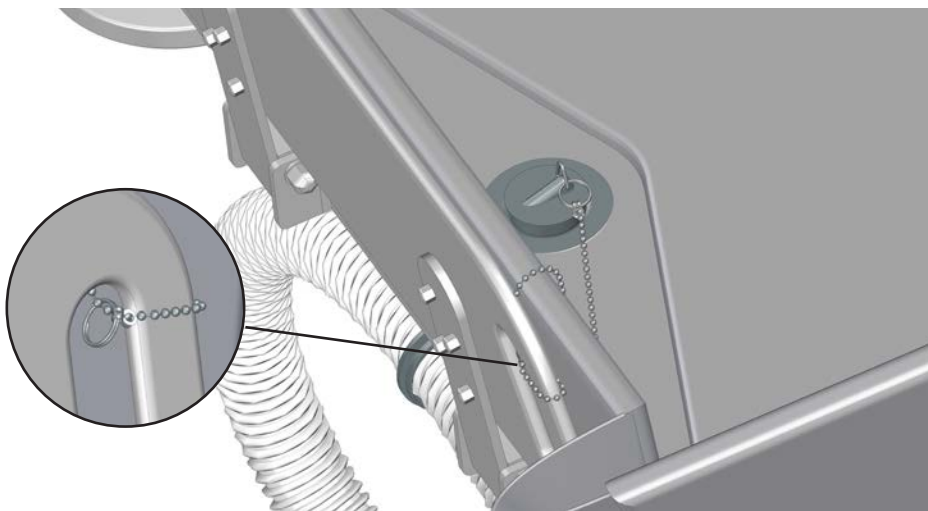Coil up the hose and attach it to the hose hanger.

en


Attach one key ring of the chain to the plug. Make a loop around one of the footboard handles with the other ring.

## IMPORTANT

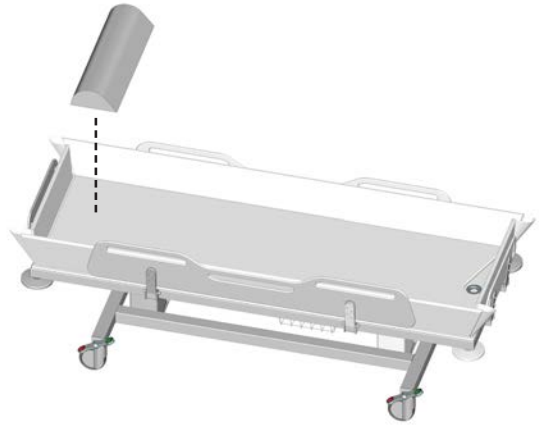
Do not move the trolley when there is water in the shower mattress.

## Head rest

The shower mattress can be used with the optional headrest for optimal comfort (R844521).

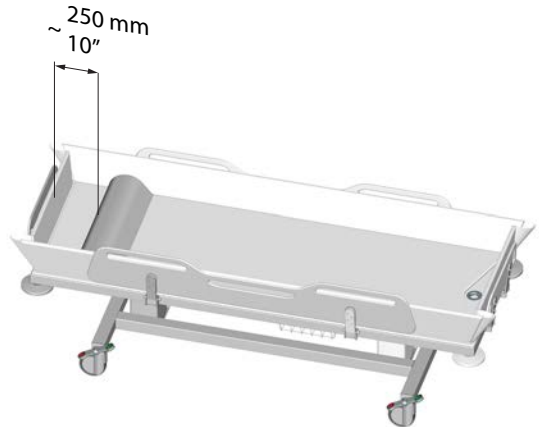Place the headrest at the head end of the shower trolley, approx. 250 mm (10") from the headboard.

The mattress and the head rest are to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents/disinfectants tested by Pressalit can be found on [pressalit.com](http://pressalit.com)


## IMPORTANT

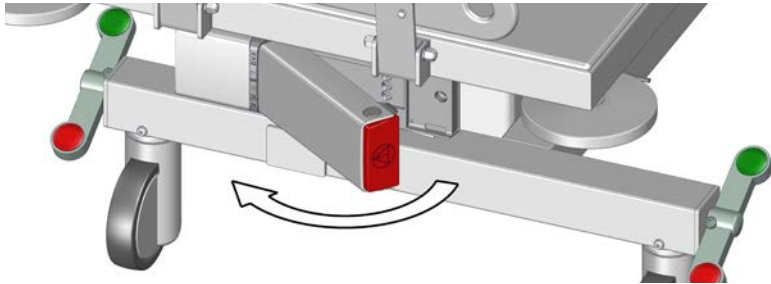To maintain hygiene, it is recommended to check for any visible damages or signs of wear and tear on the head rest regularly. If any damages are found, it should be replaced immediately to prevent any risk of contamination or infection.

en

# Cleaning

Remove the battery before cleaning the shower change trolley.

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit [Pressalit.com](http://Pressalit.com) and refer to the product's item number. Pressalit recommends cleaning the product after installation.

## CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

## IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

## CAUTION

Do not perform any type of cleaning or disinfection of the shower change trolley while the patient is lying on it.

# Electromagnetic compatibility

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

## Electromagnetic emissions

en

<b>Test standard</b>	<b>Test level / details</b>	<b>Test results</b>
60601-1-2 ch. 7.1.7 - CISPR 14	30MHz – 1GHz	Passed

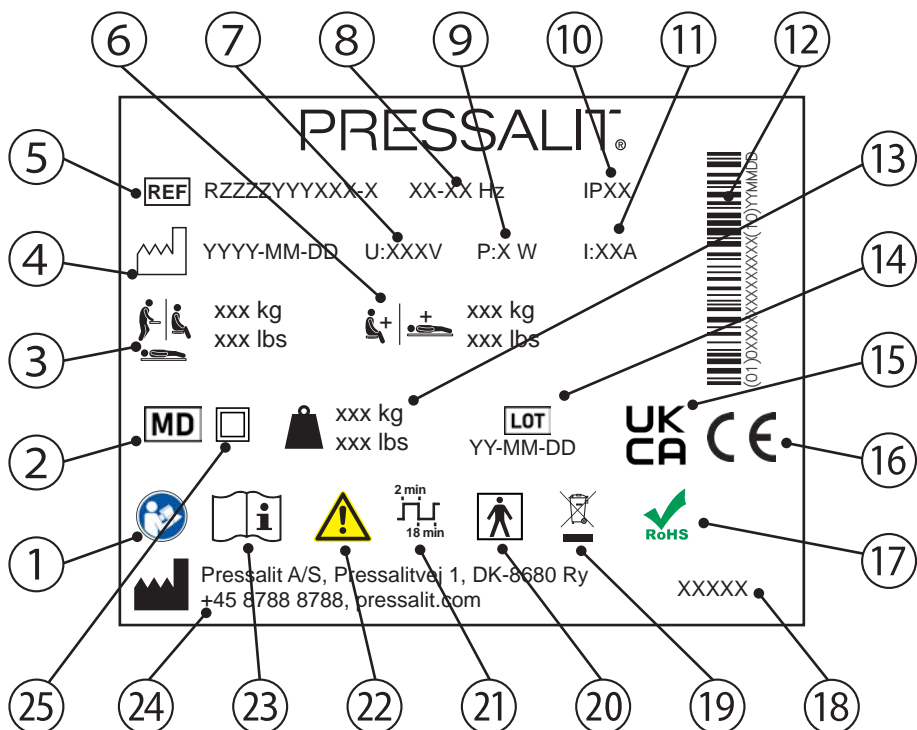
## Electromagnetic immunity

Test standard	Compliance level	Test level / details	Test results
EN60601-1-2 and EN17966 (Home healthcare environment)	=> 10V/m 2,7GHz	15V/m Frequency range: 80MHz- 6GHz 1% step size	Passed
EN60601-1-2 (Proximity fields from RF wireless communications equipment)	=> 10V/m 2,7GHz	15V/m Frequency range: 385 - 5785 MHz	Passed
IEC 61000-4-4 (Electrical fast transients / bursts)		Quick pulse simulation	Passed
Conducted disturbances induced by RF	Pulse subjection, 6V	Pulse subjection, 10V	Passed
Electrostatic discharge		Statically charged person discharge on unit, 8kV and 15kV	Passed (*)

\* If a statically charged person discharges on or near the shower trolley control box, a fault may occur in the lifting columns, and a reset must be performed. See "**Resetting**".

# Product label

en

- ① Consult manual for use

---

- ② Medical device

---

- ③ Maximum user weight

---

- ④ Production date

---

- ⑤ Article number

---

- ⑥ Max load

---

- ⑦ Mains voltage

---

- ⑧ Frequency

---

- ⑨ Maximum power consumption

⑩ IP class

---

⑪ Electric Current

---

⑫ GS1-128 barcode

---

⑬ Product weight

---

⑭ Batch code

---

⑮ UK Conformity Assessed

---

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
  - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
  - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at [pressalit.com](http://pressalit.com)

---

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

---

⑱ Product name (if any)

---

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

---

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function  
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

---

㉑ Duty cycle

---

㉒ Consult the manual for important cautionary information such as warnings and precautions

---

㉓ Consult manual for use

---

㉔ Name and address of manufacturer

---

㉕ Class II Product, double insulated

---

# Storage and transportation

- The temperature for transportation and storage of the unit must be between 10 and 50 degrees Celsius.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

# Troubleshooting

Symptom	Cause	Action
No power.	Battery depleted.	Recharge the battery. See " <b>Charging the battery</b> ".
The shower trolley is skewed even though the tilt function is not activated.	The lifting columns are out of sync.	Reset the lifting columns. See " <b>Resetting</b> ".
The shower trolley cannot be raised or lowered.	If a statically charged person discharges near the shower trolley control box, a fault may occur in the lifting columns.	Reset the lifting columns. See " <b>Resetting</b> ".

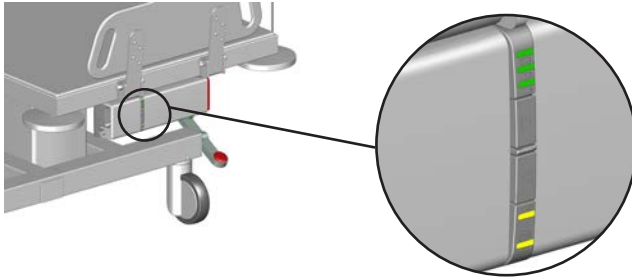
Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Expected service life of the shower change trolley is 10 years.

en

## System LED indicators

The two yellow LEDs on the internal charger show the system status.

LEDs	Cause	Status	Action
	Emergency stop activated	The unit cannot be raised or lowered	Release the emergency stop. See " <b>Releasing the emergency stop</b> "
	System error	The unit cannot be raised or lowered	Reset the lifting columns. See " <b>Resetting</b> "
	Unit overloaded	The unit cannot be raised or lowered	Reduce the load
	Duty cycle guard	The unit cannot be raised or lowered	Wait until the unit is ready
	Service needed	The unit is still functioning	Contact Pressalit A/S or your local dealer

en

# Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

<b>Recyclable components</b>	<b>Material</b>
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.
Batteries	Disposal of the battery must be in accordance with local regulations.

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