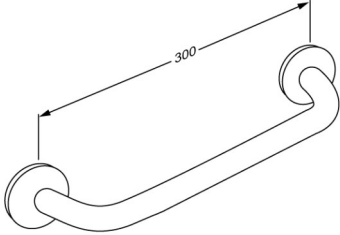




Product	PLUS handrail 300 mm
Item number	RT100035
Colour	VVS:777565135 NRF:6317161 EAN:5708590145822 HMI: 19081 NBS:N13:436 FDA:ILS/D135866 Red (RAL 3003)
Product description	 <p>PLUS handrails are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS handrail, diameter 33 mm, length 300 mm, for fixed mounting. The projection of 50 mm from the wall makes it easy to take hold of the rail. Inkl. wall rosette, which covers the fixings. Material: Powder-coated aluminium and plastic. Excl. mounting kit.</p>
Product capacity	Maximum load: 235 kg. When the handrail is loaded with 235 kg, each wall fastener is loaded with 56 kg. <i>Handrails are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i>
Materials	Basic material: Aluminium Surface material: Polyester lacquer Cover cap: Cycoloy plastic
Ergonomy	The 33 mm diameter enables the rail to be held comfortably and firmly. The projection of 50 mm from the wall makes it easy to take hold of the rail.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.

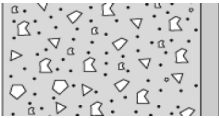
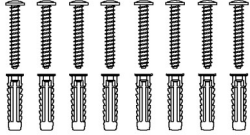

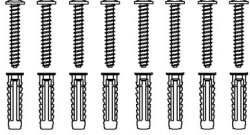
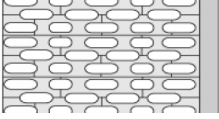
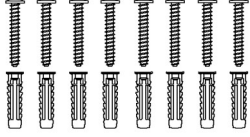
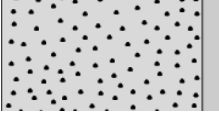
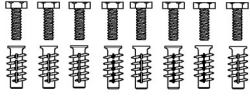
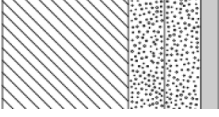
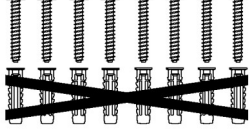


Product	PLUS handrail 300 mm
Item number	RT100035
Cleaning	<p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>

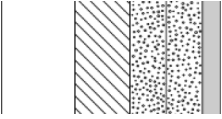
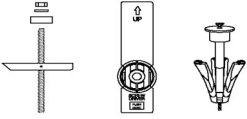

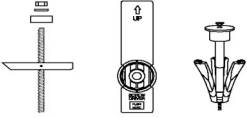
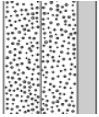


Product	PLUS handrail 300 mm
Item number	RT100035

Installation in different wall types

	Concrete >B25 V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Solid Brick >= Mz12 V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Aerated Concrete P4 V8666 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8616 - Mounting kit, 8 screws Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.	



Product Item number	PLUS handrail 300 mm RT100035	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>	