INDIVO

Indivo kitchen – Operation and maintenance manual

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Content

Mounting instruction	3
Safety	4
Usage	5
Safety labels.....	5
Operating instructions	8
Safety systems.....	10
The worktop safety bar	10
The cupboard safety plates	11
Maintenance.....	12
Resetting.....	12
Cleaning.....	12
Dimensions (mm)	13
Dimensions (inch)	17
Technical data.....	21
Trouble-shooting	22
Disposal and recycling	24

en

Mounting instruction

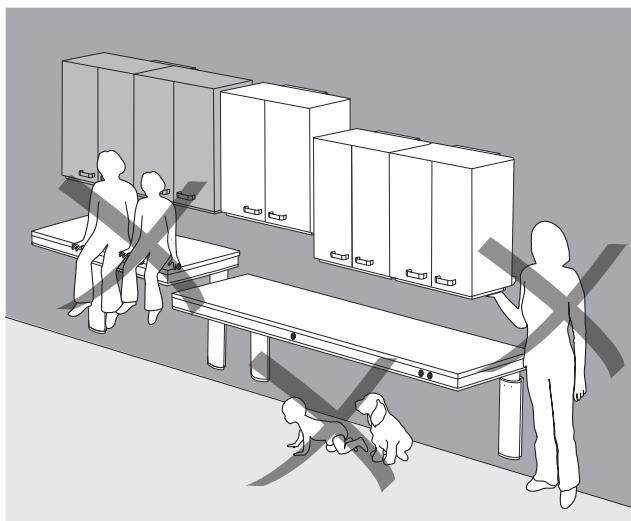
A mounting instruction is included with the product. It can also be downloaded from www.pressalitcare.com.

Original instructions

Safety

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Never use or handle this product in other ways than are specified in this manual.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.
- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.

- Never allow children to play with the product.
- Never allow children or pets to stay under the table
- Keep hands clear of the lifting mechanism.

Pressalit cannot be held liable if the product is used in any way which differs from that described in this guide.

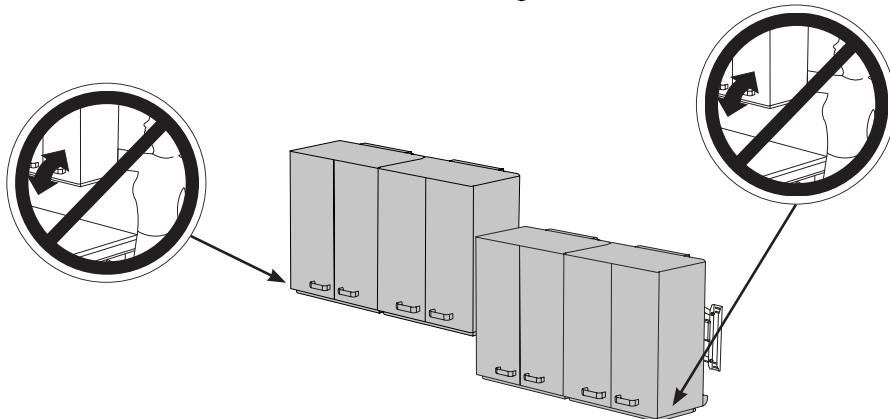
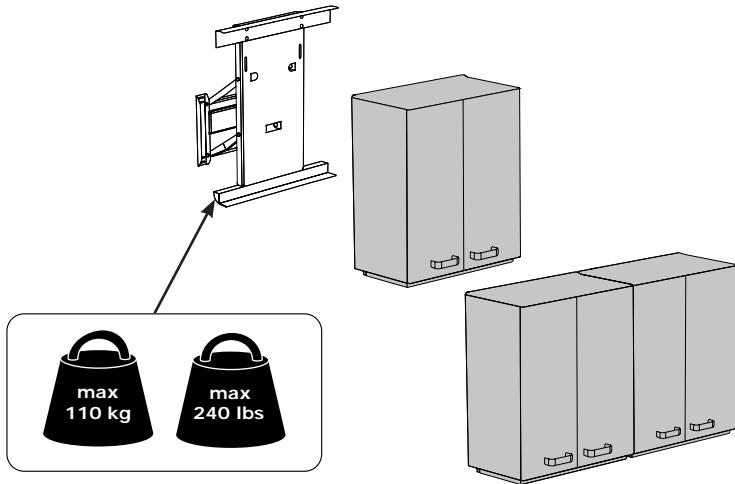
We reserve the right to amend this manual without prior notice.

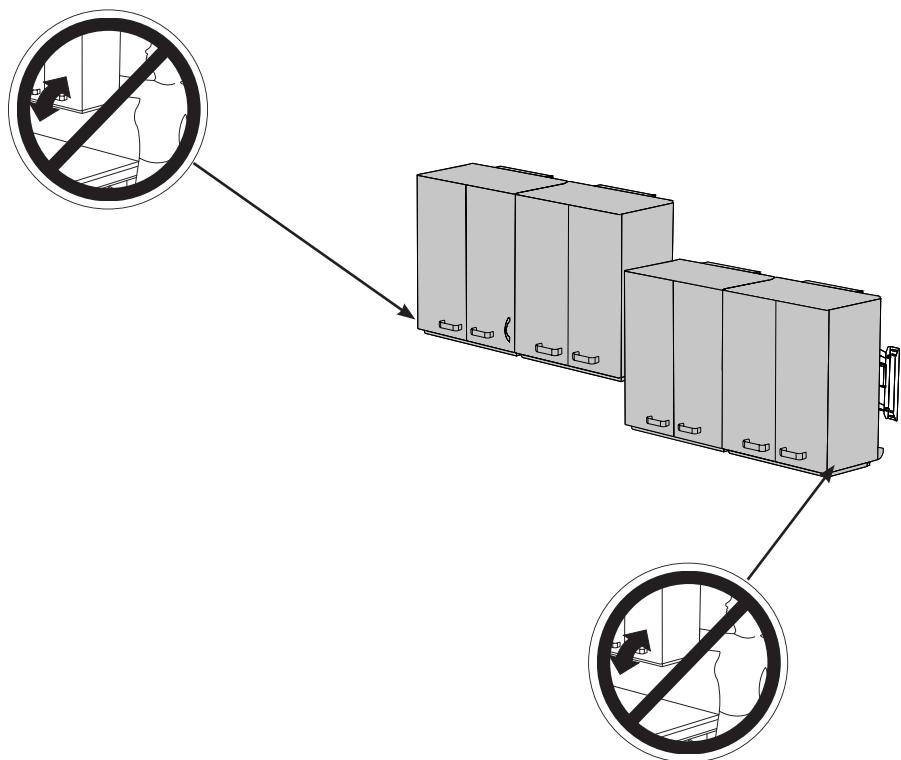
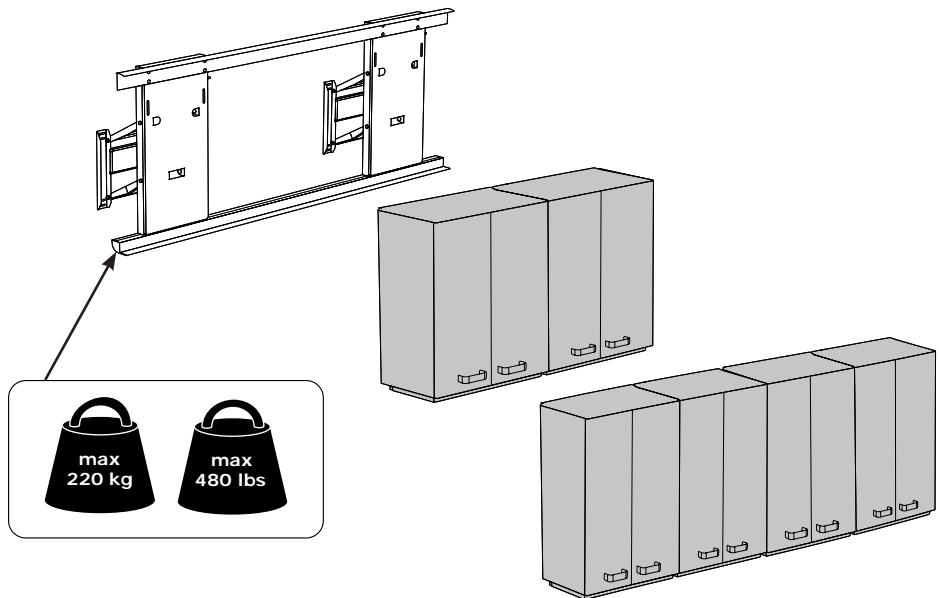
Usage

The height adjustable kitchen lifting units allow both seated and standing users to work comfortably in the same kitchen. The units are for indoor use only.




Safety labels

Before initial operation, please affix the safety labels supplied along with this booklet.


en

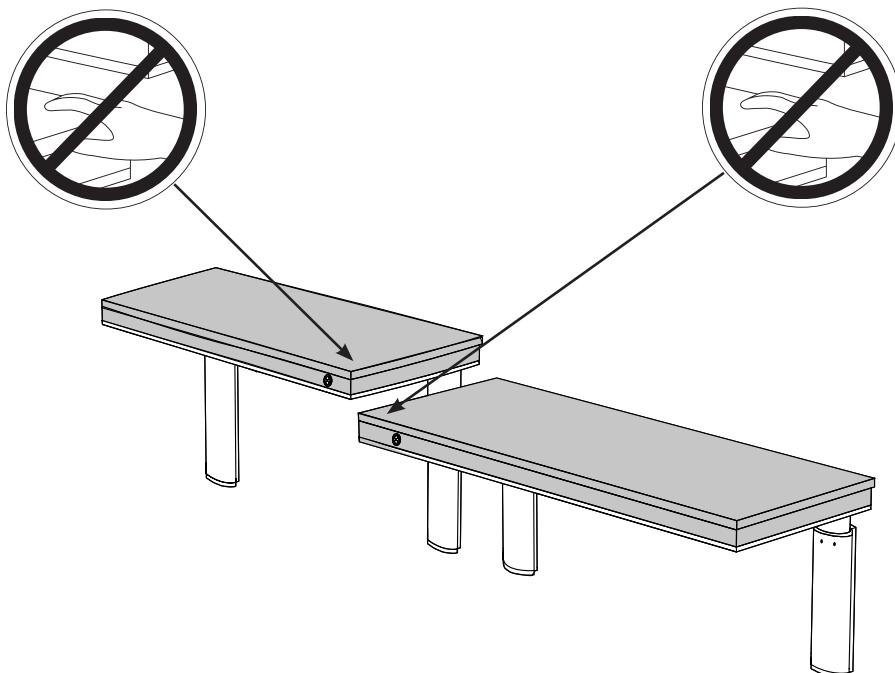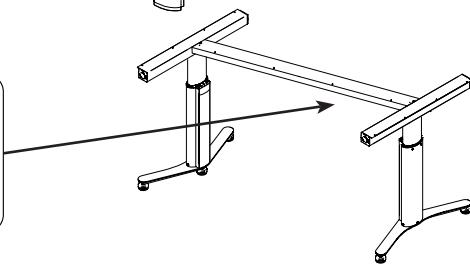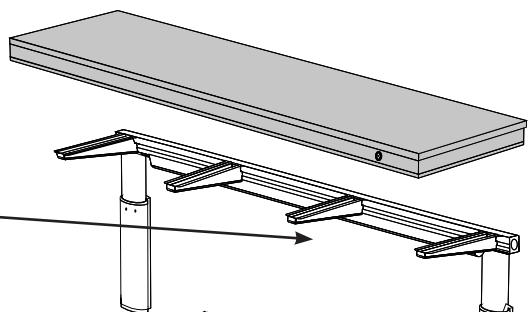Operating instructions


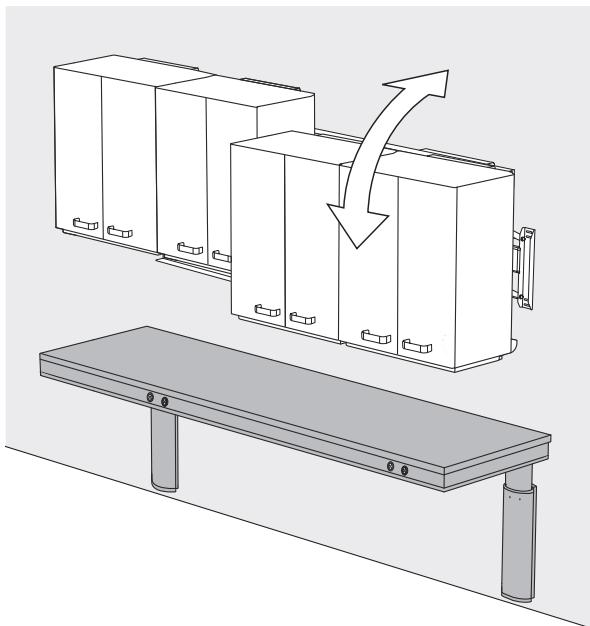
Press and hold the Up button to raise the cupboards, and press and hold the Down button to lower them. When the button is released, the motion stops.

An optional hand-held controller may be purchased as an alternative or supplement to the operating buttons.


The height adjustment is controlled by operating the red buttons on the fascia of the worktop.

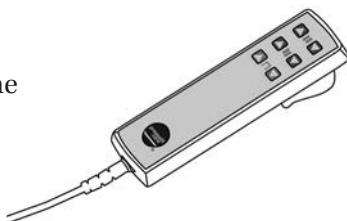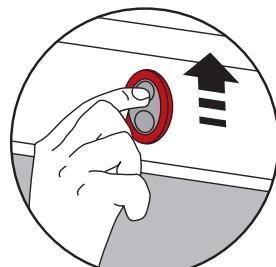The cupboards can be adjusted 430 mm up and down and 180 mm towards the user.

CAUTION


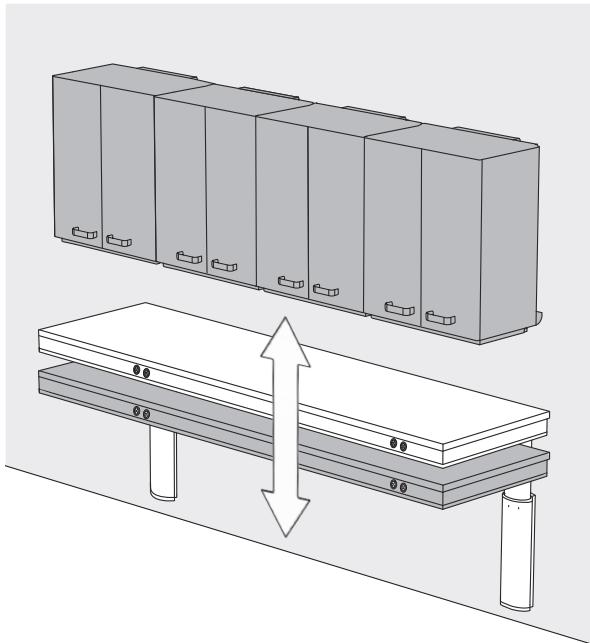
Do **NOT** use the safety plates and safety bars under the wall cupboards and the worktop as a stop function!


Press and hold the Up button to raise the worktop, and press and hold the Down button to lower it. When the button is released, the motion stops.

The height adjustment is controlled by operating the aluminum buttons on the fascia of the worktop.

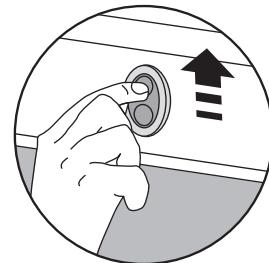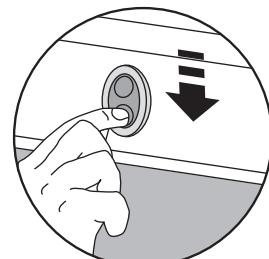The worktop can be adjusted 280 mm up and down.

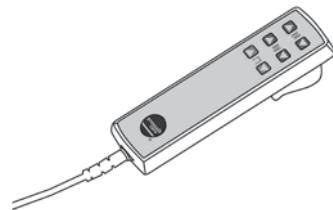An optional hand-held controller may be purchased as an alternative or supplement to the operating buttons.

en

CAUTION

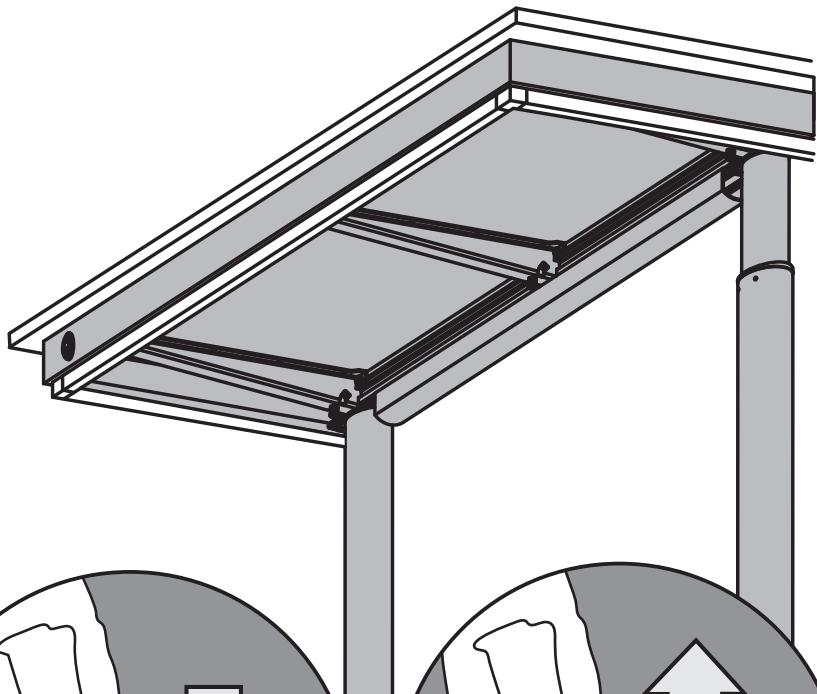
If two worktops are mounted side by side: Do not allow objects to rest on both worktops when adjusting the height.

Safety systems



Some versions of the lifting units are equipped with a safety system to prevent crushing injuries.

The worktop safety bar

If the safety bar under the worktop comes in contact with an object when the worktop is being lowered, the motion stops immediately and the worktop moves up 10 mm.


en

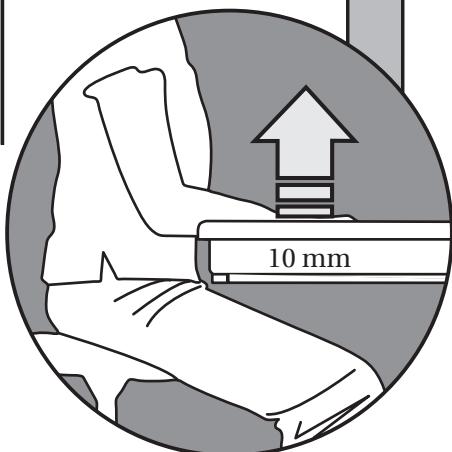
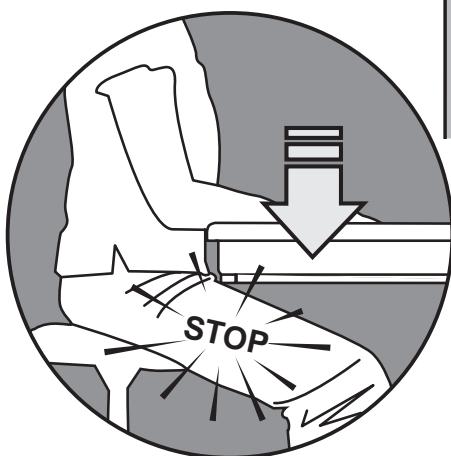
The cupboard safety plates

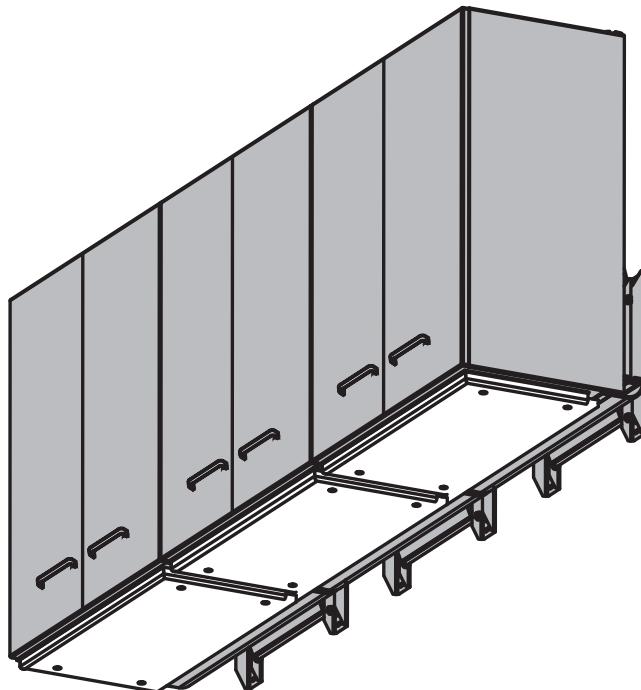If any of the safety plates under the cupboards come in contact with an object when the cupboards are being lowered, the motion stops immediately and the cupboards moves up 10 mm.

en

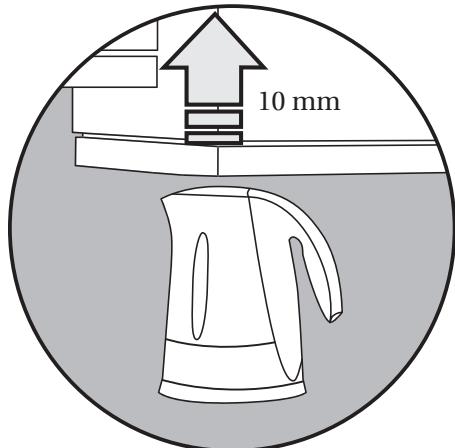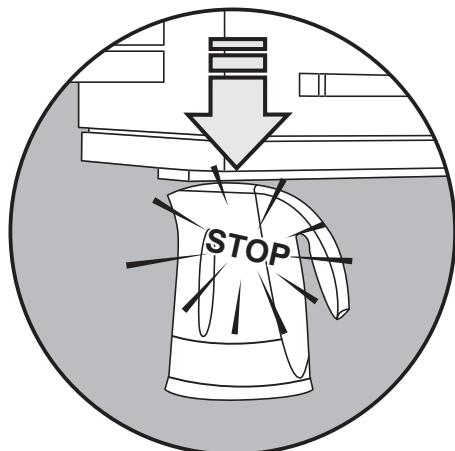
Maintenance

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

Resetting

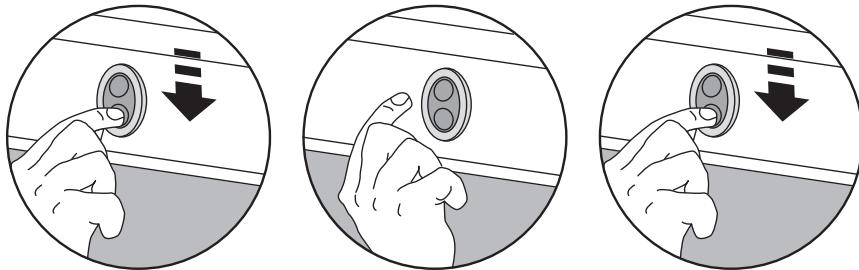It is also recommended on a weekly basis to reset the lifting mechanism.


- Hold in the switch until the table top stops in the lowest position.
- Release the switch.
- Press and hold in the switch once more for 5 seconds. Perform the same operation for the wall cupboards.

en


Cleaning

Use ordinary soapy water and dry with a soft cloth. Avoid the use of abrasive, etching or chloride cleaning agents.

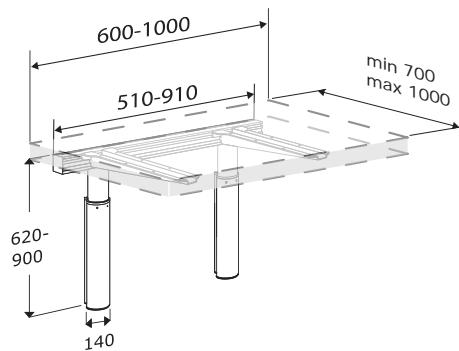Dimensions (mm)


RK1X10

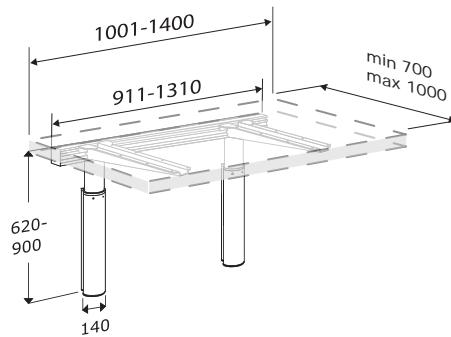
RK1X11

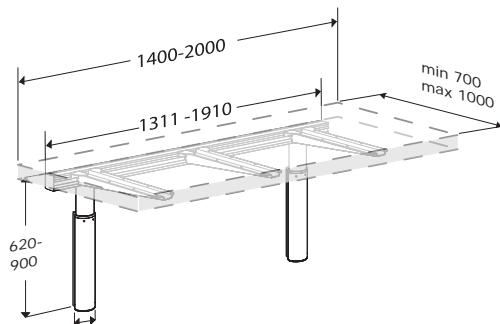
en


RK1X12

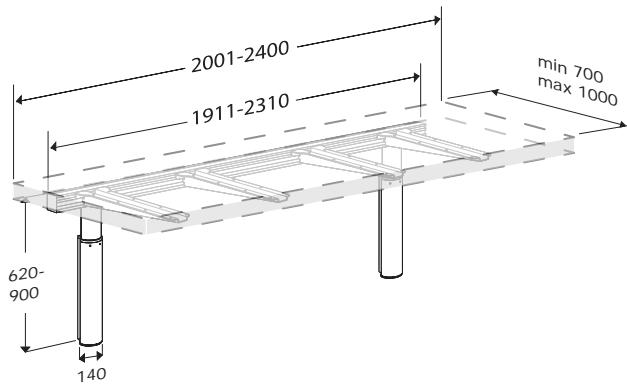
RK1X13

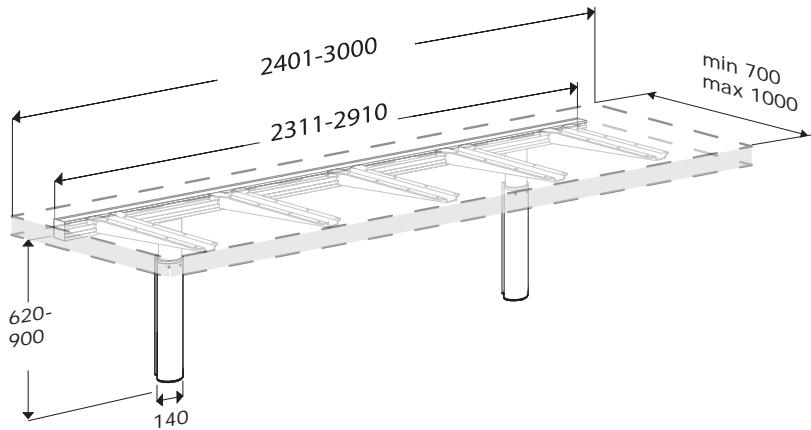RK1X14

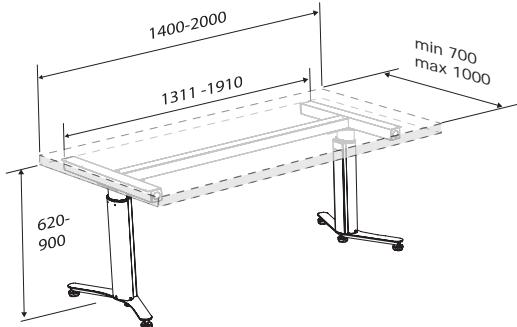
RK1081

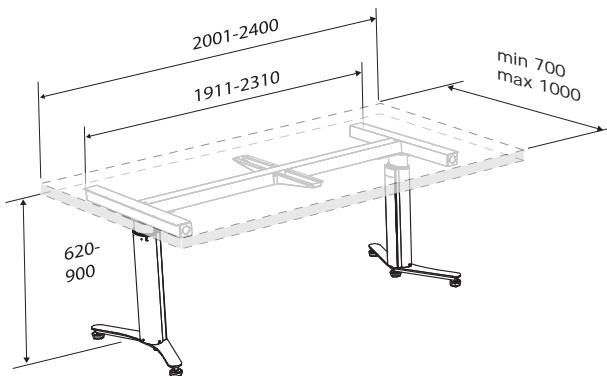
RK1082

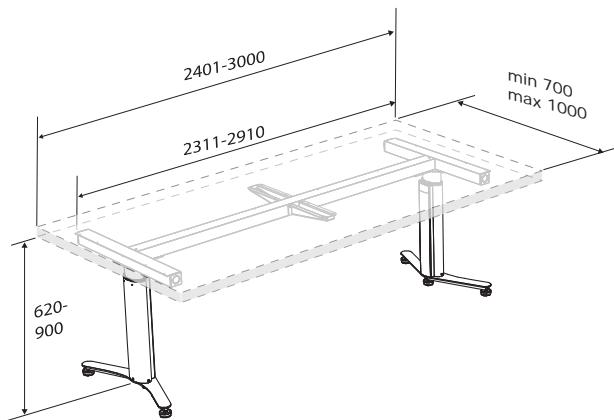
en

RK1083

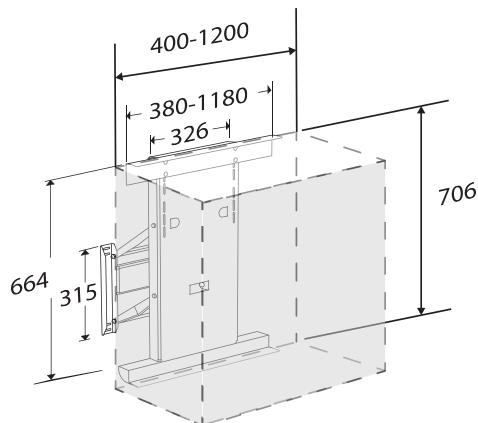
RK1X20

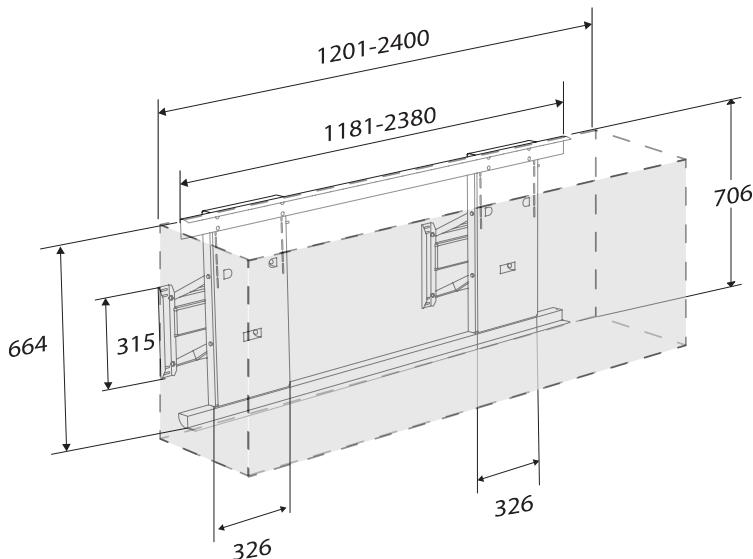
RK1X21

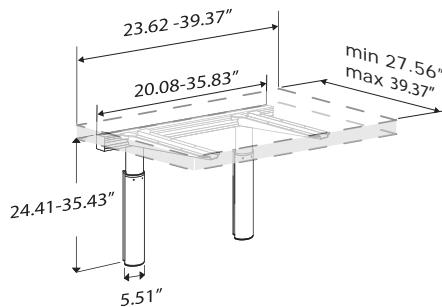Dimensions (inch)


RK1X10

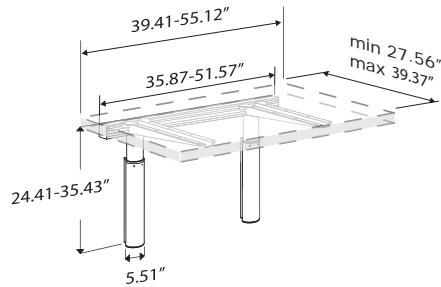
RK1X11

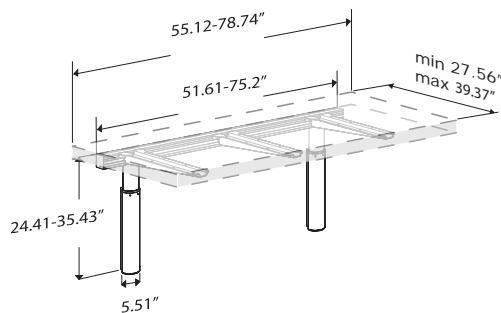
en


RK1X12

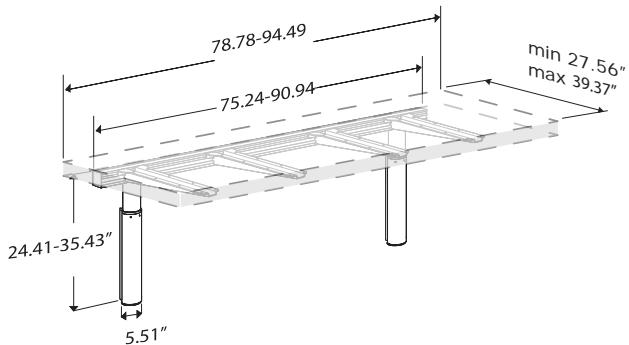
RK1X13

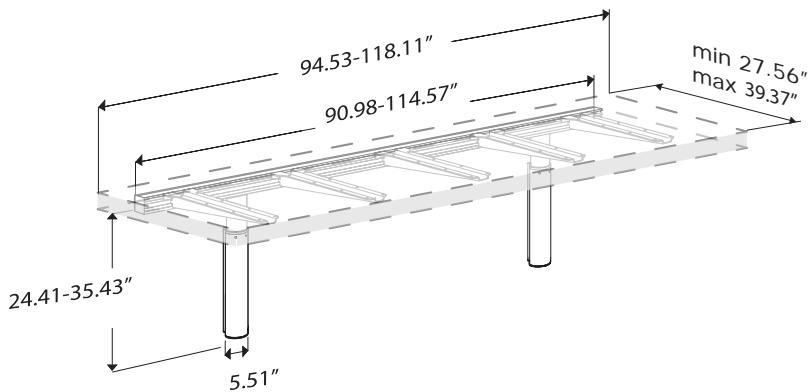RK1X14

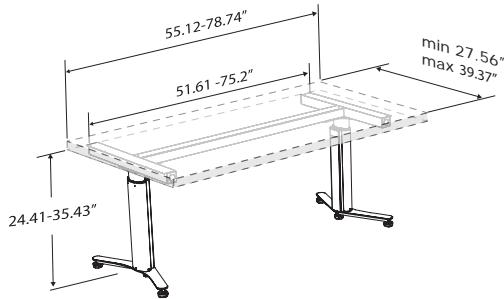
RK1081

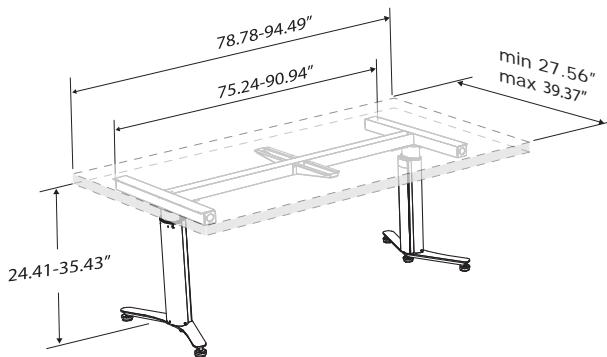
RK1082

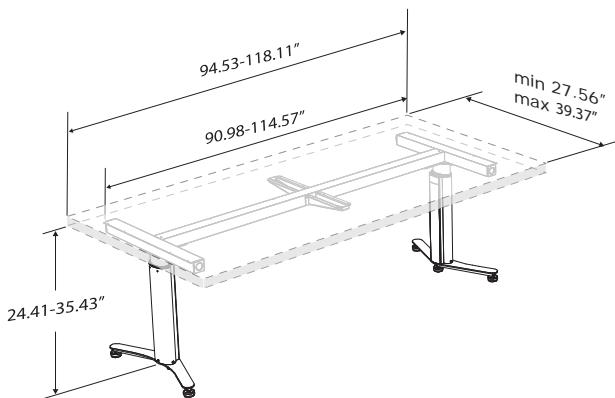
en

RK1083

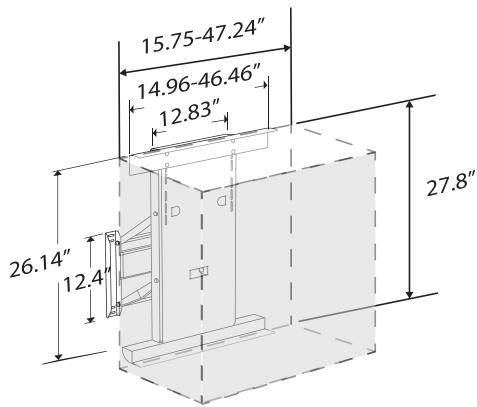RK1X20

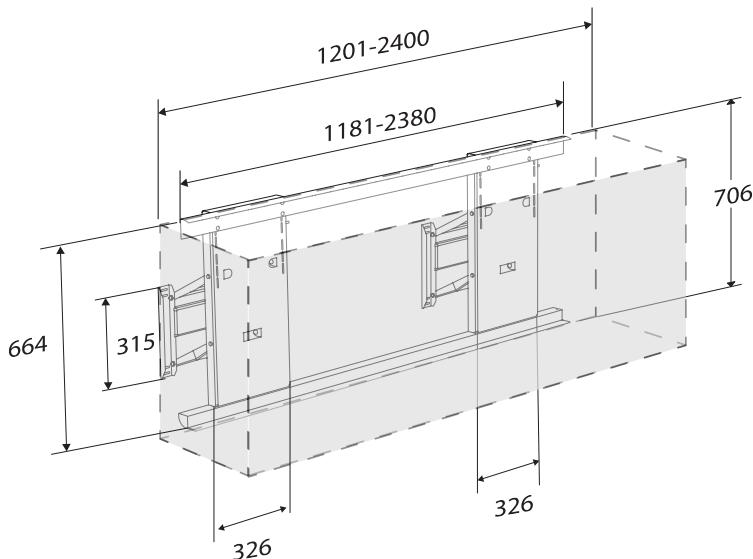RK1X21

Technical data

Cupboard lift	
Max load (incl. cupboard)	RK1X20 - 1 lift motor: 110 kg / 242,5 lbs RK1X21 - 2 lift motors: 220 kg / 485 lbs
Materials	Steel

Worktop lift	
Max load (incl. worktop)	100 kg / 220 lbs
Materials	Aluminum

Trouble-shooting

Symptom	Action
The height of the table top or wall cupboards cannot be adjusted	<p>Check that the safety bar under the table top or the safety plates under the wall cupboards have not been activated.</p> <p>Try to activate the safety bars or safety plates. They should move about 5 mm.</p> <p>If a sink and installation cover have been fitted, check that the installation cover does not collide with anything. If the problem persists, do the following:</p> <ul style="list-style-type: none">• Check that all operation switches can be operated without obstruction and that they return to a neutral position. Movement may be hampered by dirt or grease. If the problem persists, do the following:<ul style="list-style-type: none">• Reset the lifts, see "Resetting". <p>If height adjustable cupboards have been installed, reset the cupboards one at a time. Please note that the wall cupboards may be lowered very slowly until resetting has been performed.</p> <p>It may be necessary to reset the lifts several times. If the problem persists, do the following:</p> <ul style="list-style-type: none">• Check that all control boxes are provided with power, and that all plugs have been connected. The control box will typically be located behind the wall cupboards and under the table tops. If the problem persists, do the following:<ul style="list-style-type: none">• Disconnect the power if possible. Reconnect power after 5 seconds. After reconnection of power make a reset.

Symptom	Action
Cupboards or table top stop and will not go up any further.	Check that the cupboards or table top are not overloaded. If this is the case, remove some weight. Subsequently, the table top or wall cupboards should be reset, see "Resetting".
Safety bar or safety plate does not function.	Contact supplier.
Light under wall cupboards does not function.	Remove the cover with a slight tug, and replace any defective tubes. If this does not solve the problem, contact the supplier.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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Pressalit A/S