R9211
R9221

R1081, R1181
R102685, R102885
R112685, R112885
R372670, R372670112
R372870, R372870112
R372685, R372685112
R372885, R372885112

SELECT TL5

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Pressalit's Environmental and Quality Management Systems are certified by LRQA

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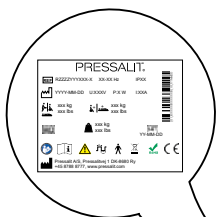
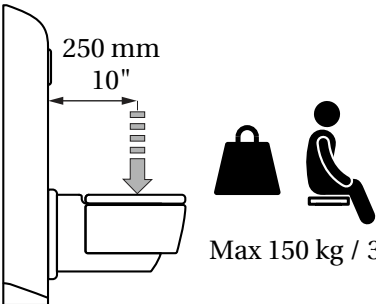
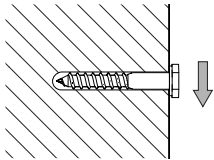
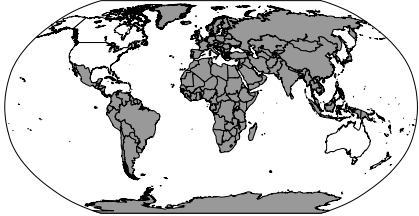
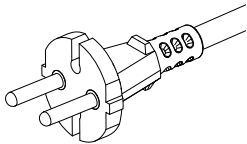
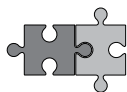
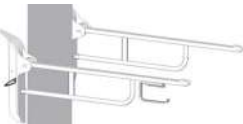
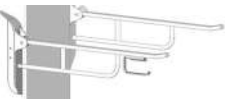
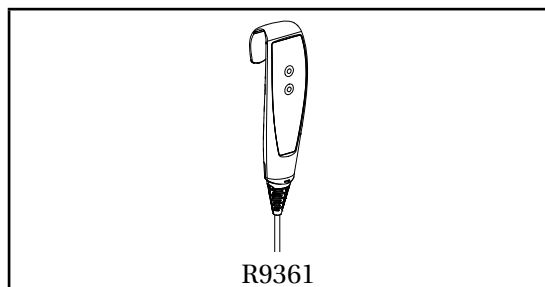
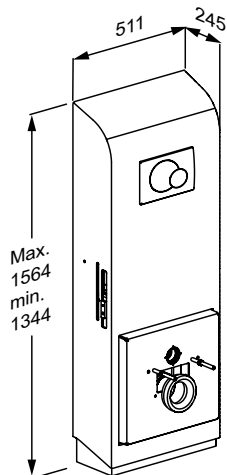
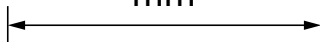
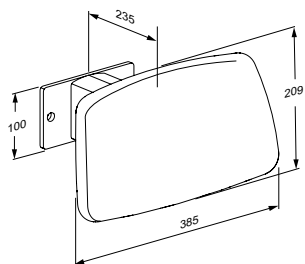
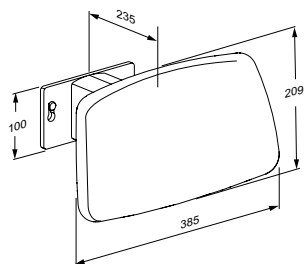
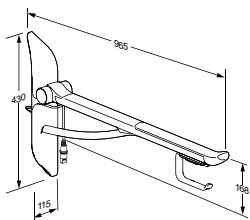
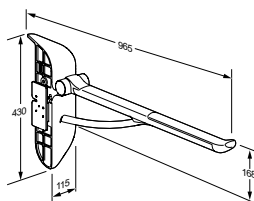
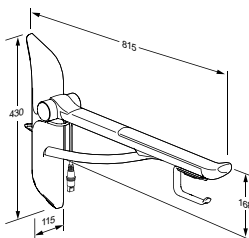
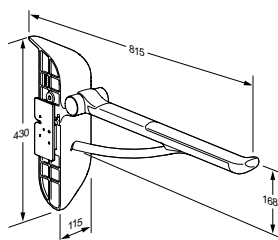
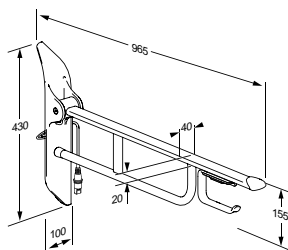
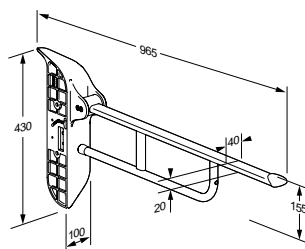
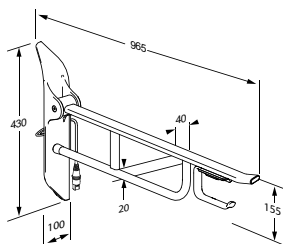
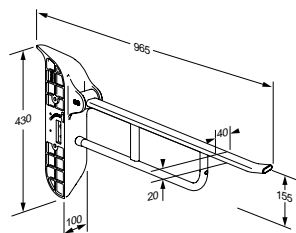
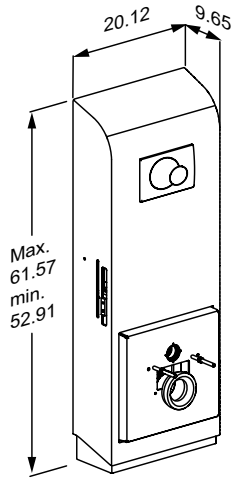
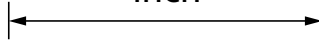
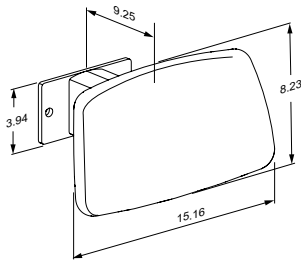
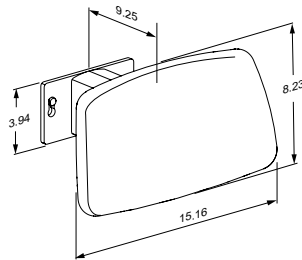
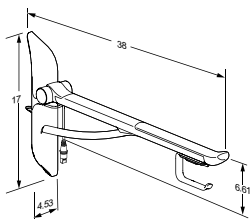
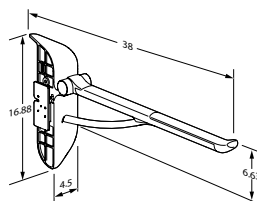
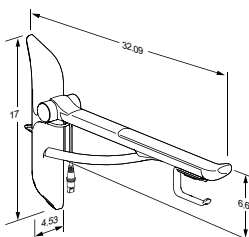
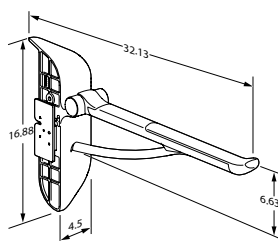
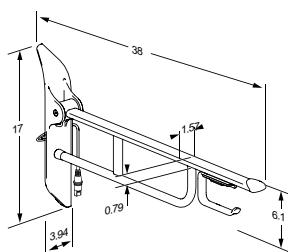
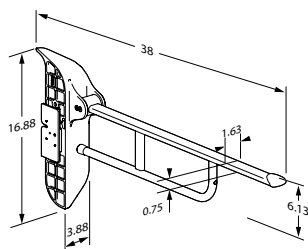
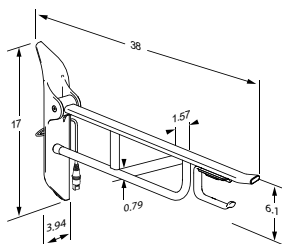
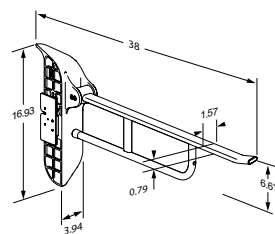
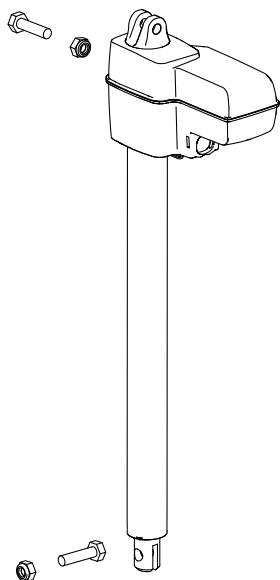
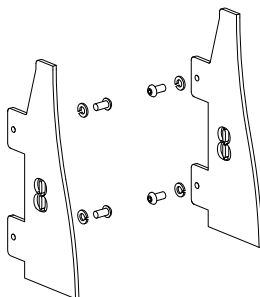
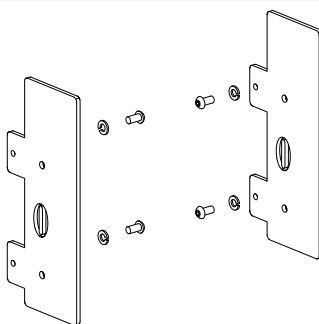
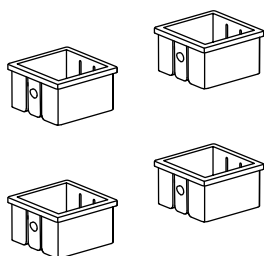
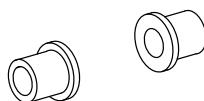
2 x V3633

Content

Mounting instruction	11
Symbols used in this manual	12
General safety	13
Intended use	14
Intended operator profile	14
Operating environment	14
Contraindications for use	14
Biocompatibility	14
Operating environment	15
Operating instructions	16
Tilting function	17
Support arms	18
Remote control	19
Cleaning	20
Disinfection	20
Maintenance	21
Maintenance inspection	21
Product label	26
Technical data	28
Electromagnetic compatibility	28
Electromagnetic immunity	29
Electromagnetic emission	32
Storage and transportation	33
Resetting	34
Trouble-shooting	34
Disposal and recycling	36

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

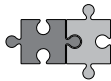
IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

en

			
Hand injury warning	Head injury warning	Feet injury warning	Electricity / voltage warning
			
Compatible products	Power specifications	EMC related symbols	Spare parts
			
Article number / versions	Product weight	Max. user weight	Safe working load

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

This product is intended for optimum ergonomics at a toilet, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by adults whose weight does not exceed the maximum load specified on the product label. Children can use it under supervision of an adult.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used in private homes, nursing homes and hospitals. If the relative humidity is above 70%, the expected life of the unit will be significantly reduced.

Contraindications for use

People who are not capable of keeping their balance in a seated position are not recommended to use this toilet lifter. Use of this product is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions. Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the product only after consulting with the attending physician.

Do not use if the user's weight exceeds 150 kg / 330 lbs.

Safe working load (SWL) is 235 kg / 518 lbs: weight + support arms + toilet + water

Biocompatibility

The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in EN ISO 10993-1:2020 Biological evaluation of medical devices Part 1 and are therefore considered suitable for this purpose.

The product must not come into contact with chlorine-containing fumes, as such exposure could lead to a degradation of its overall structural integrity, posing a risk of significant harm to both the user and the product.

CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.


Operating environment

en

Environmental factor	Operating range
Temperature	5°C to 40°C (41°F to 104°F)
Relative humidity	20% to 70% @ 30°C (86°F)
Atmospheric pressure	700 to 1060 hPa
Meters above sea level	Max. 3000 m / 9 800 feet
Duty cycle	2 minutes of continuous use, followed by an 18-minute pause.

Operating instructions

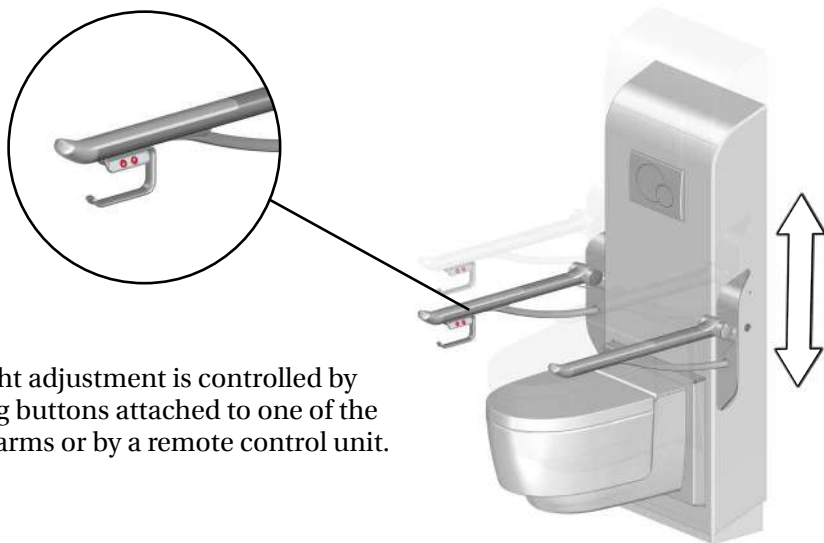The toilet lifter can be adjusted 220 mm / 8.66" up and down.

The height adjustment is controlled by operating buttons attached to one of the support arms or by a remote control unit.

en

Press and hold the up button to raise the toilet. When the button is released, the motion stops.

Press and hold the down button to lower the toilet. When the button is released, the motion stops.

Ensure that the cord of the hand control does not wrap around the user's neck, as this poses a suffocation risk. When not in use, place the corded hand control in its holder.

Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

CAUTION

- Always be careful when moving a person to and from the unit
- Never allow children to operate the unit without supervision of an adult. The children are at risk of injuries, and the product may be damaged.
- Keep pets and pests away from the unit. The product may be damaged and/or contaminated.

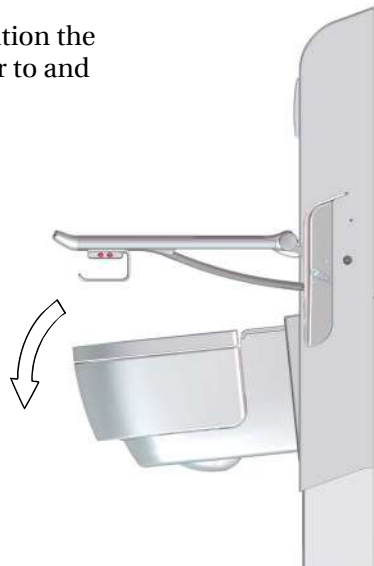
CAUTION

Do not place any objects beneath the toilet, and always ensure that the area underneath the toilet is clear when adjusting its height.

Tilting function

When adjusting the toilet lifter to upper position the toilet will tilt to accommodate easier transfer to and from the lifting unit.

NOTICE

When the operator is not the same as the user, be aware of the users comfort and balance when using the tilting function.

Support arms

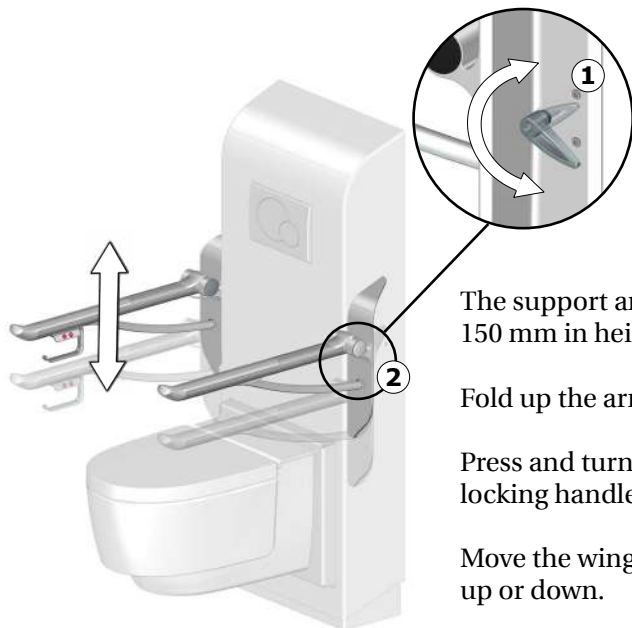
The support arms can be folded up and down. Fold them up when not in use.

For support arms R3724, R3725, R3726 and R3728: You can hear a click when the support arm is correctly folded up.

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

en

The support arms can be adjusted 150 mm in height.

Fold up the arm.

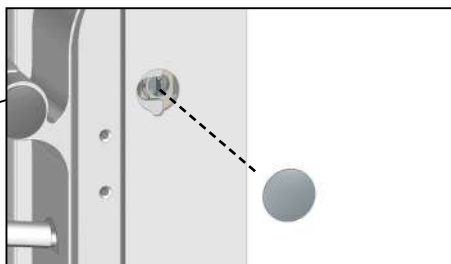
Press and turn the height adjustment locking handle ①.

Move the wing ② with the support arm up or down.


Tighten the locking handle again. It will slide back into place by itself.

Remote control


The unit can be operated with a wired remote control (optional).


Remove the cap.

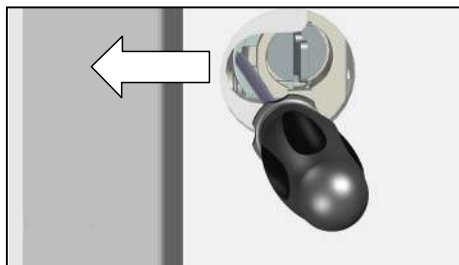Move the locking ring.

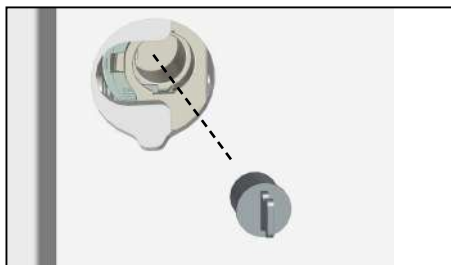Remove the cover plug.

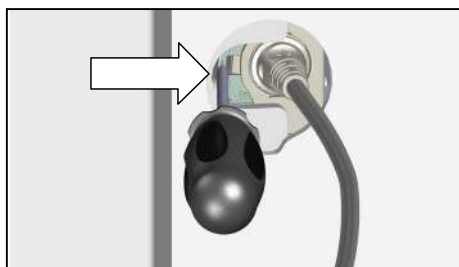Insert the plug for the remote control into the socket, and push the locking ring back into place.

en

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive or corrosive cleaning or disinfecting products.
- Avoid the use of chlorine solution higher than up to 1500 ppm.

Disinfection

When using the toilet for multiple patients, it is necessary to both clean and disinfect it between uses of different patients to prevent the risk of cross infection.

When disinfection is needed, use disinfectant wipes with up to a 85% solution of isopropyl, or a damp cloth with warm water and a disinfectant cleaner, e.g. an chlorine solution up to 1500 ppm.

If other chemicals and/or liquids with higher concentration should be used to clean or disinfect these products, please contact Pressalit providing the item's safety sheet chemical composition for consideration.

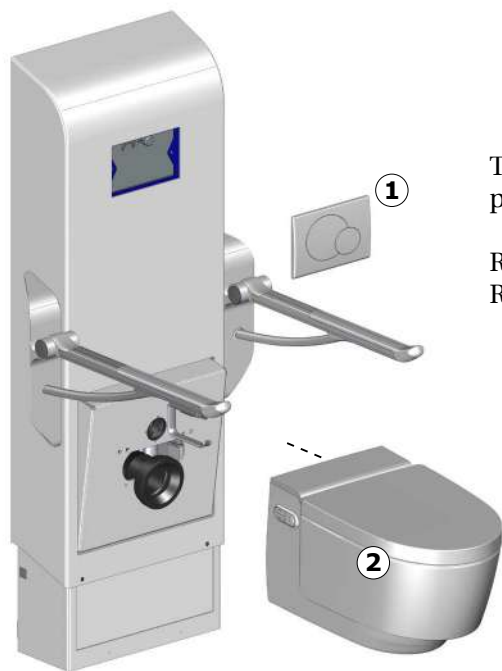
Maintenance

Expected service life of the toilet lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.


It is recommended that maintenance inspection is done once a year.

Maintenance inspection

Take the toilet to its highest position.

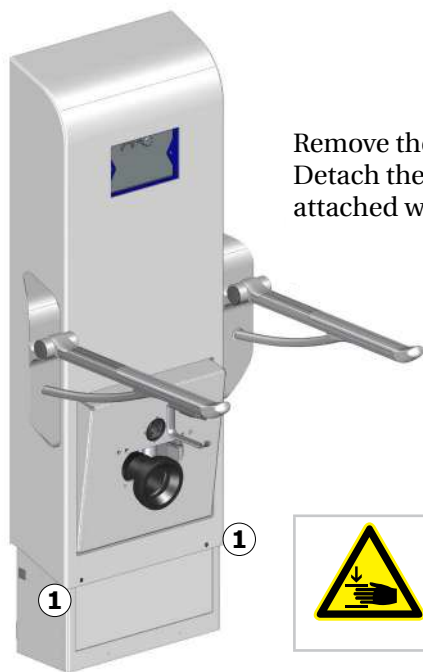Remove the flushing panel ①.
Remove the toilet ②.

Remove the two screws for the cover curtain ②.
Detach the cover curtain ②. It is additionally
attached with a Velcro strip.

The cover curtain is spring-loaded. Make sure that it doesn't snap down uncontrolled.

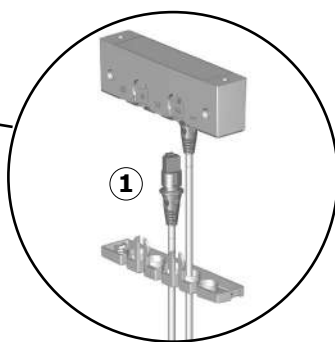
Remove the lower cover


Disconnect the mains power connection, whether
internal or external.

Reach up under the front cover to unplug the wire ① from the control buttons to the control box inside the cabinet.

en

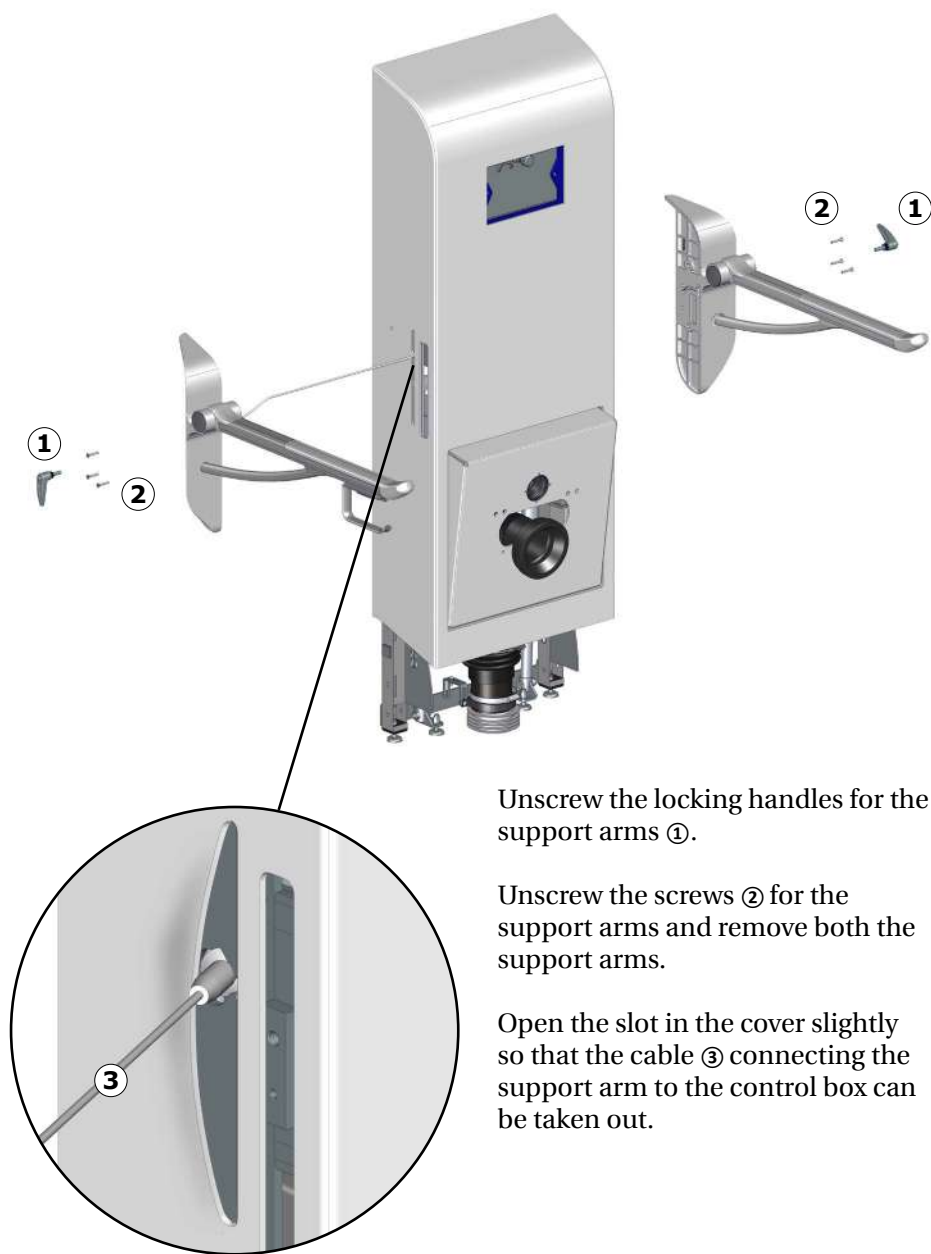
Unscrew the locking handles for the support arms ①.

Unscrew the screws ② for the support arms and remove both the support arms.

Open the slot in the cover slightly so that the cable ③ connecting the support arm to the control box can be taken out.

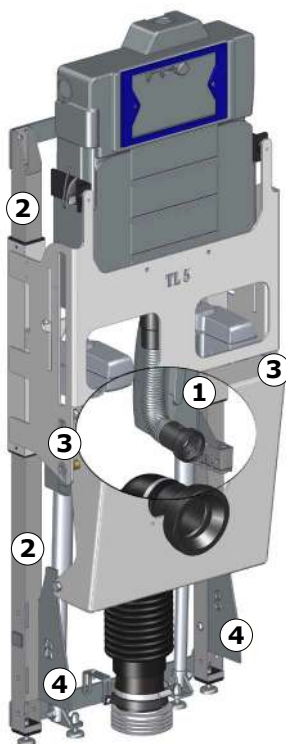Remove the front cover.

Check all electrical connections ①.

Lubricate the inner square tubes ②.

Check the suspension system ③ and the curve plate ④ for corrosion.

Reattach the front and lower front cover. Mount the support arms, the control box connector and the toilet.

Reconnect mains.

Take the toilet to its lowest position, and then run it to the top position again to distribute the lubricant.

en

en

- en

⑩	IP class
⑪	Electric Current
⑫	UDI carrier GS1-128 barcode
⑬	Product weight
⑭	Batch code
⑮	UK Conformity Assessed
These products are CE-marked in accordance with relevant requirements in: • Regulation (EU) 2017/745 (5 April 2017) on medical devices. • Council Directive 2006/42/EC (of 17 May 2006) on machinery. • Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at pressalit.com	
⑰	This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
⑱	Product name (if any)
⑲	Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
⑳	Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock.
㉑	Duty cycle
㉒	Consult the manual for important cautionary information such as warnings and precautions.
㉓	Consult manual for use
㉔	Name and address of manufacturer
㉕	Class II Product, double insulated

Technical data

Materials	Powder laquered stainless steel Plastics (PVC, Polystyrene)
IP class	The electrical parts have an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.
Measured sound pressure level	50 dBA

Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In case of loss or degradation of performance due to electromagnetic disturbances, the user can still get on and off the product.

Cable	Length
Power cable	3,2 m (126")
Hand control spiral cable	Approx. 5 m (197") extended

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Electromagnetic immunity

Electrostatic discharge	
	The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-2	+/- 8 kV contact +/- 2, 4, 8, 15 kV air

Radiated RF electromagnetic field	
	The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.
Test standard	Immunity test level
EN 61000-4-3	15 V/m 80 MHz to 1 GHz 15 V/m 1,01 GHz to 6 GHz 80 % AM 1 kHz

en

Immunity to proximity fields from RF wireless communication equipment

The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.

Test standard

Immunity test level

EN 61000-4-3*

385 MHz, 27 V/m, 18 Hz PM (50 % duty cycle square wave)
 450 MHz, 28 V/m, FM +/- 5 kHz deviation, 1 kHz sine
 710 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)
 745 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)
 780 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)
 810 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave)
 870 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave)
 930 MHz, 28 V/m, 18 Hz PM (50 % duty cycle square wave)
 1720 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)
 1845 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)
 1970 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)
 2450 MHz, 28 V/m, 217 Hz PM (50 % duty cycle square wave)
 5240 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)
 5500 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)
 5785 MHz, 9 V/m, 217 Hz PM (50 % duty cycle square wave)

Electrical fast transient/burst

The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.

Test standard

Immunity test level

EN 61000-4-4


+/- 1 kV , +/- 2 kV
 100 kHz repetition frequency

Surge immunity	
	The product must not be affected to initiate actions or to render components defective as a result of lightning.
Test standard	Immunity test level
EN 61000-4-5	Line-to-line: +/- 0.5 kV, +/- 1 kV Line-to-ground: +/- 0.5 kV, +/- 1 kV, +/- 2 kV

Immunity to conducted disturbances induced by RF fields	
	The wires must not be affected by other RF fields so as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-6	3 V (6 V in ISM bands and amateur radio bands) 0,15 to 80 MHz , 80 % AM at 1 kHz

en

Voltage dips	
	The product must not initiate or limit actions as a result of a dip in the power supply.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % U_T ; 1 cycle at 0°, 70 % U_T ; 25 cycles at 0° Single phase at 0°

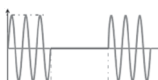Voltage interruptions	
	The product must not initiate or limit actions as a result of a short-term power supply stop.
Test standard	Immunity test level
EN 61000-4-11	0 % U_T ; 300 cycle

Electromagnetic emission

Conducted RF emission	
	The product must not send too much noise into the grid. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1




Radiated RF emission	
	The product must not emit RF signals in the room. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

en

Harmonic current emissions	
	The product must not unintentionally draw power from the grid.
Test standard	Emission test class and group
IEC 61000-3-2	Class A

Voltage changes, voltage fluctuations and flicker	
	The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room.
Test standard	Emission test class and group
EN 61000-3-3	Tested according to the standard.

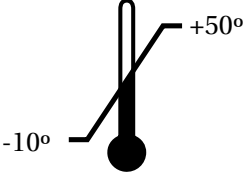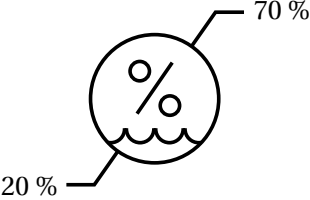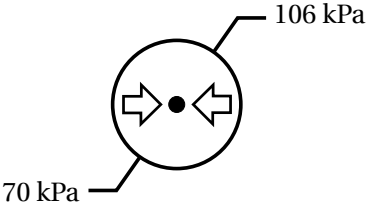Storage and transportation

	The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.
	The relative humidity for transportation and storage of the unit must be between 20 and 70% (RH).
	The air pressure for transportation and storage of the unit must be between 70 and 106 kPa.

en

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Resetting

If the height adjustment does not work, and the control box beeps and/or flashes, the unit must be reset.

Press and hold the up button and the down button simultaneously. The control box emits a beep sound. When the beep stops, release both buttons.


Take the unit to its lowest position.
The unit is now reset.

Trouble-shooting

Symptom	Cause	Action
Operating buttons does not work.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Refer to the section "Maintenance". Contact Pressalit A/S or your local dealer if the problem cannot be remedied.
The product does not work, and the control box beeps and/or flashes.	The product needs to be reset.	See "Resetting".
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".
	The inner square tubes might be slightly skewed.	Contact Pressalit A/S or your local dealer.

Height adjustment of support arms is not possible.	The screws ① holding the support arm in place are tightened too hard.	Loosen the screws until height adjustment is possible. Check mounting instruction.
For products with tilting function: The tilting function does not work or runs uneven.	The tilting mechanism is worn or warped.	Check that the bearings run freely on the curve plate ②. Check if the plate is intact or if it needs to be replaced.

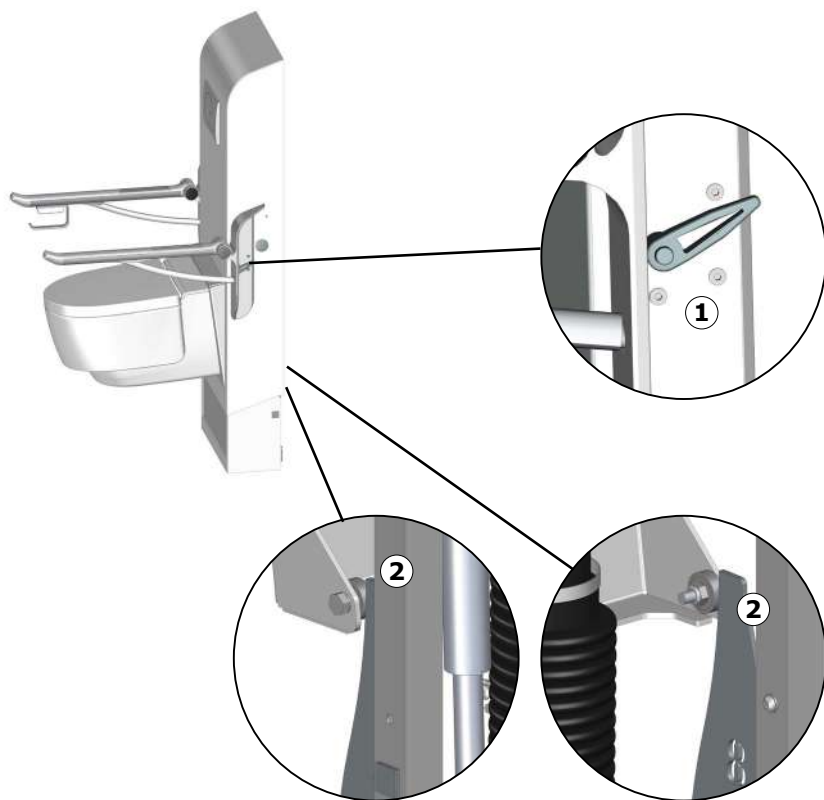Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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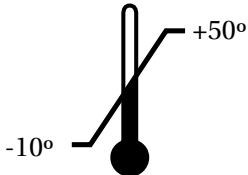
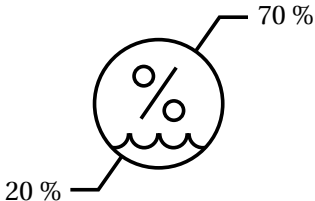
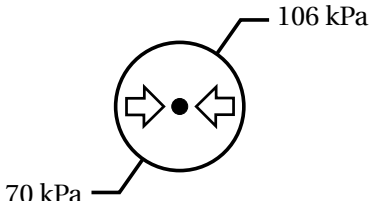
Opbevaring og transport

 A thermometer icon with a vertical scale. A line points from the label '-10°' to the bottom of the scale, and another line points from the label '+50°' to the top of the scale.	Temperaturen for transport og opbevaring af enheden skal være mellem -10 og 50 grader Celsius.
 A circular icon containing a percentage sign (%). A wavy line is drawn across the bottom of the circle. A line points from the label '20 %' to the bottom of the circle, and another line points from the label '70 %' to the top of the circle.	Den relative luftfugtighed for transport og opbevaring af enheden skal være mellem 20 og 70% (RH).
 A circular icon containing a central dot. Two arrows point horizontally towards the dot from the left and right sides. A line points from the label '70 kPa' to the left side of the circle, and another line points from the label '106 kPa' to the top of the circle.	Lufttrykket for transport og opbevaring af enheden skal være mellem 70 og 106 kPa.

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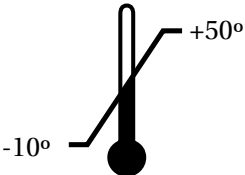
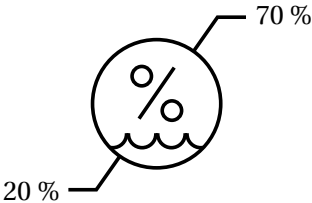
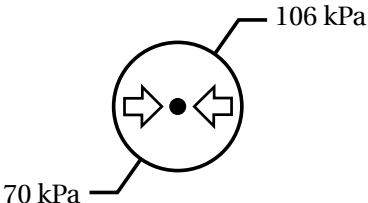
 A thermometer icon with a vertical scale. A line points from the text '-10°' to the bottom of the scale, and another line points from the text '+50°' to the top of the scale.	Temperaturen for transport og oppbevaring av enheten må være mellom -10 og 50 grader Celsius.
 A circular icon containing a percentage sign (%). A wavy line is at the bottom. Two lines point from the text '20 %' and '70 %' to the left and right sides of the circle respectively.	Den relative fuktigheten for transport og oppbevaring av enheten må være mellom 20 og 70 % (RF).
 A circular icon containing a central dot and two arrows pointing towards it from the left and right. Two lines point from the text '70 kPa' and '106 kPa' to the left and right sides of the circle respectively.	Lufttrykket for transport og oppbevaring av enheten må være mellom 70 og 106 kPa.

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