

# PRESSALIT.

Product	PLUS wall track, 2400 mm		
Item number			
	VVS:615841124		
	NRF:7501019	<u> </u>	
	EAN:5708590305851		
	HMI: 71745	128	
Colour	Anthracite grey		
		- 33 -	
Product description	PLUS wall tracks are used for mounting of support arms, shower seats and wash basin brackets, and allows sideways adjustment of these products. See below a selection of products for fitting to the wall track.  PLUS wall track, 2400 mm, for horizontal mounting. The cover plate hides bolts and wires and facilitates cleaning. Materials: Aluminium and polystyrene. See below a selection of products for fitting to the wall track. Excl. mounting kit.		
Adjustability	Fixed mounting		
Product capacity	Maximum load: Up to 1000 kg depending on the wall type.		
Materials	The wall track is primarily made of 10-micrometre anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer		
Ergonomy	The track distributes the weight of the individual bathroom product, for example a wash basin bracket, along its entire length. The products can be adjusted and locked on to the track with intuitive locking functions.		
Features	The horizontal track makes it possible to shift the product along the wall to constantl optimize the available space in the bathroom. The front cover plate conceals mounting bolts and any cables.		



## PRESSALIT®

Product	PLUS wall track, 2400 mm		
Item number	R9824112		
Cleaning	PLUS wall tracks The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.  The following cleaning agents have been tested by Pressalit and may be used, as		
	long as they are diluted according to the instructions on the container.		
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%		
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro		
	The product must be wiped down with a soft rag wrung out in clear water immediatel after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.		
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the part must be wiped over with a damp cloth wrung out in clear water.		
Declaration of conformity	MDR-ACC This product fulfils all relevant requirements in Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an <b>accessory</b> for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical		

devices to be used in accordance with their intended purposes.



## PRESSALIT®

Product			
Item number	r		

PLUS wall track, 2400 mm

R9824112

#### Installation in different wall types

Concrete >B25  V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.  To be ordered separately.	
Solid Brick >= Mz12  V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.  To be ordered separately.	
Solid Sand Lime Brick >= HIz12  V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.  To be ordered separately.	
Aerated Concrete P4 V8670 - Mounting kit, 3 bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood  V8620 - Mounting kit, 3 coach screws Ø8 x 70, plugs SX10.  NB! Plugs are not needed.  To be ordered separately.	



#### PRESSALIT®

Product PLUS wall track, 2400 mm

Item number R9824112

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com