


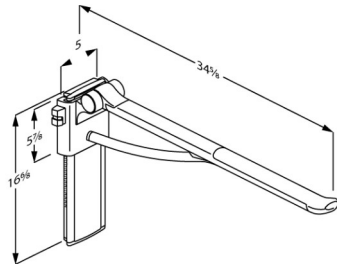

# PRESSALIT®

|                                                                             |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                        |
|-----------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Product</b><br><b>Item number</b>                                        | PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated<br>R370385112                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                       |
| <b>Color</b>                                                                | VVS:615875212<br>NRF:6191087<br>RSK:8487443<br>EAN:5708590368139<br>HMI: 125881<br>FDA: ILS/D135866<br>Anthracite gray                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                 |
| <b>Product description</b>                                                  |  <p>All PLUS fold down grab bars come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions. Suitable for use with elongated toilet pans or floor standing toilets. Replaces discontinued models R3175 og RF175.</p> <p><i>PLUS fold down grab bar, counter-balanced. Projection 34.7", ergonomically shaped to provide a clear sense of where the fold down grab bar ends. Left hand operated. Height adjustable by 9.5" without using tools. Detachable and foldable. Anthracite cover plate for vertical wall track.</i></p> <p><i>Materials: The arms are constructed of 10-micrometer anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core</i></p> <p><i>Excl. mounting kit.</i></p> |
| <b>Adjustability</b><br><b>Adjustment height</b><br><b>Product capacity</b> | Height adjustable<br>9,46 inch<br>Maximum load: 330 lbs.<br>When the fold down grab bar is loaded with 330 lbs, each wall fastener in the vertical wall track is loaded with 110 lbs.<br><br><i>PLUS fold down grab bars have been tested statically with 1540 lbs (close to the wall), 935 lbs (in the middle) and 495 lbs (the outermost point).</i>                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                 |



# PRESSALIT®

|                                                                |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             |
|----------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Product</b><br><b>Item number</b>                           | PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated<br>R370385112                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            |
| <b>Materials</b><br><br><b>Ergonomy</b><br><br><b>Features</b> | <p>The grab bars are primarily constructed in 10-micrometer anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.</p> <p>The hinge arm is also made of soft SEBS.</p> <p>Internal components and fasteners are primarily made of stainless steel.</p> <p>The hinge barrel is capped with ABS polymer.</p> <p>The oval shape gives an easy and comfortable surface to lean and push against.</p> <p>The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.</p> <p>Soft and slip-resistant material that has a warm feel.</p> <p>Folds against the wall.</p> <p>The underside is shaped for the mounting of various accessories.</p> |
| <b>Warranty</b>                                                | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |



# PRESSALIT®

|                    |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                           |
|--------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Product</b>     | PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                        |
| <b>Item number</b> | R370385112                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                |
| <b>Cleaning</b>    | <p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningssprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>Oxyplus-acid 2% (Diversey)<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b><br/>Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p> |



# PRESSALIT<sup>®</sup>

|                    |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                 |
|--------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Product</b>     | PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                              |
| <b>Item number</b> | R370385112                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
|                    | <p><b>Intended use</b><br/>The fold down grab bar is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b><br/>For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals.<br/>For institutional use, the expected life time is 10 years.<br/><i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to operation and maintenance manual, which is enclosed with the product, and which can also be found on the website under "Downloads".</p> |









PRESSALIT®

|                    |                                                                                    |
|--------------------|------------------------------------------------------------------------------------|
| <b>Product</b>     | PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated |
| <b>Item number</b> | R370385112                                                                         |

## Mounting in different wall types

|                                                                                     |                                                                                                                                                                                         |                                                                                       |
|-------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|    | <b>Concrete &gt;B25</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>To be ordered separately.                                                                          |    |
|   | <b>Solid Brick &gt;= Mz12</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>To be ordered separately.                                                                    |   |
|  | <b>Solid Sand Lime Brick &gt;= Hlz12</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>To be ordered separately.                                                         |  |
|  | <b>Aerated Concrete P4</b><br>V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8.<br>To be ordered separately.                                                            |  |
|  | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b><br>V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10.<br>NB! Plugs are not needed.<br>To be ordered separately. |  |

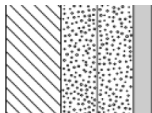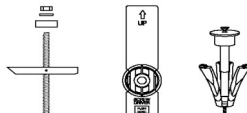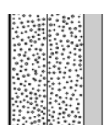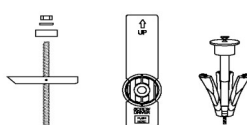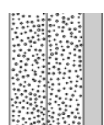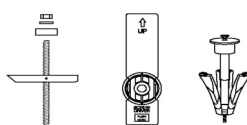

|                                                                                     |                                                                                                                                                                                                                                                                                                                |
|-------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Product</b><br><b>Item number</b>                                                | PLUS fold down grab bar with integrated counter-balance, 33.5", left hand operated<br>R370385112                                                                                                                                                                                                               |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.            |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.  |
|  | <b>Drywall (2 x 12,5 mm) water-resistant</b><br>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.                                          |