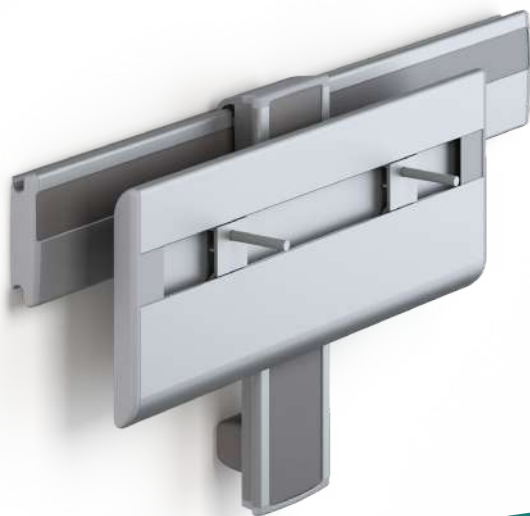
R4550 - R4560
R4580 - R4650
R4680

PLUS wash basin bracket – Operation and maintenance manual	en
PLUS vaskophæng – Brugs- og vedligeholdelsesvejledning	dk
PLUS Waschtisch-Lifter – Gebrauchs- und Pflegeanleitung	de
PLUS support de lavabo – Manuel d'utilisation et d'entretien	fr
PLUS muurframe voor wastafel – Gebruiks- en onderhoudshandleiding	nl
PLUS upphängning till tvättställ – Bruks- och underhållsanvisning	sv
PLUS oppheng til servant – Bruks- og vedlikeholdsveiledning	no
PLUS soporte de lavabo - Instrucciones de uso y mantenimiento	es
PLUS wspornik umywalki – Instrukcja obsługi i konserwacji	pl
Držák umyvadla PLUS – Návod k obsluze a údržbě	cs
PLUS 洗面盆支架 - 操作和维护手册	中文

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
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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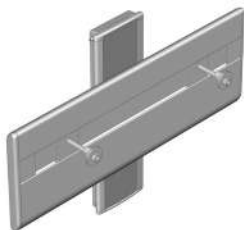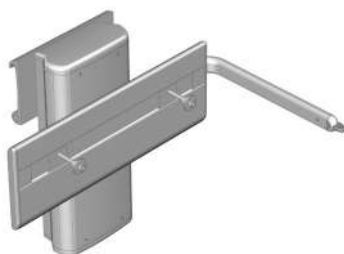
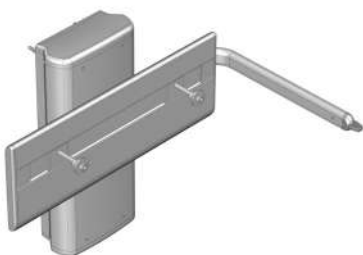
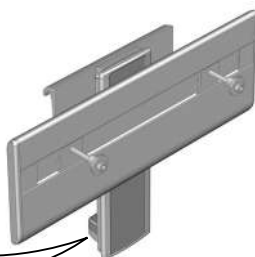
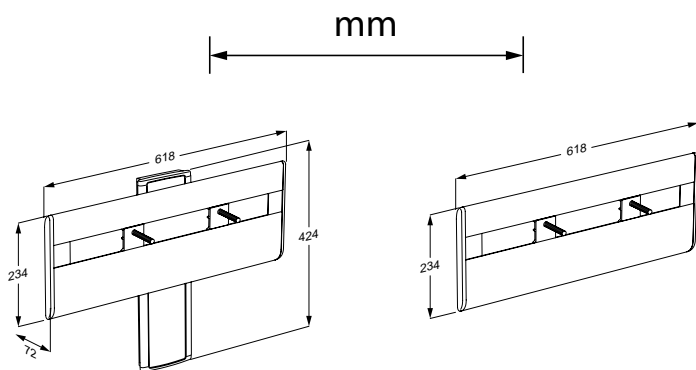
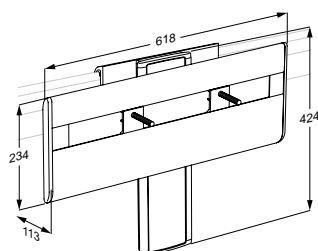
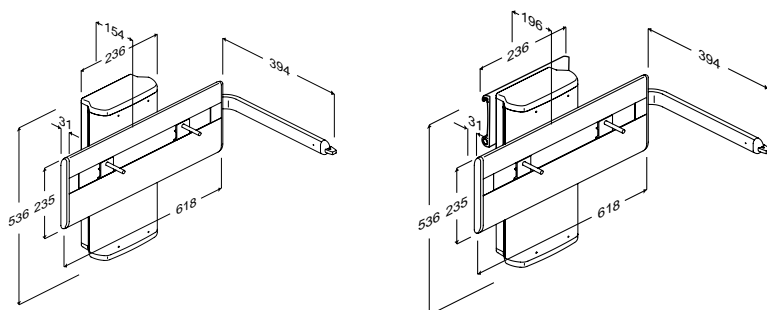
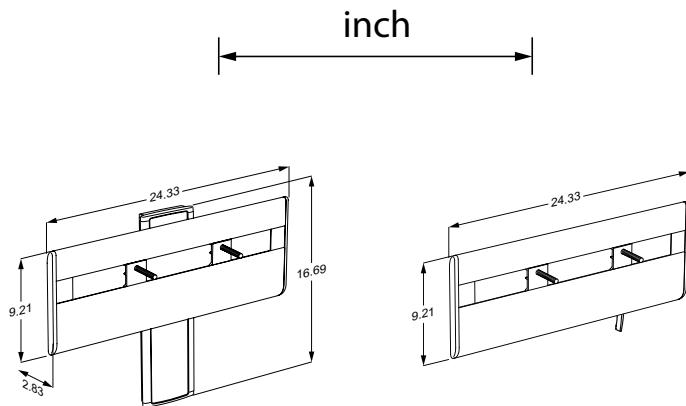
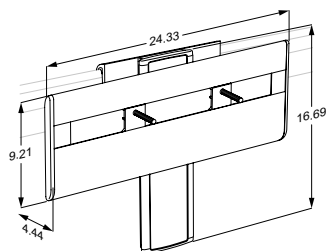
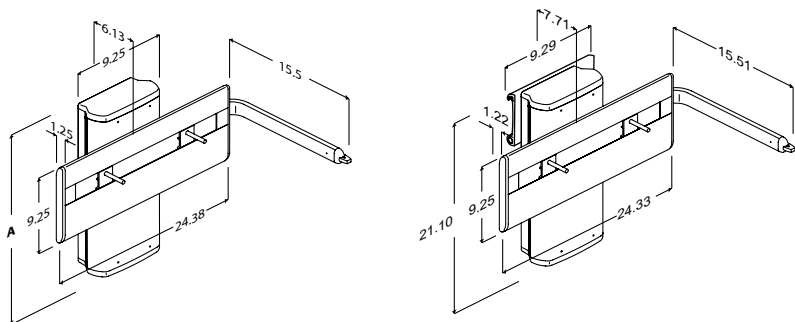
R4650, R4680

Content

Mounting instruction	6
Operational signage	6
Symbols used in this manual	7
General safety	8
Product label	9
Intended use and operator profile	10
Operating environment	10
Operating instructions	11
Height adjustment - R4550 and R4580	11
Height adjustment - R4650 and R4680	12
Sideways adjustment - R4560, R4580 and R4680	12
Cleaning	13
To remove limescale	13
Maintenance - R4650 and R4680	14
Service inspection	14
Maintenance - R4550, R4560 and R4580	16
Maintenance inspection	16
Trouble-shooting	16
Technical data	17
Disposal and recycling	17

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

An operational signage can be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Pinching of hands. Take care to avoid injury to hands when in the vicinity of closing mechanical parts.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

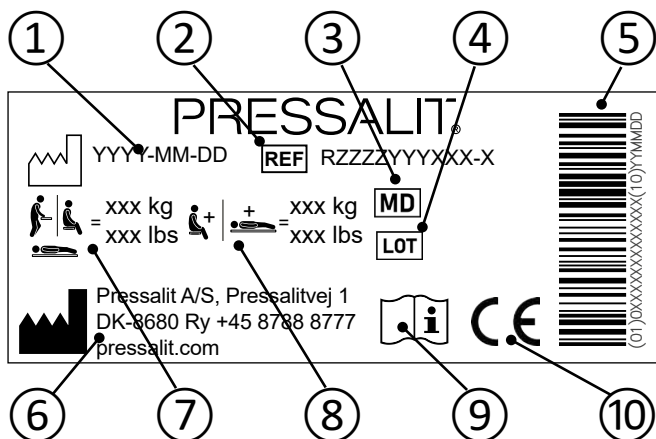
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

⑨ Consult manual for use

These products are CE-marked in accordance with relevant requirements in:

- ⑩
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
- A declaration of conformity can be found at pressalit.com

Intended use and operator profile

The wash basin bracket is intended for optimum ergonomics at a wash basin.

Version	Suitable use	Operator profile
R4550	For very infrequent height adjustment.	Height adjustment requires two persons.
R4580	For very infrequent height adjustment and where the possibility of adjusting the product sideways is a necessity.	Height adjustment requires two persons.
R4560	When height adjustment is never required, but where the possibility of adjusting the product sideways is a necessity.	Sideways adjustment can be done by a standing person.
R4650	For frequent height adjustment.	Height adjustment can be done by some seated and standing users dependent on their upper body strength and dexterity.
R4680	For frequent height adjustment and where the possibility of adjusting the product sideways is a necessity.	Height adjustment can be done by some seated and standing users dependent on their upper body strength and dexterity. Sideways adjustment can be done by a standing person.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room.
The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions.

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Operating instructions

The wash basin bracket (R4550 and R4580) can be adjusted 240 mm / 11.44 inches up and down. R4650 and R4680 can be adjusted 300 mm / 11.81 inches up and down. **Max load is 135 kg / 298 lbs.**

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

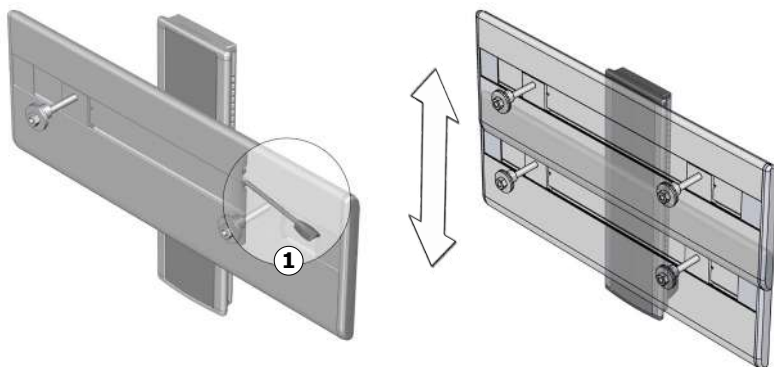
CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

Height adjustment - R4550 and R4580


NOTICE

The height adjustment of the versions R4550 and R4580 involves supporting the weight of the sink and part of the bracket. Therefore, two persons are required for height adjustment, and the bracket is not suitable for frequent adjustment.

To adjust the wash basin bracket, pull the handle ① upwards and lift the basin upwards to release the lock. Adjust the height. At the desired height, release the handle again to engage the lock.

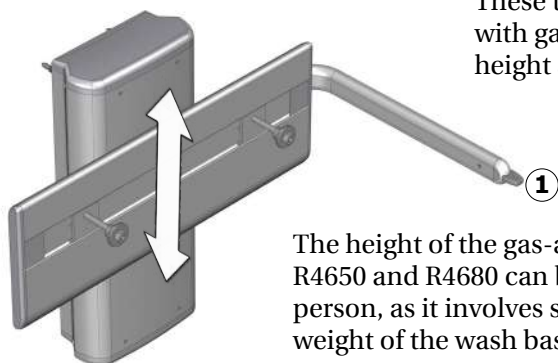Height adjustment - R4650 and R4680


These two models are equipped with gas cylinders to ease the height adjustment.

The height of the gas-assisted wash basin bracket R4650 and R4680 can be adjusted by a standing person, as it involves supporting part of the weight of the wash basin.

To adjust the wash basin bracket, push the button ① downwards and lift the basin upwards or push the basin downwards. Release the button to lock the basin in position.

Sideways adjustment - R4560, R4580 and R4680

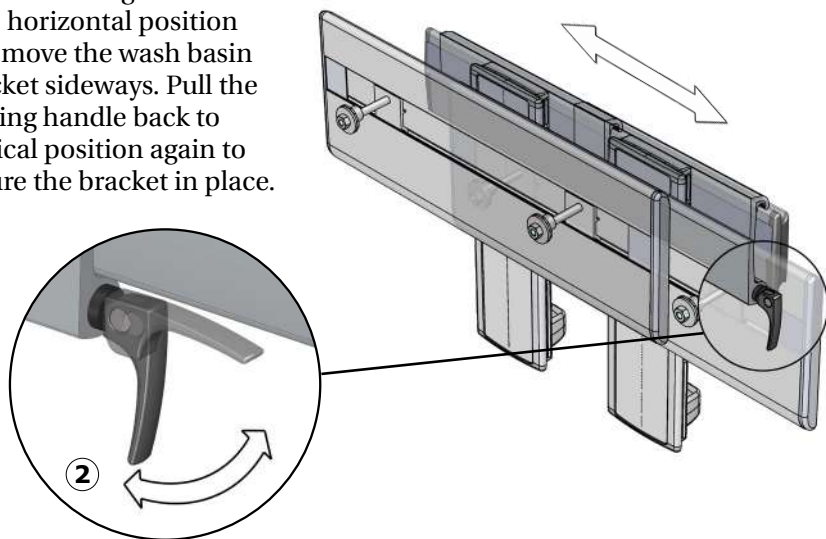en Pull the locking handle ② to horizontal position and move the wash basin bracket sideways. Pull the locking handle back to vertical position again to secure the bracket in place.

CAUTION

Never place objects underneath the wash basin.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

Maintenance - R4650 and R4680

Expected service life of the wash basin bracket is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to adjust the product to its maximum up and down positions, so that the internal parts are lubricated.

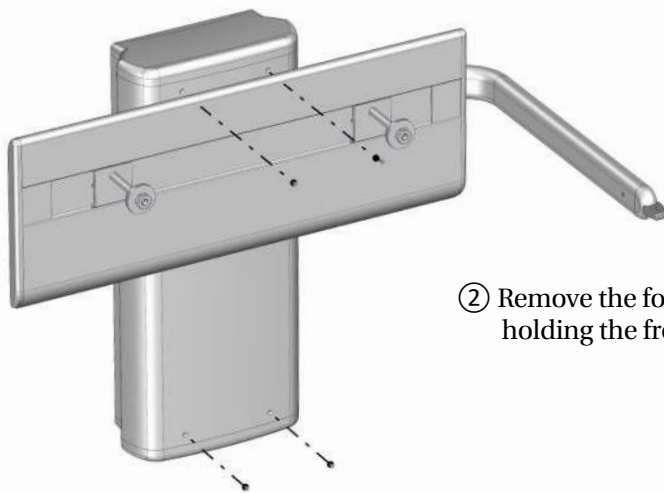
It is recommended that service inspection is done once a year.

Service inspection


Take the wash basin bracket to its highest position.


① Loosen the two top screws ①.

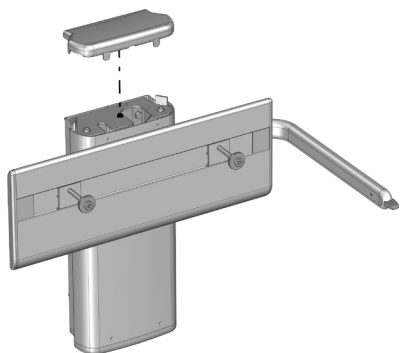
② Remove the four screws holding the front cover.

- ③ Lift off the top cover.

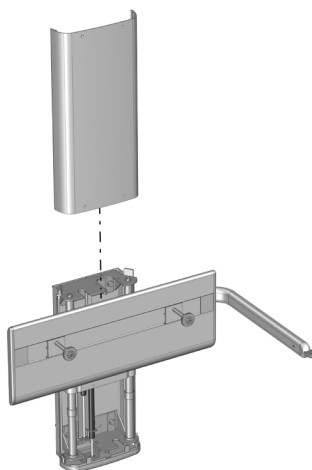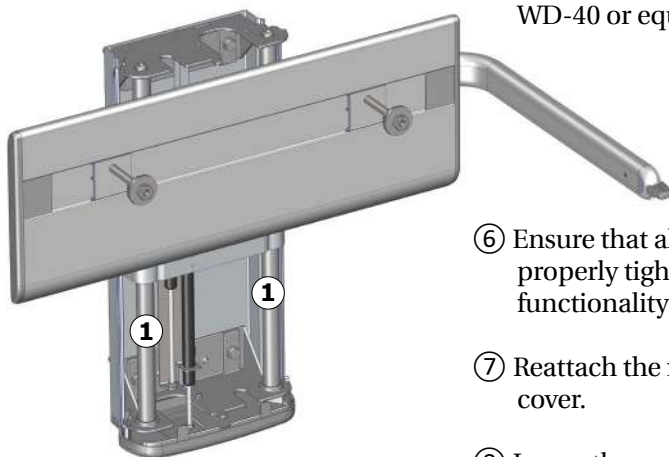- ④ Remove the front cover by sliding it upwards.


- ⑤ Lubricate the inner tubes ① with WD-40 or equivalent.

- ⑥ Ensure that all screws and bolts are properly tightened, and check the functionality of the product.

- ⑦ Reattach the front and the top cover.

- ⑧ Lower the wash basin bracket to its lowest position, and take it to the top position again to distribute the lubricant.

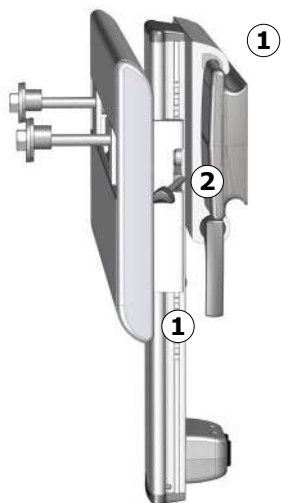Maintenance - R4550, R4560 and R4580

Remove dust from the wall tracks ① on a regular basis. Wipe the wall tracks with silicone. Lubricate the locking handles ② with WD 40 or equivalent.

It is recommended on a weekly basis to adjust the product to its maximum up, down and sideways positions.

en

Maintenance inspection

Ensure that all screws and bolts are properly tightened, and check the functionality of the product.

Trouble-shooting

Symptom	Cause	Action
The product can't be moved sideways or up and down.	There might be dirt or dust on the walltracks.	Clean the walltracks and lubricate with silicone.
The locking handle for horizontal adjustment doesn't lock.	The lock is not properly adjusted.	See mounting instruction.

Technical data

Max load	135 kg / 298 lbs
Materials	Stainless steel, Aluminium, Plastics
Weight	R4550: 5,56 kg / 12,25 lbs
	R4560: 5,35 kg / 11,79 lbs
	R4580: 8,90 kg / 19,62 lbs
	R4650: 11,70 kg / 25,79 lbs
	R4680: 13,50 kg / 29,76 lbs

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

en

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.

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