PRESSALIT®

R1170, R1171
R1190, R1191

TMA 1, TMA 2

Arm support – Operation and maintenance manual	en
Toiletstøtte – Brugs- og vedligeholdelsesvejledning	da
Stützklappgriff – Gebrauchs- und Pflegeanleitung	de
Barre de relèvement – Manuel d'utilisation et d'entretien	fr
Toiletsteun – Gebruiks- en onderhoudshandleiding	nl
Toalettarmstöd – Bruks- och underhållsanvisning	sv
Toalettstøtte – Bruks- og vedlikeholdsveiledning	no
Apoyabrazos - Instrucciones de uso y mantenimiento	es
Maniglione - Manuale d'uso e manutenzione	it
Podpora – Instrukcja obsługi i konserwacji	pl
臂撑 - 操作和维护手册	中文

English	6
Dansk	15
Deutsch	24
Français	33
Nederlands	42
Svenska	51
Norsk	60
Español	69
Italiano	78
Język polski	87
中文	96

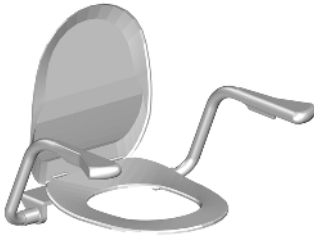
Pressalit's Environmental and Quality Management Systems are certified by LRQA

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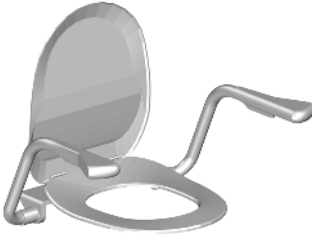
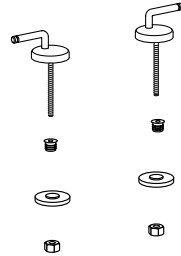
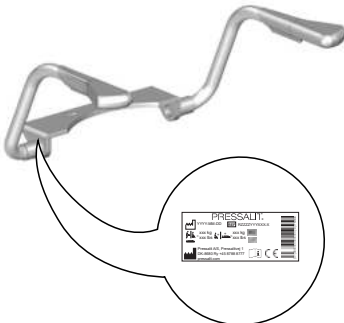
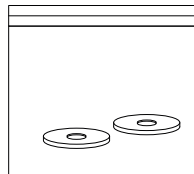
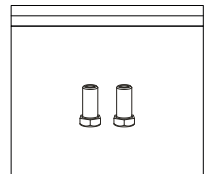
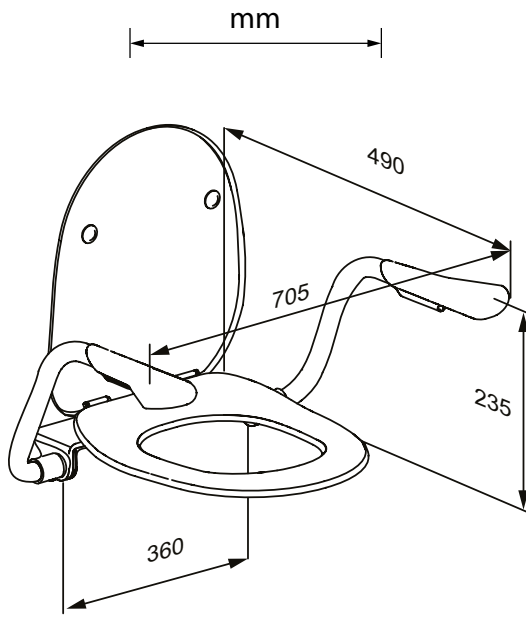
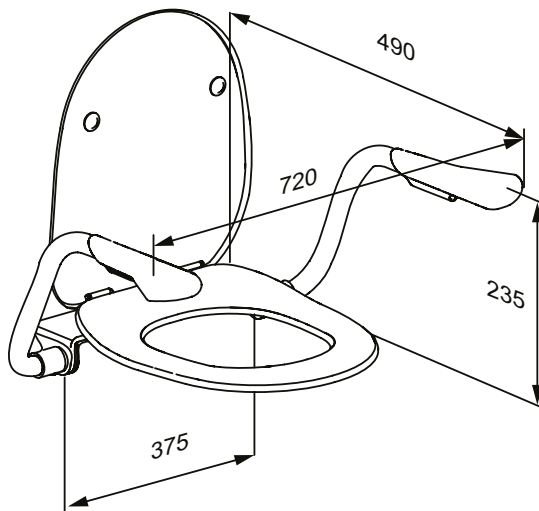
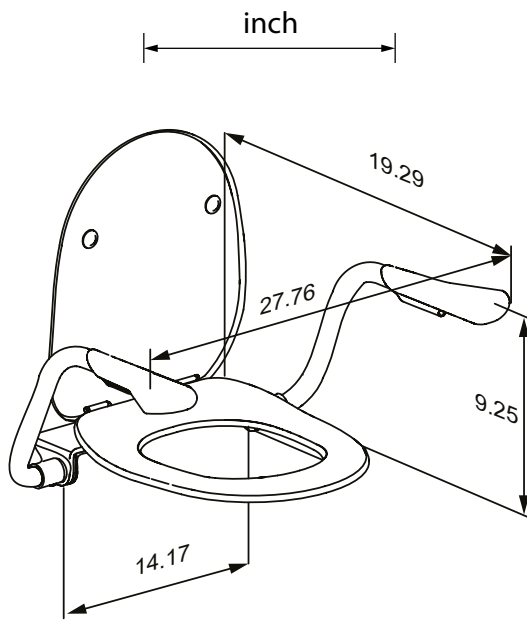
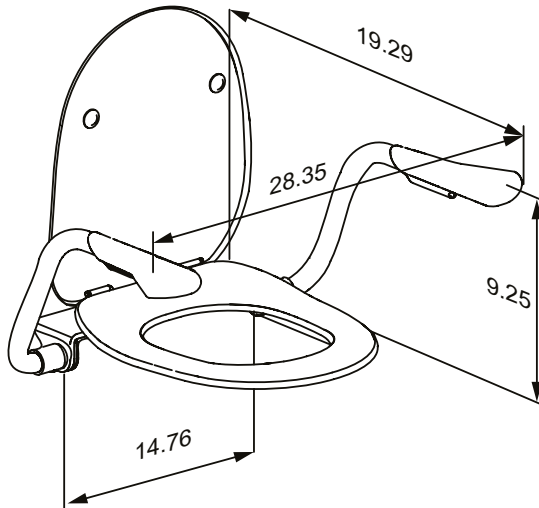
R1191 (R1190)

Content

Mounting instruction	6
Symbols used in this manual	7
General safety	8
Product label	9
Intended use	10
Intended user profile	10
Operating environment	10
Operating instructions	11
Maintenance	12
Maintenance inspection	12
Cleaning	13
Technical data	13
Disposal and recycling	14

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Spare parts

Product weight

Max. user weight

Article number/
versions

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

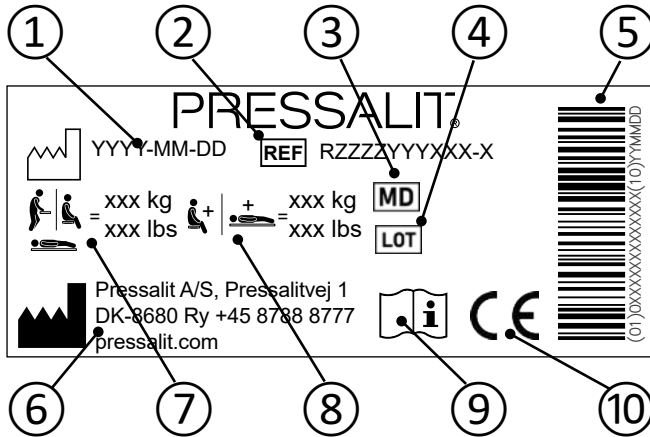
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

⑨ Consult manual for use

⑩ These products are CE-marked in accordance with relevant requirements in regulation (EU) 2017/745 (5 April 2017) on medical devices.

A declaration of conformity can be found at pressalit.com.

Intended use

The arm support is used as support for a person when he or she sits down or gets up from a toilet.

Intended user profile

The product can be used by everyone - adults, children, teenagers, and the elderly who need support when getting up or sitting down on the toilet.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room, in private homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years.

Life cycle test performed according to ISO 17966:2016.

The surface of the product may become hot and/or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

Fire hazard. Keep open flame away from the grey handles.

Any serious incident that has occurred in relation to the product should be reported to Pressalit and the health authorities.

A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

Operating instructions

The two arm supports can be folded up and down individually. Fold them both up when not in use.

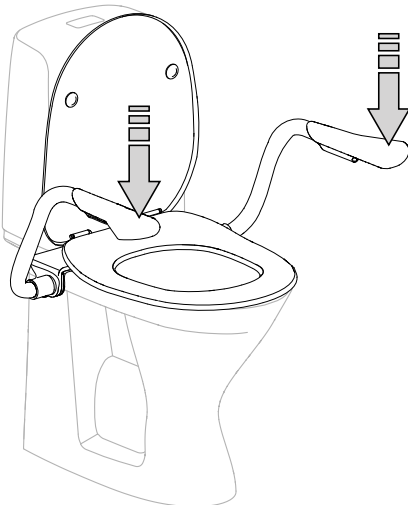
Take care to avoid injury to hands when in the vicinity of closing moveable parts.

en

max 70 kg
165 lbs

max 70 kg
165 lbs

140 kg
308 lbs

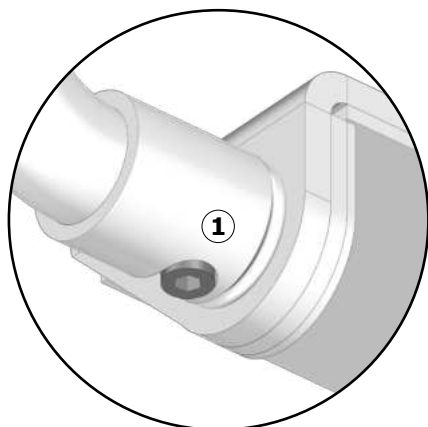
Maintenance

It is recommended that maintenance inspection is done once a year.

Maintenance inspection

Check the screws ① for wear.
Replace them if necessary.

Do not use hinges other than
the original ones.

CAUTION

- Do not use the product if it is broken or damaged.
- Do not repair items not described under maintenance.

NOTICE

If the product is installed in an environment with a high concentration of salt or chloride, the warranty period will be reduced from 3 years to 1 year. Salt and chloride have a corrosive effect on the product's components, which will result in a reduced lifespan.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.

The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

Technical data

Max user weight	140 kg /312 lbs	
Weight	R1170, R1190	7.8 kg / 17.2 lbs
	R1171, R1191	4.4 kg / 9.7 lbs
Materials	Steel, Polyurethane	

en

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Polyurethane	Like other plastics, polyurethane can be recycled in two primary ways: mechanical recycling, in which the material is reused in its polymer form, and chemical recycling that takes the material back to its various chemical constituents.

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