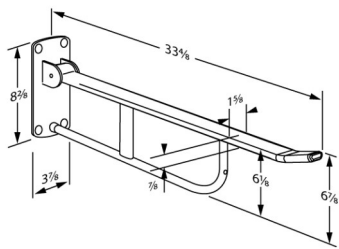




PRESSALIT®

Product Item number	VALUE II fold down grab bar, fixed height R1109000
Color	VVS:615891100 NRF:6191057 RSK:8487434 EAN:5708590367576 HMI: 123560 FDA: ILS/D135866 White
Product description	 VALUE II fold down grab bar, fixed, foldable, length 33.5". The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately. VALUE II fold down grab bar, fixed, foldable, length 33.5". The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Materials: Rilsan-coated steel, pressure-cast aluminium, plastic. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.
Adjustability Product capacity	Fixed mounting Maximum load: 265 lbs. When the fold down grab bar is loaded with 265 lbs, each mounting bolt is loaded with 320 lbs.
Materials Ergonomy Features	Basic material: Steel Surface material: Rilsan. Provides firm support when you are sitting, standing, or transferring to and from a toilet. Good, broad support surface for the hand and arm. The ergonomic form gives a firm and comfortable grip. The grab bar can fold up when not in use or to make more space.
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



PRESSALIT®

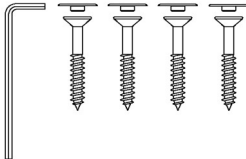
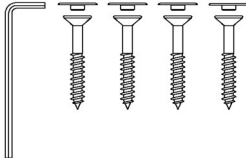
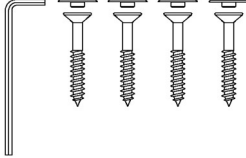
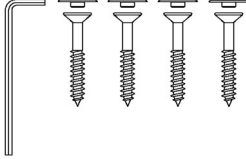
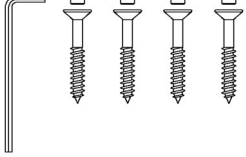
Product	VALUE II fold down grab bar, fixed height
Item number	R1109000
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean) NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.



PRESSALIT®

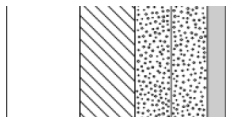
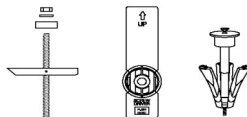
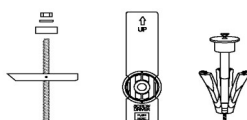
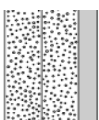
Product	VALUE II fold down grab bar, fixed height
Item number	R1109000

Mounting in different wall types

	Concrete >B25 V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Solid Brick >= Mz12 V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Aerated Concrete P4 V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8593 - 4 countersunk bolts Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	



PRESSALIT®

Product Item number	VALUE II fold down grab bar, fixed height R1109000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar. 
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.

pressalit.com

Pressalit A/S
 Pressalitvej 1
 DK-8680 Ry
 Tel.: +45 8788 8788
 Fax: +45 8788 8789
 E-mail: pressalit@pressalit.com

PRESSALIT®