




|   |   |
|---|---|
| <p><b>Product</b></p> <p><b>Item number</b></p>   | <p>SELECT TL3 toilet lifter, electrical, for floor outlet</p> <p>R9133112</p>   |
| <p><b>Color</b></p> <p><b>Product description</b></p>   | <p>EAN:5708590372686</p> <p>FDA:IKX/D319729</p> <p>Anthracite gray</p>  <p>SELECT toilet lifts are available in several models and colors for both wall outlet and floor outlet. They are available in electrically or manually height-adjustable versions, with or without side profiles and for standard toilet bowls and shower toilets. All developed with a focus on hygiene and easy cleaning.</p> <p>SELECT TL3 toilet lifter, for floor outlet.<br/>         Incl. cUPC-approved cistern with dual flushing system preset at 0.8/1.6 gallons (can be changed to 0.5/1.1 gallons).<br/>         Manually height adjustable 8" with control panel on fold down grab bar.<br/>         Toilet pan, toilet seat and fold down grab bars (required for height adjustment) to be ordered separately.<br/>         Materials: Stainless steel and PMMA/ABS.<br/>         Excl. wall mounting kit.</p> |
| <p><b>Adjustability</b></p> <p><b>Adjustment height</b></p> <p><b>Product capacity</b></p> <p><b>Speed up</b></p> <p><b>Noise level</b></p> | <p>Height adjustable</p> <p>7,88 inch</p> <p>Maximum user weight: 440 lbs.<br/>         Maximum load: 518 lbs. (user weight + accessories).<br/>         When the toilet lifter is loaded with 518 lbs, each wall fastener is loaded with 143 lbs.<br/> <i>The toilet lifter has been tested statically with 1320 lbs, which is a requirement according to the applied assistive products standards. Please note that most toilets compatible with this lifter have been tested up to 880 lbs.</i></p> <p>Speed up: 0,39 inch/s<br/>         Speed down: 0,39 inch/s</p> <p>48 dB</p>   |
| <p><b>Materials</b></p>   | <p>The toilet lifter is constructed of stainless steel, the cover fairing is made of PMMA/ABS plastics.</p>   |




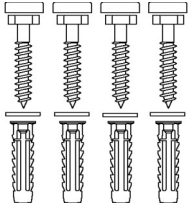
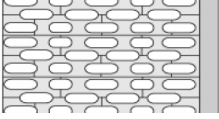
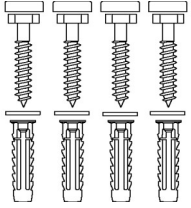
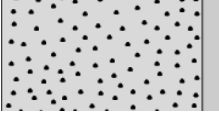
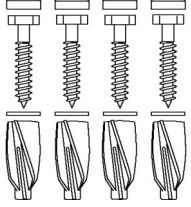
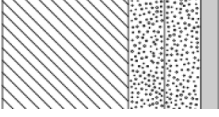
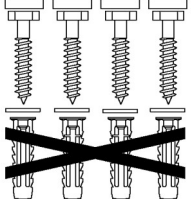


|   |  |
|---|--|
| <b>Product</b>                          | SELECT TL3 toilet lifter, electrical, for floor outlet   |
| <b>Item number</b>                      | R9133112   |
| <b>Mains connection (V)</b>             | 230  |
| <b>Power consumption (maximum load)</b> | Power consumption by maximum load: 70 W<br>Power consumption by standby: 2 W   |
| <b>Warranty</b>                         | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.  |
| <b>Cleaning</b>                         | <p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningsprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> |
| <b>WEEE</b>                             | <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.</p>  |





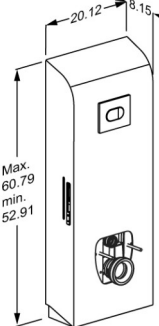



|                    |  |
|--------------------|--|
| <b>Product</b>     | SELECT TL3 toilet lifter, electrical, for floor outlet |
| <b>Item number</b> | R9133112   |

## Mounting in different wall types
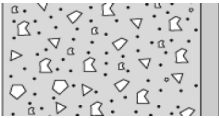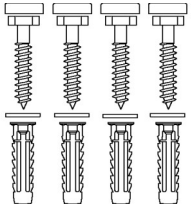
|   |  |   |
|---|--|---|
|    | <p><b>Concrete &gt;B25</b></p> <p>V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers.<br/>To be ordered separately.</p>  |    |
|   | <p><b>Solid Brick &gt;= Mz12</b></p> <p>V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers.<br/>To be ordered separately.</p>  |   |
|  | <p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers.<br/>To be ordered separately.</p>   |  |
|  | <p><b>Aerated Concrete P4</b></p> <p>V8655 - Mounting kit, 4 lag bolts Ø10 x 70, plugs GB14, with grey covers.<br/>To be ordered separately.</p>   |  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with white covers. NB! Plugs are not needed.<br/>To be ordered separately.</p> |  |

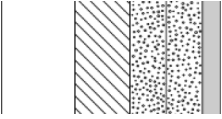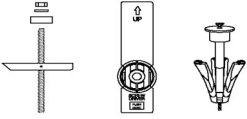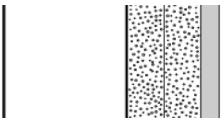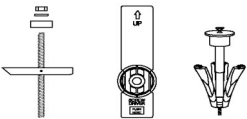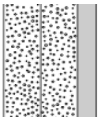

|   |  |
|---|--|
| <b>Product</b><br><b>Item number</b>  | SELECT TL3 toilet lifter, electrical, for floor outlet<br>R9133112   |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b><br>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar.<br>           |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b><br>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar.<br> |
|  | <b>Drywall (2 x 12,5 mm) water-resistant</b><br>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.<br>  |