

PRESSALIT.

Product	PLUS shower seat 310, manually height adjustable				
Item number	R7334000				
	VVS:777513200				
	NRF:6317089				
	EAN:5708590359342				
	HMI: 120479				
	FDA: INN/D135865				
Color	White				
Product description	PLUS shower seats are offered in a number of colours. They are available as both fixed height, electrically adjustable and manually adjustable, and with an option of back and armrests. They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.				
	PLUS shower seat 310, with back- and armrests. Manually height adjustable by 9.8", without the use of tools. Armrests are independently height adjustable. The seat and the armrests are foldable. Anthracite cover plate for vertical wall track. Materials: Comfortable polyurethane foam over a plywood core, fitted to an aluminium profile. The non-slip gripping surface of the armrests are made of soft SEBS incorporating a glass-reinforced nylon core. Excl. mounting kit.				
Adjustability	Height adjustable				
Adjustment height	9,85 inch				
Product capacity	Maximum user mass (seat): 440 lbs. Maximum load for armrests is 55 kg. When the seat is loaded with 440 lbs, each wall fastener is loaded with 37 lbs.				
	The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.				



PRESSALIT.

Product	PLUS shower seat 310, manually height adjustable			
Item number	R7334000			
Materials	The shower seat is made of polyurethane (PU) foam with a plywood core and this is covered with a moulded coated surface that is protected using a clear PU lacquer. The backrest is made of polyurethane (PU) foam protected using a clear PU lacquer The arms are primarily constructed of 3/8" anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core. The hinge arm is also made of soft SEBS. Internal components and fasteners are primarily made of stainless steel. The hinge barrel is capped with ABS			
Ergonomy	Ergonomically designed seat - contact surface are 12.2" x 14.4". Soft and slip-resistant material that has a warm feel. The oval shape of the armrests gives an eas and comfortable surface to lean on and push against.			
Features	The seat, backrest and armrests can be adjusted simultaneously thanks to a safe ar easy click system. The armrests can also be adjusted independently of the seat and backrest. The seat and armrests can fold up when not in use or to maximize floor space.			
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.			



PRESSALIT®

Product	PLUS shower seat 310, manually height adjustable R7334000			
Item number				
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.			
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.			
	DISINFECTANTS: Erisan OXY+ Tvättsprit 70% CaviCide surface disinfectant decontaminant cleaner Rodalon Desinfektionsspray Terralin protect (SCHÜLKE+) Dismonzon Plus (BODE Chemie GmbH)			
	NON-DISINFECTANTS: Wetrok Caledor (Wetrok) Petrosol Citro (Büfa)			
	Important: Polyurethane surfaces of PLUS shower seats and PLUS backrests contains an antibacterial surface, which will be compromised if cleaned with anything other than soap and water.			
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.			
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this			

solution, the parts should be wiped over with a damp cloth wrung out in plain water.



PRESSALIT.

Product				
Item number	r			

PLUS shower seat 310, manually height adjustable


R7334000


Mounting in different wall types

α.	٧.	0:[۲.,		0
1.0	Δ	۵.	0.	٠۵٠	
∴	Ω.	ΰ.	ß. 7	۲. [ا ا ک
0	\cdot). D·		· · · ·	
	å•.D	1:13			2

Concrete >B25


V8612 - Mounting kit, 10 lag bolts $\emptyset 8$ x 70, plugs SX10. To be ordered separately.

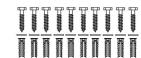


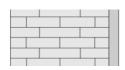

Solid Brick >= Mz12

V8612 - Mounting kit, 10 lag bolts $\emptyset 8$ x 70, plugs SX10. To be ordered separately.

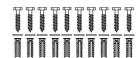

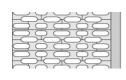

Solid Sand Lime Brick >= Hlz12

V8612 - Mounting kit, 10 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.







Aerated Concrete P4

V8662 - Mounting kit, 10 countersunk bolts M8 x 35, plugs FTP M 8.

To be ordered separately.

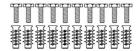


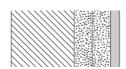

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8612 - Mounting kit, 10 lag bolts $\emptyset 8$ x 70, plugs SX10. NB! Plugs are not needed.

To be ordered separately.

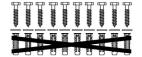



PRESSALIT®

Product Item number	PLUS shower seat 310, manually height adjustable R7334000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant Wall fasteners (10 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	

pressalit.com