


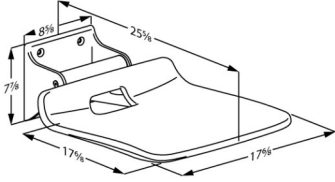

<p>Product Item number</p>	<p>VALUE IV shower seat, fixed height R1620000112</p>
<p>Color Product description</p>	<p>VVS:777525312 NRF:6314097 RSK:8565069 EAN:5708590377711 HMI: 124483 DB: 2296038 FDA:INN/D135865 Anthracite gray</p>  <p>VALUE IV folding shower seat and backrest create a simple solution for users who need to sit down while showering.</p> <p>VALUE IV shower seat. Fixed height. The seat is foldable. Materials: Diecast aluminium, Rilsan-coated stainless steel and vacuum-moulded polyethylene (PE). Incl. mounting bolts for brick wall. Plugs depend on wall type and must be ordered separately.</p>
<p>Adjustability Product capacity</p>	<p>Fixed mounting</p> <p>Maximum user mass: 298 lbs. When the seat is loaded with 298 lbs, each wall fastener is loaded with 276 lbs.</p> <p><i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i></p>
<p>Materials</p>	<p>The wall profile is made of diecast aluminium, and the frame is made of Rilsan-coated stainless steel. Rilsan provides good protection against corrosion and is comfortable to touch and easy to clean. The seat is made of vacuum-moulded polyethylene (PE).</p>
<p>Warranty</p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



Product	VALUE IV shower seat, fixed height
Item number	R1620000112
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kilto Clean)</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>