


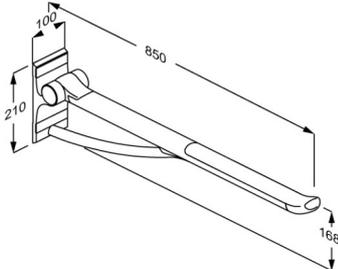

|  |  |
|--|--|
| <p><b>Product</b><br/><b>Item number</b></p>             | <p>PLUS support arm with integrated counter-balance, 850 mm<br/>R360085112</p>   |
| <p><b>Colour</b><br/><br/><b>Product description</b></p> | <p>VVS:615871212<br/>NRF:6191075<br/>RSK:8487439<br/>EAN:5708590367958<br/>HMI: 123553<br/>DB: 2509942<br/>FDA: ILS/D135866</p> <p>Anthracite grey</p>  <p>All PLUS support arms come with an integrated counter-balance, which ensures a controlled movement between horizontal and vertical positions. Suitable for use with toilet pans projecting 700 mm or floor standing toilets. <i>Replaces discontinued models R3085 and RF085.</i></p> <p>PLUS support arm, counter-balanced. Projection 850 mm, ergonomically shaped to provide a clear sense of where the support arm ends. For fixed mounting. Foldable. Materials: The arms are constructed of 10-micrometre anodized aluminium, with a non-slip gripping surface made of soft SEBS incorporating a glass-reinforced nylon core<br/>Excl. mounting kit.</p> |
| <p><b>Adjustability</b><br/><b>Product capacity</b></p>  | <p>Fixed mounting</p> <p>Maximum load: 150 kg.<br/>When the support arm is loaded with 150 kg, each wall fastener is loaded with 196 kg.</p> <p><i>PLUS support arms have been tested statically with 700 kg (close to the wall), 425 kg (in the middle) and 225 kg (the outermost point).</i></p>   |
| <p><b>Materials</b></p>                                  | <p>The arms are primarily constructed in 10-micrometre anodized aluminium, with a non-slip gripping surface at the end made of soft SEBS and incorporating a glass-reinforced nylon core.<br/>The hinge arm is also made of soft SEBS.<br/>Internal components and fasteners are primarily made of stainless steel.<br/>The hinge barrel is capped with ABS polymer.</p>   |



# PRESSALIT®

|                                      |   |
|--------------------------------------|---|
| <b>Product</b><br><b>Item number</b> | PLUS support arm with integrated counter-balance, 850 mm<br>R360085112  |
| <b>Ergonomy</b>                      | <p>The oval shape gives an easy and comfortable surface to lean and push against.</p> <p>The end of the support arm is enlarged so that it is easier to pull forward from a sitting position and also provides a clear sense of where the support arm ends.</p> <p>Soft and slip-resistant material that has a warm feel.</p> |
| <b>Features</b>                      | <p>Folds against the wall.</p> <p>The underside is shaped for the mounting of various accessories.</p>  |
| <b>Warranty</b>                      | Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.   |



|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS support arm with integrated counter-balance, 850 mm   |
| <b>Item number</b> | R360085112   |
| <b>Cleaning</b>    | <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b><br/>Clinimax Difficil-S<br/>Erisan OXY+<br/>Sprit (husholdningsprit)<br/>Suma Bac D10<br/>Virkon<br/>Rodalon Desinfektionsspray<br/>Lysoformin Spezial<br/>CaviCide<br/>Sufanios Premium<br/>Tvättsprit 70%<br/>Oxyplus-acid 2% (Diversey)<br/>WipeClean UniversalCleaning<br/>Chlorine Disinfection Wipes</p> <p><b>NON-DISINFECTANTS:</b><br/>Suma Star free D1<br/>Taski 100<br/>Taski 200<br/>Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b><br/>Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p> |



PRESSALIT®

|                                      |  |
|--------------------------------------|--|
| <b>Product</b><br><b>Item number</b> | PLUS support arm with integrated counter-balance, 850 mm<br>R360085112   |
|                                      | <p><b>Intended use</b><br/>The support arm is used as a support for sitting down or getting up from a toilet, or for sideways transfer from a wheelchair.</p> <p><b>Operating environment</b><br/>For indoor use in a wet environment, such as a bathroom or a washroom, in private homes, nursing homes and hospitals.<br/>For institutional use, the expected life time is 10 years.<br/><i>Life cycle test performed according to ISO 17966:2016.</i></p> <p>For maintenance, please refer to user guide, which is enclosed with the product, and which can also be found on the website under "Downloads".</p> |








|                    |  |
|--------------------|--|
| <b>Product</b>     | PLUS support arm with integrated counter-balance, 850 mm |
| <b>Item number</b> | R360085112   |

## Installation in different wall types

|  |  |  |
|--|--|--|
|  | <p><b>Concrete &gt;B25</b><br/> V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10.<br/> To be ordered separately.</p>   |  |
|  | <p><b>Solid Brick &gt;= Mz12</b><br/> V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10.<br/> To be ordered separately.</p>   |  |
|  | <p><b>Solid Sand Lime Brick &gt;= Hlz12</b><br/> V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10.<br/> To be ordered separately.</p>  |  |
|  | <p><b>Aerated Concrete P4</b><br/> V8654 - Mounting kit, 4 countersunk bolts M8 x 35, plugs FTP M 8.<br/> To be ordered separately.</p>  |  |
|  | <p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b><br/> V8604 - Mounting kit, 4 countersunk screws Ø8 x 70, plugs SX10. NB! Plugs are not needed.<br/> To be ordered separately.</p> |  |

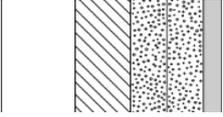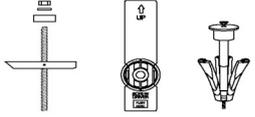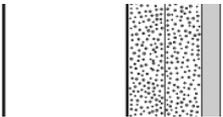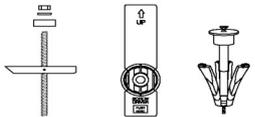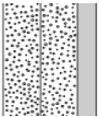

|   |  |   |
|---|--|---|
| <b>Product</b><br><b>Item number</b>  | PLUS support arm with integrated counter-balance, 850 mm<br>R360085112   |   |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b><br>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.       |    |
|    | <b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b><br>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. |    |
|  | <b>Drywall (2 x 12,5 mm) water-resistant</b><br>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.   |  |