


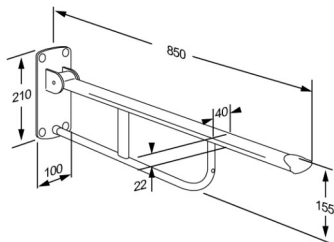

PRESSALIT®

Product Item number	VALUE support arm, fixed height R1100000
Colour Product description	VVS:615722300 NRF:6191642 RSK:7972410 EAN:5708590176406 HMI: 25411 NBS:N13:436 DB: 2076622 FDA:ILS/D135866 White  VALUE support arm, fixed, foldable, projection 850 mm. The ergonomic design gives a secure and comfortable grip as well as a good sense of where the support arm ends. Materials: Rilsan-coated steel, pressure-cast aluminium, plastic. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.
Adjustability Product capacity	Fixed mounting Maximum load: 120 kg. When the support arm is loaded with 120 kg, each wall fastener is loaded with 145 kg.
Materials Ergonomy Features	The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA). Provides firm support when you are sitting, standing, or transferring to and from a toilet. Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip. The support arm can fold up when not in use or to make more space.



PRESSALIT®






Product	VALUE support arm, fixed height
Item number	R1100000
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kiilto Clean) NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.







PRESSALIT®

Product	VALUE support arm, fixed height
Item number	R1100000

Installation in different wall types

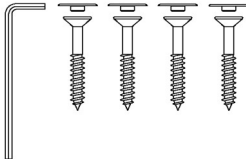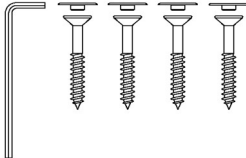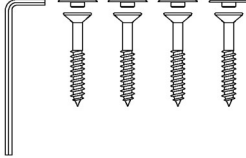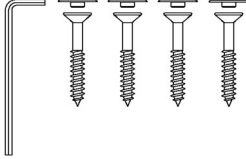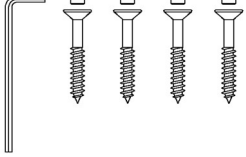	Concrete >B25 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Solid Brick >= Mz12 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Aerated Concrete P4 V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8593 - 4 countersunk screws Ø10 x 80, with white screw covers are included. Plugs to be ordered separately.	

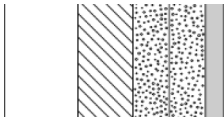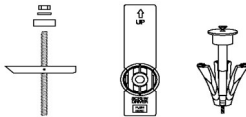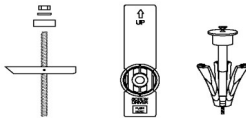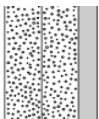

Product Item number	VALUE support arm, fixed height R1100000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls. 

Bolts for wall fixing are included. It is the responsibility of the installer to ensure that suitable plugs are used. Please see mounting instruction.