R9013 - R9023
R9014 - R9024

R1081 - R1085
R372370 - R372385
R372470 - R372485
R1181 - R1185

SELECT TL3 & TL4

Toilet lifter – Operation and maintenance manual	en
Toiletløfter – Brugs- og vedligeholdelsesvejledning	da
WC-Lifter – Gebrauchs- und Pflegeanleitung	de
Système de réglage en hauteur de wc – Manuel d'utilisation et d'entretien	fr
Hoog-laag-toiletsysteem – Gebruiks- en onderhoudshandleiding	nl
Upphängning till toalett – Bruks- och underhållsanvisning	sv
Oppheng til toalett – Bruks- og vedlikeholdsveiledning	no
Elevador del inodoro - Instrucciones de uso y mantenimiento	es
Podnośnik toalet – Instrukcja obsługi i konserwacji	pl
座便器升降装置 - 操作和维护手册	中文

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Pressalit's Environmental and Quality Management Systems are certified by LRQA

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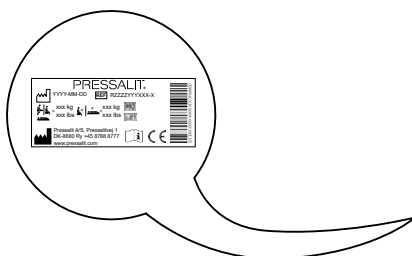
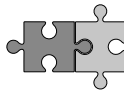
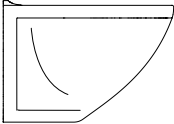
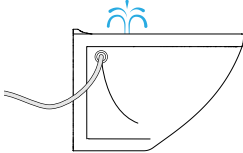
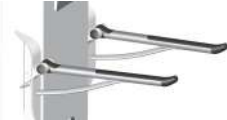
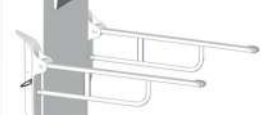
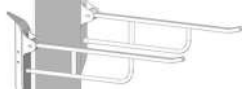
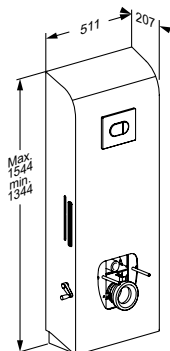
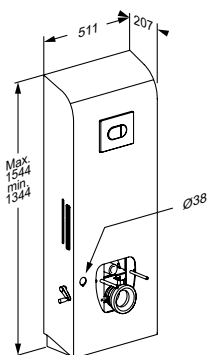
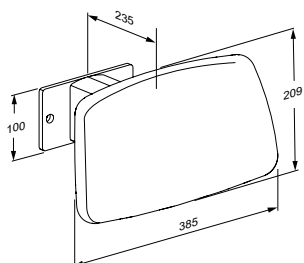
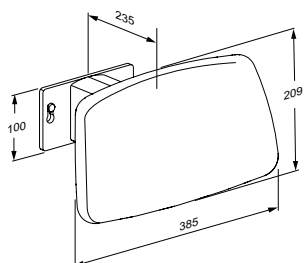
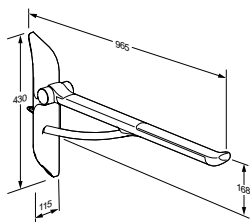
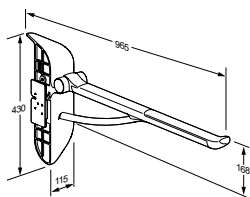
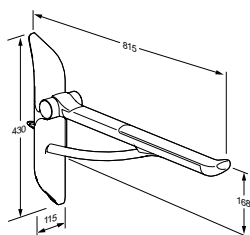
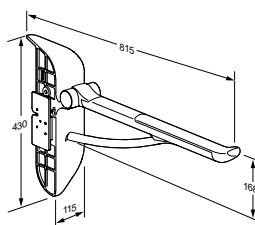
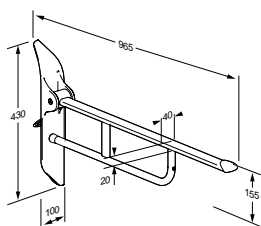
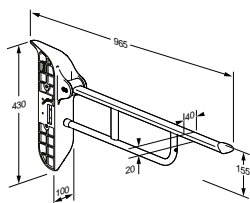
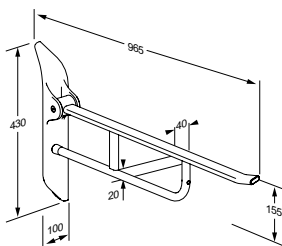
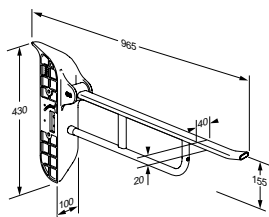
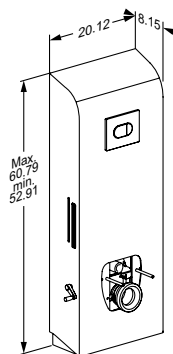
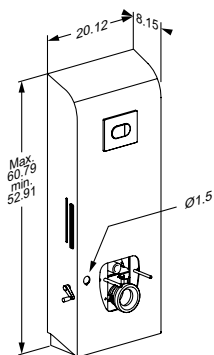
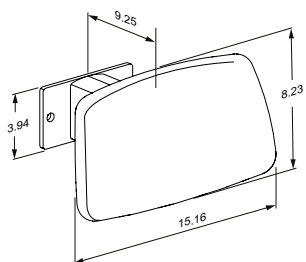
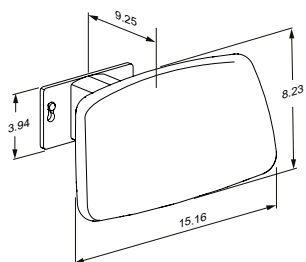
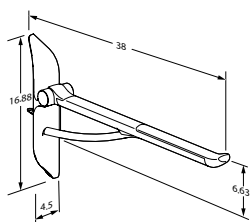
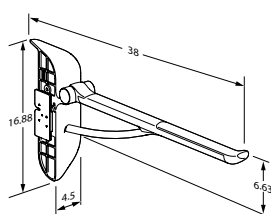
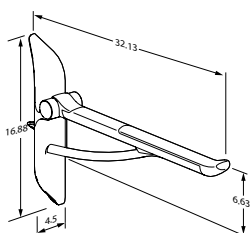
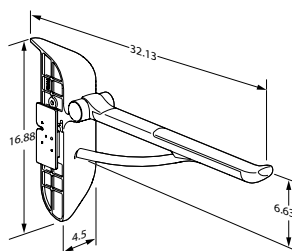
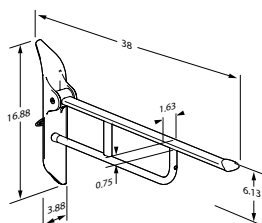
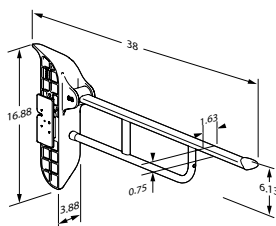
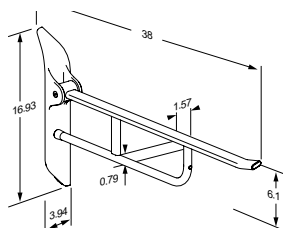
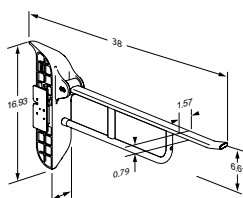
R1181

Content

Mounting instruction	9
Symbols used in this manual	10
General safety	11
Intended use	12
Intended operator profile	12
Operating environment	12
Contraindications for use	12
Operating instructions	13
Support arms	14
Cleaning	15
Maintenance	16
Maintenance inspection	16
Product label	18
Technical data	19
Trouble-shooting	20
Disposal and recycling	21

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

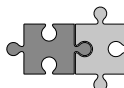
Hand injury warning

Head injury warning

Feet injury warning

Compatible products

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

This product is intended for optimum ergonomics at a toilet, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

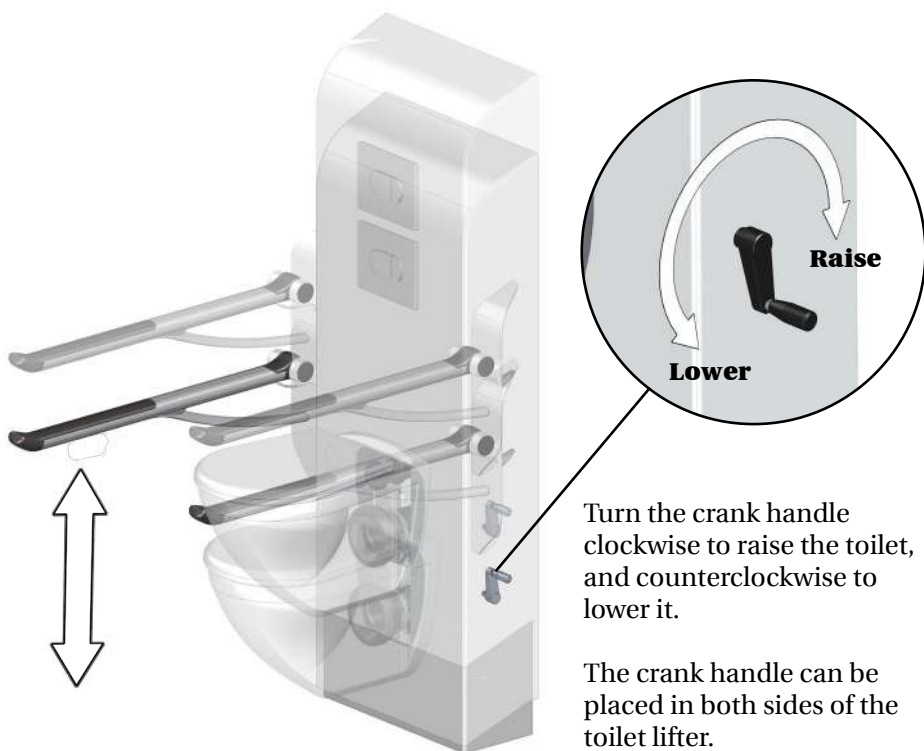
For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

Contraindications for use

People who are not capable of keeping their balance in a seated position are not recommended to use this toilet lifter.

Operating instructions

The toilet lifter can be adjusted 200 mm up and down.

Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

CAUTION

Never place objects underneath the toilet.

Support arms

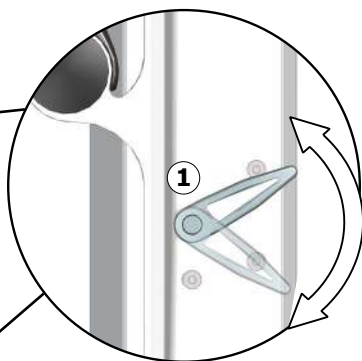
The support arms can be folded up and down. Fold them up when not in use.

For support arms R3723 and R3724:
You can hear a click when the support arm is correctly folded up.

Take care to avoid injury to hands when in the vicinity of closing moveable parts.

The support arms can be adjusted 150 mm in height.

Fold up the arm.

Press and turn the height adjustment locking handle ①.

Move the wing ② with the support arm up or down.

Tighten the locking handle again. It will slide back into place by itself.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

en

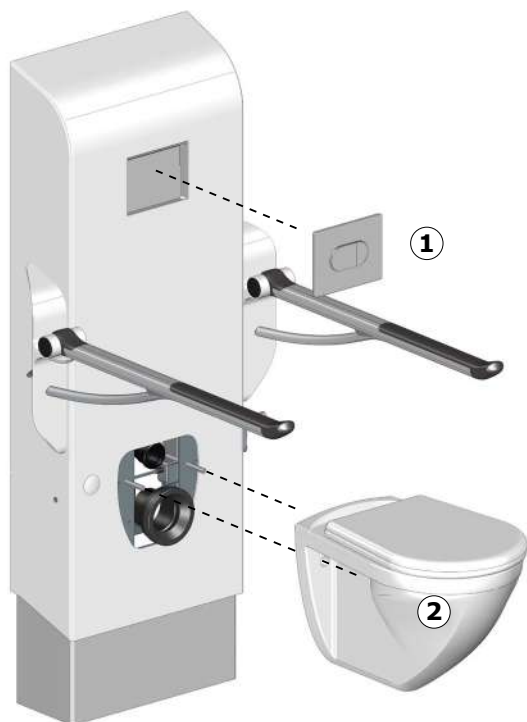
Maintenance

Expected service life of the toilet lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a monthly basis to adjust the product to its maximum positions up and down to lubricate the internal parts.

It is recommended that maintenance inspection is done once a year.

Maintenance inspection


Move the toilet to its highest position.

Remove the control panel ①.
Remove the toilet ②.

Unscrew the locking handles for the support arms ①.

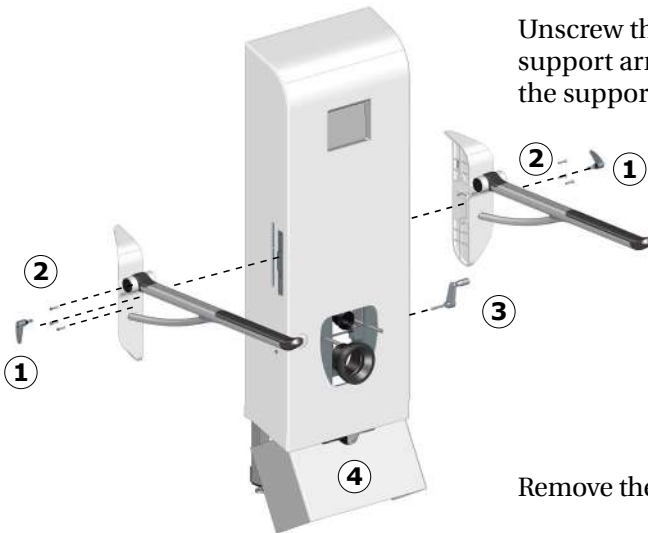Unscrew the screws for the support arms ② and remove both the support arms.

Remove the crank handle ③.

Remove the lower cover ④. It is a click-fit.

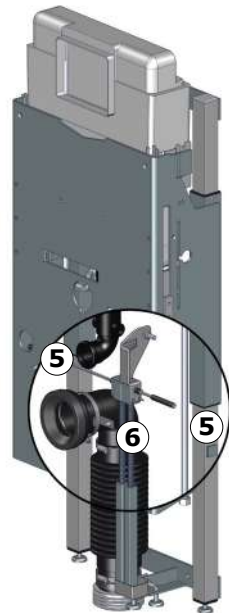
Remove the front cover.

Lubricate the inner square tubes ⑤ and the hoisting spindle mechanism ⑥.


Reattach the front and lower front cover.

Mount the support arms and the toilet.

Take the toilet to its lowest position, and then run it to the top position again to distribute the lubricant.

Product label

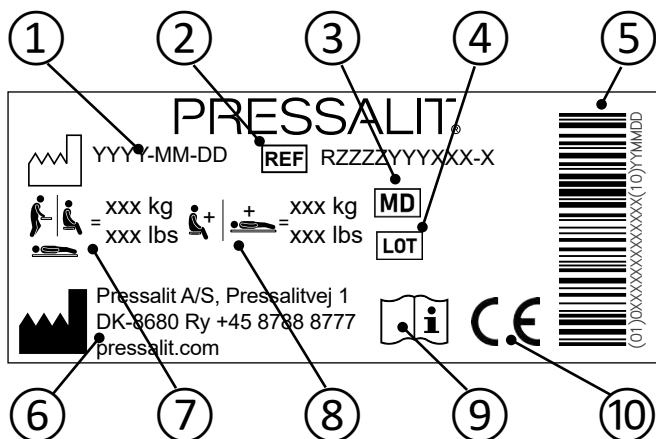- ① Production date
- ② Article number
- ③ Medical device
- ④ Batch code
- ⑤ GS1-128 barcode
- ⑥ Name and address of manufacturer
- ⑦ Maximum user weight
- ⑧ Max load
- ⑨ Consult manual for use

These products are CE-marked in accordance with relevant requirements in:


- ⑩
 - Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
- A declaration of conformity can be found at pressalit.com

Technical data

Maximum load	
Toilet lifter	Maximum user mass: 200 kg / 440 lbs Maximum load: 235 kg / 518 lbs
Support arms	Maximum load: R372x: 150 kg / 330 lbs R1xxx: 120 kg / 264 lbs
R8273, R9013, R9014	52,2 kg / 115 lbs
R8083, R9023, R9024	52 kg / 114.6 lbs
R372385, R372485	5,24 kg / 11.5 lbs
R372370, R372470	4,9 kg / 10.8 lbs
R1081, R1085	3,15 kg / 6.9 lbs
R1181, R1185	3,15 kg / 6.9 lbs
R6408, R6409	2,78 kg / 6,12 lbs

Trouble-shooting

Symptom	Cause	Action
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".
	The inner square tubes might be slightly skewed..	Contact Pressalit A/S or your local dealer.
Height adjustment of support arms is not possible.	The screws ① holding the support arm in place are tightened too hard.	Loosen the screws until height adjustment is possible. Check mounting instruction.
The product does not move down.	The bottom cover may have shifted under the lifter due to incorrect mounting of the product, blocking the downward movement.	Check that the bottom cover and all related parts are correctly positioned. If necessary, refit or adjust them to allow proper downward movement.

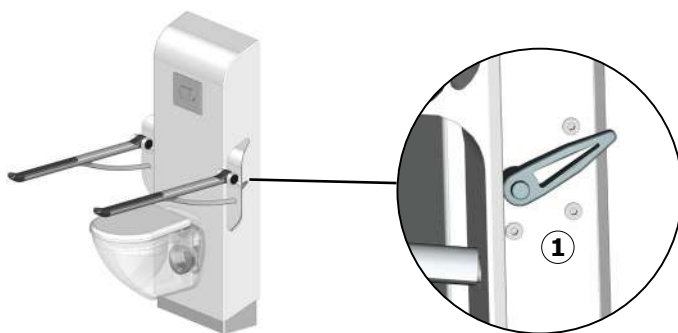Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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