



<p><b>Product</b> <b>Item number</b></p>	<p>PLUS wall track, grooved, 8.3". For left hand operated fold down grab bars R9861112</p>
<p><b>Color</b>   <b>Product description</b></p>	<p>VVS:615842132 EAN:5708590352275</p> <div data-bbox="1034 607 1374 719" data-label="Image"> </div> <p>Anthracite gray</p> <p>PLUS wall track, grooved, for vertical mounting, 8.3". For height adjustable, left-hand operated fold down grab bars. Below a selection of products for fitting to the wall track. The cover plate hides bolts and wires and facilitates cleaning. Excl. mounting kit. See Technical datasheet (Downloads) for information on mounting kits suitable for various wall types.</p>
<p><b>Adjustability</b></p>	<p>Fixed mounting</p>
<p><b>Materials</b>  <b>Ergonomy</b>  <b>Features</b></p>	<p>The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.</p> <p>The vertical track enables height adjustment of the different products. If combined with the horizontal track, the available space in the bathroom can be utilized optimally. In environments with multiple users, the vertical track makes it possible to adjust the different products to individual user heights.</p> <p>The front cover plate conceals mounting bolts and any cables.</p>
<p><b>Warranty</b></p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



<b>Product</b>	PLUS wall track, grooved, 8.3". For left hand operated fold down grab bars
<b>Item number</b>	R9861112
<b>Cleaning</b>	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p><b>DECALCIFICATION OF ALUMINUM PARTS:</b> Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>









<b>Product</b>	PLUS wall track, grooved, 8.3". For left hand operated fold down grab bars
<b>Item number</b>	R9861112

## Mounting in different wall types

	<p><b>Concrete &gt;B25</b>  V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10.  To be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b>  V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10.  To be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b>  V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10.  To be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b>  V8653 - Mounting kit, 4 bolts M8 x 35, plugs FTP M 8.  To be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b>  V8603 - Mounting kit, 4 lag bolts Ø8 x 70, plugs SX10. NB!  Plugs are not needed.  To be ordered separately.</p>	

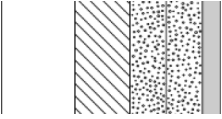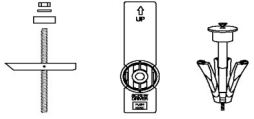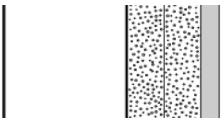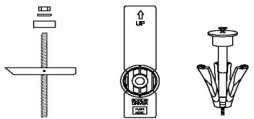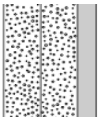

<b>Product</b> <b>Item number</b>	PLUS wall track, grooved, 8.3". For left hand operated fold down grab bars R9861112	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b> Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</b> Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.