R9113 - R9123
R9114 - R9124
R9133 - R9143
R9134 - R9144

R1081 - R1083
R372470 - R372485
R372570 - R372585
R1181 - R1183

SELECT TL3 & TL4

Toilet lifter – Operation and maintenance manual **en**

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Pressalit's Environmental and Quality Management Systems are certified by LRQA

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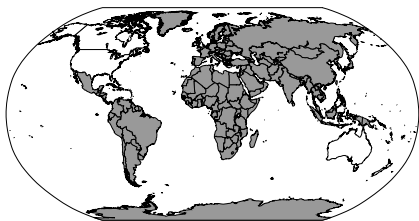
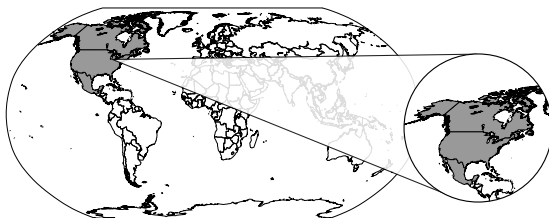
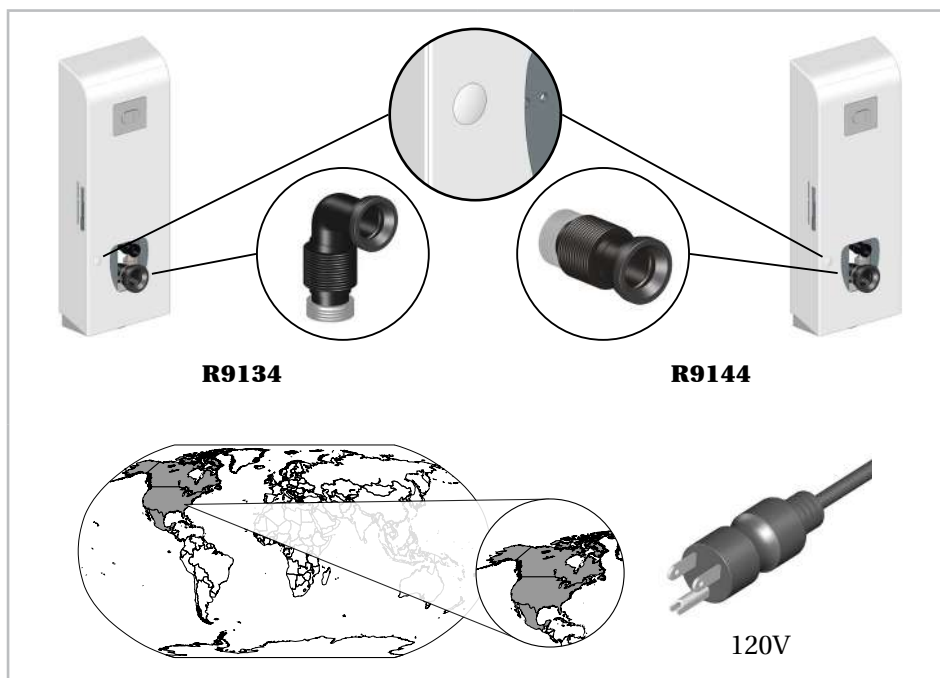
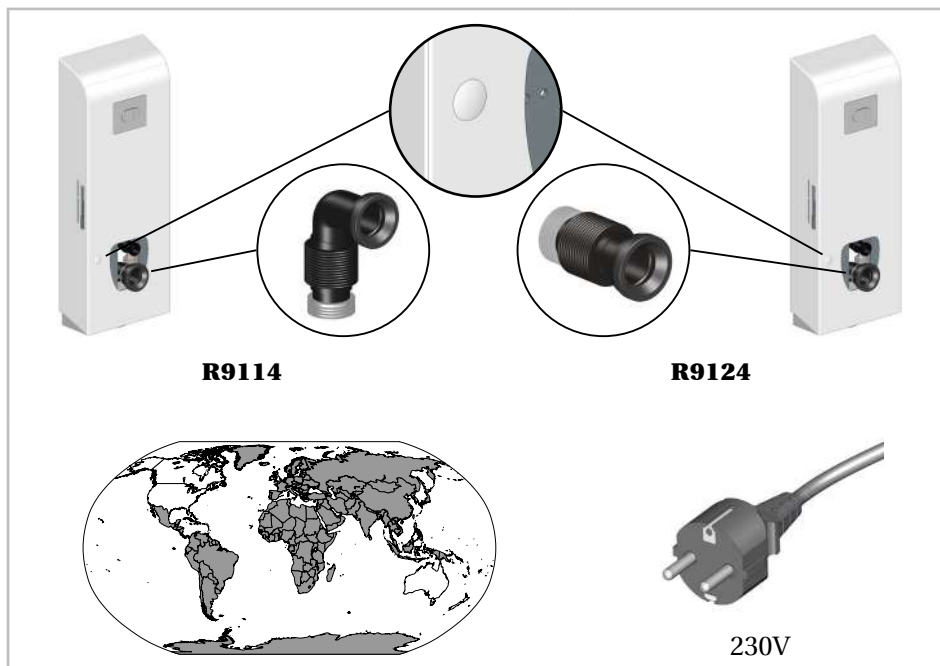
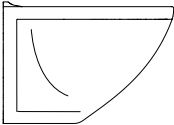
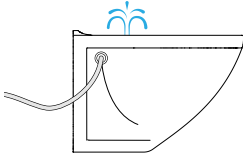
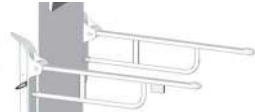
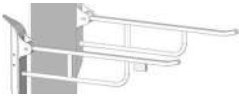
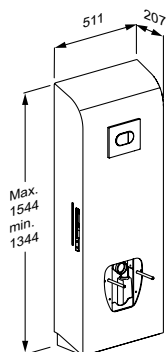
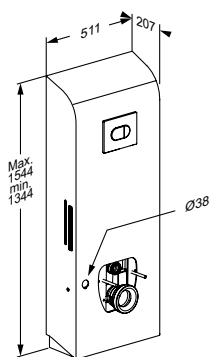
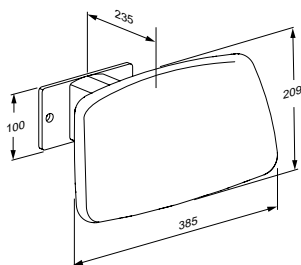
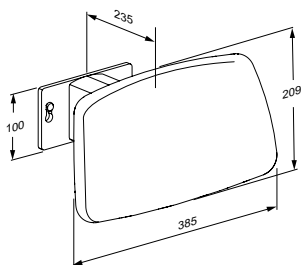
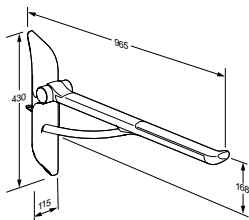
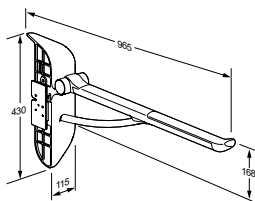
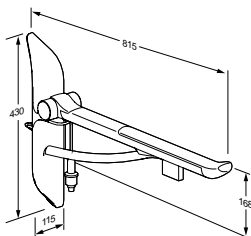
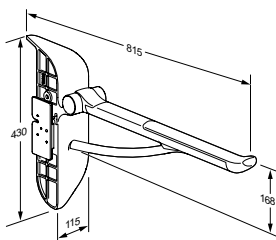
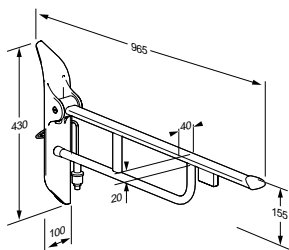
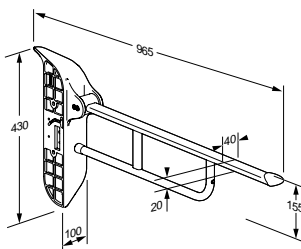
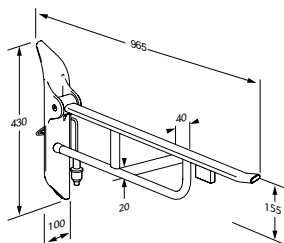
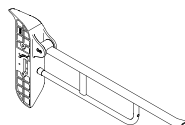
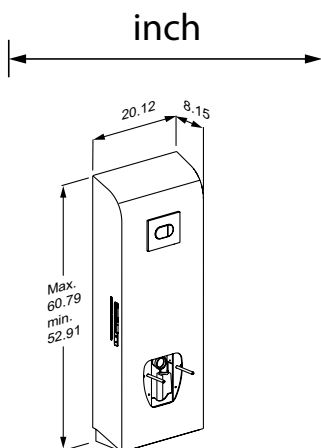
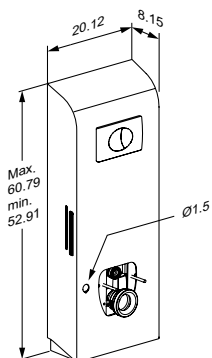
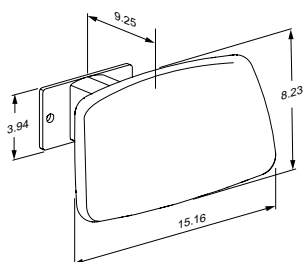
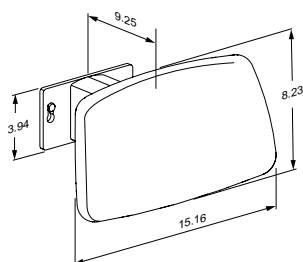
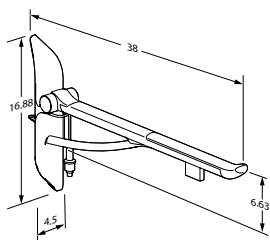
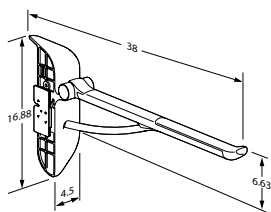
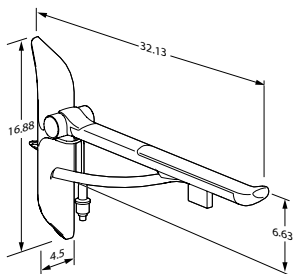
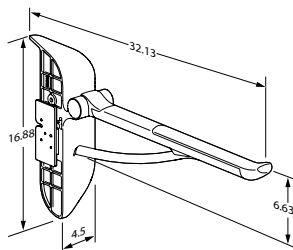
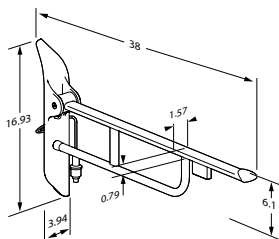
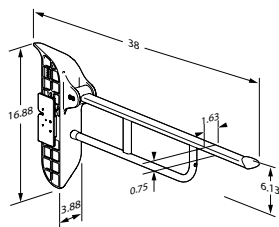
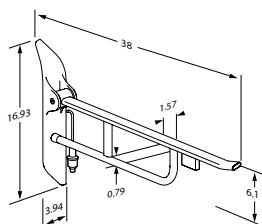
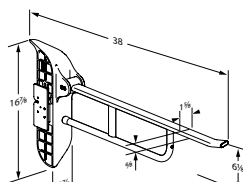
R1181

Content

Mounting instruction	10
Symbols used in this manual	11
General safety	12
Intended use	13
Intended operator profile	13
Operating environment	13
Contraindications for use	13
Operating instructions	14
Support arms	15
Cleaning	16
Maintenance	17
Maintenance inspection	17
Product label	21
Technical data	23
Trouble-shooting	24
Disposal and recycling	25

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

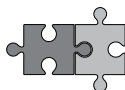
Head injury warning

Feet injury warning

Electricity or potentially dangerous voltage.

Compatible products

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use

This product is intended for optimum ergonomics at a toilet, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

Contraindications for use

People who are not capable of keeping their balance in a seated position are not recommended to use this toilet lifter.

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

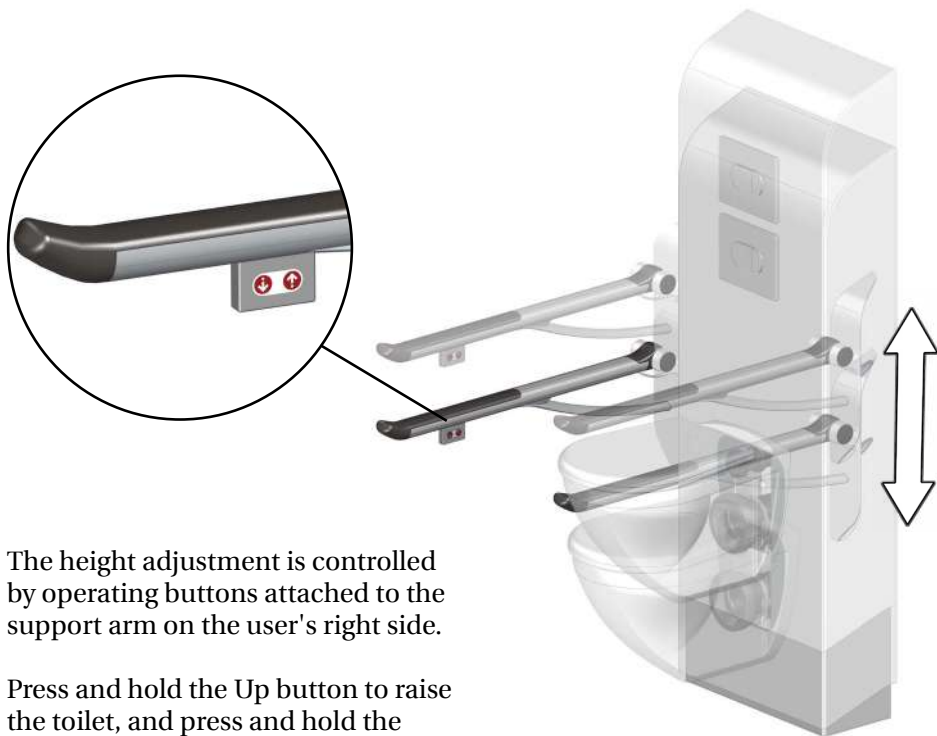
en

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Operating instructions

The toilet lifter can be adjusted 200 mm up and down.

The height adjustment is controlled by operating buttons attached to the support arm on the user's right side.

Press and hold the Up button to raise the toilet, and press and hold the Down button to lower it. When the button is released, the motion stops.

Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

CAUTION

Never place objects underneath the toilet.

Support arms

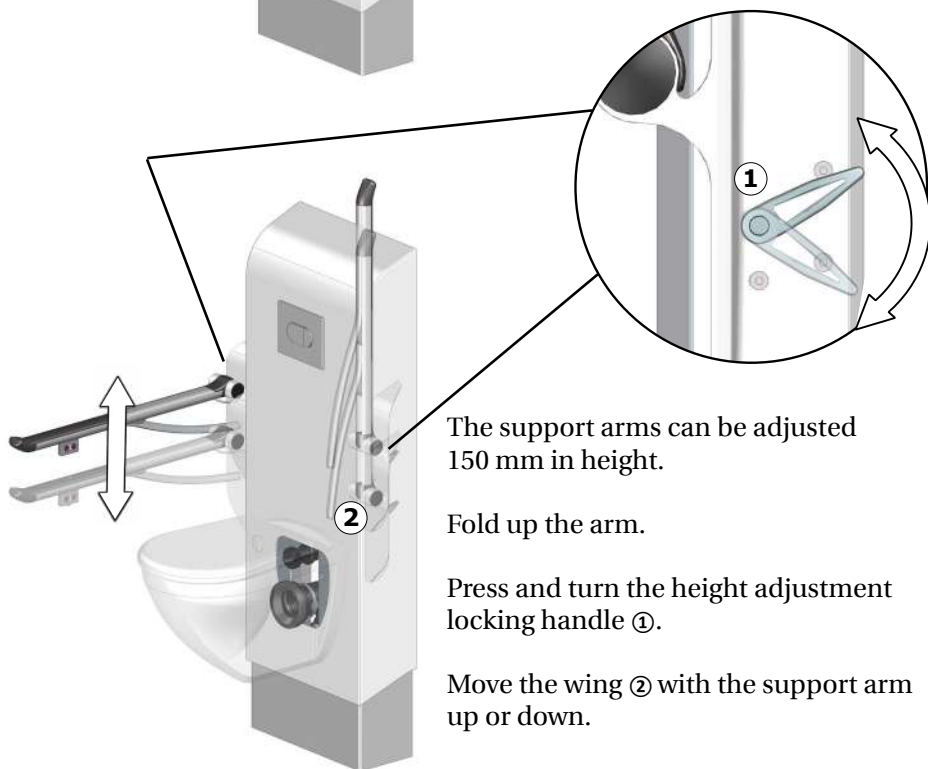
The support arms can be folded up and down. Fold them up when not in use.

For support arms R3724 and R3725:
You can hear a click when the support arm is correctly folded up.

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

The support arms can be adjusted 150 mm in height.

Fold up the arm.

Press and turn the height adjustment locking handle ①.

Move the wing ② with the support arm up or down.

Tighten the locking handle again. It will slide back into place by itself.

en

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used, provided that the disinfectant is suitable for this product and that the disinfectant manufacturer's safety instructions are followed.

Maintenance

Expected service life of the toilet lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that maintenance inspection is done once a year.

Maintenance inspection

Move the toilet to its highest position. Ensure it is fully raised by pressing and holding the up button for 20 seconds.

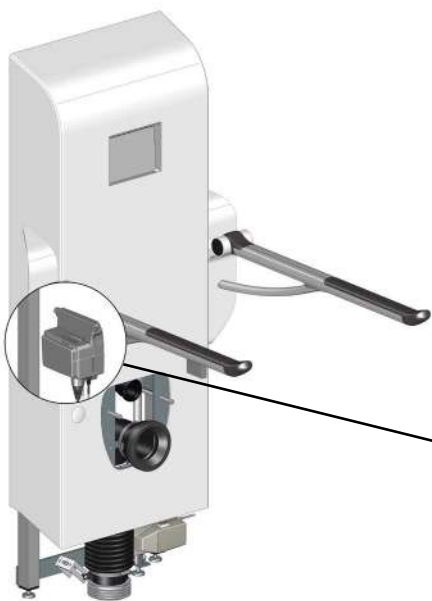
**Disconnect mains
before removing
the covers.**

en


Remove the flush panel ①.
Remove the toilet ②.


Remove the lower cover ①. It is a click-fit.

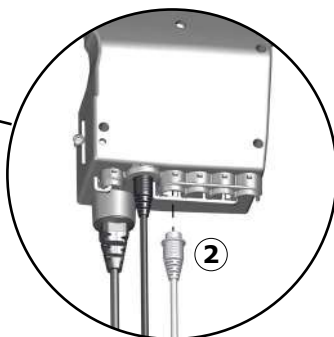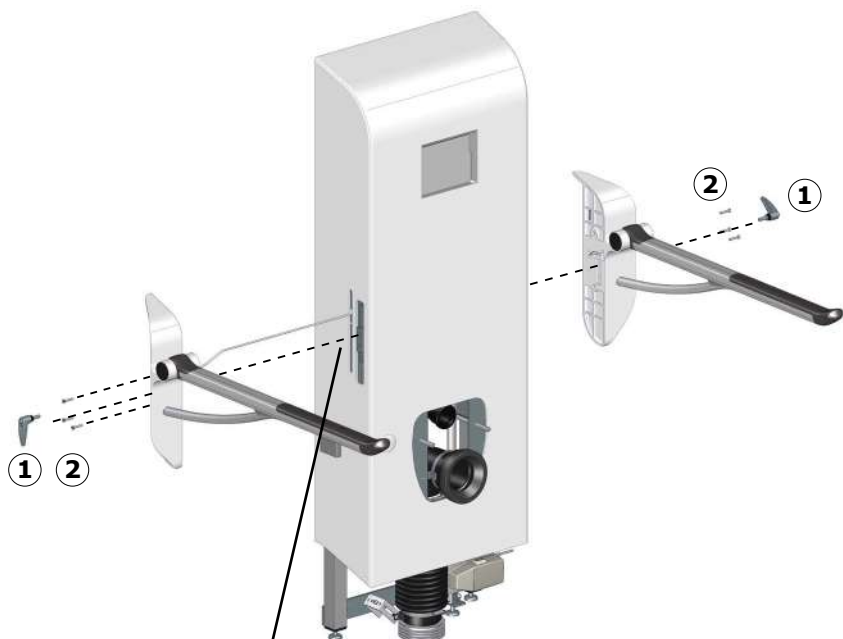Reach up under the front cover to unplug the wire ② from the control buttons to the control box inside the cabinet.


Unscrew the locking handles for the support arms ①.

Unscrew the screws ② for the support arms and remove both the support arms.

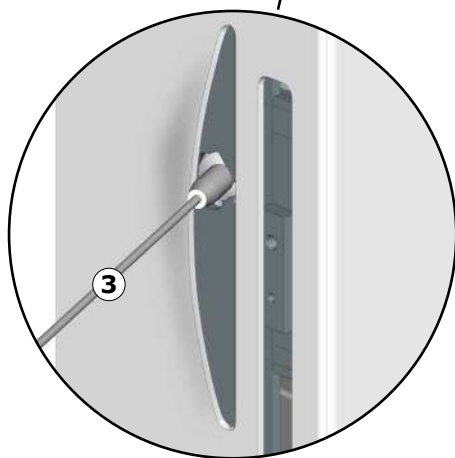
Open the slot in the cover slightly so that the cable ③ connecting the support arm to the control box can be taken out.

en

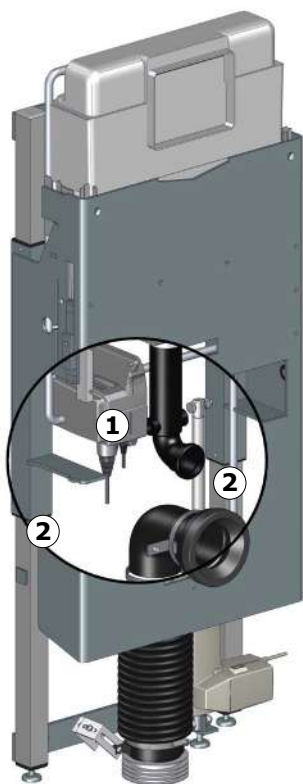Remove the front cover.

Check all electrical connections ①.

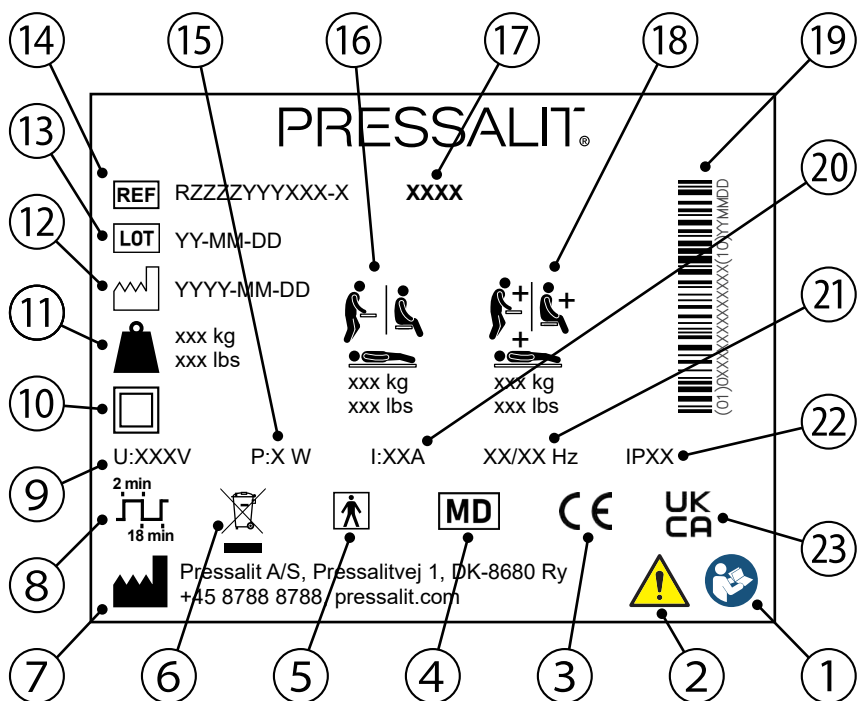
Lubricate the inner square tubes ②.

Reattach the front and lower front cover. Mount the support arms, the control box connector and the toilet.

Reconnect mains.

Take the toilet to its lowest position, and then run it to the top position again to distribute the lubricant.

Product label

- 1 Read the manual before use
- 2 Consult the manual for important cautionary information such as warnings and precautions
- 3 These products are CE-marked in accordance with relevant requirements in applicable legislation.
A declaration of conformity can be found at pressalit.com
- 4 Medical device
- 5 Type BF applied part. Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock
- 6 Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑦	Name and address of manufacturer
⑧	Duty cycle
⑨	Mains voltage
⑩	Class II Product, double insulated
⑪	Product weight
⑫	Production date
⑬	Batch code
⑭	Article number
⑮	Maximum power consumption
⑯	Maximum user weight
⑰	Model name
⑱	Max load
⑲	GS1-128 barcode
⑳	Electric Current
㉑	Frequency
㉒	IP class
㉓	UK Conformity Assessed. UK responsible representative: Pressalit Ltd., 100, Longwater Avenue, Green Park, Reading, Berkshire, RG2 6GP, United Kingdom.


Technical data

Maximum load	
Toilet lifter	Maximum user mass: 200 kg / 440 lbs Maximum load: 235 kg / 518 lbs
Support arms	Maximum load: R372x: 150 kg / 330 lbs R1xxx: 120 kg / 264 lbs
Product weight	
R9113, R9123, R9133, R9143 R9114, R9124, R9134, R9144	55,9 kg / 123.2 lbs
R372470	4,9 kg / 10.8 lbs
R372485	5,24 kg / 11.5 lbs
R372570	5,0 kg / 11.0 lbs
R372585	5,34 kg / 11.7 lbs
R1081/R1181	3,15 kg / 6.9 lbs
R1083/R1183	4,57 kg / 10.0 lbs
R6408, R6409	2,78 kg / 6,12 lbs

en

Trouble-shooting

Symptom	Cause	Action
Operating buttons does not work.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".
	The inner square tubes might be slightly skewed.	Contact Pressalit A/S or your local dealer.
Height adjustment of support arms is not possible.	The screws ① holding the support arm in place are tightened too hard.	Loosen the screws until height adjustment is possible. Check mounting instruction.
The product does not move down when pressing the down button.	The bottom cover may have shifted under the lifter due to incorrect mounting of the product, blocking the downward movement.	Before inspection, always ensure the motor is in the highest position by pressing and holding the up button for 20 seconds. This ensures that there is no remaining downward movement of the lifter that could cause it to drop when the bottom cover is removed.

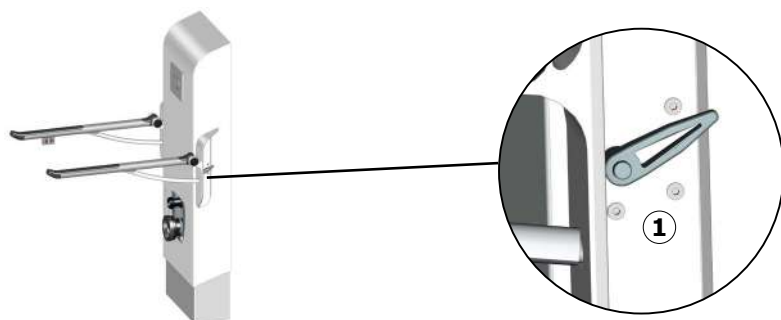Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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