




<p>Product Item number</p>	<p>VALUE fold down grab bar, 33.5", incl. control panel for TL3/TL4 R1083000</p>
<p>Color Product description</p>	<p>VVS:615892302 NRF:6191055 RSK:7828177 EAN:5708590346854 HMI: 138195 FDA: ILS/D135866 White</p>  <p>VALUE and VALUE II fold down grab bars with the ALBA wing mounting profile are specifically designed for use with SELECT TL3 and TL4 toilet lifters. The height adjustment of the toilet lifter is done only with the control panel on the fold down grab bar.</p> <p>VALUE fold down grab bar, incl. control panel for TL3/TL4 toilet lifter, 33.5", foldable, height adjustable 5.9" without the use of tools (left hand operated). Provides firm support when sitting, standing, or transferring to and from a toilet. Materials: Rilsan-coated steel, diecast aluminum and end plug in nylon (PA). For fitting to TL3/TL4 electrical toilet lifter, ALBA wing mounting profile included. For mounting on user's right side.</p>
<p>Adjustability Adjustment height Product capacity</p>	<p>Height adjustable 5,91 inch Maximum load: 265 lbs</p>
<p>Materials</p>	<p>The fold down grab bar is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA). The mounting profile is made from diecast aluminum.</p>
<p>Warranty</p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>

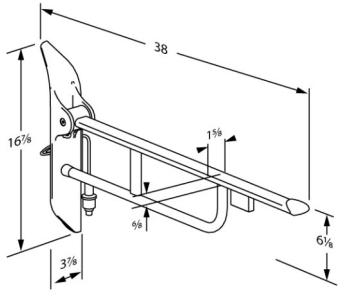

Product	VALUE fold down grab bar, 33.5", incl. control panel for TL3/TL4
Item number	R1083000
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% WipeClean UniversalCleaning Chlorine Disinfection Wipes Klorilli (Kilto Clean)</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
WEEE	<p>Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.</p>



PRESSALIT®

Product	VALUE fold down grab bar, 33.5", incl. control panel for TL3/TL4
Item number	R1083000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®

12. juni 2026