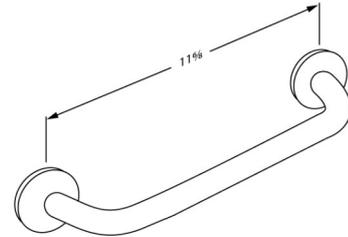




Product	PLUS grab bar 11.8"
Item number	RT100112
Color	Anthracite gray
Product description	<p>VVS:777565112 NRF:6311207 RSK:8980539 EAN:5708590318653 HMI: 19081 NBS:N13:436 DB: 2511001 FDA:ILS/D135866</p> <p>PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS grab bar, diameter 1.3", 11.8", for fixed mounting. Material: Powder-coated aluminium and plastic. Excl. mounting kit.</p>
Product capacity	<p>Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. <i>Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i></p>
Materials	<p>Basic material: Aluminium</p> <p>Surface material: Polyester lacquer</p> <p>Cover cap: Cycoloy plastic</p>
Ergonomy	<p>The 1.3" diameter enables the rail to be held comfortably and firmly. The projection of 2" from the wall makes it easy to take hold of the rail.</p>
Warranty	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



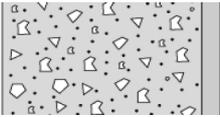
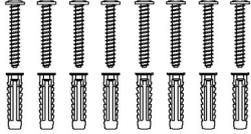
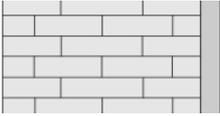
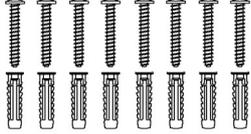
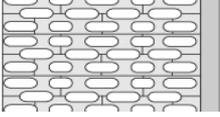
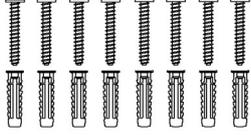
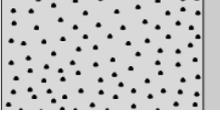
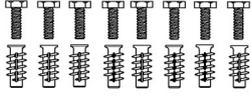
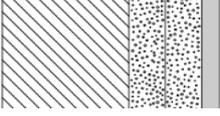
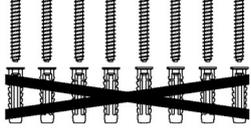


Product	PLUS grab bar 11.8"
Item number	RT100112
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>

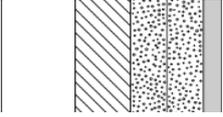
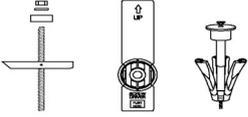
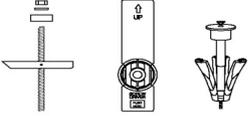
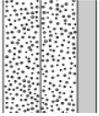


Product	PLUS grab bar 11.8"
Item number	RT100112

Mounting in different wall types

	Concrete >B25 V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Solid Brick >= Mz12 V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Aerated Concrete P4 V8666 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8616 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.	



Product Item number	PLUS grab bar 11.8" RT100112	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</p> <p>Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or Wingt anchors or similar.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>	