

## PRESSALIT.

Product	PLUS wash basin bracket with wired hand control, electrically height adjustable and manually sideways adjustable
Item number	R4781
	VVS:654003220
	NRF:6132538
	RSK:7972596
	EAN:5708590309347
	HMI: 71755
	NBS:N13:30,335
	FDA: ILW/D142473
Product description	PLUS wash basin brackets are available in electrically, pneumatically and manually height adjustable versions.  They are suitable for wall-mounting or for fitting to Pressalit PLUS wall tracks.  PLUS wash basin bracket, powered with wired remote control, electrically height adjustable by 300 mm and sideways adjustable. Recommendation: Flexible feed and
	waste system R2064/R2065/R2067. (UK markets R2074/R2075/R2077).  Materials: Aluminium and ABS polymer.  Incl. fixing bolts for wash basin. For fitting to horizontal PLUS wall track (not included)
Adjustability	Height and sideways adjustable
Adjustment height	300 mm
Product capacity	Maximum load: 100 kg. Suitable for wash basins weighing no more than 22 kg. When the wash basin is loaded with 100 kg, each wall fastener will be carrying 22 kg. Wash basin brackets need to be able to carry the weight of the wash basin, as well as support the weight of the person who is leaning against the wash basin or who is

using the wash basin to rise from seated position.

**Neither wash basin nor wash basin bracket are intended to carry the full body weight of a person**, and the test methods applicable to wash basin brackets reflect this: ISO 17966 prescribes a static test on the front edge of the wash basin at 75% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 100 kg and the wash basin weighs 22 kg, it is — according to ISO 17966 — the equivalent of a user weighing 104 kg leaning on the front edge of the wash basin.



## PRESSALIT.

PLUS wash basin bracket with wired hand control, electrically height
adjustable and manually sideways adjustable
R4781
Speed up mm: 20 mm/s Speed down mm: 23 mm/s
45 dB
The wash basin bracket is primarily made of 10-micrometre anodized aluminium. The ends of the bracket cover are capped with ABS polymer. The brackets are available with a wired hand control encased in polypropylene.
The height of the wash basin is easily adjusted by means of the wired hand control. The wash basin bracket suits all types of bolt-mounted wash basin. Its rounded, sleek design greatly facilitates cleaning.
An independant user will be able to make the necessary adjustments to an electrically operated wash basin bracket themselves without any problems.
230
LA40 (Linak) 4000N, IPX6
SMPS14002
HB3 (Linak) IPX6
Power consumption by maximum load, watt: 75 W Power consumption by standby: 1,5 W
Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.



## PRESSALIT®

Product Item number	PLUS wash basin bracket with wired hand control, electrically height adjustable and manually sideways adjustable  R4781
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro
	The product must be wiped down with a soft rag wrung out in clear water immediated after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.
	DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the part must be wiped over with a damp cloth wrung out in clear water.
Declaration of conformity	CE UK MDR-MASK-RoHS This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices - Council Directive 2006/42/EC on machinery

Council Directive 2011/65/EC on RoHS. For the UK:
Medical Device Regulation (MDR) 2002 on medical devices.

Equipment Regulations 2012.

Supply of Machinery (Safety) Regulations 2008Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic



## PRESSALIT®

Product	PLUS wash basin bracket with wired hand control, electrically height adjustable and manually sideways adjustable
Item number	R4781

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com