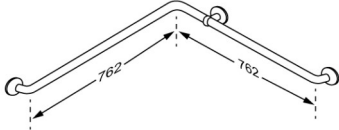




<p>Product Item number</p>	<p>PLUS handrail corner 762 x 762 mm RT126035</p>
<p>Colour Product description</p>	<p>VVS:777568335 NRF:6317183 EAN:5708590146362 HMI: 25426 NBS:N13:436 FDA:ILS/D135866 Red (RAL 3003)</p>  <p>PLUS handrails are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom. A reliable solution with many different options, which can easily lay the grounds for increased independence.</p> <p>PLUS handrail, corner, 33 mm, reversible right/left A: 762 mm x B: 762 mm, for fixed mounting. The projection of 50 mm from the wall makes it easy to take hold of the rail. Inkl. wall rosette, which covers the fixings. Material: Powder-coated aluminium and plastic. Excl. mounting kit.</p>
<p>Product capacity</p>	<p>Maximum load: 235 kg. When the handrail is loaded with 235 kg, each wall fastener is loaded with 56 kg. <i>Handrails are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.</i></p>
<p>Materials Ergonomy</p>	<p>Basic material: Aluminium Surface material: Polyester lacquer Cover cap: Cycoloy plastic Strap: Nylon (PA)</p> <p>The 33 mm diameter enables the rail to be held comfortably and firmly. The projection of 50 mm from the wall makes it easy to take hold of the rail.</p>
<p>Warranty</p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>



Product	PLUS handrail corner 762 x 762 mm
Item number	RT126035
Cleaning	<p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70%</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.</p>

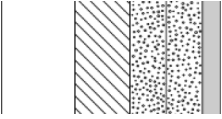
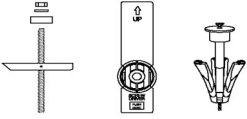

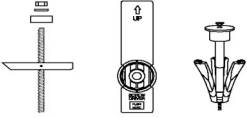
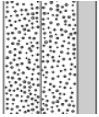


Product	PLUS handrail corner 762 x 762 mm
Item number	RT126035

Installation in different wall types

	Concrete >B25 V8617 - Mounting kit, 12 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Solid Brick >= Mz12 V8617 - Mounting kit, 12 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8617 - Mounting kit, 12 screws Ø5 x 50 - plugs SX8, for brick wall. Must be ordered separately.	
	Aerated Concrete P4 V8667 - Mounting kit, 12 bolts M5 x 50 - plugs FTP K 6. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8617 - Mounting kit, 12 screws Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.	



Product Item number	PLUS handrail corner 762 x 762 mm RT126035	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</p> <p>Wall fasteners (12 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>	