




<p>Product Item number</p>	<p>MCT 3 change trolley, electrically height adjustable R9662721000</p>
<p>Color Product description</p>	<p>EAN:5708590390000 FDA: INQ/D124319</p> <p>White</p>  <p>Pressalit offers a variety of mobile and wall-hung, height-adjustable and fixed height shower and changing tables of high quality. They are designed to provide a stable platform for showering and changing adults and children with reduced mobility. These tables can be used in schools, institutions, hospitals and private homes and are all designed to be durable, comfortable and easy to clean. The design and flexibility of the tables ensures optimal working conditions for the carer as well as a high level of comfort and safety for the user.</p> <p>MCT 3 change trolley, 27.6" x 82.7", two fold-down sides. Electrically height adjustable with wired remote control. 19.7" lifting range, from 17.7" to 37.4". Mattress to be ordered separately. Note: The MCT 3 must be plugged into a main power supply to use the height adjustment function.</p>
<p>Adjustability Adjustment height Product capacity Speed up Noise level</p>	<p>Height adjustable</p> <p>19,7 inch</p> <p>Maximum user weight (MUM): 441 lbs. Maximum load (SWL): 452 lbs. <i>The maximum load is to be understood as the total safe working load of the product, including both the weight of the user, and the weight of the mattress</i></p> <p><i>This product has been tested statically with a load of 1130 lbs as the IEC 60601-1 standard requires testing with a safety factor 2.5.</i></p> <p>Speed up: 0,67 inch/s Speed down: 0,67 inch/s</p> <p>50 dB</p>
<p>Materials</p>	<p>Plastics, Compact Density Fiberboards (CDF) and powder-lacquered steel.</p>

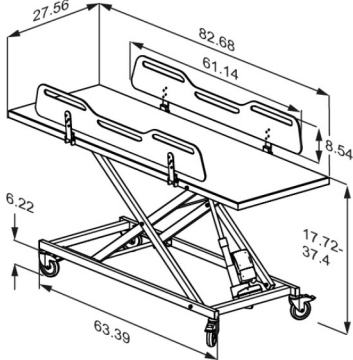

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9662721000
Ergonomy	<ul style="list-style-type: none">• The ergonomically designed safety rails have a practical lower central section, which provides better access and work environment for the carer while transferring, changing and dressing the user.• One hand operated safety rails allows the carer to maintain eye contact as well as physical contact with the user during the whole process, the foldable safety rails also provide easy access from all four sides of the table.• The cut-out grips in the safety rails enable users to assist when being turned, which motivates physical training.• Easy to clean, smooth contact surfaces which do not collect dirt and bacteria.
Features	<ul style="list-style-type: none">• The trolley can be moved from room to room for easy transport and multiple users.• Intuitive, easy-to-use drop down rails with one hand operation.
Mains connection (V)	120
Actuator (motor)	LA40 (Linak) IPX6
Control box	CA30 (Linak) IPX6
Wired remote control	HB30 (Linak) IPX6
Power consumption (maximum load)	Power consumption by maximum load: 232 W Power consumption by standby: 0,5 W
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is placed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.



Product	MCT 3 change trolley, electrically height adjustable
Item number	R9662721000
Cleaning	<p>The product is to be cleaned with a damp cloth that has been wrung out in a mild soap mixture, such as universal cleaning solution. After cleaning, it is essential to wipe the product with a dry, soft cloth. This step is crucial to prevent the materials from discoloration. Additionally, it helps prevent any potential degradation of the surface paint,</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningsprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% Accel PREvention Wipes Lingettes OxiGenesis from ENVIROX</p> <p>NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
WEEE	<p>Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.</p>



PRESSALIT®

Product	MCT 3 change trolley, electrically height adjustable
Item number	R9662721000

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com

PRESSALIT®