

PRESSALIT.

Product	PLUS grab bar, corner 30" x 17.9"	
Item number	RT128000	
	VVS:777568400	
	NRF:6311228	
	RSK:8980584	
	EAN:5708590146379	
	HMI: 25427	
	NBS:N13:436	
	FDA: ILS/D135866	
Color	White	
Product description	PLUS grab bars are high-quality products and they can be combined in countless ways and they provide the final touch to furnishing a functional and flexible bathroom A reliable solution with many different options, which can easily lay the grounds for increased independence. PLUS grab bar, 1.3", reversible right/left, A:30" x B:17.9".	
	Material: Powder-coated aluminium and plastic. Excl. mounting kit.	
Product capacity	Maximum load: 440 lbs. When the grab bar is loaded with 440 lbs, each wall fastener is loaded with 123 lbs. Grab bars are tested with a 1.5 safety factor in accordance with the requirements in ISO 17966:2016.	
Materials	Core material: Aluminium Surface material: Polyester lacquer Cover cap: Cycoloy plastic Strap: Nylon (PA)	
Ergonomy	The 1.3" diameter enables the rail to be held comfortably and firmly. The projection o 2" from the wall makes it easy to take hold of the rail.	
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.	



PRESSALIT.

Product Item number	PLUS grab bar, corner 30" x 17.9" RT128000
Cleaning	This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro

The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.

DECALCIFICATION OF ALUMINUM PARTS:

Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.



PRESSALIT.

Product				
Item	number			

PLUS grab bar, corner 30" x 17.9"

RT128000

Mounting in different wall types

Concrete >B25

V8617 - Mounting kit, 12 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.






Solid Brick >= Mz12

V8617 - Mounting kit, 12 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.



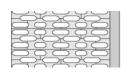

Solid Sand Lime Brick >= Hlz12

V8617 - Mounting kit, 12 bolts \emptyset 5 x 50 - plugs SX8, for brick wall. Must be ordered separately.






Aerated Concrete P4

V8667 - Mounting kit, 12 bolts M5 x 50 - plugs FTP K 6, for aerated concrete. To be ordered separately.



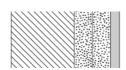

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8617 - Mounting kit, 12 bolts \emptyset 5 x 50 - plugs SX8. NB! Plugs are not needed.

To be ordered separately.





PRESSALIT®

Product Item number	PLUS grab bar, corner 30" x 17.9" RT128000	
	Drywall (2 x 12,5 mm) water-resistant, re-er 22 mm plywood Wall fasteners (12 pcs) must be bought separatry drywall it is possible to use e.g. SnapTight or vanchors or similar.	ately, for
	Drywall (2 x 12,5 mm) water-resistant, re-ensteel plates (.08 inch) Wall fasteners (12 pcs) must be bought separatry drywall it is possible to use e.g. SnapTight or vanchors or similar.	ately, for
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended fatal injury to persons and walls.	d - risk of

pressalit.com