




<p>Product Item number</p>	<p>PLUS wall track, 15.7", right-grooved, for vertical mounting R9874112</p>
<p>Color Product description</p>	<p>VVS:615843104 NRF:7501073 EAN:5708590365961 HMI: 124108</p> <p>Anthracite gray</p>  <p>Below a selection of products for fitting to the wall track.</p> <p>PLUS wall track, 15.7", right-grooved, for vertical mounting. For height adjustable folding seats and for height adjustable, right-hand operated fold down grab bars. The cover plate hides bolts and wires and facilitates cleaning. Materials: Aluminium and polystyrene. Excl. fixing kit.</p>
<p>Adjustability</p>	<p>Fixed mounting</p>
<p>Materials Ergonomy Features</p>	<p>The wall track is primarily made of 10-micrometer anodized aluminium. The cover plate is made of polystyrene and the ends of the track are capped with ABS polymer.</p> <p>The vertical track enables height adjustment of the different products. If combined with the horizontal track, the available space in the bathroom can be utilized optimally. In environments with multiple users, the vertical track makes it possible to adjust the different products to individual user heights.</p> <p>The front cover plate conceals mounting bolts and any cables.</p>
<p>Warranty</p>	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.</p>

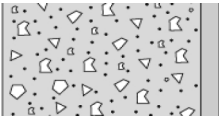
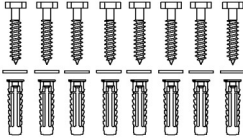

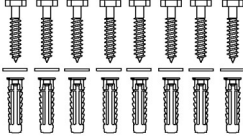
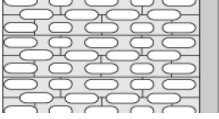
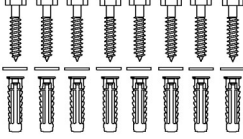
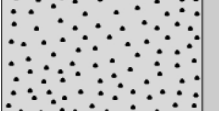
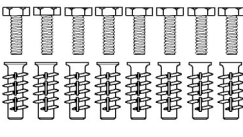
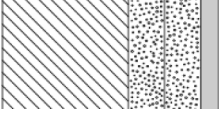
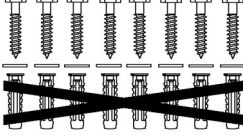


Product Item number	PLUS wall track, 15.7", right-grooved, for vertical mounting R9874112
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p> <p>DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that water deposits (Calcium, lime etc.) be removed from wall tracks and other aluminum parts using a solution of three parts of 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts should be wiped over with a damp cloth wrung out in plain water.</p>
Declaration of conformity	<p>This product fulfils all relevant requirements in European Regulation (EU) 2017/745 on medical devices in risk class I. This product is considered to be an accessory for a medical device, as it is not itself a medical device, but is intended to be used together with several particular medical devices to specifically enable the medical devices to be used in accordance with their intended purposes.</p>

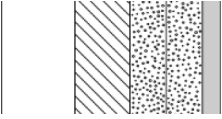
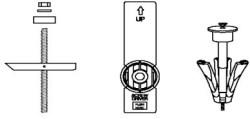
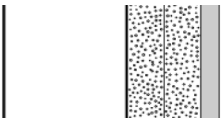
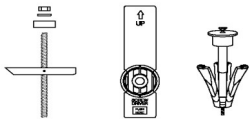
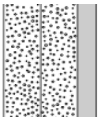



Product	PLUS wall track, 15.7", right-grooved, for vertical mounting
Item number	R9874112

Mounting in different wall types

	Concrete >B25 V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Brick >= Mz12 V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. To be ordered separately.	
	Aerated Concrete P4 V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8610 - Mounting kit, 8 lag bolts Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



Product Item number	PLUS wall track, 15.7", right-grooved, for vertical mounting R9874112	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.