R8574572/R8575572
R8584572/R8585572
R8594572/R8595572

CT 4000/CT 4100

Change table – Operation and maintenance manual en

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Pressalit's Environmental and Quality Management Systems are certified by LRQA

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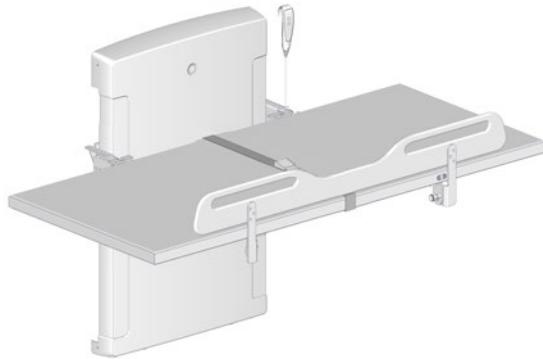
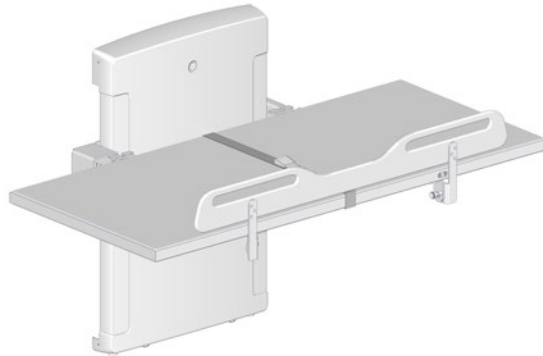
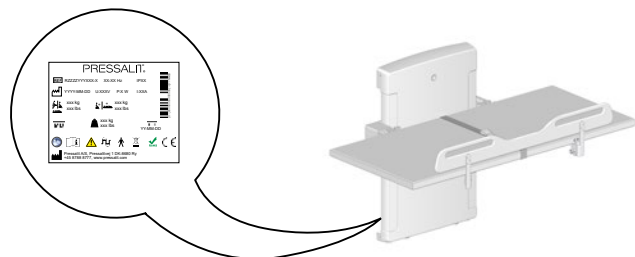
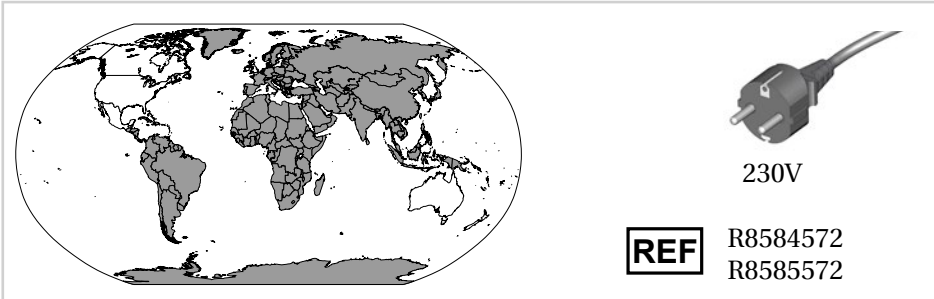
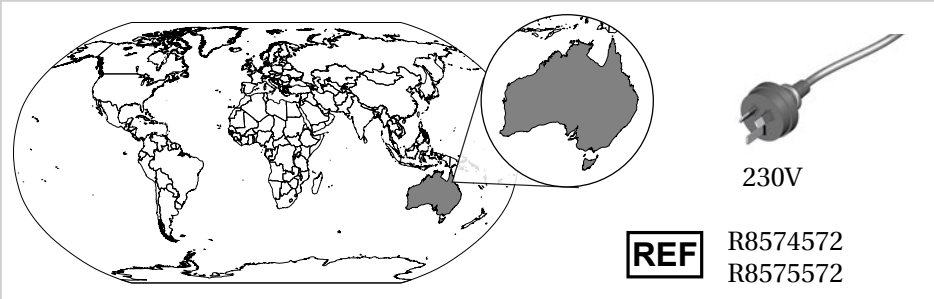
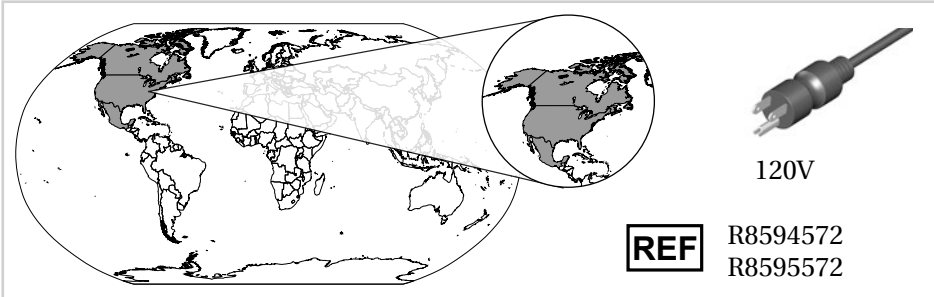
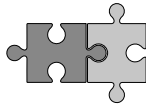
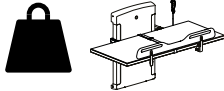
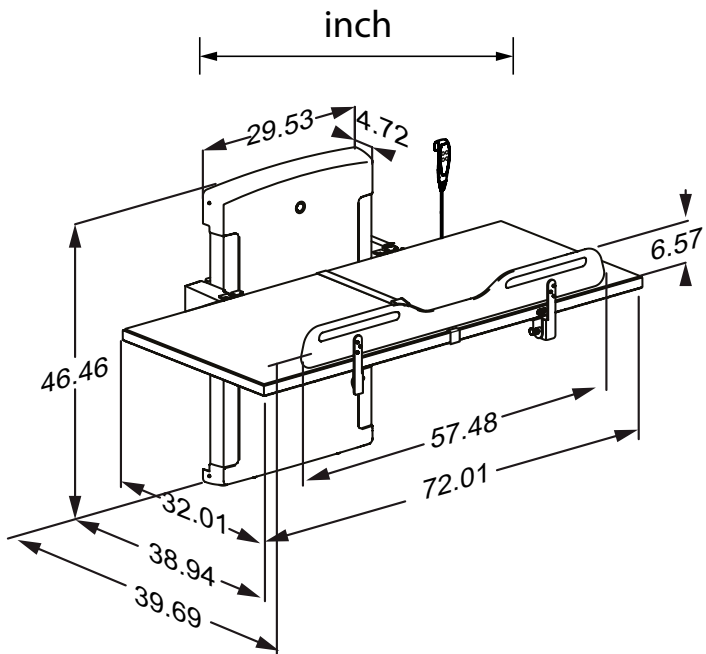
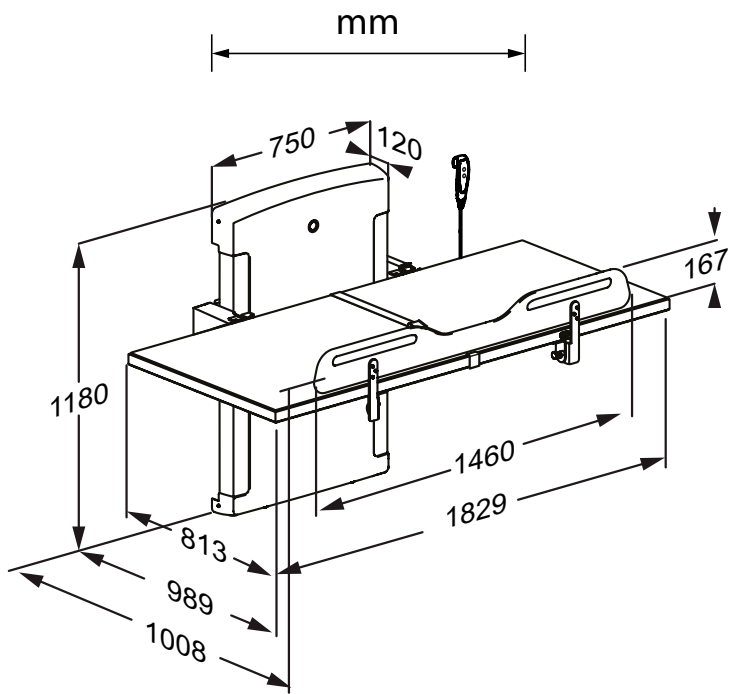
Content

Mounting instruction	8
Operational signage	8
Symbols used in this manual	9
General safety	10
Intended use	11
Using alone (no helper)	11
Using with a helper	12
Intended operator profile	13
Intended user profile	13
Operating environment	13
Contraindications for use	14
Biocompatibility	14
Operating instructions	15
Folding up and down, CT 4000	15
Folding up and down, CT 4100	16
Height adjustment	17
The safety rail	18
The body belt	18
Cleaning	19
Disinfection	19
Maintenance	20
Maintenance inspection	20
Product label	22
Resetting	24
Trouble-shooting	24
Electromagnetic compatibility	26
Electromagnetic immunity	27
Electromagnetic emission	30
Storage and transportation	31
Technical data	32
Disposal and recycling	33

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Operational signage

A Quick-guide is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

 WARNING	<p>WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>		
 CAUTION	<p>CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>		
IMPORTANT	<p>IMPORTANT indicates important information about handling and use of the product.</p>		
NOTICE	<p>NOTICE is used to address practices not related to personal safety.</p>		
 <p>Hand injury warning</p>	 <p>Head injury warning</p>	 <p>Electricity / voltage warning</p>	 <p>Other specific warning</p>
 <p>Compatible products</p>	 <p>Spare parts</p>	 <p>Power specifications</p>	 <p>EMC related symbols</p>
 <p>Product weight</p>	 <p>Max. user weight</p>	 <p>Article number / versions</p>	 <p>Do not leave patient unattended</p>

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

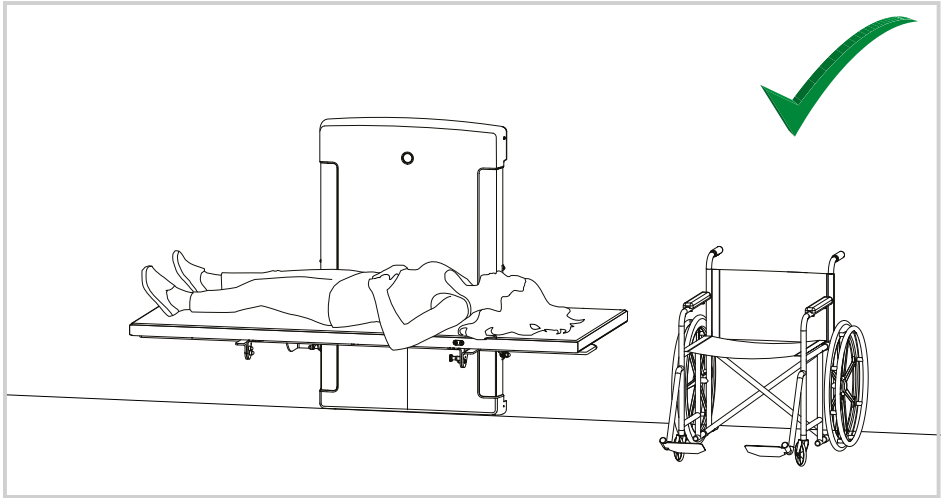
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Intended use


Using alone (no helper)


 **CAUTION**

en

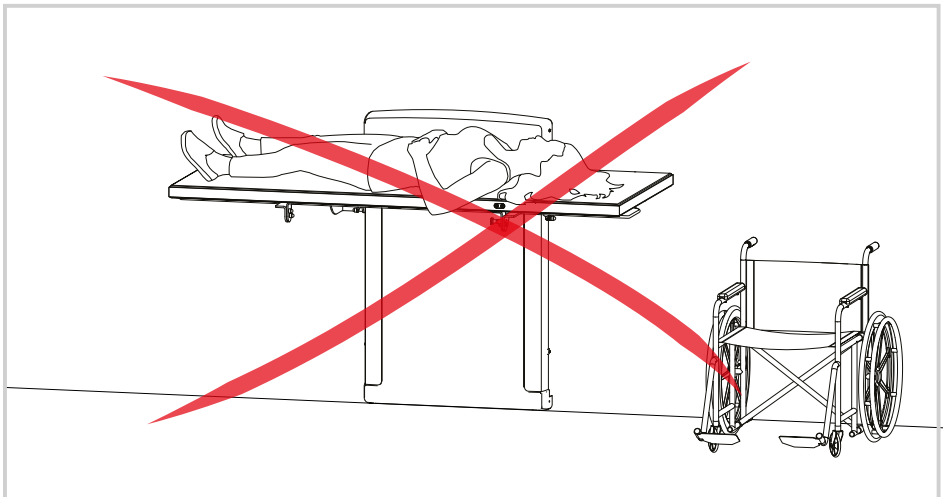This product has no manual lowering function, so in case of a power outage, the lying surface will stay in its current position. It is therefore not recommended to take the lying surface to the highest position when using the product alone.


Using with a helper.

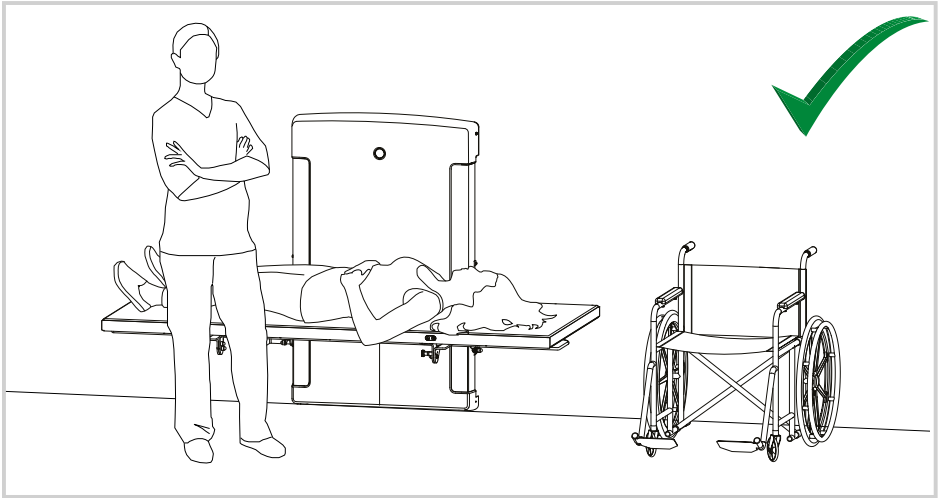en

 **CAUTION**

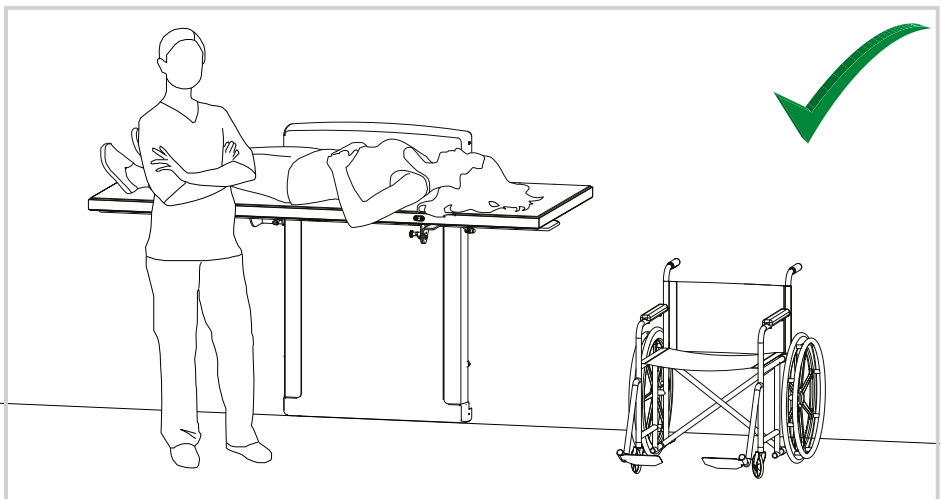It is only recommended to take the lying surface to its highest position when using this product in the presence of a helper.

This product is intended for people who:

- need to get changed on a change table.
- are above the usual change table age.
- are caregivers and need a customized work position.
- need to use a change table in combination with person lifts.

Intended operator profile

The product is to be used by a person trained in health care and/or trained in changing a person in their care. It can also be used by self-reliant users.

Intended user profile

The product can be used by adults whose weight does not exceed the **maximum load of 227 kg / 500 lbs.**

Operating environment

For indoor use in public changing facilities and schools. Can also be used in nursing homes and hospitals under normal working conditions.

Environmental factor	Operating range
Temperature	5°C to 40°C (41°F to 104°F)
Relative humidity	20% to 70% @ 30°C (86°F)
Atmospheric pressure	700 to 1060 hPa
Meters above sea level	Max. 3000 m / 9 800 feet
Duty cycle	2 minutes of continuous use, followed by an 18-minute pause.

- Never allow children to operate the unit without supervision of an adult.
- Keep pets and pests away from the unit.

Contraindications for use

Use of the change table is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions. Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the change table only after consulting with the attending physician. Do not use if the patient's weight exceeds 227 kg / 500 lbs.

Biocompatibility

The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in BS EN ISO 10993-1:2020 Biological evaluation of medical devices – Part 1 and are therefore considered suitable for this purpose.

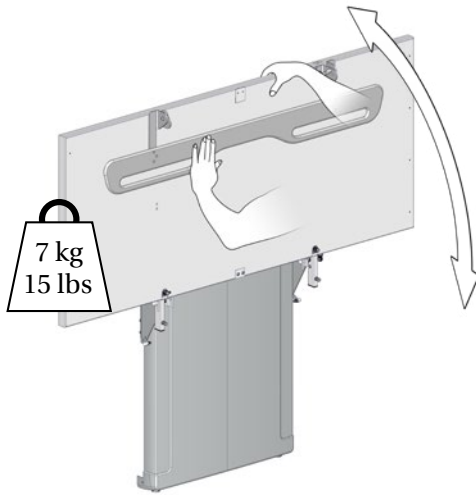
NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities. A serious incident is defined as an incident that leads to the death of a patient, user, or any other person, or causes a temporary or permanent deterioration in the person's health. Any incident that has resulted in or has the potential to result in a serious incident must be reported.

en

Operating instructions

Folding up and down, CT 4000

Support the table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the table up against the wall after use. Be sure to fold the safety rail away first.

Do not leave the patient unattended.

 **CAUTION**

Always be careful when moving a person to and from the table.

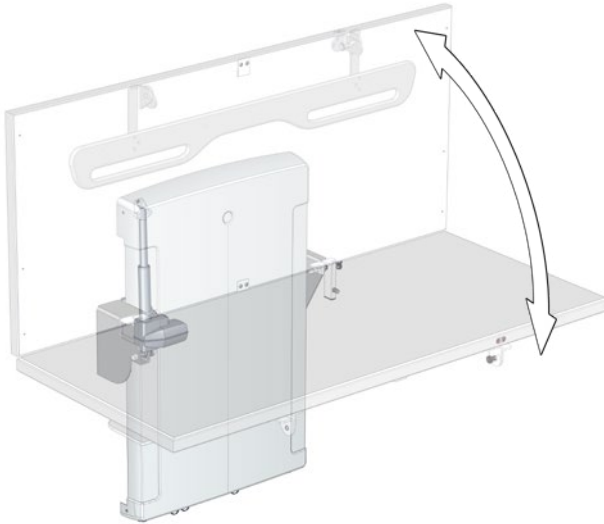
Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

 **CAUTION**

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (e.g. sunlight or frost).

Folding up and down, CT 4100

en

The table folds up and down electrically

On the wired remote control for this model, there are two additional buttons that control the fold-up function.

Press and hold the fold up button ② to fold the table up, and press and hold the fold down button ③ to fold the table down. The movement stops when the button is released.

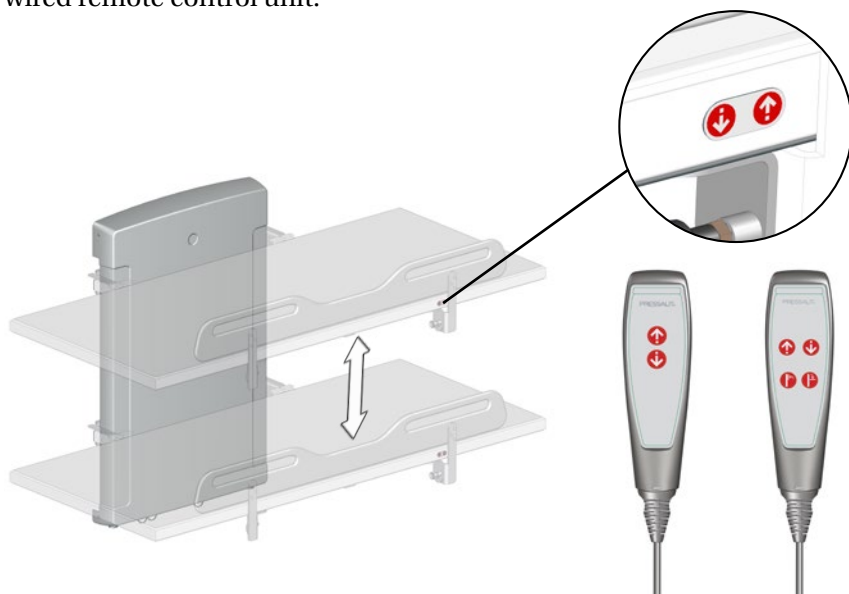
Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.

Height adjustment

When the change table is installed at the recommended height, it can be adjusted between 300 and 1000 mm (11.81 and 39.37 inches) above the floor.

The height adjustment is done by pressing the buttons on the frame or on the wired remote control unit.

Press and hold the Up button to raise the table, and press and hold the Down button to lower the table. The movement stops when the button is released.

The height adjustment may only be used at temperatures between 15 and 30 degrees Celsius.

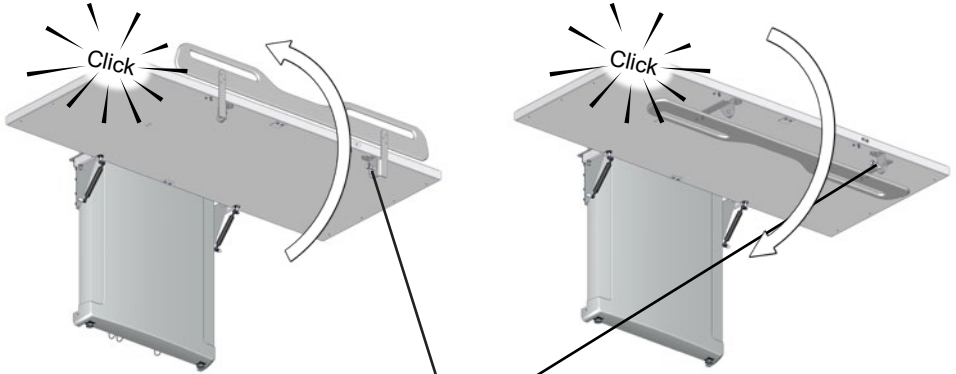
Make sure that there are no objects under the change table when adjusting the height.


To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.

The safety rail

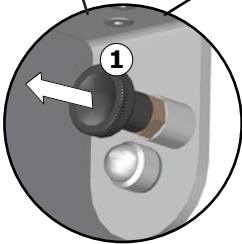
To pull the safety rail up: Release the lock ① and pull the safety rail from the underside of the table, and swing it upwards. The safety rail will automatically lock into position.

To fold the safety rail away: Release the lock ① and swing the safety rail down to the underside of the table. The safety rail will automatically lock into position.

en

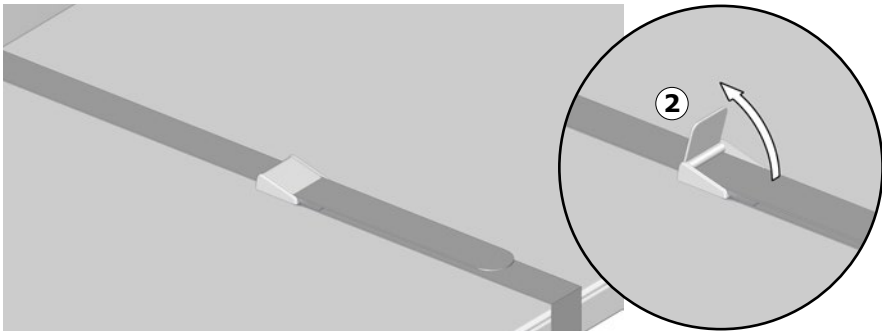The body belt

The product is equipped with a body belt, that can be used to secure the user. Pull the belt lock lever ② up to release the belt lock. Press the lever down again to lock the body belt.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number. Pressalit recommends cleaning the product after installation.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

Disinfection

When using the shower table for multiple patients, it is necessary to both clean and disinfect the shower table between uses on different patients to prevent the risk of cross infection. A list of suitable disinfectants can be found at pressalit.com.

NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty of the nursing bench will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product.

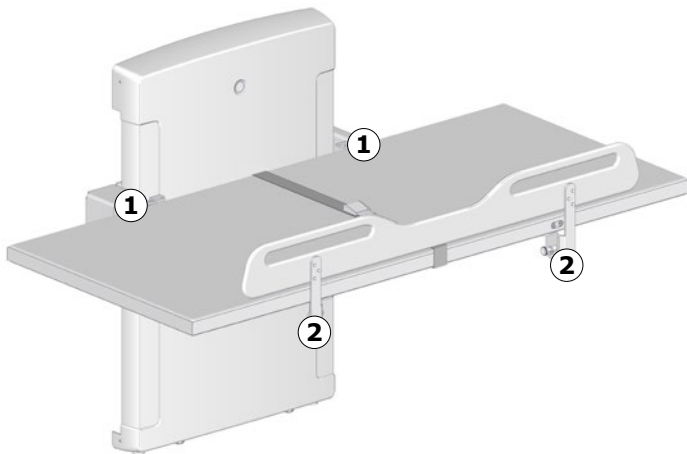
Maintenance

Expected service life of the change table is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the table to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that inspection is done once a year.


Maintenance inspection

Check the hinges and other fittings ① on the table and the safety rail ②. Tighten if necessary.

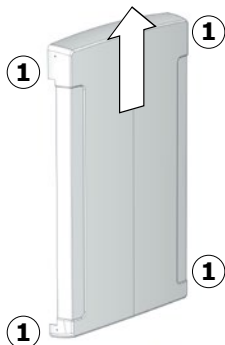
Do not perform any type of maintenance inspection on the change table while a patient is lying on it.


Disconnect mains before removing the covers.

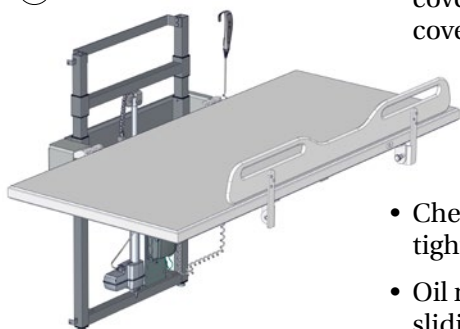- Remove the upper and lower retaining screws ① on the lift unit cover and carefully remove the cover.

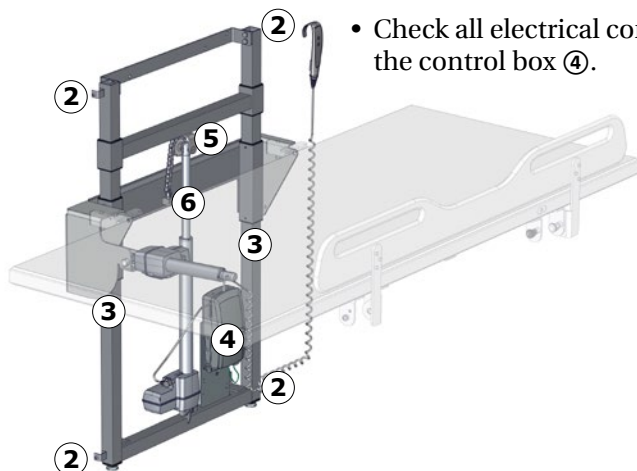
- Check the wall brackets ② and tighten if necessary.
- Oil moving parts, especially the sliding brackets ③.

- Check all electrical connections to the control box ④.

- Check the chain wheel ⑤ for wear.
- Visually inspect the chain ⑥ and other steel parts for penetrating rust.
- Oil the chain ⑥.
- Put the lift unit cover back on and tighten the retaining screws.
- Reconnect the power supply.

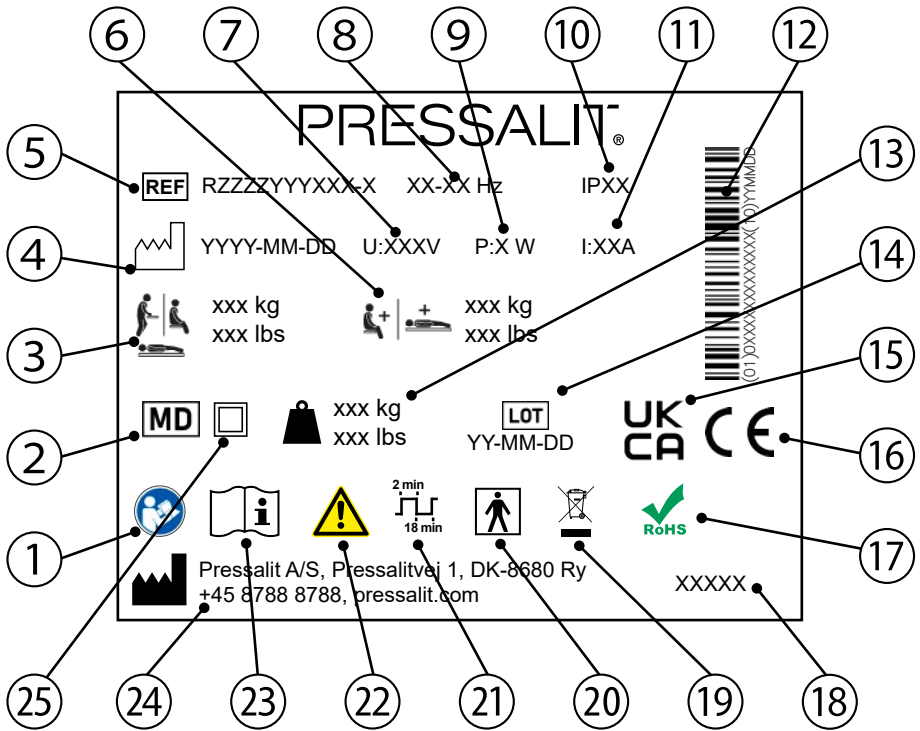Product label

- ① Consult manual for use

- ② Medical device

- ③ Maximum user weight

- ④ Date of manufacture

- ⑤ Catalogue number

- ⑥ Max load

- ⑦ Mains voltage

- ⑧ Frequency

- ⑨ Maximum power consumption

⑩ IP class

⑪ Electric Current

⑫ UDI carrier GS1-128 barcode

⑬ Product weight

⑭ Batch code

⑮ UK Conformity Assessed

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
 - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at pressalit.com

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

⑱ Product name (if any)

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function
The product complies with requirements of IEC 60601-1 to provide protection against electric shock.

㉑ Duty cycle

㉒ Consult the manual for important cautionary information such as warnings and precautions.

㉓ Consult manual for use

㉔ Name and address of manufacturer

㉕ Class II Product, double insulated

Resetting

If the height adjustment does not work, and the control box beeps and/or flashes, the unit must be reset.

Press and hold the up button and the down button simultaneously. The control box emits a beep sound. When the beep stops, release both buttons.


Take the unit to its lowest position.
The unit is now reset.

Trouble-shooting

Symptom	Cause	Action
The unit does not stay in upright position.	One or more gas cylinders are defective.	Contact Pressalit A/S or your local dealer.
The product will not work, even though the remote control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product does not work, and the control box beeps and/or flashes.	The product needs to be reset.	See "Resetting".

The unit runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under “Maintenance”.
The table top is not level.	Adjustment screws too high or too low.	Adjust the adjustment screws ①.

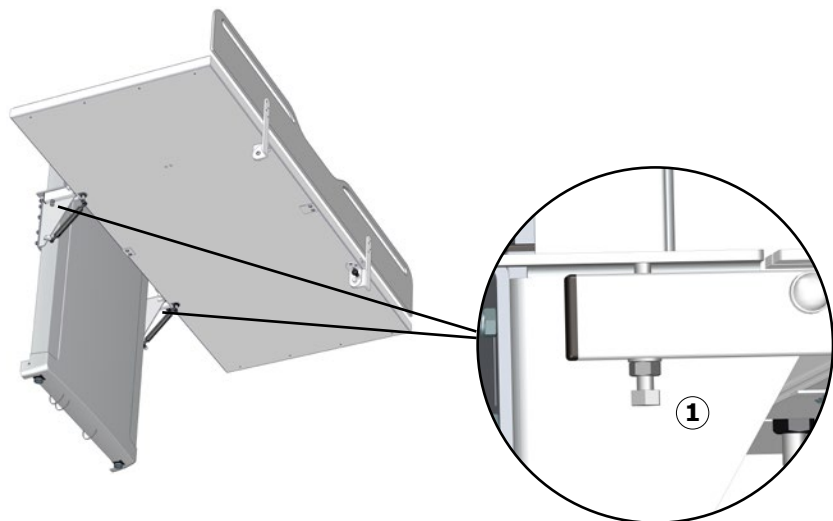Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In case of loss or degradation of performance due to electromagnetic disturbances, the user still can get on and off the product.

Cable	Length
Power cable	3,2 m (126")
Hand control spiral cable	Approx. 5 m (197") extended

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Electromagnetic immunity

Electrostatic discharge	
	The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-2	+/- 8 kV contact +/- 2, 4, 8, 15 kV air


Radiated RF electromagnetic field	
	The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.
Test standard	Immunity test level
EN 61000-4-3	10 V/m 80 MHz to 2,7 MHz 80 % AM 1 kHz

en


Immunity to proximity fields from RF wireless communication equipment


	<p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-3*</p>	<p>385 MHz, 27V/m, 18 Hz PM (50 % duty cycle square wave) 450 MHz, 28V/m, FM +/- 5 kHz deviation, 1 kHz sine 710 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 745 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 780 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 810 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 870 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 930 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave) 1720 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1845 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 1970 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 2450 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave) 5240 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5500 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave) 5785 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)</p>

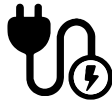Electrical fast transient/burst

	<p>The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.</p>
<p>Test standard</p>	<p>Immunity test level</p>
<p>EN 61000-4-4</p>	<p>Signal line: +/- 1 kV, AC power line: +/- 2 kV 100 kHz repetition frequency</p>

Surge immunity	
	The product must not be affected to initiate actions or to render components defective as a result of lightning.
Test standard	Immunity test level
EN 61000-4-5	Line-to-line: +/- 0.5 kV, +/-1 kV Line-to-ground: +/- 0.5 kV, +/-1 kV, +/- 2 kV

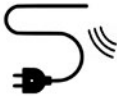Immunity to conducted disturbances induced by RF fields	
	The wires must not be affected by other RF fields so as to initiate actions or render components defective.
Test standard	Immunity test level
EN 61000-4-6	3 V (6 V in ISM bands and amateur radio bands) 0,15 to 80 MHz , 80 % AM at 1 kHz

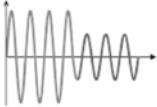Voltage dips	
	The product must not initiate or limit actions as a result of a dip in the power supply.
Test standard	Immunity test level
EN 61000-4-11	0 % U _T ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % U _T ; 1 cycle at 0°, 70 % U _T ; 25/30 cycles at 0° Single phase at 0°

Voltage interruptions	
	The product must not initiate or limit actions as a result of a short-term power supply stop.
Test standard	Immunity test level
EN 61000-4-11	0 % U _T ; 300 cycle. 250/300 cycles

Electromagnetic emission




Conducted RF emission	
	The product must not send too much noise into the grid. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

Radiated RF emission	
	The product must not emit RF signals in the room. This could lead to a disruption of other products.
Test standard	Emission test class and group
EN 55011	Class B, group 1

Harmonic current emissions	
	The product must not unintentionally draw power from the grid.
Test standard	Emission test class and group
IEC 61000-3-2	Class A

Voltage changes, voltage fluctuations and flicker	
	The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room.
Test standard	Emission test class and group
EN 61000-3-3	Tested according to the standard.

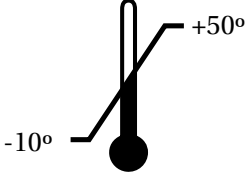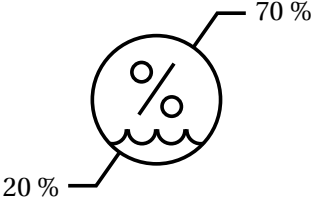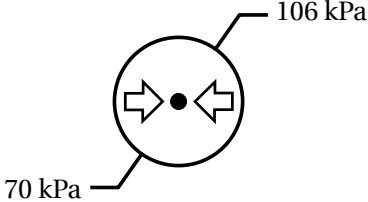Storage and transportation

 <p>A thermometer icon with a vertical scale. The top of the scale is labeled '+50°' and the bottom is labeled '-10°'. The liquid level is shown between these two marks.</p>	<p>The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.</p>
 <p>A circular icon containing a percentage sign (%) and wavy lines representing water. A line points from the top right to '70 %' and another line points from the bottom left to '20 %'.</p>	<p>The relative humidity for transportation and storage of the unit must be between 20 and 70% (RH).</p>
 <p>A circular icon containing a central dot and two arrows pointing towards each other. A line points from the top right to '106 kPa' and another line points from the bottom left to '70 kPa'.</p>	<p>The air pressure for transportation and storage of the unit must be between 70 and 106 kPa.</p>

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

en

Technical data

Max load	227 kg / 500 lbs	
Materials	Powder laquered stainless steel Plastics (PVC, Polystyrene)	
Weight	R8574572 R8584572 R8594572	75 kg / 165 lbs
	R8575572 R8585572 R8595572	85 kg / 187 lbs
IP class	The electrical parts have an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.	

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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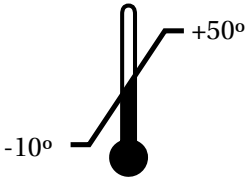
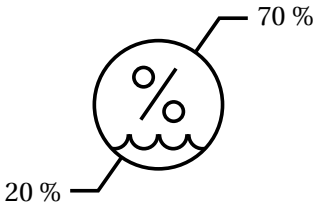
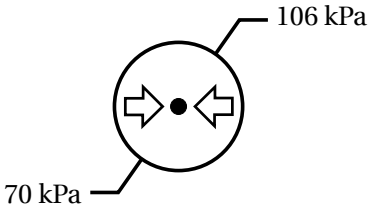
Opbevaring og transport

 <p>A thermometer icon with a scale. The top of the scale is labeled '+50°' and the bottom is labeled '-10°'. The liquid level is shown between these two marks.</p>	<p>Temperaturen for transport og opbevaring af enheden skal være mellem -10 og 50 grader Celsius.</p>
 <p>A circular icon containing a percentage sign (%) and wavy lines representing water vapor. A line points from the top of the circle to the text '70 %' and another line points from the bottom of the circle to the text '20 %'.</p>	<p>Den relative luftfugtighed for transport og opbevaring af enheden skal være mellem 20 og 70% (RH).</p>
 <p>A circular icon containing a central dot and two arrows pointing towards each other. A line points from the top of the circle to the text '106 kPa' and another line points from the bottom of the circle to the text '70 kPa'.</p>	<p>Lufttrykket for transport og opbevaring af enheden skal være mellem 70 og 106 kPa.</p>

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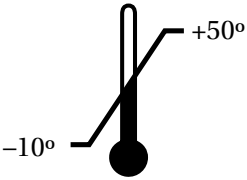
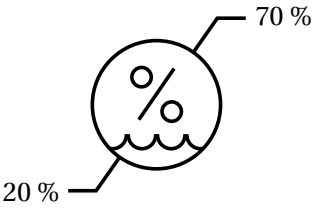
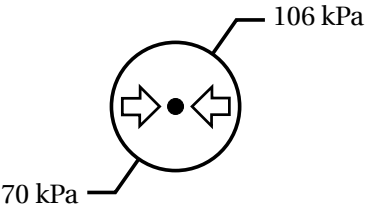
 <p>A thermometer icon with a black bulb and a white scale. The scale has a tick mark at -10° on the left and +50° on the right. The liquid level is shown between these two marks.</p>	<p>Temperaturen vid transport och förvaring måste ligga mellan -10 och 50 grader Celsius.</p>
 <p>A circular icon containing a percentage sign (%). Below the circle is a wavy line representing water. Two lines point from the text '20 %' on the left and '70 %' on the right to the circle.</p>	<p>Den relativa luftfuktigheten vid transport och förvaring av enheten måste ligga mellan 20 och 70 % (RH).</p>
 <p>A circular icon containing a central black dot. Two arrows point towards the dot from the left and right sides. Two lines point from the text '70 kPa' on the left and '106 kPa' on the right to the circle.</p>	<p>Luftrycket vid transport och förvaring måste ligga mellan 70 och 106 kPa.</p>

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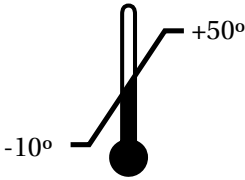
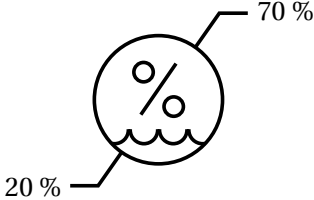
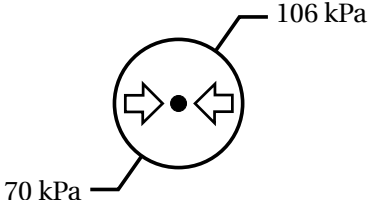
 <p>A thermometer icon with a scale. The top of the scale is labeled '+50°' and the bottom is labeled '-10°'. The liquid level is shown at approximately the 0-degree mark.</p>	<p>Temperaturen for transport og oppbevaring av enheten må være mellom -10 og 50 grader Celsius.</p>
 <p>A circular icon containing a percentage sign (%). A wavy line at the bottom represents water. A line points from the top of the circle to '70 %' and another line points from the bottom to '20 %'.</p>	<p>Den relative fuktigheten for transport og oppbevaring av enheten må være mellom 20 og 70 % (RF).</p>
 <p>A circular icon with a central dot and two arrows pointing towards it from the left and right. A line points from the top of the circle to '106 kPa' and another line points from the bottom to '70 kPa'.</p>	<p>Lufttrykket for transport og oppbevaring av enheten må være mellom 70 og 106 kPa.</p>

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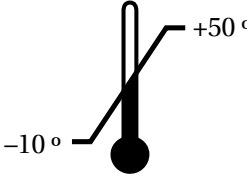
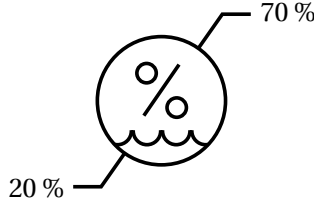
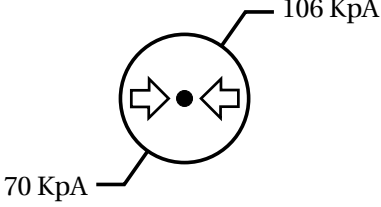
 <p>A thermometer icon with a black bulb and a white scale. The scale has a tick mark at -10° and another at +50°. The liquid level is shown between these two marks.</p>	<p>La temperatura para el transporte y el almacenamiento de la unidad debe estar comprendida entre -10 y 50 grados.</p>
 <p>A circular icon containing a percentage sign (%) and a wavy line representing water. Two lines point from the text '20 %' and '70 %' to the bottom and top of the circle, respectively.</p>	<p>La humedad relativa para el transporte y el almacenamiento de la unidad debe estar comprendida entre 20 y 70 % (HR).</p>
 <p>A circular icon with a central black dot and two white arrows pointing towards it from the left and right. Two lines point from the text '70 KpA' and '106 KpA' to the bottom and top of the circle, respectively.</p>	<p>La presión de aire para el transporte y el almacenamiento de la unidad debe estar comprendida entre 70 y 106 KpA.</p>

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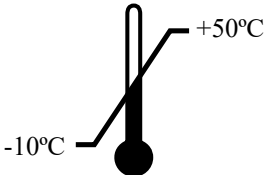
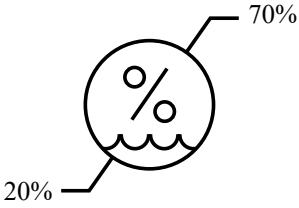
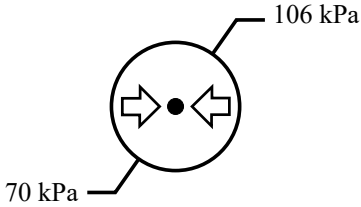
 <p>A thermometer icon with a black bulb and a white scale. The scale has two points marked: -10°C on the left and +50°C on the right. A vertical line indicates the temperature range between these two points.</p>	<p>Zakres dopuszczalnych temperatur transportu i przechowywania urządzenia wynosi od -10 do 50°C.</p>
 <p>A circular icon containing a percentage sign (%). Below the circle are wavy lines representing water or humidity. Two lines point to the top and bottom of the circle, labeled 70% and 20% respectively.</p>	<p>Wilgotność względna podczas transportu i przechowywania urządzenia musi pozostawać w zakresie od 20 do 70%.</p>
 <p>A circular icon containing a central black dot. Two arrows point horizontally towards the dot from the left and right sides. Two lines point to the top and bottom of the circle, labeled 106 kPa and 70 kPa respectively.</p>	<p>Ciśnienie powietrza podczas transportu i przechowywania urządzenia musi pozostawać w zakresie od 70 do 106 kPa.</p>

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