


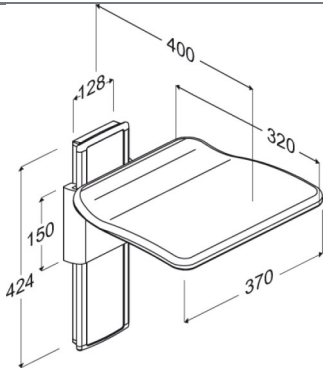

Product Item number	PLUS folding seat, manually height adjustable R5520112000
Colour	VVS:777531200 NRF:7501003 RSK:7973927 EAN:5708590306360 HMI: 130158 NBS:N13:448 FDA:INN/D135865 White
Product description	 PLUS folding seat. Manually height adjustable by 240 mm. Anthracite cover plate for vertical wall track. The seat is foldable. Excl. mounting kit. The folding seat is not suitable for use in the shower area. PLUS folding seat. Manually height adjustable by 240 mm. Anthracite cover plate for vertical wall track. Materials: Aluminium, thermoset plastic, plastic. Excl. mounting kit.
Adjustability Adjustment height Product capacity	Height adjustable 240 mm Maximum user mass: 265 kg. When the seat is loaded with 265 kg, each wall fastener is loaded with 23 kg. <i>The seat has been tested statically with safety factor 1.5, which is a requirement according to the applied assistive products standards.</i>
Materials Ergonomy Features	The folding seat is made out of thermoset plastic, and the mounting and suspension bracket are made from 10 my anodised aluminium. The end caps are made from ABS plastic. Ergonomically shaped seat, which offers good and comfortable support. The seat is suitable for situations where additional seating is required and space is limited, - e.g. in kitchens, lifts and corridors or on the landings. The seat can be folded away when not in use, - or to free up space. Stays up of itself when folded away.



PRESSALIT®

Product	PLUS folding seat, manually height adjustable
Item number	R5520112000
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
Cleaning	The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Sprit (husholdningssprit) Suma Bac D10 Virkon Rodalon Desinfektionsspray Lysoformin Spezial CaviCide Sufanios Premium Tvättsprit 70% NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200 Milizid Citro The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate. DECALCIFICATION OF ALUMINUM PARTS: Pressalit recommends that chalky deposits be removed from wall tracks and other aluminum parts using a solution of three parts 30% acetic acid to seven parts tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth wrung out in clear water.



PRESSALIT®

Product	PLUS folding seat, manually height adjustable
Item number	R5520112000

Installation in different wall types

	Concrete >B25 V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Brick >= Mz12 V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Solid Sand Lime Brick >= Hlz12 V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. To be ordered separately.	
	Aerated Concrete P4 V8660 - Mounting kit, 8 countersunk bolts M8 x 35, plugs FTP M 8. To be ordered separately.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8610 - Mounting kit, 8 coach screws Ø8 x 70, plugs SX10. NB! Plugs are not needed. To be ordered separately.	



Product Item number	PLUS folding seat, manually height adjustable R5520112000
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 
	Drywall (2 x 12,5 mm) water-resistant Wall fasteners (8 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar. 