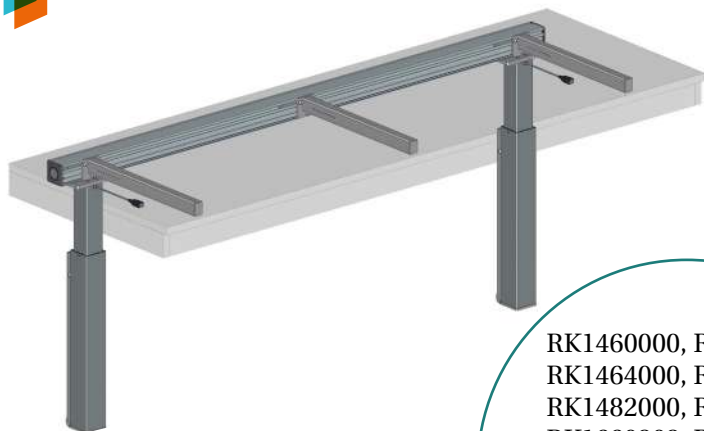
RK1460000, RK1462000
RK1464000, RK1480000
RK1482000, RK1484000
RK1660202, RK1662202
RK1664202, RK1680202
RK1682202, RK1684202
RK1860202, RK1862202
RK1864202, RK1880202
RK1882202, RK1884202


Indivo Pack – Operation and maintenance manual	en
Indivo Pack – Brugs- og vedligeholdelsesvejledning	dk
Indivo Pack – Gebrauchs- und Pflegeanleitung	de
Indivo Pack – Manuel d'utilisation et d'entretien	fr
Indivo Pack – Gebruiks- en onderhoudshandleiding	nl
Indivo Pack – Bruks- och underhållsanvisning	sv
Indivo Pack – Bruks- og vedlikeholdsveiledning	no
Indivo Pack – Instrucciones de uso y mantenimiento	es
Indivo Pack – Manuale d'uso e manutenzione	it
Indivo Pack – käyttö- ja huolto-opas	fi
Indivo Pack – Instrukcja obsługi i konserwacji	pl

English	7
Dansk	19
Deutsch	31
Français	43
Nederlands	55
Svenska	67
Norsk	79
Español	91
Italiano	103
Suomi	115
Język polski	127


Pressalit's Environmental and Quality Management Systems are certified by LRQA

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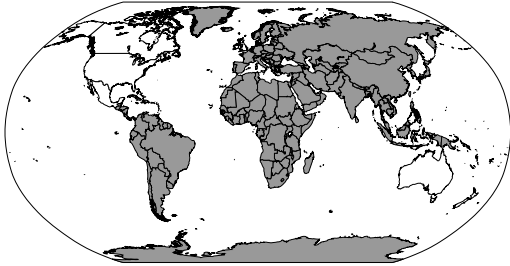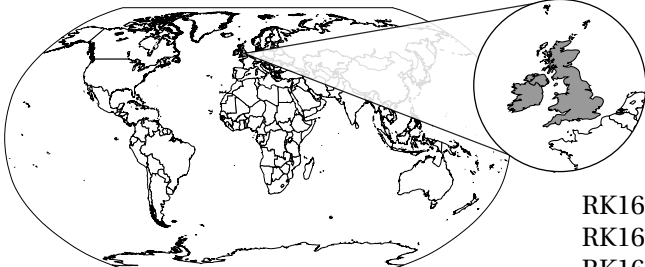
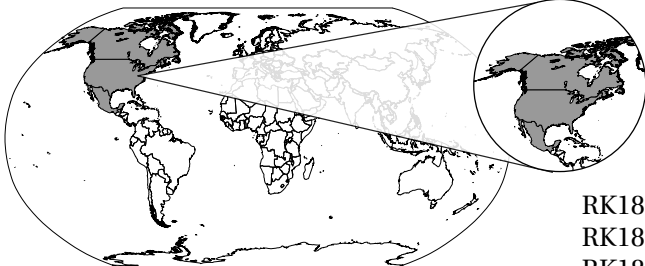
120V


RK1860202, RK1862202
RK1864202, RK1880202
RK1882202, RK1884202


	Max P	U	Max I
RK1460000, RK1462000, RK1464000, RK1480000, RK1482000, RK1484000, RK1660202, RK1662202, RK1664202, RK1680202, RK1682202, RK1684202	450 W	230 V	2,0 A
RK1860202, RK1862202, RK1864202 RK1880202, RK1882202, RK1884202	450 W	120 V	3,8 A

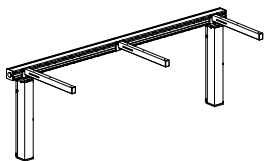Max 100 kg / 220 lbs

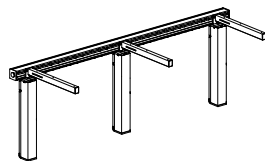
Max 150 kg / 330 lbs

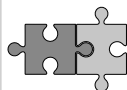RK1043

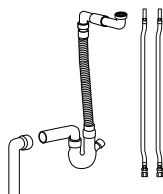
RK1344


RK9061 (EU, US), RK9071 (UK)

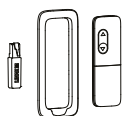RK1342

RK1047

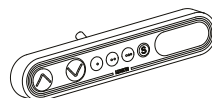
RK1049


RK1054

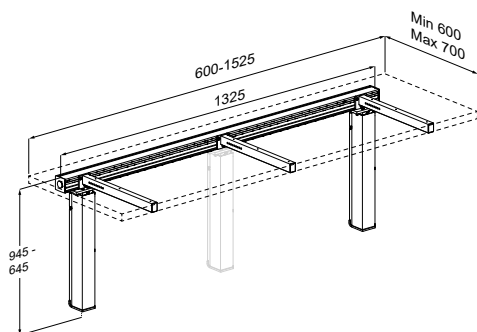
mm

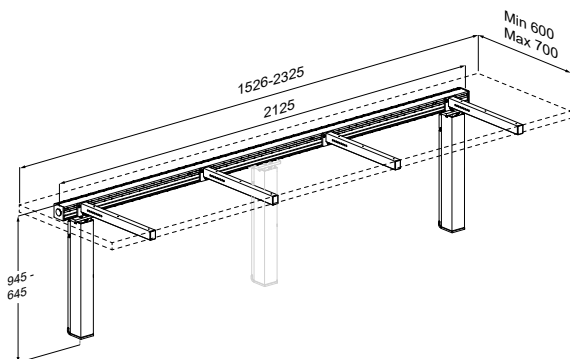RK1460000, RK1660202, RK1860202
RK1480000, RK1680202, RK1880202

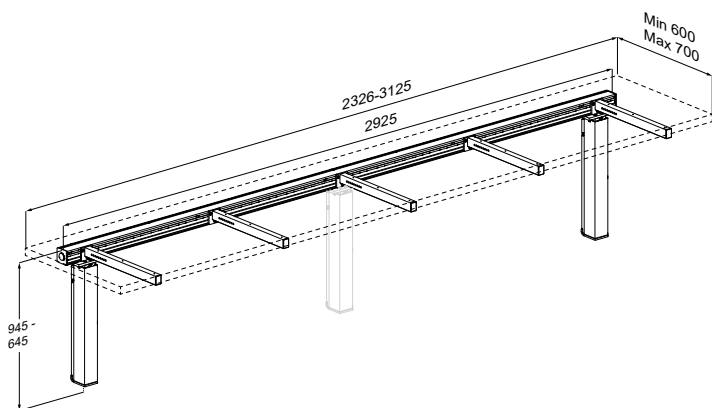
RK1462000, RK1662202, RK1862202
RK1482000, RK1682202, RK1882202


RK1464000, RK1664202, RK1864202
RK1484000, RK1684202, RK1884202

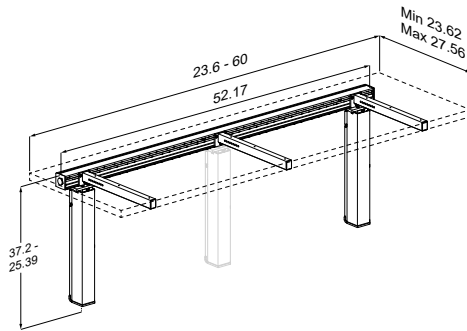
inch

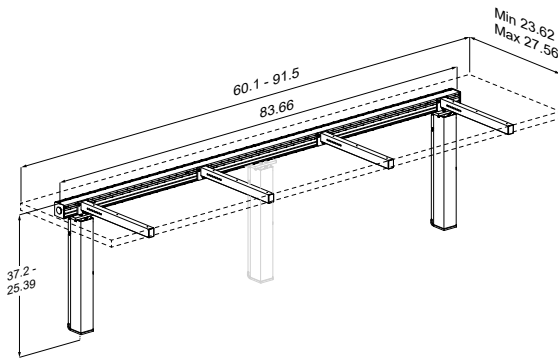RK1460000, RK1660202, RK1860202
RK1480000, RK1680202, RK1880202

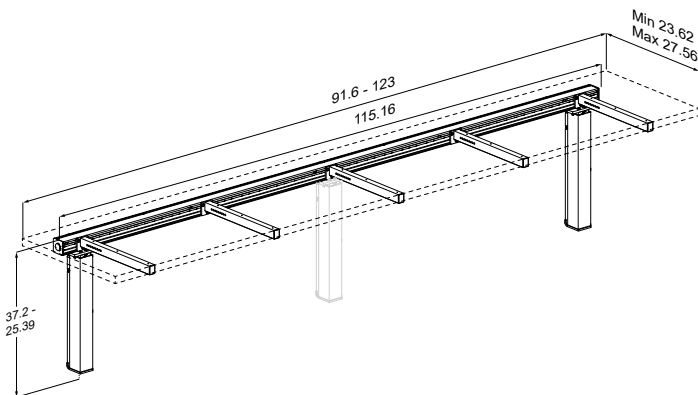RK1462000, RK1662202, RK1862202
RK1482000, RK1682202, RK1882202

RK1464000, RK1664202, RK1864202
RK1484000, RK1684202, RK1884202

Content

Mounting instruction	7
Symbols used in this manual	8
General safety	9
Usage	10
Safety labels	10
Operating instructions	11
Safety system	12
Accessories	13
Maintenance	14
Resetting	14
Cleaning	14
Trouble-shooting	15
Technical data	16
Operating environment	16
Storage and transportation	17
Disposal and recycling	18

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

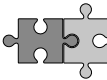
IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

en

		
Compatible products	Power specifications	Safe working load
		
Hand injury warning	Head injury warning	Spare parts

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

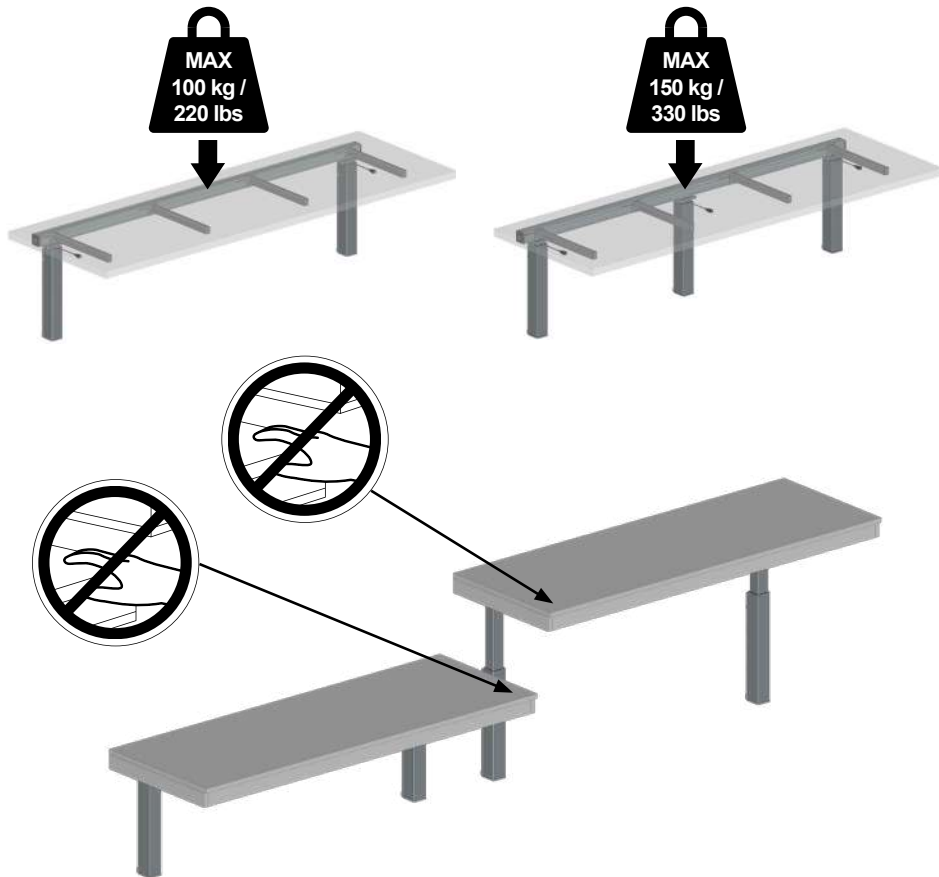
- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Usage

The height adjustable lifting units allow both seated and standing users to work comfortably at the same table. The units are for indoor use only.

Safety labels


Before initial operation, please affix the safety labels supplied along with this booklet.


When two worktops are mounted side by side, ensure that no objects span across both surfaces during height adjustment.

Operating instructions

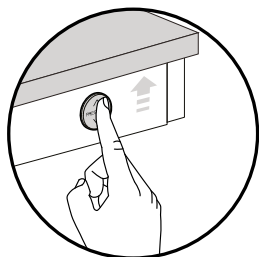The height adjustment is controlled by operating the buttons on the fascia of the worktop. The table can be adjusted 300 mm/ 11,8" up and down.

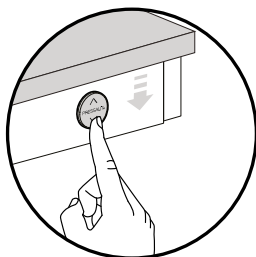Press and hold the up button to raise the table. When the button is released, the motion stops.


Press and hold the down button to lower the table. When the button is released, the motion stops.

WARNING

- Do not leave kitchen appliances, such as a blender or food processor, running while the worktop is being height-adjusted.
- It is recommended that heavy items that are permanently positioned on the work top are placed closest to the wall.

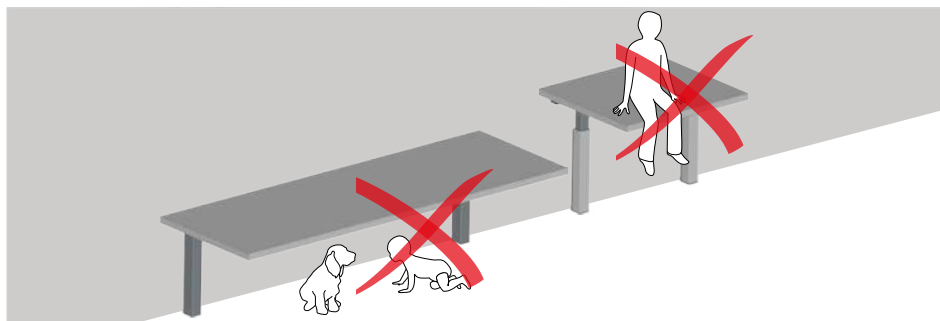en

CAUTION

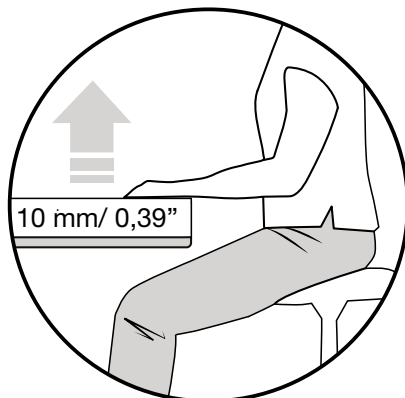
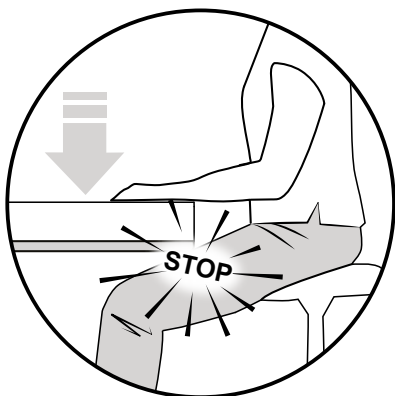
This product is not for play. Be cautious of moving parts. Keep the table clear of objects and pets. Never allow children or pets to remain under the table.

Take care to avoid injury to hands and head when in the vicinity of moving mechanical parts.


Safety system

The table lifting unit is equipped with a safety rail to prevent crushing injuries. If the safety rail under the table comes in contact with an object when the table is being lowered, the motion stops immediately, and the tabletop automatically rises 10 mm.

Accessories

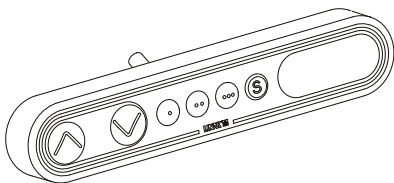The INDIVO Pack is compatible with various accessories, which are illustrated on the accessories page in this manual. One notable accessory is the DESKLINE Desk Panel DPF1C.

A mounting instruction is included with the accessories and can also be downloaded from pressalit.com.

The DESKLINE Desk Panel offers two primary functions: Parallel drive and memory drive.

Parallel Drive

The arrow buttons allow for manual adjustment of the lift height.

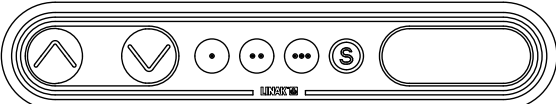
Memory Drive

The panel supports saving and recalling up to three predefined height settings. This function is suitable for repeated use of specific positions.

Height Display

The display shows the current height in centimeters or inches, allowing for accurate monitoring of the lift position.

en

	Up	
	Down	
•	Memory 1	
••	Memory 2	
•••	Memory 3	
	Store memory	


Error codes	Description
E01	The desk has an unknown position and needs to be initialised.
E02	Overload in upward direction has occurred.
E03	Overload in downward direction has occurred.
E16	Illegal keypress

Maintenance


It is recommended to adjust the table monthly by raising it to its maximum height and then lowering it to its lowest position. This ensures that the internal components do not seize up. Then reset the lifting mechanism.

Check that the safety rail is fully functional and securely fastened to the table. Inspect for defective parts or cables, and ensure that they are replaced to maintain functionality and safety. Replacements must only be performed by authorized personnel. Contact your local dealer or Pressalit A/S.

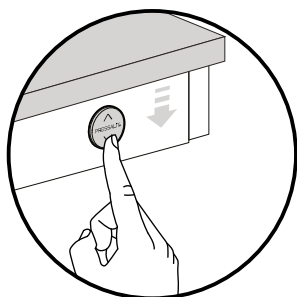
Resetting

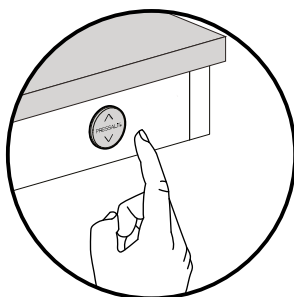Hold in the down button until the table top stops in the lowest position.

Release the button.

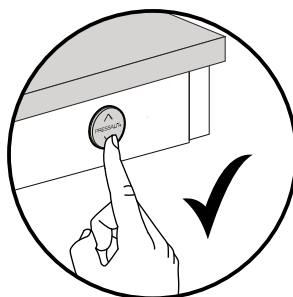Press and hold in the down button once more for 5 seconds. The table is now reset.

Cleaning

Clean with soapy water and dry using a soft cloth. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit Pressalit.com and refer to the product's item number.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive or corrosive cleaning or disinfecting products.
- Avoid the use of chlorine solution higher than up to 1500 ppm.

Trouble-shooting

Symptom	Action
The height of the table cannot be adjusted	• Safety components check Check that the safety rail under the table has not been activated. • Obstruction check If a sink and installation cover have been fitted, check that the installation cover does not collide with anything. • Operation button check Ensure the operation button operates without obstruction and that it returns to a neutral position. Movement may be hampered by dirt or grease. • Reset the lifts Perform a reset of the lifting mechanism. See the section “Resetting” under Maintenance for detailed instructions. It may be necessary to reset it several times. • Power and connections check Check that all control boxes are receiving power and that all plugs are securely connected. The control box is located under the tabletop. • Power cycle If the problem persists, disconnect the power. Wait for 5 seconds before reconnecting the power. After reconnection, perform a reset.
The table top stops and will not go up any further.	Check that the table top is not overloaded. If it is, remove some of the weight. Then reset the table top – see the section “Resetting” under Maintenance.
Safety rail does not function.	Contact supplier.

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Technical data



Materials	Powder laquered steel Aluminium Plastics Electronics	
Weight	RK1460000, RK1660202, RK1860202	32,46 kg / 71,56 lbs
	RK1462000, RK1662202, RK1862202	35,55 kg / 78,37 lbs
	RK1464000, RK1664202, RK1864202	42,18 kg / 92,99 lbs
	RK1480000, RK1680202, RK1880202	43,9 kg / 96,78 lbs
	RK1482000, RK1682202, RK1882202	45,2 kg / 99,65 lbs
	RK1484000, RK1684202, RK1884202	52,6 kg / 115,96 lbs
Noise level	< 55 dB(A)	

en

Operating environment

Environmental factor	Operating range
Temperature	10°C to 40°C (50°F to 104°F)
Relative humidity	20% to 80% @ 30°C (86°F)
Atmospheric pressure	700 to 1060 hPa
Meters above sea level	Max. 2000 m / 6 600 feet
Duty cycle	2 minutes of continuous use, followed by an 18-minute pause.

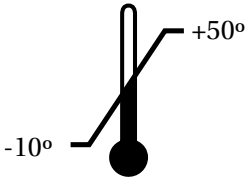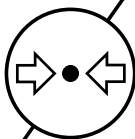Storage and transportation

 -10° +50°	The temperature for transportation and storage of the unit must be between -10 and 50 degrees Celsius / 14 and 122 degrees Fahrenheit.
 20 % 80 %	The relative humidity for transportation and storage of the unit must be between 20 and 80% (RH).
 70 kPa 106 kPa	The air pressure for transportation and storage of the unit must be between 70 and 106 kPa.

IMPORTANT

- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product:

Recyclable components	Material
Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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