

Product Item number	SELECT TL4 toilet lifter, manual, for wall outlet R9024112		
	VVS:617989112		
	NRF:6191185		
	EAN:5708590372983		
	HMI: 138247		
	FDA: IKX/D319729		
Color	Anthracite gray		
Product description	SELECT toilet lifts are available in several models and colors for both wall outlet ar floor outlet. They are available in electrically or manually height-adjustable versions with or without side profiles and for standard toilet bowls and shower toilets. All developed with a focus on hygiene and easy cleaning.		
	SELECT TL4 toilet lifter, for wall outlet. Incl. cUPC-approved cistern with dual flushing system preset at 0.8/1.6 gallons (ca be changed to 0.5/1.1 gallons). Manually height adjustable 8" with crank handle. Suitable for douche toilet or douche toilet seat. Toilet pan, toilet seat and fold down grab bars to be ordered separately. Materials: Stainless steel and PMMA/ABS. Excl. wall mounting kit.		
Adjustability	Height adjustable		
Adjustment height	7,88 inch		
Product capacity	Maximum user weight: 440 lbs. Maximum load: 518 lbs. (user weight + accessories). When the toilet lifter is loaded with 518 lbs, each wall fastener is loaded with 143 lbs The toilet lifter has been tested statically with 1320 lbs, which is a requirement according to the applied assistive products standards. Please note that most toilets compatible with this lifter have been tested up to 880 lbs.		
Materials	The toilet lifter is constructed of stainless steel, the cover fairing is made of PMMA/ABS plastics.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a		



Product	SELECT TL4 toilet lifter, manual, for wall outlet R9024112		
ltem number			
Cleaning			
	This product should be cleaned with hot water or a mild cleaning solution. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.		
	For surface disinfection, chlorine-based disinfectants may be used according to th product specification and safety precautions for these products.		
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.		
	DISINFECTANTS:		
	Clinimax Difficil-S Erisan OXY+		
	Sprit (husholdningssprit)		
	Suma Bac D10		
	Virkon Rodalon Desinfektionsspray		
	Lysoformin Spezial		
	CaviCide		
	Sufanios Premium Tvättsprit 70%		
	NON-DISINFECTANTS: Suma Star free D1		
	Taski 100		
	Taski 200		
	Milizid Citro		
	The product must be wiped down with a soft rag wrung out in clear water immedia		
	after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.		



Product

Item number

SELECT TL4 toilet lifter, manual, for wall outlet R9024112

### Mounting in different wall types

<b>Concrete &gt;B25</b> V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
Solid Brick >= Mz12 V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
Solid Sand Lime Brick >= HIz12 V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with grey covers. To be ordered separately.	
Aerated Concrete P4 V8655 - Mounting kit, 4 lag bolts Ø10 x 70, plugs GB14, with grey covers. To be ordered separately.	
Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8605 - Mounting kit, 4 lag bolts Ø10 x 70, plugs SX12, with white covers. NB! Plugs are not needed. To be ordered separately.	



Product Item number	SELECT TL4 toilet lifter, manual, for wall outlet R9024112	
	<b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with</b> <b>22 mm plywood</b> Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch) Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.	
	<b>Drywall (2 x 12,5 mm) water-resistant</b> Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	<u>.</u>

#### pressalit.com

Pressalit A/S Pressalitvej 1 DK-8680 Ry Tel.: +45 8788 8788 Fax: +45 8788 8789 E-mail: pressalit@pressalit.com

